

- 1. Press and hold down on cross bar (circled) to move depth stop position.**




**2. Pull depth stop entirely out of stop mechanism.**



**3. Remove black irrigation clip from back of head.**



**4. Twist neck of handpiece about 30 degrees to left or right.**

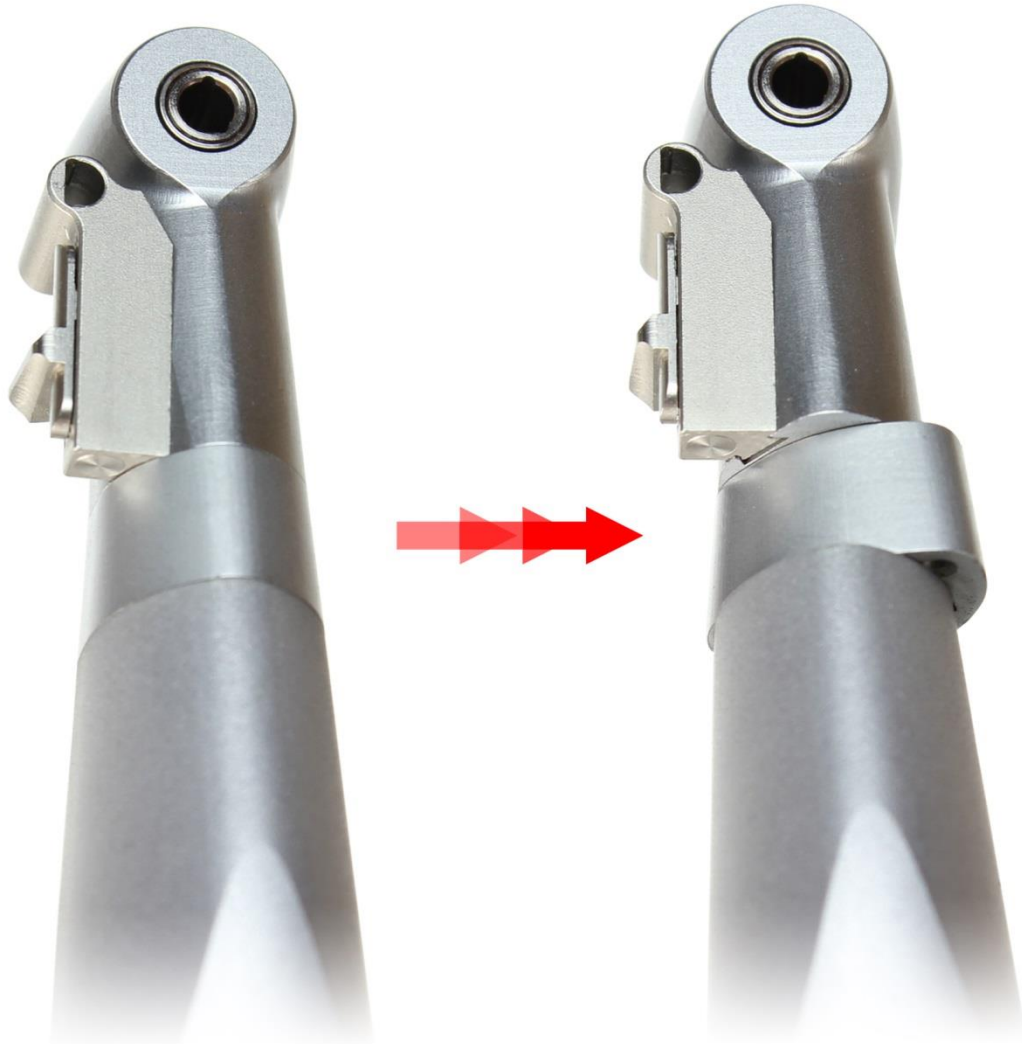

**5. Pull up on head to remove from handpiece body.**



## 6. Slide depth stop mechanism down to remove from head.



**7. Reattach head and drive shaft to handpiece body, be sure to align correctly.**



**8. With head and drive shaft reattached, twist neck back to center.**





**11. Replace black irrigation clip. Note the upside-down “U” shaped channel where the depth stop mechanism was installed.**



## ***To Reattach***

**To reattach the depth gauge, simply reverse the order of these directions.**



P.O. Box 1548 • Woodinville, WA 98072  
8333 216<sup>th</sup> Street S.E. • Woodinville, WA 98072  
International: (425) 487-3157 • Toll Free: (800) 426-5913  
[www.aseptico.com](http://www.aseptico.com) • [info@aseptico.com](mailto:info@aseptico.com)