Cavitron® Products

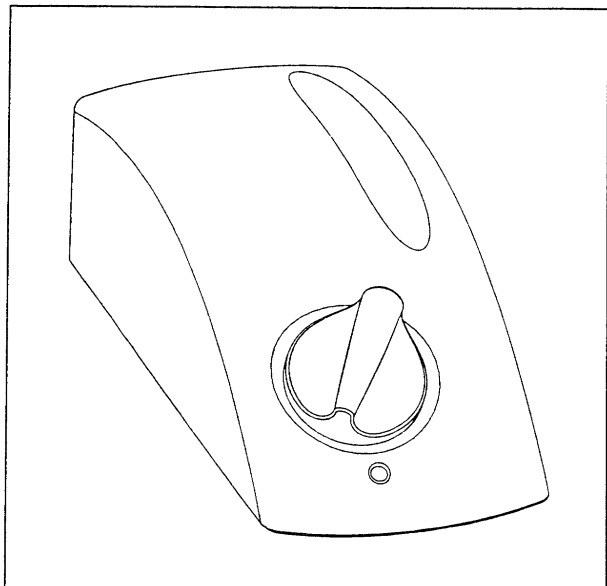
Cavitron® Select Ultrasonic Scaler
Directions For Use

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
Cavitron® Select Ultrasonic Scaler
with Cavitron® Select Reservoir System
Directions For Use

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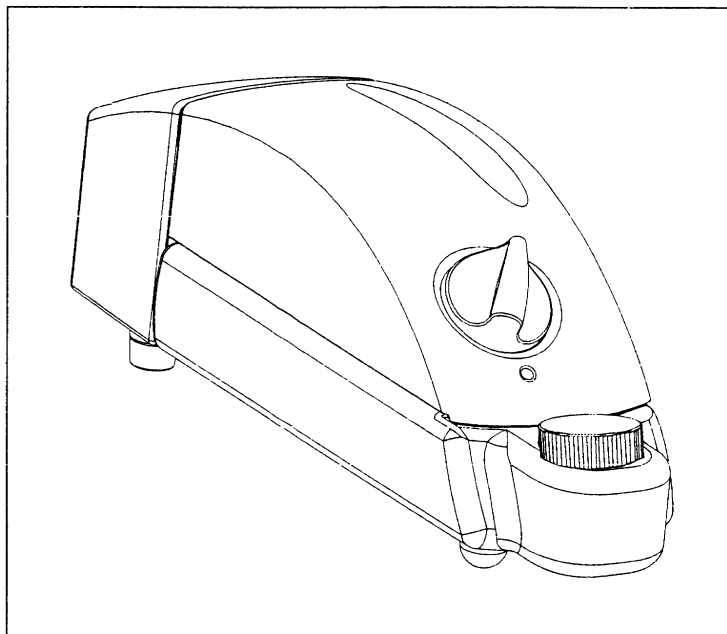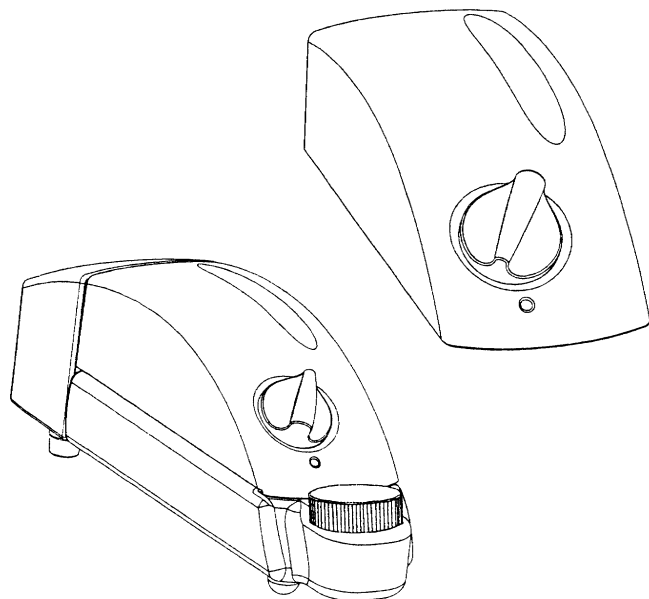
Cavitron® Select Ultrasonic Scaler

Cavitron® Select Ultrasonic Scaler with Cavitron® Select Reservoir System

Directions For Use

Please read carefully and completely before operating unit.

Introduction

Cavitron® Select Ultrasonic Scaler with Cavitron® Select Reservoir System

DENTSPLY® Professional Care is an ISO9001 certified company. The Cavitron® Select System is classified by Underwriters Laboratories Inc. with respect to electric shock, fire, mechanical hazards only in accordance UL 2601-1 1 and Can/CSA C22.2 NO. 601.1.

The Scaling System operates by converting ordinary AC house current into high frequency current. The ultrasonic system consists of two parts: an insert, and the electronic system. The system incorporates two closed loops. One loop provides automatic tuning (operating frequency is adjusted to be at resonance for each insert), the second loop automatically controls the tip stroke over different working conditions. The scaler produces 25,000 microscopically small strokes per second at the insert's working tip. This combined with acoustic effects of the coolant water, produces a synergistic action that literally "powers away" the heaviest calculus deposits while providing exceptional operator and patient comfort.

The Pump/Reservoir accessory for the Cavitron® Select is an optional lavage dispenser. Footswitch plug, handpiece plug, water tube and power supply plug are removed from the Scaler before mating with the Pump/Reservoir. The Scaler is snapped together with the Pump/Reservoir. The footswitch plug, handpiece plug and power supply plug are inserted into mating receptacles at the rear of the Pump/Reservoir. Ultrasonic signal, footswitch and DC power connections between the units are made via an edgcard connector at the rear of the Scaler and the

front of the Pump module. The Reservoir has a capacity of 450 ml. Fluid from the Reservoir and is propelled from the Reservoir Bottle with 15–19 psig compressed air generated by an internal diaphragm pump. ON/OFF of the lavage is controlled through an internal solenoid valve. Fluid to the Handpiece passes directly from the Pump/Reservoir to the Handpiece. No fluid passes through the Cavitron® Select when used with this accessory. Flow through any insert in the Handpiece will be greater than 40 cc/m water when adjusted to maximum.

Technical Support

For technical support and repair assistance in the U.S., call 1-800-989-8826 Monday through Friday, 8:00 AM to 5:00 PM (Eastern Time). For other areas, contact your local DENTSPLY® representative.

Supplies & Replacement Parts

To order supplies or replacement parts in the U.S., contact your local DENTSPLY® Distributor or call 1-800-989-8826 Monday through Friday, 8:00 AM to 5:00 PM (Eastern Time). For other areas, contact your local DENTSPLY® representative.

Caution: U.S. Federal Law restricts this device to sale by or on the order of a Dentist.

Table of Contents

Section Number	Section Title Description of Contents	Page Number
1	Indications for use	3
2	Contraindications and Warnings	3
	2.1 Contraindications	
	2.2 Warnings	
3	Precautions	3
	3.1 Precautions for All Systems	
	3.2 Precautions for Ultrasonic Prophylaxis Procedures	
	3.3 Precautions for Cavitron® Select Reservoir System	
4	Infection Control	4
	4.1 General Infection Control Recommendations	
	4.2 Water Supply Recommendations	
5	Installation Instructions for Cavitron® Select Ultrasonic Scaler	4-5
	5.1 General Information	
	5.2 Water Line Requirements	
	5.3 Electrical Requirements	
	5.4 Unpacking the System	
	5.5 Power Cord/Power Connection	
	5.6 Foot Control Cable Assembly Connection	
	5.7 Water Supply Line Connection	
	5.8 Lavage Control Detachable Handpiece Cable Assembly Connection	
6	Installation Instructions for Cavitron® Select Scaler with Cavitron® Select Reservoir System	6-7
	6.1 General Information	
	6.2 Electrical Requirements	
	6.3 Unpacking the System	
	6.4 Assembling the System	
	6.5 Power Cord/Power Connection	
	6.6 Foot Control Cable Assembly Connection	
	6.7 Lavage Control Detachable Handpiece Cable Assembly Connection	
7	Cavitron® Select Ultrasonic Scaler Description	8-9
	7.1 System Controls for Cavitron® Select Ultrasonic Scaler	
	7.2 System Controls for Cavitron® Select Scaler with Cavitron® Select Reservoir System	
	7.3 Handpiece	
	7.4 Cavitron® 25K™ Ultrasonic Inserts	
	7.5 Foot Control Information & Operation	
8	Accessories	10
	8.1 Accessories List	
	8.2 Medicaments	
9	Techniques for Use	10
	9.1 Patient Positioning	
	9.2 Performing Ultrasonic Scaling Procedures	
	9.3 Patient Comfort Considerations	
10	System Care	11-12
	10.1 Daily Maintenance — Start-Up Procedures, Between Patients, Shut-Down Procedures	
	10.2 Weekly Maintenance	
	10.3 Water Line Filter Maintenance	
	10.4 Pump Lavage Filter Maintenance	
11	Troubleshooting	13
	11.1 Troubleshooting Guide	
	11.2 Technical Support and Repairs	
12	Specifications	14
13	Classifications	14
14	Disposal of Unit	14

Section 1: Indications for Use

Ultrasonic procedures:

- All general supra and subgingival scaling applications.
- Periodontal debridement for all types of periodontal diseases.
- Endodontic procedures.

Section 2: Contraindications and Warnings

2.1 Contraindications

- Ultrasonic Systems should not be used for restorative dental procedures involving the condensation of amalgam.

2.2 Warnings

- Persons fitted with cardiac pacemakers, defibrillators and other active implanted medical devices, have been cautioned that some types of electronic equipment might interfere with the operation of the device. Although no instance of interference has ever been reported to DENTSPLY[®], we recommend that the handpiece and cables be kept at least 6 to 9 inches (15 to 23 cm) away from any device and their leads during use.
- There are a variety of pacemakers and other medically implanted devices on the market. Clinicians should contact the device manufacturer or the patient's physician for detailed information about the device.

Section 3: Precautions

3.1 Precautions for All Systems

- Cavitron[®] Inserts manufactured by DENTSPLY[®] Professional should be used for optimal performance.
- Do not place the System on or next to a radiator or other heat source. Excessive heat may damage the System's electronics. Place the System where air is free to circulate on all sides and beneath it.
- The System is portable, but must be handled with care when moving.
- Equipment flushing and dental water supply system maintenance are strongly recommended. See Sections 4.2: Water Supply Recommendations and 10: System Care.
- Close the water shut-off valve to the System every night before leaving the office.

3.2 Precautions for Ultrasonic Prophylaxis Procedures

- Like a toothbrush, ultrasonic inserts "wear out" with use. Inserts with just 2 mm of wear lose about 50% of their scaling efficiency. In general, it is recommended that ultrasonic inserts be discarded and replaced after 2 mm of wear to maintain optimal efficiency and avoid breakage.
- If excessive wear is noted, or the insert has been bent, reshaped or otherwise damaged, discard the insert immediately.

- Ultrasonic insert tips that have been bent, damaged, or reshaped are susceptible to in-use breakage and should be discarded and replaced immediately.
- Retract the lips, cheeks and tongue to prevent contact with the insert tip whenever it is placed in the patient's mouth.
- Only the medically approved DENTSPLY[®] power supply should be used with this System.

3.3 Precautions for Cavitron[®] Select Reservoir System

- Do not place the System on or next to a radiator or other heat source. Excessive heat may damage the System's electronics. Place the System where air is free to circulate on all sides and beneath it.
- The System is portable, but must be handled with care when moving.
- Equipment flushing and dental water supply system maintenance are strongly recommended. See Sections 4.2: Water Supply Recommendations and 10: System Care.
- Pour liquid into the cap opening of the reservoir bottle. To prevent potential leaking at the cap, do not overfill bottle with liquid into the neck of the bottle. Ensure cap is tightly sealed.
- Turn unit off before refilling reservoir bottle to prevent potential leaking of fluid. Turn cap a 1/4 revolution to release air pressure and then remove the cap completely.

Section 4: Infection Control

4.1 General Infection Control Recommendations

- As with all dental procedures, use standard personal protection equipment (i.e., wear face mask, eyewear, or face shield, gloves and protective gown).
- For maximal operator and patient safety, carefully follow the Infection Control Information procedures detailed in the Cavitron® Infection Control Information booklet enclosed with your system, Part Number 81158.
- As with high speed handpieces, and other dental devices, the combination of water and ultrasonic vibration from your Cavitron® Select Scaler will create aerosols. With proper technique, much of the Cavitron® Select Scaler's aerosol dispersion can be effectively controlled and minimized. Please carefully follow the procedural guide lines in this manual regarding the use of your System.

4.2 Water Supply Recommendations

It is highly recommended that all dental water supply systems conform to applicable CDC (Centers for Disease Control and Prevention) and ADA (American Dental Association) standards, and that all recommendations be followed in terms of flushing, chemical flushing, and general infection control procedures.

Section 5: Installation Instructions for Cavitron® Select Ultrasonic Scaler

5.1 General Information

If the installation of your Cavitron® Select Scaler is performed by someone other than trained DENTSPLY® Distributor personnel, care should be taken to observe the following requirements and recommendations.

5.2 Water Line Requirements

- Incoming water supply line pressure to the Scaler must be 20 psi (138 kPa minimum) to 40 psi (276 kPa maximum). If your dental water system's supply line pressure is above 40 psi, install a water pressure regulator on the water supply line to your Cavitron® Select Scaler.
- A manual shut-off valve on the dental water system supply line should be used so that the water can be completely shut-off when the office is unoccupied.
- After the above installations are completed on the dental water supply system, the dental office water line should be thoroughly flushed prior to connection to the Scaler.

5.3 Electrical Requirements

Refer to Section 12: Specifications.

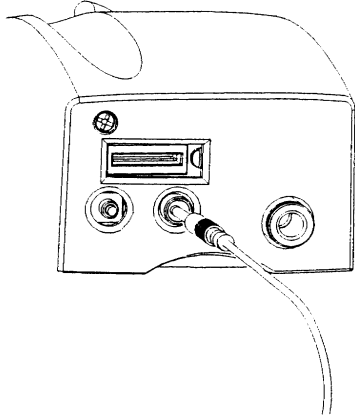
5.4 Unpacking the System

Carefully unpack your Cavitron® Select Scaler and verify that all components and accessories are included:

1. Cavitron® Select Scaler.
2. Detachable AC Power Cord set.
3. Cavitron® Select Lavage Control Detachable Handpiece Cable Assembly.
4. Foot Control Assembly.
5. Hose / Filter / Quick Disconnect Assembly.
6. Additional Water Line Filter.
7. Additional Pump Lavage Filters.
8. Medically Approved Power Supply.
9. Literature Packet.


5.5 Power Cord/Power Connection

- Verify the Power Control knob is set to the OFF position before proceeding.
- Plug the detachable AC cord into the back of the Medically Approved Power Supply.
- Plug the power supply outlet into the back of the System.
- Plug the 3 prong grounded plug into an approved outlet.

5.6 Foot Control Cable Assembly Connection

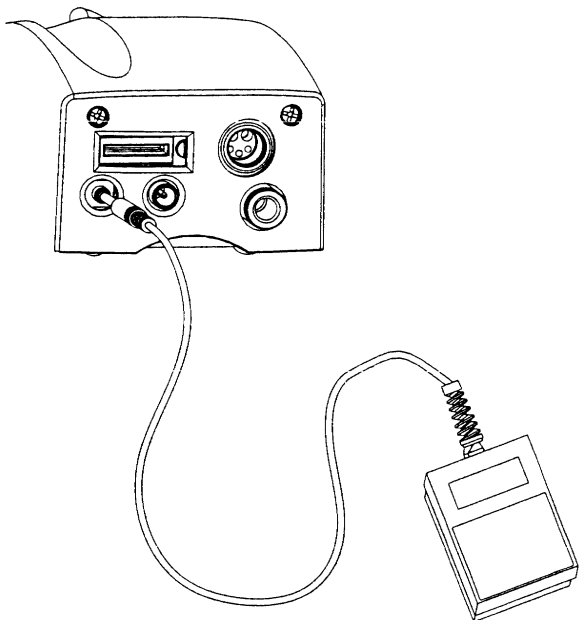Align the Foot Control plug with the receptacle on the back of the System and push it in until firmly seated.


5.7 Water Supply Line Connection

- Push the blue hose into the orange water receptacle until the hose cannot be pushed in any further.
- Connect the free end of the system's water supply line to the office water supply line.
- Inspect all connections to make certain there are no leaks.
- To remove water line from Cavitron® Select Scaler, turn off or disconnect water supply line FIRST, then push on outer orange ring of the outlet and gently pull out water hose.

5.8 Lavage Control Detachable Handpiece Cable Assembly Connection

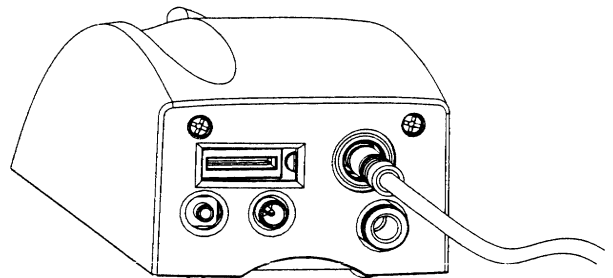- Plug the Handpiece outlet into the receptacle on the back of the System.
- Holding the Handpiece Cable Assembly seat male connector into appropriate female connector.
- Once seated, snap male connector into female connector by pushing on the end of the male connector.
- Rotate Lavage Control knob on the handpiece to middle setting.
- Turn on the on/off switch (power control knob).

Section 6: Installation Instructions for Cavitron® Select Scaler with Cavitron® Select Reservoir System

6.1 General Information

If the installation of your Cavitron® Select Ultrasonic Scaler with Cavitron® Select Reservoir System is performed by someone other than trained DENTSPLY® Distributor personnel, care should be taken to observe the following requirements and recommendations.

6.2 Electrical Requirements

Refer to Section 12: Specifications.

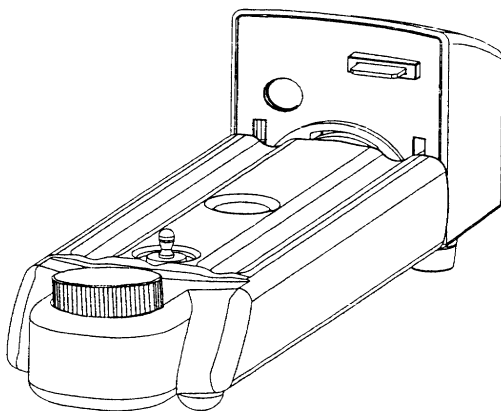
6.3 Unpacking the System

Carefully unpack your Cavitron® Select Scaler and verify that all components and accessories are included:


1. Cavitron® Select Scaler.
2. Cavitron® Select Reservoir Pump.
3. Detachable AC Power Cord set.
4. Cavitron® Select Lavage Control Detachable Handpiece Cable Assembly.
5. Foot Control Assembly.
6. Cavitron® Select Reservoir Bottle.
7. Hose / Filter / Quick Disconnect Assembly.
8. Additional Water Line Filter.
9. Additional Pump Lavage Filters.
10. Medically Approved Power Supply.
11. Literature Packet.

6.4 Assembling the System

- Remove protective sticker from edge card connector.
- Rinse Cavitron® Select Reservoir Bottle before using.
- Insert Cavitron® Select Reservoir Bottle into Cavitron® Select Reservoir Pump and rotate clockwise until bottle locks into place.

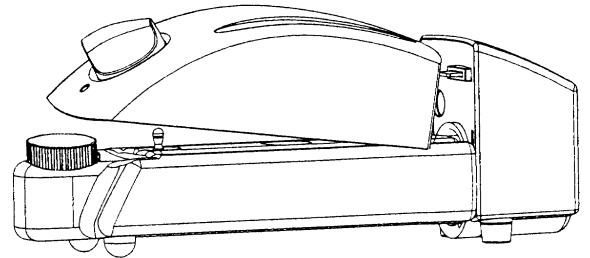
- Connect Cavitron® Select Scaler to Cavitron® Select Reservoir System. Insert male edge card connector on Cavitron® Select Reservoir Pump into Cavitron® Select Scaler opening on the back of the Cavitron® Select Scaler. Press down on the top of the Cavitron® Select Scaler


(directly above power control knob) until the black striker knob located on the Reservoir Bottle locks into place.

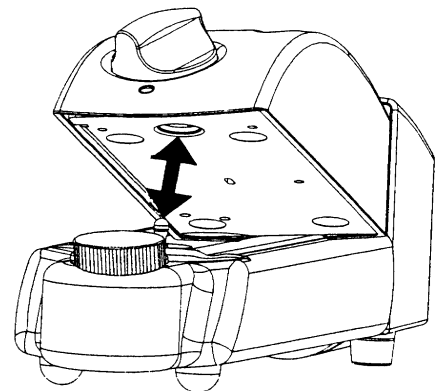- To detach Cavitron® Select Scaler from Cavitron® Select Reservoir Pump and Bottle, lift up on the front of the Cavitron® Select Scaler (below power control knob and

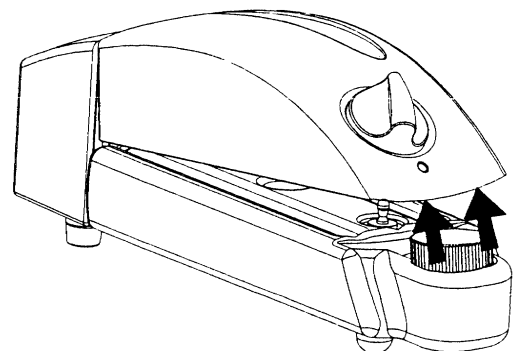
power indicator light). After the striker knob is unsnapped, pull the Cavitron® Select Scaler away from the Cavitron® Select Reservoir Pump.

NOTE: Always disengage striker knob before pulling the Cavitron® Select Scaler away from the Cavitron® Select Reservoir Pump.


6.5 Power Cord/Power Connection

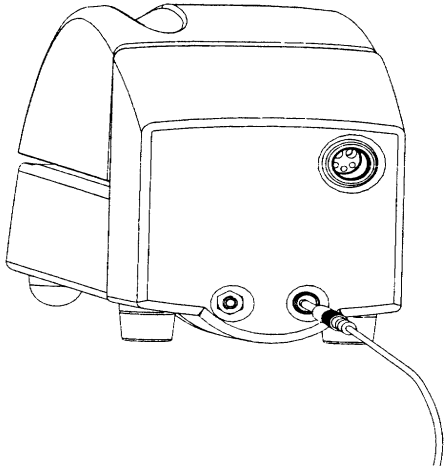- Verify the Power Control knob is set to the OFF position before proceeding.
- Plug the detachable AC cord into the back of the Medically Approved Power Supply.
- Plug the power supply outlet into the back of the Cavitron® Select Reservoir Pump.
- Plug the 3 prong grounded plug into an approved outlet.


6.6 Foot Control Cable Assembly Connection

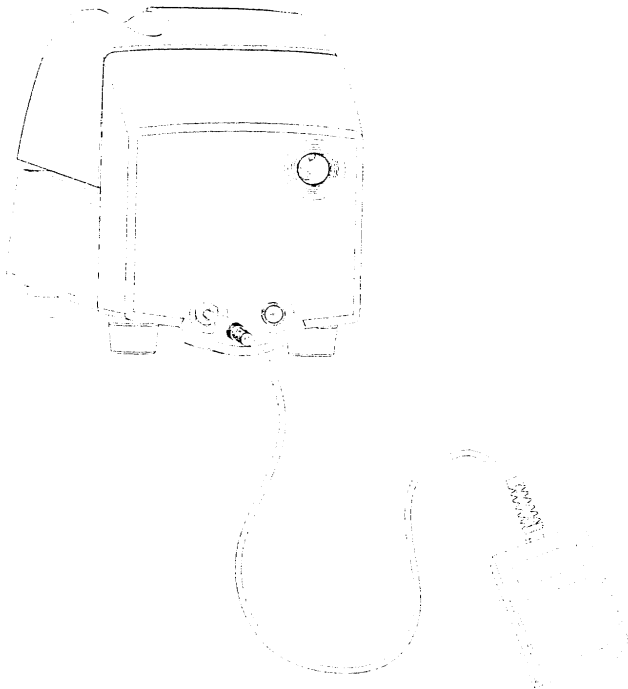Align the Foot Control plug with the receptacle on the back of the Cavitron® Select Reservoir Pump and push it in until firmly seated.


6.7 Lavage Control Detachable Handpiece Cable Assembly Connection

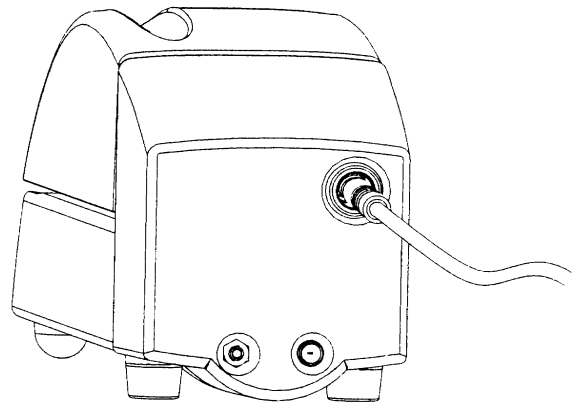- Plug the Handpiece outlet into the receptacle on the back of the Cavitron® Select Reservoir Pump.
- Holding the Handpiece Cable Assembly seat male connector into appropriate female connector.
- Once seated, snap male connector into female connector by pushing on the end of the male connector.
- Rotate Lavage Control knob on the handpiece to middle setting. Turn on the on/off switch (power control knob). This energizes the system and the ultrasonic unit is ready for operation.


Section 7: Cavitron® Select Ultrasonic Scaler Description

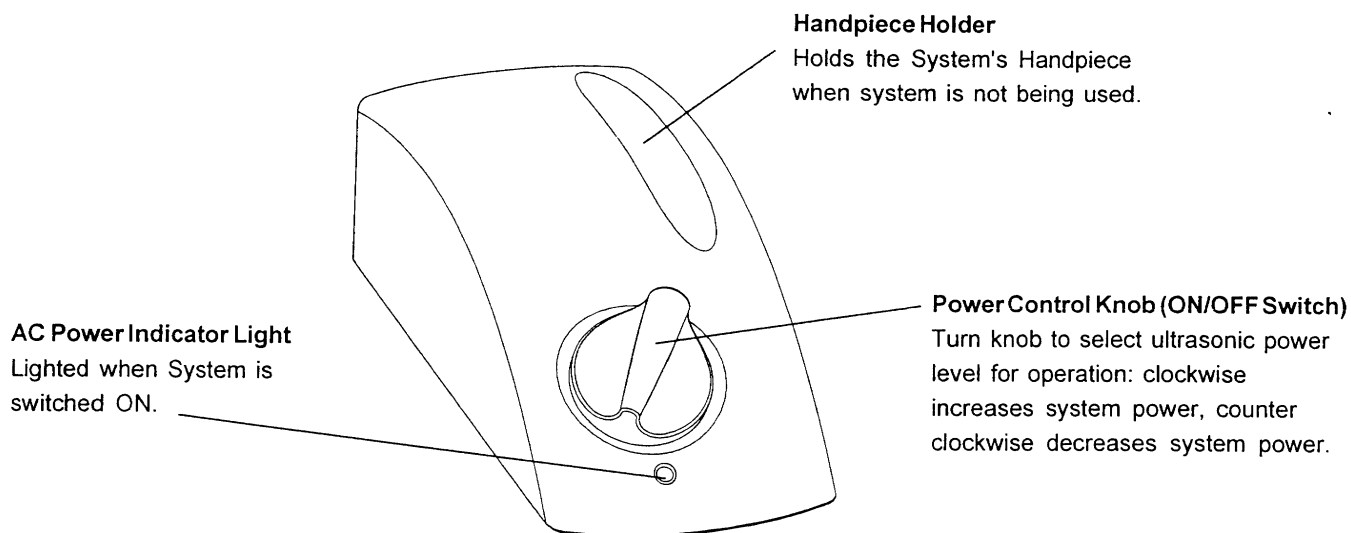7.1 System Controls: Cavitron® Select Ultrasonic Scaler

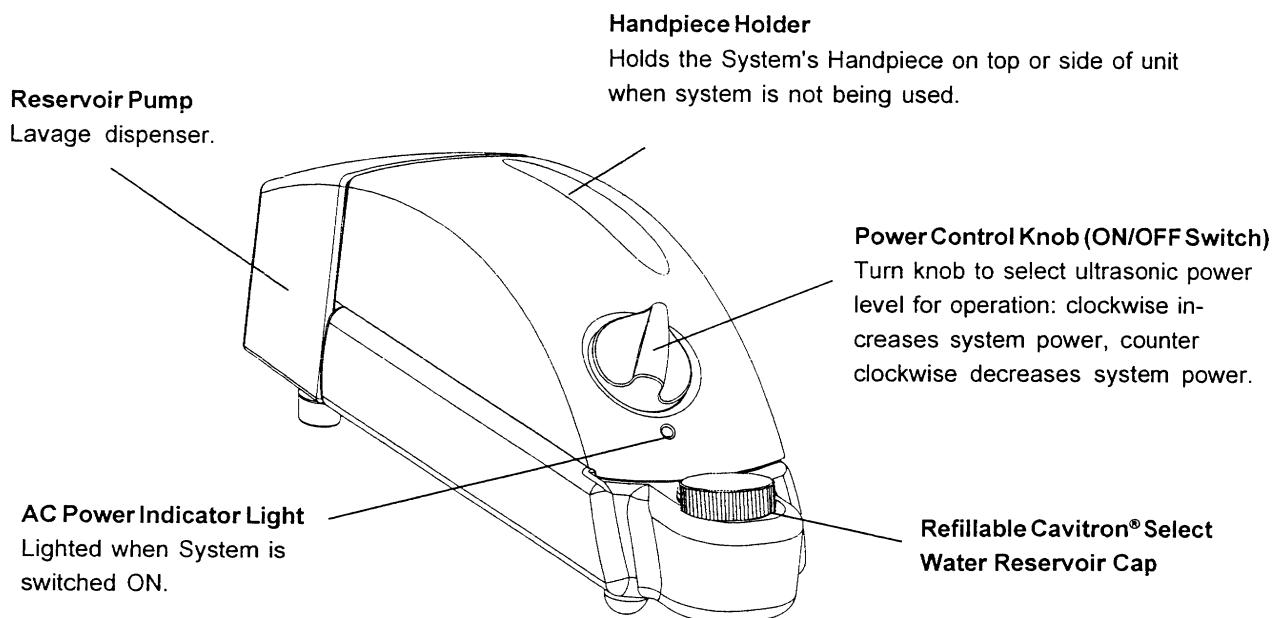
7.2 System Controls: Cavitron® Select Ultrasonic Scaler with Cavitron® Select Reservoir System

7.3 Handpiece

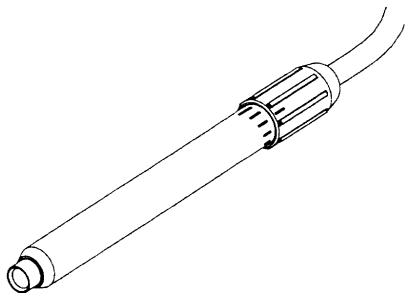The open end of the handpiece accepts all Cavitron® 25K™ Inserts. Holding open end of handpiece upright, lavage can be increased by holding the handpiece and gently twisting the blue Lavage Control knob clockwise.


NOTE:

Do not disassemble or autoclave handpiece. Refer to Section 10: System Care for cleaning instructions.

7.4 Cavitron® 25K™ Ultrasonic Inserts

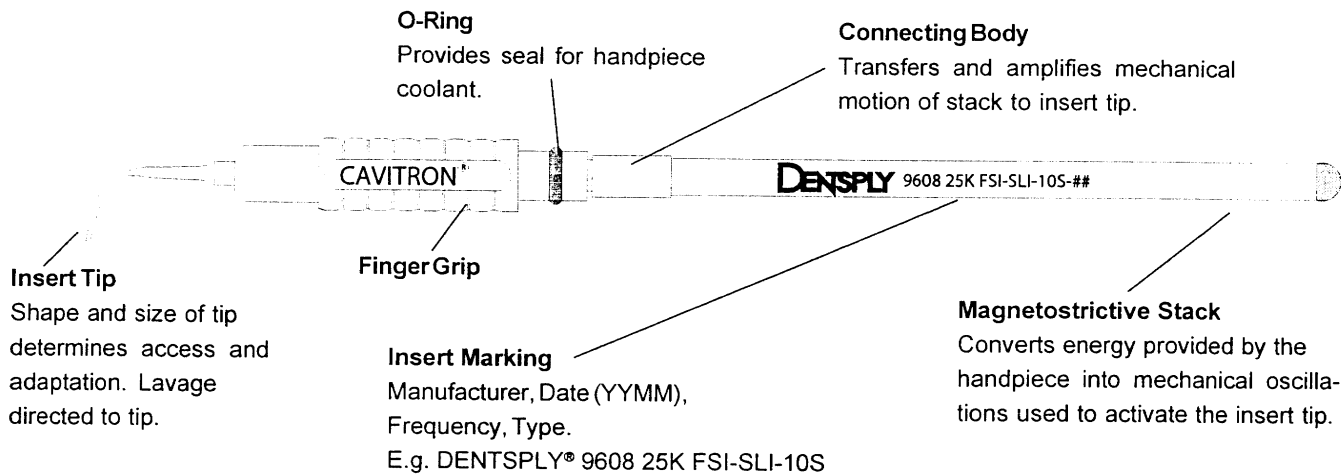The many styles of Cavitron® 25K™ Ultrasonic Inserts are easily interchangeable for various procedures and applications. See enclosed booklet for specific information.


Hold the handpiece in an upright position. Activate the foot control, fill handpiece up with water to bleed any air bubbles that might be trapped inside the handpiece. Lubricate the rubber O-ring on the insert with water before placing it into the handpiece. Fully seat insert with a gentle push-twist motion. **DO NOT FORCE.**

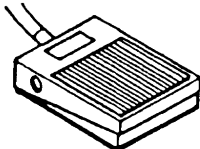7.5 Foot Control Information & Operation

The Foot Control is a single-position momentary switch, which activates both ultrasonic energy and lavage flow at the insert tip.

<p>Foot Control released...</p> 	<p>Foot Control depressed.</p> 
<p>Both ultrasonic activation and irrigating flow stop.</p> 	<p>The ultrasonic insert tip is activated and irrigating water flows.</p> 

Section 8: Accessories

8.1 Accessories List

1. Cavitron® Select Lavage Control Detachable Handpiece Cable Assembly
2. Medically Approved Power Supply
3. Cavitron® Ultrasonic Inserts
4. Foot Control Assembly
5. DualSelect Dispensing System
6. Cavitron® Select Reservoir Bottle
7. Cavitron® Select Reservoir Pump

*For detailed information, contact your local DENTSPLY® Representative or authorized DENTSPLY® Distributor.

8.2 Medicaments


The following medicaments are compatible with the Cavitron® Select with Cavitron® Select Reservoir Pump:

- Chlorhexidine Gluconate 0.12% (Oris™ CHX Antimicrobial Irrigating Solution)
- Cetylpyridinium Chloride (Cepacol®)
- Essential Oils (Listerine)
- Hydrogen Peroxide, 3% USP
- Povidone Iodine, 10% Solution
- Saline Solution
- Sanguinaria Extract (Viadent®)
- Sterile Water, USP (For irrigating and System maintenance purposes)

Section 9: Techniques for Use

9.1 Patient Positioning

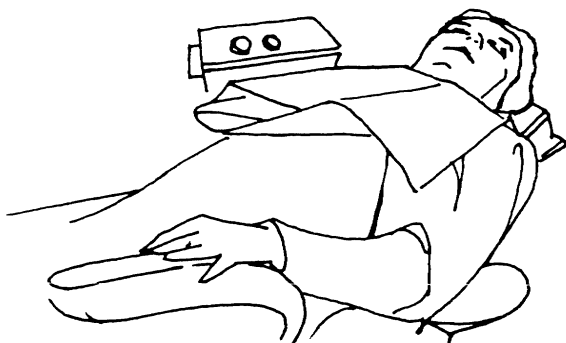- For optimal access to both the upper and lower arches, the backrest of the chair should be adjusted as for other dental procedures. This assures patient comfort and clinician visibility.

- Have the patient turn their head to the right or left. Also position their chin up or down depending upon the quadrant and surface being treated. Evacuate irrigant using either a saliva ejector or High Volume Evacuator (HVE).

9.2 Performing Ultrasonic Scaling Procedures

- NOTE: Refer to the Infection Control Information booklet supplied with your System for general procedures to be followed at the beginning of each day and between patients.
- The edges of Cavitron® Ultrasonic Inserts are intentionally rounded so there is little danger of tissue laceration with proper ultrasonic scaling technique. Whenever the insert tip is placed in the patient's mouth, the lips, cheek and tongue should be retracted to prevent accidental prolonged contact with the activated tip.
- Turn unit on.

- Hold the empty Handpiece in an upright position. Activate the Foot Control until fluid exits. Once fluid exits, deactivate foot controller.
- Lubricate the rubber O-ring on the insert with water before placing it into the handpiece. Fully seat insert with a gentle push-twist motion. DO NOT FORCE.
- Activate the Foot Control. Hold the handpiece over a sink or drain. Check spray temperature to verify fluid is reaching the working end of the insert tip. Adjust the Lavage Control knob to ensure adequate flow for the selected power setting. Greater flow settings provide cooler irrigant.
- In general, it is suggested that a “feather-light-touch” be used both supra- and subgingivally. The motion of the activated tip and acoustic effects of the irrigating fluid, in most cases, is adequate to remove even the most tenacious calculus.
- Periodically check the Cavitron® Ultrasonic Insert for wear with the DENTSPLY® Insert Efficiency Indicator.
- The use of a saliva ejector or High Volume Evacuator (HVE) is recommended during all procedures.
- Set the System's Power Control knob to the lowest power setting for the application and the selected insert.

9.3 Patient Comfort Considerations

Reasons for sensitivity

- Incorrect tip placement. Point should be directed away from root surfaces.
- Not keeping tip in motion on tooth. Do not allow the insert to remain in a static position on any one area of the tooth. Change the insert's path of motion.
- Applying pressure. Use extremely light grasp and pressure, especially on exposed cementum.
- If sensitivity persists, decrease power setting and/or move from the sensitive tooth to another and then return.

Section 10: System Care

It is recommended that you perform the following maintenance procedures to help minimize bio-film formation in the lavage path of your Cavitron Select Ultrasonic Scaler and Cavitron Select Reservoir dispensing system which could affect the lavage flow to the ultrasonic insert tip and scaling performance.

10.1 Daily Maintenance

Start-Up Procedures at the beginning of the day (Cavitron® Select Ultrasonic Scaler):

1. Open the manual shut-off valve on the dental office water supply system.
2. Turn the System ON using the Power Control knob (see illustration in Section 7). Verify the Power Indicator Light is lighted.
3. Set the Power Control knob to minimum power.
4. Set the Lavage Control knob on Handpiece to maximum.
5. Hold the Handpiece (without the Cavitron® Insert installed) upright over a sink or drain. Activate the Foot Control and flush the water line for at least 2 minutes. Do NOT exceed 3 minutes.
6. Place a sterilized Cavitron® Insert into the Handpiece. Set the Power and Lavage Control knobs to your preferred operating position.

Start-Up Procedures at the beginning of the day (Cavitron® Select Ultrasonic Scaler with Cavitron® Select Reservoir System):

1. Carefully lock Cavitron® Select Reservoir Bottle into Cavitron® Select Reservoir Pump.
2. Fill Cavitron® Select Reservoir Bottle with tap, distilled water, or medicament. Replace Cavitron® Select Bottle cap.
3. Connect Cavitron® Select Scaler to Cavitron® Select Reservoir System (see Section 6 for complete instructions).
4. Turn the System ON using the Power Control knob (see illustration in Section 7). Verify the Power Indicator Light is lighted.
5. Set the Power Control knob to minimum power.
6. Set the Lavage Control knob on the Handpiece to maximum.
7. Hold the Handpiece (without the Cavitron® Insert installed) upright over a sink or drain. Activate the Foot Control and flush the water line for at least 2 minutes. Do NOT exceed 3 minutes.
8. Place a sterilized Cavitron® Insert into the Handpiece. Set the Lavage Control knob and Power Control knobs to your preferred operating position.

Between Patients (Cavitron® Select Ultrasonic Scaler):

1. Remove all ultrasonic inserts used, clean and sterilize.
2. Clean and disinfect the surfaces of the cabinet, Power Cord, Handpiece and cable assembly, Foot Control and cable assembly by applying an approved non-immersion type disinfectant solution* carefully following the instructions provided by the disinfectant solution manufacturer. To clean system, generously spray disinfectant solution on a clean towel and wipe all surfaces. Discard used towel. To disinfect system, generously spray disinfectant on a clean towel and wipe all surfaces. Allow disinfectant solution to air dry.

3. Hold the Handpiece over a sink or drain and flush the water line of the unit as above at maximum water flow for 30 seconds. Do NOT exceed 3 minutes.
4. Place a freshly sterilized insert into the handpiece.

*NOTE: Water-based disinfection solutions are preferred. Some alcohol-based disinfectant solutions may be harmful and may discolor plastic materials.

Between Patients (Cavitron® Select Ultrasonic Scaler with Cavitron® Select Reservoir System):

1. Follow steps 1–3 listed in *Between Patients (Cavitron® Select Ultrasonic Scaler)*.
2. With the power OFF, unscrew cap and fill Cavitron® Select Reservoir Bottle with tap, distilled water, or medicament.
3. Do not overfill Bottle with liquid into the neck of the Bottle. Securely screw cap back on Bottle.
4. Place a freshly sterilized insert into the handpiece.

Shut-Down Procedures at the end of the day (Cavitron® Select Ultrasonic Scaler):

1. Remove all ultrasonic inserts used, clean and sterilize.
2. Turn the System OFF.
3. Clean and disinfect the surfaces of the cabinet, Power Cord, Handpiece and cable assembly, Foot Control and cable assembly by applying an approved non-immersion type disinfectant solution* carefully following the instructions provided by the disinfectant solution manufacturer. To clean system, generously spray disinfectant solution on a clean towel and wipe all surfaces. Discard used towel. To disinfect system, generously spray disinfectant on a clean towel and wipe all surfaces. Allow disinfectant solution to air dry.
4. Close the manual shut-off valve on the dental water supply system.

Shut-Down Procedures at the end of the day (Cavitron® Select Ultrasonic Scaler with Cavitron® Select Reservoir System):

1. Remove Cavitron® Insert from Handpiece.
2. Turn the System OFF.
3. Detach Cavitron® Select Scaler from Reservoir System.
4. Detach Cavitron® Select Reservoir Bottle from Pump (see Section 6 for complete instructions).
5. Empty entire contents of Cavitron® Select Reservoir Bottle.
6. Replace empty Cavitron® Select Reservoir Bottle to Pump.
7. Carefully lock Cavitron® Select Reservoir Bottle into Cavitron® Select Reservoir Pump.
8. Connect Cavitron® Select Scaler to Cavitron® Select Reservoir System (see Section 6 for complete instructions).
9. Turn unit ON. Set Power Control knob to minimum and Lavage Control knob to maximum.
10. Hold the Handpiece (without the Cavitron® Insert installed) upright over a sink or drain.
11. To air purge the water line, activate Foot Control to operate the Cavitron® Select Scaler for at least 30 seconds. Do NOT exceed 3 minutes. (This will eliminate fluid to prevent it from settling in the Cavitron® Select System for extended periods of time. This may also help decrease the

chance of bio-film formations in the Cavitron® Select water lines.)

12. Follow steps 2–4 listed in *Shut-Down Procedures at the end of the day (Cavitron® Select Ultrasonic Scaler)*.

10.2 Weekly Maintenance: Cavitron® Select Ultrasonic Scaler with Cavitron® Select Reservoir System

1. Remove Cavitron® Insert from Handpiece.
2. Turn the System OFF.
3. Detach Cavitron® Select Scaler from Reservoir System.
4. Detach Cavitron® Select Reservoir Bottle from pump (see Section 6 for complete instructions).
5. Empty entire contents of Cavitron® Select Reservoir Bottle.
6. Replace empty Cavitron® Select Reservoir Bottle to pump.
7. Carefully lock Cavitron® Select Reservoir Bottle into Cavitron® Select Reservoir Pump.
8. Connect Cavitron® Select Scaler to Cavitron® Select Reservoir System (see Section 6 for complete instructions).
9. Turn the System ON. Set Power Control knob to minimum and Lavage Control knob to maximum.
10. Hold the Handpiece (without the Cavitron® Insert installed) upright over a sink or drain.
11. To air purge the water line, activate the Foot Control for at least 30 seconds. Do NOT exceed 3 minutes. (This will eliminate fluid to prevent it from settling in the Cavitron® Select System for extended periods of time. This may also help decrease the chance of bio-film formations in the Cavitron® Select water lines.)
12. Turn the System OFF.
13. Prepare a 1:10 hypochlorite (NaOCL) solution by mixing 1 part 5.25% sodium hypochlorite with 10 parts tap water.
14. Remove cap and pour this 1:10 solution into the Cavitron® Select Reservoir Bottle. Do not overfill with solution (see Section 3.3 for complete instructions).
15. Securely replace cap on Bottle.
16. Turn the System ON. Set Power Control knob to minimum and Lavage Control knob to maximum.
17. Hold the Handpiece (without the Cavitron® Insert installed) upright over a sink or drain.
18. Activate the Foot Control for at least 10 seconds. Do NOT exceed 60 seconds.
19. Turn the System OFF.
20. *Leave the unit undisturbed for at least 10 minutes, BUT FOR NO MORE THAN 30 MINUTES to allow sodium hypochlorite solution to soak into the lines. As a suggestion, place a sign which states that the System is being disinfected and should not be used.*
21. Unscrew cap and discard disinfectant solution.

22. Fill Cavitron® Select Reservoir Bottle with tap or distilled water. Do not overfill with liquid (see Section 3.3 for complete instructions).
23. Turn the System ON. Set Power Control knob to minimum and Lavage Control knob to maximum.
24. Activate the Foot Control for at least 60 seconds. Do NOT exceed 3 minutes.
25. Turn off system. Detach Cavitron® Select scaler from Select System.
26. Detach Cavitron® Select Reservoir Bottle from pump (see Section 6 for complete instructions).
27. Autoclave Cavitron® Select Reservoir Bottle.

NOTE: ALL CHEMICALS MUST BE FLUSHED FROM THE SYSTEM BEFORE IT IS READY FOR PATIENT USE.

10.3 Water Line Filter Maintenance

When the water line filter becomes discolored, the filter should be replaced to prevent reduced water flow to the Cavitron® Select System. A 10-pack of replacement filters is available by ordering Part Number 90158 from your local DENTSPLY® distributor.

1. Disconnect the water supply hose from the water source. If a quick-disconnect connector is attached to the end of the hose, relieve the water pressure by pressing the tip of the connector in an appropriate container and drain the water.
2. Grasp the fittings on either side of the filter disk and twist counterclockwise. Remove the filter section from either side of the water hose.
3. Install the replacement filter onto the water hose fittings. The filter should be positioned to match up with the correct hose fitting.
4. Hand tighten the two hose fittings in a clockwise direction. Reconnect the water supply hose, operate the unit to bleed the air and test for leaks.

10.4 Pump Lavage Filter

When the pump lavage filter becomes discolored, the filter should be replaced to prevent reduced water flow to the Cavitron® Select™ System. A 10-pack of replacement filters is available by ordering Part Number 90191 from your local DENTSPLY® distributor.

1. Empty the contents of the Cavitron Select Reservoir bottle and disconnect it from the pump.
2. Remove the old filter and discard.
3. Insert the small end of the new filter until filter is seated.

Section 11: Troubleshooting

Although service and repair of the Cavitron® Select should be performed by DENTSPLY® personnel, the following are some basic trouble shooting procedures that will help avoid unnecessary service calls. Generally, check all lines and connections to and from the System, a loose plug or connection will often create problems. Check the settings on the System's knobs.

11.1 Troubleshooting Guide

System will not operate:

(Power Indicator Light is not lighted.)

1. Check that the Power Control knob is in the ON position, and that the detachable Power Cord is fully seated in the receptacle on back of the Select™ System.
2. Check that the System's three-prong plug is fully seated in an appropriate AC receptacle, and that AC current is present.
3. Check that the DC power plug is properly inserted into the DC input located on the back of the System.

(Power Indicator Light is lighted.)

1. Check that the Foot Control Connector is fully seated in the Foot Control Receptacle on the back of the System.
2. Check that the Handpiece Cable Assembly is fully seated in the Handpiece Receptacle.

System operates:

(No lavage flow at insert tip.)

1. Assure that Lavage Control knob is properly adjusted.
2. Check that irrigant supply control valve/s (dental office water supply) are open.
3. Check that the Handpiece Cable Assembly is fully seated in the Handpiece Receptacle.
4. Check to see if Water Line Filter or Pump Lavage Filter is discolored—if so, replace.
5. Check to assure that liquid is available in reservoir bottle.

11.2 Technical Support and Repairs

For technical support and assistance call 1-800-989-8826 Monday through Friday, 8:00 AM to 5:00 PM (Eastern Time). For other areas, contact your local DENTSPLY® representative.

Section 12: Specifications

Cavitron® Select: 100-240 VAC Systems

Electrical Voltage	30 VDC +/- 5%
Current	2 amps DC Max
Frequency	50/60 Hertz
Water Pressure	20 – 40 psig
Flow Rate	Minimum setting (CCW) 0 – 15 ml/min Maximum setting (CW) >45 ml/min
Weight	425 g / 15 oz.
Dimensions	Height: 2.4 in. Width: 3-1/8 in. Depth: 7 ½ in. Handpiece Cable: 72 in. Foot Control Cable: 72 in. Power Cord: 2 m Water Supply Line: 96 in.

 DIRECT CURRENT

Cavitron® Select Power Supply (674022001):

Electrical Voltage	100 – 240 VAC
Current	1,6 amps Max @ 100 – 240 VAC
Insulation Requirement	Reinforce
Frequency	47 – 63 Hertz
Weight	550 g / 19.4 oz. Max
Dimensions	Height: 14,3 cm / 5.7 in. Width: 7,6 cm / 3.0 in. Depth: 14,3 cm / 5.7 in. Power Cord: 213 cm / 84 in.

TYPE B EQUIPMENT

FOOTSWITCH

Footswitch Not for operating theatres
Protection Class IPX1

Cavitron® Select Reservoir Pump:

Electrical Voltage	30 VDC +/- 5%
Pump Pressure	15 – 19 psig
Flow Rate	40 ml/min
Weight	513 g / 18 oz.
Dimensions	Height: 10,2 cm / 4.0 in. Width: 8,4 cm / 3.3 in. Length: 30 cm / 11.8 in.

REFER TO INSTRUCTION
MANUAL FOR INFORMATION

Operating Environment:

Temperature:	15 to 40 Deg. Celsius
Relative Humidity:	30% to 75% (non-condensing)

Transport and Storage Conditions:


Temperature:	0 to 70 Deg. Celsius
Relative Humidity:	10% to 95% (non-condensing)
Atmospheric Pressure:	500 to 1060 hPa

Section 13: Classification

• Type of protection against electric shock:	Class 1
• Degree of protection against electric shock:	Type B
• Degree of protection against the harmful ingress of water:	Ordinary
• Mode of operation:	Continuous
• Degree of safety of application in the presence of a flammable anaesthetic mixture with air or with oxygen or nitrous oxide:	Equipment not suitable for use in the presence of flammable anesthetics or oxygen.
• According to medical device directive:	IIA (rule 9)

Section 14: Disposal of Unit

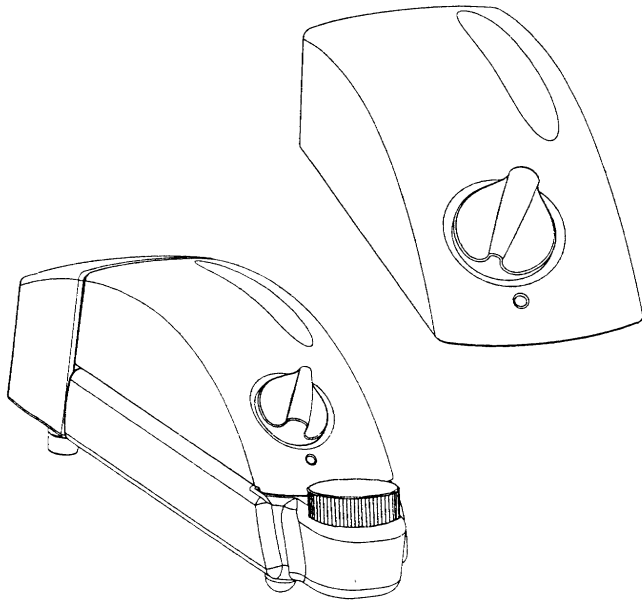- Accordance with local and state laws.

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U. S. A.

DENTSPLY Professional
Technical Service and Repair Department
1301 Smile Way
York, PA 17404
Phone: (800) 989-8826

Deutschland

DENTSPLY DeTrey GmbH
De-Trey-Strasse 1
78467 Konstanz
Germany
Phone: 7531 583 0

France

DENTSPLY DeTrey
17 Michael FARADAY
78380 Montigny Le Bretonneux
France
Phone: (1) 30 14 77 77

Australia

DENTSPLY (Australia) Pty. Ltd.
204-206 Gipps Street
Abbotsford, Victoria 3067
Australia
Phone: (3) 9417 1666
(3) 9417 1777

United Kingdom

DENTSPLY Ltd.
Hamm Moor Lane
Addlestone, Weybridge
Surrey KT15 2SE
England
Phone: (0) 1932 853422

Italia

DENTSPLY DeTrey Italia s.r.l.
Via A. Cavaglieri, 26
I-00173 Roma
Italia
Phone: (06) 723 3626

Canada

DENTSPLY Canada
161 Vinyl Court
Woodbridge, Ontario
L4L 4A3 Canada
Phone: (905) 851-6060

Swiss Representative

DENTSPLY DeTrey Sàrl
Baar Office
Oberdorfstrasse 11
6342 Baar
Switzerland

Printed in U.S.A. Form #90127 Rev. D 02/03

DENTSPLY
PROFESSIONAL

Manufactured by
DENTSPLY Professional
DENTSPLY International
1301 Smile Way
York, PA 17404

CE
0086

EU-Authorized Representative:
DENTSPLY DeTrey GmbH
De-Trey-Str.1
78467 Konstanz
Germany

Distributed by
DENTSPLY Canada
Woodbridge, Ontario
L4L 4A3