
#### **SPIRIT 2000 DENTAL STOOL**

#### **GENERAL DESCRIPTION**

The seats contoured shape should help to eliminate pinching the thigh of the operator. The back shall contain a full back-rest which provides support to the lower back and lumbar area. A pneumatic cylinder shall provide vertical seat adjustment from a minimum of 16" to a maximum of 22" doctor's, minimum of 20" to a maximum of 27" assistant's. If activated while the operator is seated, the seat shall gently lower. If activated when weight is removed the seat shall rise. Two easy to reach adjustment levers for seat height and back tilt shall provide adjustment to the stool without requiring the operator to get out of the stool.

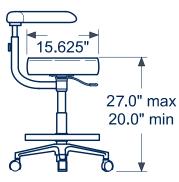



#### **CLEANING DENTAL STOOL UPHOLSTERY**

## NOTE: As with all cleaning products, first clean a small inconspicuous area to ensure the material will not discolor or fade.

It is recommended that each stain be cleaned in a stepwise manner using the sequence below:

#### 1. Regular Cleaning

A solution of 10% household liquid dish soap with warm water applied with a soft damp cloth. Rinse with clean water and wipe dry. Cleaning frequency depends upon use. It is recommended that upholstery be cleaned between patients.

#### 2. Stubborn Stains

Use a detergent cleaner. Wipe using a soft cloth or bristle brush. Rinse with clean water and wipe dry.

#### 3. More Difficult Stains

Carefully clean the stained area rubbing alcohol. Apply with a soft white cloth and rub gently. Rinse with clean water and wipe dry.

#### 4. Ultra Leather Upholstery

- · Clean spots with mild soap and water or an ordinary household cleaner. Wipe off any soap residue with a clean damp cloth.
- · Air dry or dry quickly with a warm setting of a hair dryer.
- · For stubborn stains use a mild solvent.
- · Disinfect ultra leather upholstery with a 5:1 bleach solution.
- Dry cleanable by conventional methods using commercial dry cleaning solvent.

#### **Other Tips**

- · Always apply cleaners with a soft white cloth. Avoid the use of paper towels.
- · When using strong cleaning solutions such as alcohol, it is advisable to first test in an inconspicuous area.
- · Never use harsh solvents or cleaners that are intended for industrial use.
- · To restore luster, a light coat of spray furniture wax may be used. Apply to chair; allow to set for 30 seconds. Lightly buff dry with a clean, dry cloth.

#### **Obtaining technical Literature**

The manufacturer will make available on request component parts lists, descriptions, or other information that will assist technical personnel to repair and replace serviceable items.

#### **Product Disposal**


Contact your local authorized dealer for proper disposal of the device to ensure compliance with your local environmental regulations. **WARNING:** Only authorized service technicians should attempt to service this equipment. Use of other than authorized technicians will void the warranty.

©2011, Pelton & Crane, 11727 Fruehauf Drive, Charlotte, NC, 28273 USA We reserve the right to make alterations which may be due to any technical improvements.

P/N 042669 Rev. 3 1/12

# Spirit 2000 Stools



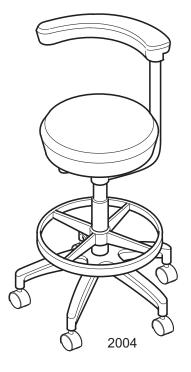


# Pelton & Crane

Pelton & Crane

## Doctor's Stool (Model 2003) Installation Instructions


### **Stool Assembly Instructions**

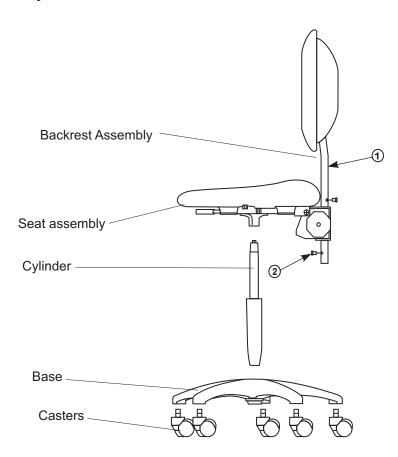

- If the casters are pre-installed into the base, remove base from the carton and place on the floor upright. If the base and casters are shipped separately, remove base from the carton and place on the floor in an inverted position.
- 2. Firmly push the shaft of the casters into the small outer holes of the base until the casters are completely embedded in the holes. Turn the base over onto the casters.
- Remove the cylinder from the rolled cardboard insert.
- 4. Place the large end of the cylinder into the center hole of the base.
- 5. Place the open hole in the control plate over the top end of the cylinder.
- 6. Apply pressure to the top to the seat to lock the base, cylinder, and seat together.
- 7. Insert Backrest Bar Item #1 into control plate socket. Install screw item #2 into hole at bottom of backrest



## Assistant's Stool (Model 2004) Installation Instructions

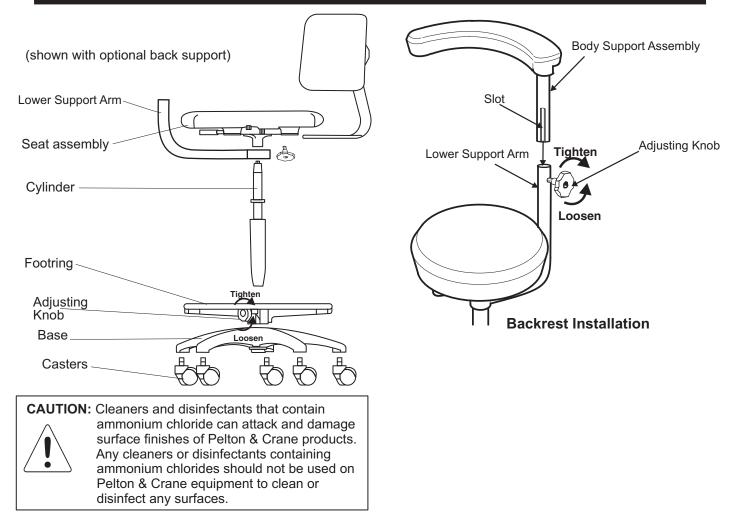

- 1. If the casters are pre-installed into the base, remove base from the carton and place on the floor upright. If the base and casters are shipped separately, remove base from the carton and place on the floor in an inverted position.
- 2. Firmly push the shaft of the casters into the small outer holes of the base until the casters are completely embedded in the holes. Turn the base over onto the casters.
- 3. Remove the cylinder from the rolled cardboard insert.
- 4. If a footring is included, slide the large end of the cylinder through the footring.
- 5. Place the large end of the cylinder, with attached footring into the center hole of the base. Slide the footring to the desired height and tighten adjusting knob to fix in position.
- 6. Place hub of lower arm support over the top end of the cylinder.
- 7. Place the open hole in the control plate over the top end of the cylinder.
- 8. Apply pressure to the top to the seat to lock the base, cylinder, and seat together.
- 9. Install body support assembly aligning the slot in shaft with the nut located inside of lower support arm. Slide body support assembly in the lower support arm and tighten adjusting knob clockwise.
- 10. If a backrest is included, remove the hand knob and washers from the bottom of the seat plate. Slide the slotted end of the backrest bar into the rectangular end slot. Reassemble the hand knob and washer through the slot in the bar in the same order as removed.