





Invacare® **Leo**

Scooter User manual



How can you get in touch with Invacare®?

If you have any questions or need support, please contact your authorised Invacare® Dealer, who has the necessary know-how and equipment plus the special knowledge concerning your Invacare® product, and can offer you all-round satisfactory service. Should you wish to contact Invacare® directly, you can reach us in Europe at the following addresses and phone numbers.

A	Invacare Austria GmbH Herzog Odilostrasse 101 A-5310 Mondsee Austria	雷: Fax: @: WWW:	+43 6232 5 53 50 +43 6232 5 53 54 info@invacare-austria.com www.invacare.at	
$\mathbb{B}(\exists)$	Invacare n.v. Autobaan 22 B-8210 Loppem (Brugge) Belgium	☎: Fax: @: WWW:	+32 (0)50 83 10 10 +32 (0)50 83 10 11 belgium@invacare.com www.invacare.be	
CH	Invacare AG Benkenstraße 260 CH-4108 Witterswil Switzerland	雷: Fax: @: WWW:	+41 (0)61487 70 80 +41 (0)61487 70 81 switzerland@invacare.com www.invacare.ch	
D	Invacare GmbH Alemannenstraße 10 88316 Isny Deutschland	音 Fax @: WWW:	+49 (0)7562 70 00 +49 (0)7562 7 00 66 kontakt@invacare.com www.invacare.de	
(DK)	Invacare A/S Sdr. Ringvej 37 DK-2605 Brøndby Danmark	雷 (Kundeservice): Fax (Kundeservice): @: WWW:	+45 (0)36 90 00 00 +45 (0)36 90 00 01 denmark@invacare.com www.invacare.dk	
E	Invacare® SA c/ Areny s/n Polígon Industrial de Celrà E-17460 Celrà (Girona) ESPAÑA	雷: Fax: @: WWW:	+34 (0)972 49 32 00 +34 (0)972 49 32 20 contactsp@invacare.com www.invacare.es	

F	Invacare® Poirier SAS Route de St Roch F-37230 Fondettes France	雷: Fax: @: WWW:	+33 (0)247 62 64 66 +33 (0)247 42 12 24 contactfr@invacare.com www.invacare.fr	
GB	Invacare® Ltd Pencoed Technology Park Pencoed Bridgend CF35 5AQ United Kingdom	☎ (Customer services): Fax (Customer services @: WWW:	` ,	
	Invacare Mecc San s.r.l. Via dei Pini, 62 I - 36016 Thiene (VI) ITALIA	雷: Fax: @: WWW:	+39 0445 38 00 59 +39 0445 38 00 34 italia@invacare.com www.invacare.it	
Œ	Invacare Ireland Ltd. Unit 5 Seatown Business Campus Seatown Rd, Swords County Dublin Ireland	雷: Fax: @: WWW:	+353 18 10 70 84 +353 18 10 70 85 ireland@invacare.com www.invacare.ie	
N	Invacare® AS Grensesvingen 9 Postboks 6230 Etterstad N-0603 Oslo Norge	☎ (Kundeservice):Fax (Kundeservice):@:@:WWW:	+47 (0)22 57 95 00 +47 (0)22 57 95 01 norway@invacare.com island@invacare.com www.invacare.no	
(NL)	Invacare® B.V. Celsiusstraat 46 NL-6716 BZ Ede Nederland	雷: Fax: @: @: WWW:	+31 (0)318 69 57 57 +31 (0)318 69 57 58 nederland@invacare.com csede@invacare.com www.invacare.nl	
P	Invacare Lda Rua Estrada Velha, 949 P-4465-784 Leça do Balio Portugal	音: , 音: Fax: @: WWW:	+351 225 10 59 46 +351 225 10 59 47 +351 225 10 57 39 portugal@invacare.com www.invacare.pt	

FIN 5	Återförsäljare: nvacare® AB -agerstagatan 9 S-163 91 Spånga Sverige	T (Kundtjänst): Fax (Kundtjänst): @: @: WWW:	+46 (0)8 761 70 90 +46 (0)8 761 81 08 sweden@invacare.com finland@invacare.com www.invacare.se	
 - -	Fillverkare: nvacare® Deutschland GmbH Kleiststraße 49 D-32457 Porta Westfalica Deutschland	MÖLNDAL ☎: Fax: @:	+46 (0)31 86 36 00 +46 (0)31 86 36 06 ginvacare@invacare.com	
	Jeutschlanu	LANDSKRONA . 3 : Fax: @:	+46 (0)418 2 85 40 +46 (0)418 1 80 89 linvacare@invacare.com	
		OSKARSHAMN . ☎: Fax: @:	+46 (0)491 1 01 40 +46 (0)491 1 01 80 oinvacare@invacare.com	
Eastern european countries	European Distributor Organisation (EDO) Kleiststraße 49 D-32457 Porta Westfalica Deutschland	Fax @: WWW:	+49 (0)5731 75 45 40 +49 (0)5731 75 45 41 edo@invacare.com www.invacare.de	

Table of Contents

Cha	pter		Page
1	Intro	oduction	10
	1.1 1.2 1.3 1.4 1.5	Important symbols in this manual	13 13 14
2	Safe	ty notes	15
	2.1 2.2 2.3 2.4	General safety notes Safety information with regard to care and maintenance Safety information on electromagnetic interference Safety information on driving and freewheel mode	18 19
3	Key	features	22
4	The	position of the labels on the product	23
5	Drivi	ing	25
	5.1 5.2 5.3 5.4 5.5	Before driving for the first time	26 27 27
6	Push	ning the scooter by hand	28
	6.1	Disengaging Motors	28

7 Ор	erating console	29
7.1	Operating console arrangement	30
7.2	Driving the Scooter	
7.3	Activating and deactivating acoustic signals	
7.4	Diagnosis and fault repair	
	7.4.1 Error diagnosis	
7.5	Error Codes and Diagnostic Codes	38
3 Ad	justment facilities	41
8.1	Moving the seat position forwards or backwards	41
8.2	Adjusting the armrest width	
8.3	Disengaging the seat to rotate it or remove it	
8.4	Adjusting the seat height	44
8.5	Postural belts	
	8.5.1 Types of postural belts	
	8.5.2 Adjusting the postural belt correctly	
	8.5.3 Fixing the containment belt to the scooter	
8.6	Rollator bracket	
	8.6.1 Attaching the rollator	
	8.6.2 Removing the rollator bracket	
9 Ele	ectrical system	55
9.1	Electronics protection system	
9.2	9.1.1 The main fuse	
9.2	Batteries	
	9.2.2 Charging the batteries	

		9.2.3 Removing and fitting batteries	60
		9.2.3.1 Removing the batteries	61
		9.2.3.2 How to handle damaged batteries correctly	63
	9.3	Charger	64
		9.3.1 FEATURES	64
		9.3.2 CHARACTERISTICS	
		9.3.3 INSTRUCTIONS FOR USE	
		9.3.4 LED DISPLAY	
		9.3.5 TROUBLESHOOTING	
		9.3.6 CAUTION	66
10	Care	and maintenance	67
	10.1 10.2	Cleaning the mobility deviceInspection checks	67
11		air instructions	69
• •	•		
	11.1	Repairing a flat tyre	
		11.1.1 Removing/fitting tyres (4-wheel version and rear wheels of 3-wheel version)	
		11.1.2 Removing/fitting tyres (castor wheel of 3-wheel version)	
		11.1.3 Mending punctures	
12	Disn	nantling the Scooter for transport	76
		·	
	12.1 12.2	Removing the seatRemoving the battery and drive unit	/6
	12.2	12.2.1 Removing the batteries	
		12.2.2 Removing the drive unit	
13	Disp	osal	79
4.4			00
14	recr	nical specifications	80

1 Introduction

Dear user,

First we would like to thank you for purchasing our product! We hope you will enjoy your new Scooter.

This manual contains important hints and information on:

- Safety
- Operation
- Care and maintenance.

Please take care to read the operating manual thoroughly before starting out on your first journey.

This product has been designed to fit the needs of different types of users with different requirements.

The decision whether the model is suitable for the user may only be taken by medical specialists with appropriate aptitude.

Invacare® or their statutory representatives can accept no liability in cases in which the mobility product has not been adapted to suit the user's handicaps.

Some maintenance and settings can be carried out by the user. Certain adjustments do however require technical training and may only be carried out by your Invacare® specialist dealer. Damages and errors caused by nonobservance of the operating manual or as a result of incorrect maintenance are excluded from all guarantees.

This manual contains copyrighted information. It may not be reproduced or copied in whole or in part without the prior written consent of Invacare® or its authorised representative. It may also contain information that pertains to models sold only in certain countries. In this case the information will be clearly marked as pertaining to a particular country-specific version. We reserve the right to make any alterations on the grounds of technical improvements.

1.1 Important symbols in this manual



General risks

This symbol warns you of general hazards!

Always follow the instructions to avoid injury to the user or damage to the product.



EXPLOSION HAZARD!

This symbol warns you of an explosion hazard, which can be caused by excessive tyre pressure in a pneumatic tyre.

Always follow the instructions to avoid injury to the user or damage to the product.



BURN HAZARD!

This symbol warns you of the danger of chemical burns, for example due to the discharge of battery acids!

Always follow the instructions to avoid injury to the user or damage to the product.



BURN Risk!

This symbol warns of the risk of burns, for example, as a result of hot motor surfaces.

Follow the instructions in order to avoid injury or damage to the product.



RISK OF CRUSHING!

This symbol warns of a risk of crushing caused by being careless with heavy components.

Always follow the instructions to avoid injury to the user or damage to the product.



Wear eye protection

This symbol refers to the requirement for wearing eye protection, for example when working with batteries.

• You must wear safety goggles when this symbol is displayed.



Wear protective gloves

This symbol indicates the requirement to wear protective gloves, for example when working with batteries.

You must wear protective gloves when this symbol is displayed.



NOTE:

This symbol identifies general information which is intended to simplify working with your product and which refers to special functions.



Requirements:

This symbol identifies a list of various tools, components and items which you will need in
order to carry out certain work. Please do not attempt to carry out the work if you do not have
the listed tools available.



READ WELL BEFORE OPERATION!

This symbol advises you to read information carefully.

1.2 Important symbols found on the vehicle



This product has been supplied from an environmentally aware manufacturer. This product may contain substances that could be harmful to the environment if disposed of in places (landfills) that are not appropriate according to legislation.

- The 'crossed out wheelie bin' symbol is placed on this product to encourage you to recycle wherever possible.
- Please be environmentally responsible and recycle this product through your recycling facility at its end of life.

1.3 Type classification and permissible use

This vehicle was designed for persons whose ability to walk is impaired, but who are still in terms of their eyesight and physically and mentally able to operate an electric vehicle. It has been classified according to EN 12184 as a **class B mobility product** (for indoor and outdoor areas). It is therefore compact and agile enough for indoor areas, but also able to overcome many obstacles in outdoor areas.

You can find precise information about speed, turning radius, driving range, maximum safe slope, maximum obstacle height and permissible operating conditions in chapter "Technical specifications" starting from page 80.

Please also pay attention to all safety information in chapter "Safety notes" starting from page 15.

The vehicle was successfully tested according to German and international standards as to its safety. It satisfies the requirements according to DIN EN 12184 including EN 1021-1/-2. It was also tested successfully according to EN60529 IPX4 as to its resistance to spray water, and is therefore well suited for typical middle European weather conditions. When equipped with an appropriate lighting system, the vehicle is suitable for use on public roads.

1.4 Warranty

The terms and conditions of the warranty are part of the general terms and conditions particular to the individual countries in which this product is sold.

1.5 Life expectancy

We estimate a life expectancy of five years for this product, provided it is used in strict accordance with the intended use as set out in this document and all maintenance and service requirements are met. The estimated life expectancy can be exceeded if the product is carefully used and properly maintained, and provided technical and scientific advances do not result in technical limitations. The life expectancy can also be considerably reduced by extreme or incorrect usage. The fact that we estimate a life expectancy for this product does not constitute an additional warranty.

2 Safety notes



READ WELL BEFORE OPERATION!

2.1 General safety notes



Danger of injury if this scooter is used in any other way than the purpose described in this manual!

Adhere strictly to the instructions in this user manual!

Danger of injury if the scooter is driven when your ability to drive is impaired by medication or alcohol!

• Never drive any vehicle under the influence of medication or alcohol!

Danger of damage or injury if the scooter is accidentally set into motion!

- Switch the power system off before you get in, get out or handle awkward objects!
- Be aware that the motor brakes are automatically deactivated when the motors are disengaged. For this reason, freewheel operation is only recommended on flat surfaces, never on gradients. Never leave your vehicle on a gradient with its motors disengaged. Always reengage the motors immediately after pushing the vehicle.



Danger of injury if the On/Off Button is pressed while the vehicle is in motion, due to it coming to an abrupt, sharp stop!

• If you have to brake in an emergency, simply pull the handbrake until the scooter comes to a halt! Only switch the vehicle off while in motion as a last resort!

Danger of injury if the scooter is transported in another vehicle with the occupant seated in it!

Never transport the scooter with the occupant seated in it!

Danger of injury if maximum permissible load is exceeded!

Do not exceed the maximum permissible load (see "Technical specifications" on page 80)!

Danger of injury when lifting heavy components!

 When maintaining, servicing or lifting any part of your scooter, take into account the weight of the individual components, especially the batteries! Be sure at all times to adopt the correct lifting posture and ask for assistance if necessary!

Danger of injury if you fall off the scooter!

• If restraining systems are installed (such as seat belts), use them each time you drive the scooter.



Danger of injury by moving parts!

• Make sure that no injury is incurred by moving parts of the scooter, like wheels or a Seat Lifter, especially when children are around!

Risk of injury from hot surfaces!

• Do not leave the mobility device in direct sunlight for prolonged periods. Metal parts and surfaces such as the seat and armrests can become very hot.

Danger of fire or breaking down due to electric devices being connected!

• Do not connect any electric devices to your vehicle that are not expressly certified by Invacare® for this purpose! Have all electrical installations done by your authorised Invacare® Dealer!

Danger of technical failure and injury if unauthorised spare parts and components are used!

Only use original Invacare® spare parts, which have been approved for use with this vehicle!

2.2 Safety information with regard to care and maintenance



Danger of accident and loss of guarantee if maintenance is insufficient!

- For reasons of safety and in order to avoid accidents which result from unnoticed wear, it is important that this electric mobility product undergoes an inspection once every year under normal operating conditions (see inspection plan contained in service instructions).
- Under difficult operating conditions such as daily travel on steep slopes, or in the case of use in medical care cases with frequently changing wheelchair users, it would be expedient to carry out intermediate checks on the brakes, accessories and running gear.
- If the mobility product is to be operated on public roads, the vehicle driver is responsible for ensuring that it is in an operationally reliable condition. Inadequate or neglected care and maintenance of the mobility product will result in a limitation of the manufacturer's liability.

2.3 Safety information on electromagnetic interference

This electric vehicle was successfully tested in accordance with International standards as to its compliance with Electromagnetic Interference (EMI) regulations. However, electromagnetic fields, such as those generated by radio and television transmitters, and cellular phones can influence the functions of electric vehicles. Also, the electronics used in our vehicles can generate a low level of electromagnetic interference, which however will remain within the tolerance permitted by law. For these reasons we ask you to please observe the following precautions:



WARNING: Danger of malfunction due to electromagnetic interference!

- Do not switch on or operate portable transceivers or communication devices (such as radio transceivers or cellular phones) when the vehicle is switched on.
- Avoid getting near strong radio and television transmitters.
- In case the vehicle should be set in motion unintentionally or the brakes are released, switch it off immediately.
- Adding electrical accessories and other components or modifying the vehicle in any way can
 make it susceptible to electromagnetic interference. Keep in mind that there is no sure way to
 determine the effect such modifications will have on the overall immunity of the electronic
 system.
- Report all occurrences of unintentional movement of the vehicle, or release of the electric brakes to the manufacturer.

2.4 Safety information on driving and freewheel mode



Danger of injury if the vehicle tips over!

- Only ever negotiate gradients up to the maximum tilt-resistant gradient and only with the backrest in an upright position, and the seat lifter in the lowest position (if installed)!
- Only ever drive downhill at a maximum of 2/3rds of the top speed! Avoid abrupt braking or accelerating on gradients!
- If at all possible, avoid driving on slippery surfaces (such as snow, gravel, ice etc.) where there is a danger of you losing control over the vehicle, especially on a gradient! If driving on such a surface is inevitable, then always drive slowly and with the utmost caution!
- Never attempt to overcome an obstacle when on an uphill or downhill gradient!
- Never attempt to drive up or down a flight of steps!
- Always approach obstacles straight on! Ensure that the front wheels and rear wheels move
 over the obstacle in one stroke, do not stop halfway! Do not exceed the maximum obstacle
 height (see "Technical specifications" on page 80)!
- Avoid shifting your centre of gravity as well as abrupt changes of direction when the vehicle is in motion!



Danger of injury if the vehicle tips over! (Continued)

- Never use the vehicle to transport more than one person!
- Do not exceed the maximum permissible load!
- When loading the vehicle, always distribute the weight evenly! Always try to keep the centre of gravity of the vehicle in the middle, and as close to the ground as possible!
- Note that the vehicle will brake or accelerate if you change the Driving Speed while it is in motion!

Danger of injury if you collide with an obstacle when driving through narrow passages such as doorways and entrances!

• Drive through narrow passages in the lowest Driving Speed and with due caution!



CAUTION: The centre of gravity for the scooter is higher than that of an electric wheelchair! There is an increased tipping hazard when negotiating bends!

 Reduce speed before negotiating bends! Only accelerate when you have come out of the bend!





CAUTION: Danger of tipping! Anti tip wheels (stabilisers) are only effective on firm ground! They sink in on soft ground such as grass, snow or mud if the mobility device rests itself on them. They lose their effect and the mobility device can tip over.

• Only drive with extreme care on soft ground, especially during uphill and downhill journeys. In the process pay increased attention to the tip stability of the mobility device.

3 Key features

- 1) Disengaging lever
- 2) Unlocking lever for sliding seat rails (front right below seat)
- 3) Unlocking lever for swivelling and removing seat (left below seat)
- 4) Operating console
- 5) Lever for adjusting steering column inclination
- 6) Keyswitch (ON/OFF)



4 The position of the labels on the product



1) Identification label sticker on the seat support tube



2) Manufacturer label on the seat support tube



3) Distributor label on the chassis at the rear



4) Battery label under the cover at the rear

The symbols on the labels are explained in section "Important symbols found on the vehicle" on page 13.

5 Driving

5.1 Before driving for the first time...

Before you take your first trip, you should familiarise yourself well with the operation of the vehicle and with all operating elements. Take your time to test all functions and driving modes.



NOTE:

If installed, make sure to properly adjust and use the postural belt each time you use the wheelchair.

Sitting Comfortably = Driving Safely

Before each trip, make sure that:

- You are within easy reach of all operating controls.
- The battery charge is sufficient for the distance intended to be covered.
- The postural belt (if installed) is in perfect order.
- The rear mirror (if installed) is adjusted so you can look behind at all times without having to bend forward or shift your seating position.

5.2 Taking Obstacles

Your Scooter can climb obstacles and kerbs of up to 6 cm in height.



CAUTION: Danger of Tipping Over!


- Never approach obstacles at an angle but at 90 degrees as shown below.
- Put your backrest into an upright position before climbing an obstacle.

Driving up over an obstacle

 Approach the kerb or obstacle slowly head-on. Shortly before the front wheels touch the obstacle, increase the speed and reduce only after the rear wheels have also climbed the obstacle.

Driving down off of an obstacle

 Approach the kerb or obstacle slowly head-on. Before the front wheels touch the obstacle, reduce speed and keep it until also the rear wheels have come down off of the obstacle.

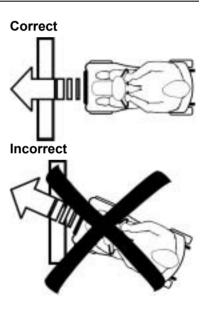

5.3 Driving up and down gradients

For information concerning the maximum safe slope, please see chapter **"Technical specifications"** starting on page **80**.



WARNING: Danger of tipping over!

- Only ever drive downhill at a maximum of 2/3rds of the top speed!
- Always return the backrest of your seat to an upright position before ascending slopes! We recommend that you lean the backrest slightly to the rear before descending slopes!
- Never attempt to ascend or descend a slope on slippery surfaces or where there is a danger of skidding (such as wet pavement, ice etc)!
- Avoid trying to get out of the vehicle on an incline or a gradient!
- Always drive in a straight direction along the road or path you are travelling on, rather than attempting to zigzag!
- Never attempt to turn around on an incline or a slope!

5.4 Parking and stationary

If you park your vehicle, or leave it at a standstill for a longer period:

• switch off the power supply (keyswitch).

5.5 Use on public roads

The wheels may bear the note "Not For Highway Use". However, the mobility device may be used on all traffic routes for which it is approved in accordance with the relevant national legislation.

6 Pushing the scooter by hand

The motors of the scooter are equipped with automatic brakes, preventing the scooter from rolling away out of control when the power supply is switched off. When pushing the scooter, the magnetic brakes must be disengaged.

6.1 Disengaging Motors



Danger of the vehicle running away!

When the motors are disengaged (for push operation whilst freewheeling), the
electromagnetic motor brakes are deactivated! When the vehicle is parked, the levers for
engaging and disengaging the motors must without fail be locked firmly into the "DRIVE"
position (electromagnetic motor brakes activated)!


The lever for engaging and disengaging the motor is located on the right-hand side at the rear.

Disengaging the drive

- Switch off the Scooter (keyswitch).
- Press the unlocking knob on the disengaging lever (1).
- Push the disengaging lever forwards. The drive is now disengaged.

Engaging the drive


Pull the lever to the rear.

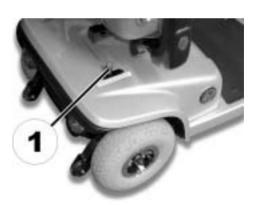

7 Operating console

7.1 Operating console arrangement

- 1) Battery charge display
- 2) Speed controller
- 3) Horn
- Left-hand direction indicator (switches itself off automatically after 30 seconds)
- 5) Lighting
- 6) Status display
- 7) Warning blinker
- Right-hand direction indicator (switches itself off automatically after 30 seconds)
- 9) Drive lever

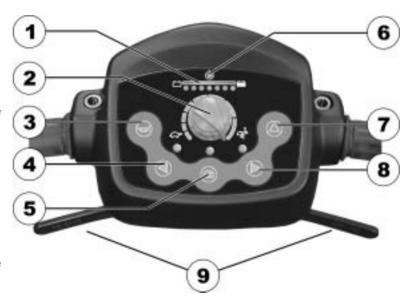

7.1.1 Status display



NOTE:

The ON/OFF diode is used as a fault display (status display). Chapter "Error Codes and Diagnostic Codes" on page **38** contains an explanation of the error codes.

7.1.2 Battery charge display

- All diodes illuminate: maximum driving range
- Only red and yellow diodes illuminate: reduced driving range. Recharge the batteries at the end of your journey.
- Only red LEDs illuminate/blink, electronics beeps 3x: battery reserve = severely restricted driving range. Recharge batteries immediately!



NOTE:

Overdischarge protection: after a certain drive time on reserve battery power the electronics system switches the drive off automatically and brings the Scooter to a standstill. If you do not drive your Scooter for a while the batteries will "recuperate" and allow a further, but short, journey. However, after a very brief journey the red diodes will illuminate again and the electronic system will beep three times. This procedure leads to battery damage and should be avoided if possible!

7.2 Driving the Scooter



WARNING: Danger from the unintended rolling of the vehicle!

When stopping the vehicle, the drive lever needs to return entirely to the middle position to activate the electromagnetic brakes. If there is any obstruction stopping the lever from returning to the middle position, the electromagnetic brakes cannot be activated. This can lead to the vehicle rolling unintentionally.

- Please make sure that the drive lever is in the middle position, if the vehicle is to remain stationary.
- Switch the power supply on (keyswitch)The operating console display illuminates. The Scooter is ready to drive.



NOTE:

If the Scooter is not ready to drive after switching on, check the status display (see Chapter "Status display" on page 30 and Chapter "Diagnosis and fault repair" on page 36.

- Set the required speed with the speed controller.
- Pull the right-hand drive lever carefully to travel forwards.
- Pull the left-hand drive lever carefully to travel in reverse.



NOTE:

The control system is programmed with standard values in the works. Your Invacare®dealer can carry out programming tailored to fit your requirements.



Caution: any changes to the drive program can affect the driving characteristics and the tipping stability of the vehicle!

- Changes to the drive program may only be carried out by trained Invacare® specialist dealers!
- Invacare® supplies all mobility products with a standard drive program ex-works. Invacare®
 can only give a warranty for safe vehicle driving behaviour especially the tipping stability for this standard drive program!



NOTE:

To brake quickly, simply let go of the drive lever. It will then automatically return to the middle position. The Scooter will brake.


7.3 Activating and deactivating acoustic signals

The Scooter electronic system can be programmed to emit an acoustic signal in the following situations:

- battery capacity low (activated in delivery status)
- direction indicators actuated (deactivated in delivery status)
- hazard lamps actuated (deactivated in delivery status)

The electronic system must be switched off in order to activate or deactivate an acoustic signal for particular functions, and a particular keystroke combination needs to be entered when switching on again.

After a signal for a particular function has been successfully activated/deactivated, a combination of LEDs on the battery charge display will blink as a confirmation. The LEDs are numbered as shown on the right:

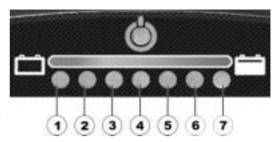

The keystroke combinations and LED codes for various options are as follows:

Function:	Keystroke combination	LED(s)	Condition
Acoustic signal at low battery capacity	"Lighting" + "direction indicator left"	D1	deactivated
		D1+D2	activated
Acoustic signal when direction	"Lighting" + "direction indicator right"	D3	deactivated
indicators actuated	J	D3+D4	activated
Acoustic signal when hazard lamps	"Lighting" + "hazard lamps"	D5	deactivated
actuated		D5+D6	activated

Activating or deactivating an acoustic signal

Please proceed as follows to activate or deactivate an acoustic signal for a particular function:

- 1) Switch off the electronic system.
- 2) Enter the keystroke combination and hold.
- 3) Switch on the electronic system
- 4) Wait two seconds until the appropriate blink code is displayed on the battery charge display, then release the keys. Do not hold the keystroke combination down for more than five seconds.
- 5) If LED D7 blinks five times subsequently the acoustic signal has been successfully activated.
- 6) The electronic system will return to normal operating status automatically.

7.4 Diagnosis and fault repair

The electronic system offers diagnostic information to support the technician during the recognition and rectification of faults on the Scooter. If there is a fault, the status display blinks several times, pauses, then blinks again. The type of fault is displayed by the number of blinks in each group, which are also known as the "blink code".

The electronic system reacts differently depending on the seriousness of the fault and its effect on user safety. It can, for example:

- simply show the blink code as a warning and allow both driving and normal operation to continue;
- display the blink code, stop the Scooter and prevent further travel until the electronic system has been switched off and switched on again;
- display the blink code, stop the Scooter and not permit further travel until the fault has been rectified.

You can find detailed descriptions of individual blink codes, including possible causes and fault repair, in the section entitled "Error Codes and Diagnostic Codes" on page **38**.

7.4.1 Error diagnosis

If the Scooter shows a failure, please use the following guide to locate the fault.



NOTE:

Before making any diagnosis, ensure that the Scooter has been switched on at the keyswitch.

If the status display is OFF:

check whether the keyswitch is SWITCHED ON.

Check whether all cables are correctly connected.

If the status bar indicator is BLINKING:

count the number of blinks and then proceed to the next section.

7.5 Error Codes and Diagnostic Codes

Blink code	FAULT	Consequence for the Scooter	Comments
1	Battery must be charged	Continues to drive	• The batteries are discharged. Charge the battery as soon as possible.
2	Battery voltage too low	Stops driving	The batteries are depleted. Charge batteries.
			 If you switch the Scooter off for a few minutes, the battery can often recuperate to such a stage that a short journey is still possible. You should only do this in emergency, however, because this causes the batteries to become excessively discharged!
3	Battery voltage too high	Stops driving	 The battery voltage is too high. If the battery charger is connected, disconnect it from the Scooter. The electronic system charges the batteries when running downhill and when braking. This fault is caused when the battery voltage becomes too high during this process. Switch the Scooter off and on again. Switch on the lights to use up some power.

Blink code	FAULT	Consequence for the Scooter	Comments
4	Power time exceeded	Stops driving	 The Scooter has used up too much power for a long period, probably because the motor has been overloaded or the Scooter has attempted to work against insurmountable resistance. Switch off the Scooter, wait a few minutes and then switch it on again. The electronic system has determined a motor short-circuit. Check the wiring harness for short-circuits and check the motor: Contact your Invacare® dealer.
5	Brake failure	Stops driving	 Ensure that the disengaging lever is in the engaged position. There is a defect in the braking coil or in the cabling. Check the wheel lock and cabling for open or short-circuited circuits. Contact your Invacare® dealer.
6	No neutral position when switching Scooter on.	Stops driving	 Drive lever is not in neutral when the keyswitch was turned. Put the drive lever in neutral, turn the power off and then turn on again. It may be necessary to recalibrate the drive lever. Contact your Invacare® dealer.
7	Fault in speed potentiometer	Stops driving	The drive lever electronics could be faulty or incorrectly connected. Contact your Invacare® dealer.

Blink code	FAULT	Consequence for the Scooter	Comments
8	Motor voltage error	Stops driving	The motor or its cabling is defective Contact your Invacare® dealer.
9	Miscellaneous internal fault	Stops driving	Contact your Invacare® dealer.
10	Push mode / free running	Continues to drive	The Scooter has exceeded the maximum speed for push mode / free running. Switch the electronics system off and on again.

8 Adjustment facilities

8.1 Moving the seat position forwards or backwards

The disengaging lever for adjusting the seat is located front right below the seat

- Pull the lever (1) to disengage the seat.
- Slide the seat forwards or backwards into the required position.
- Let go the lever again to lock the seat into its required position.



8.2 Adjusting the armrest width

The hand wheels for releasing the armrests are located under the seat (1).

- Turn the hand wheels to loosen the fixing for the armrest.
- Adjust the armrests to the required width.
- Retighten the handwheels



8.3 Disengaging the seat to rotate it or remove it

The seat can be turned to one side to make getting in and out of the scooter easier. The seat is also easier to remove from this position.

The lever for disengaging the seat is located under the seat (1) on the right.

- Pull the lever forwards to disengage the seat.
- Turn the seat to one side.
- If you want to remove the seat, hold it firmly by the backrest and front edge and remove it upwards.



8.4 Adjusting the seat height

The seat height can be adjusted to 46, 48.5, 51, 53.5 or 56 cm.



Requirements:

- 2x open-ended spanners 17 mm
- Remove the seat
- Remove the battery and motor compartment cover.
- Remove the seat pillar locking bolt using both openended spanners.



- Adjust the seat height.
- Reinsert the bolt and tighten.



8.5 Postural belts

A postural belt is an option which can either be fixed to the wheelchair ex-works or can be retrofitted by your specialist dealer. If your wheelchair is fitted with a postural belt, your specialist dealer will have informed you about fitting and usage.

The postural belt is used to help the wheelchair user keep an optimum sitting position. Correct use of the belt assists the user in sitting securely, comfortably and well-positioned in the wheelchair, especially for such users who do not have such a good sense of balance while sitting.



NOTE:

We recommend using the postural belt whenever the wheelchair is used. The belt should be tight enough to ensure that you are sitting comfortably and that your body is in the correct sitting position.

8.5.1 Types of postural belts

Your wheelchair can be fitted with the following postural belt types ex-works. If your wheelchair has been fitted with a different belt to those listed below, please ensure that you have received the manufacturer's documentation with regard to correct fitting and use.

Belt with metal buckle adjustable on one side

The belt can only be adjusted on one side, which can result in the buckle not being in the middle of the waist (across pelvic area) after adjustment has taken place.



8.5.2 Adjusting the postural belt correctly

- Ensure that you are sitting correctly, which means that you are sitting right at the back of the seat, your pelvis is positioned erect and as symmetrically as possible, not to the front, to the side or at one edge of the seat.
- Position the postural belt so that your hipbones can be easily felt above the belt.
- Adjust the belt length using one of the adjustment aids described above. The belt should be
 adjusted so that you can fit a flat hand between the belt and your body.
- The buckle should be positioned as centrally as possible. In doing so, please carry out adjustments on both sides as much as possible.
- Please check your belt every week to ensure that it is still in good working condition; to ensure
 it has no damage or wear, and that it is fixed properly to the wheelchair. If the belt is only
 fastened with a bolted connection, ensure that the connection has not loosened or undone.
 You can find more information about maintenance work on belts in the service manual, which is
 available from Invacare®.


8.5.3 Fixing the containment belt to the scooter



Requirements:

- jaw spanner 12 mm
- jaw spanner 13 mm

The fixing points (1) for attaching the belt are located under the seat (the figure shows only the left hand side).

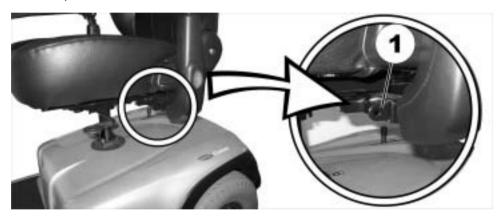

- Take hold of the belt mounting and hold it in front of the hole in the fixing.
- Position the bolt (1), screw the nut on from the other side and tighten with a jaw spanner.



 Repeat the same procedure on the other side of the seat. Check to ensure that the nut is tightened properly on the bolt.



8.6 Rollator bracket

Your scooter can be fitted with an optional rollator bracket. Only the following rollators, which have been approved by Invacare, can be transported using this bracket:

- Dolomite Jazz 600
- Dolomite Legacy 600
- Invacare Banjo P452E/3

The maximum permitted rollator weight is 9 kg.



CAUTION! Risk of tipping as a result of altered center of gravity.


The center of gravity of the scooter shifts towards the back as a result of attaching the rollator. The maximum safe angle of incline is thereby reduced by up to 2°.

• Please note that gradients that you would normally be able to negotiate may now be too steep and the scooter could tip. Do not attempt to climb or descend such gradients.

8.6.1 Attaching the rollator

Dolomite Jazz 600



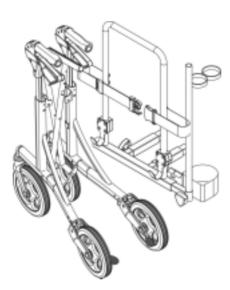

Dolomite Legacy 600






Invacare Banjo P452E/3





8.6.2 Removing the rollator bracket

- Loosen the screws (1).
- Pull the rollator bracket out of the fixtures.

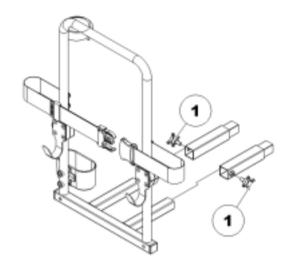


8.6.3 Positioning the rear reflector



CAUTION! Risk of accident due to poor visibility.

If you wish to use your wheelchair on public roads and a rear reflector is required by national legislation, then the rollator bracket may not cover the rear reflector.

 Make sure that the rear reflector is mounted in such a way that a sufficient amount of the reflective area is visible.

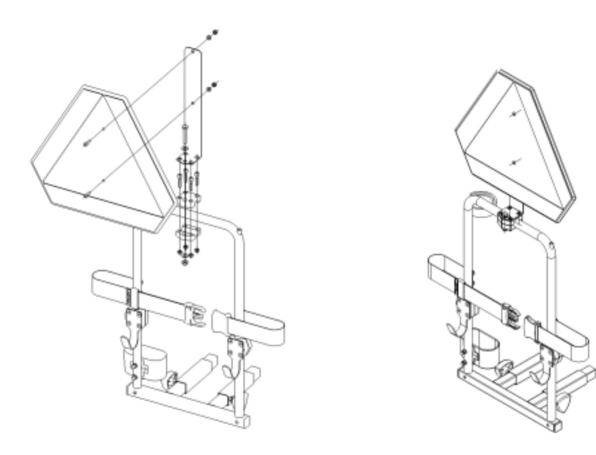

• Position the rear reflector as shown in the drawing.

9 Electrical system

9.1 Electronics protection system

The vehicle's electronics are equipped with an overload-protection system.

If the motors are put under considerable strain for a longer period of time (for example, when driving up a steep hill) and especially when the ambient temperature is high, then the electronic system could overheat. In this case the vehicle's power is reduced gradually until it finally comes to a halt. The Status Display shows a corresponding error code (see chapter "Error Codes and Diagnostic Codes" on page 38). By switching the power supply off and back on again, the error code is cancelled and the electronics are switched back on. It will take approximately five minutes until the electronics have cooled down enough for the motors to restore full power again.

When the motors are stalled by an insurmountable obstacle, such as a high kerb, and the vehicle driver allows the motors to strain against this hindrance for more than 20 seconds without moving, then the electronics will automatically switch off to prevent the motors from being damaged. The Status Display shows a corresponding error code (see chapter "Error Codes and Diagnostic Codes" on page 38). By switching off and back on again, the error code is cancelled and the electronics are switched back on.

9.1.1 The main fuse

The entire electric system is protected against overload by two master fuses. The master fuses are mounted on the positive battery cables.



NOTE

A defective main fuse may be replaced only after checking the entire electric system. An Invacare® specialised dealer must perform the replacement. You can find information on the fuse type in chapter **"Technical specifications"** starting on page **80**.

9.2 Batteries

9.2.1 What you need to know about batteries

Power is supplied by two 12 V batteries. The batteries are maintenance-free and only need regular charging.

New batteries should always be fully charged once before their first use. New batteries will be at their full capacity after having run through approx. 10 - 20 charging cycles. How fast the batteries discharge depends on many circumstances, such as ambient temperature, condition of the surface of the road, tyre pressure, weight of the driver, way of driving and utilisation of lighting.



NOTE

The batteries supplied with your electric vehicle are not hazardous goods. This classification is based on the German *GGVS Hazardous Goods Road Transport Ordinances*, and the *IATA/DGR Hazardous Goods Rail Transport / Air Transport Ordinances*. Batteries may be transported without restrictions, whether by road, rail or by air. Individual transport companies have, however, guidelines which can possibly restrict or forbid certain transport procedures. Please ask the transport company regarding each individual case.

Pay attention to the Battery Charge Indicator! Make sure to charge the batteries when the Battery Charge Indicator shows that battery charge is low. We recommend charging the batteries after each trip, as well as each night over night. Depending on the level of discharge, it can take up to 12 hours until the batteries are fully charged again.

Protect your charger from sources of heat such as heaters and direct sunlight. If the battery charger overheats, charging current will be reduced and the charging process delayed.

To avoid damaging the batteries, never allow them to be fully discharged. Do not drive on heavily discharged batteries if it is not absolutely necessary, as this will strain the batteries unduly and shorten their life expectancy.

In case your vehicle is not used for a longer period of time, then the batteries must be charged at least once a month to maintain a full charge. Alternatively, the vehicle can stay connected to the charger. The batteries cannot be overcharged with the specified charger.

Please use only charging devices in Class 2. This class of chargers may be left unattended during charging. All charging devices which are supplied by Invacare® comply with these requirements.

9.2.2 Charging the batteries

Make sure you read and understand the battery charger's User's Manual, if supplied, as well as
the safety notes on the front and rear panels of the charger!



WARNING!

Risk of explosion and destruction of batteries if the wrong battery charger is used!

• Only ever use the battery charger supplied with your vehicle, or a charger that has been approved by Invacare®.

Risk of electric shock and damage to the battery charger if it gets wet!

- Protect the battery charger from water.
- Always charge in a dry environment.

Risk of short circuit and electric shock if the battery charger has been damaged!

Do not use the battery charger if it has been dropped or damaged.

Risk of electric shock and damage to the batteries!

• NEVER attempt to recharge the batteries by attaching cables directly to the battery terminals.

Risk of fire and electric shock if a damaged extension cable is used!

• Only ever use an extension cable if it is absolutely necessary. In case you must use one, make sure it is in good condition.

Risk of injury if using the wheelchair during charging!

- DO NOT attempt to recharge the batteries and operate the wheelchair at the same time.
- DO NOT sit in the wheelchair while charging the batteries.

The charging socket is located on the left of the steering column

Connecting the battery charger

- Switch off the Scooter.
- Push the protective cap on the side of the charging socket to the side.
- Connect the battery charger to the Scooter.
- Connect the battery charger to the mains.

Disconnecting the battery charger from the Scooter

- First unplug the battery charger from the mains.
- Then unplug the battery charger from the Scooter.



9.2.3 Removing and fitting batteries



WARNING:

Danger of injury if the batteries are not handled correctly during assembly and maintenance work!

- New batteries should be installed by authorised technicians!
- Observe the warnings on the batteries!
- Take into account the heavy weight of the batteries!
- Only ever use the battery type defined in the technical specifications (see "**Technical** specifications" on page **80**)!

Danger of fire and burns if battery terminals are short-circuited!

• DO NOT short-circuit battery terminals with a tool!



WARNING:

Corrosion and burns from acid leakage if batteries are damaged!



• Remove clothes that have been soiled by acid immediately.



After contact with skin:

Immediately wash affected area with lots of water.

After contact with eyes:

• Immediately rinse eyes under running water for several minutes; consult a physician.

9.2.3.1 Removing the batteries



Requirements:

- Open spanner, 11 mm.
- Remove the seat.
- Remove the battery and motor compartment cover.
- Unplug the battery connecting plug (1).
- Release the battery holder strap.
- Remove the batteries.



- Loosen the blue cable on the negative battery terminal with the open-ended spanner, and remove the cable.
- Loosen the red cable battery clamp on the positive battery terminal with the open-ended spanner, and remove the cable.
- Repeat the procedure for the other battery





NOTE:

Replacing new batteries takes place in reverse order.

9.2.3.2 How to handle damaged batteries correctly



WARNING:

Corrosion and burns from acid leakage if batteries are damaged!

• Remove clothes that have been soiled by acid immediately.

After contact with skin:

• Immediately wash affected area with lots of water.

After contact with eyes:

- Immediately rinse eyes under running water for several minutes; consult a physician.
- Always wear safety goggles and appropriate safety clothing when handling damaged batteries.
- Place damaged batteries in an acid-resistant receptacle immediately after removing them.
- Only ever transport damaged batteries in an appropriate acid-resistant receptacle.
- Wash all objects that have come into contact with acid with lots of water.

Disposing of dead or damaged batteries correctly


Dead or damaged batteries can be given back to your dealer or directly to Invacare®.

9.3 Charger

9.3.1 FEATURES

- 1. CHARGING LED display
- 2. POWER LED display
- 3. Mains socket
- 4. (115V/230V) input voltage selector switch
- 5. Battery charging plug



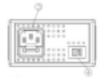

9.3.2 CHARACTERISTICS

Article	Battery charger (switch mode)			
Model	4F24040	4F24050	4F2460	
Output current (DC)	4A	5A	6A	
Charging voltage (DC)	28.8 V			
Trickle charging V (DC)	27.6 V			
Input current (AC)	4/2 A			

Input voltage (AC)	115 Vac or 230 Vac 50/60Hz (select manually)	
Degree of efficiency	AC-DC 80%	
Voltage supply	Switch mode	
Charging process	Constant current, two levels of constant voltage	
Battery application	24 V gel or AGM batteries (20 Ah ~ 60 Ah)	
Output detection	Short-circuit detection	
	Output voltage/current limit	
	Protected against feedback	
	Overheating detection	
Operating temperature	0 °C ~ 40 °C	
Dimensions L x W x H	190 mm x 100 mm x 55 mm	
Weight	965 g	
Color	Black	

9.3.3 INSTRUCTIONS FOR USE

- Select the correct input voltage (115V/230V).
- Connect the battery charger to the charging socket of the electric vehicle using the charging plug (5).
- Connect the power cord. The POWER LED (2) will light.
- The charging process begins. During charging, the CHARGING LED (1) lights orange. When it changes from orange to green, the vehicle is fully charged.

9.3.4 LED DISPLAY

POWER LED: GREEN LIGHT ON = Power on CHARGING LED: ORANGE LIGHT ON = Charging

GREEN LIGHT ON = Fully charged LIGHT OFF = Trickle charging

9.3.5 TROUBLESHOOTING

- 1. If the POWER LED is off:
 - Check that the input voltage (115V/230V) is correct and that the charging cable is correctly connected.
 - If the LED still does not light, please contact your dealer.
- 2. If the CHARGING LED is off:
 - Check that the charging cable is correctly connected.
 - If the battery is fully charged, the charger will switch to trickle charge mode and the CHARGING LED light goes out.
 - If the charging process was not started (orange LED), the battery could be faulty. Please contact your dealer.
- 3. If the CHARGING LED does not change from orange to green:
 - The battery cannot be charged correctly. It may be faulty. Please contact your Invacare dealer.
- 4. If the CHARGING LED changes from orange to green immediately:
 - The battery is either fully charged or faulty. Please contact your dealer.

9.3.6 CAUTION

- In order to extend the service life of the charger, you should not leave the electric vehicle connected to it for prolonged periods.
- Do not use the charger for totally discharged batteries or when the batteries are faulty.
- Use the charger in a well ventilated room.
- Only use the charger for gel or AGM batteries (13-60 Ah).
- Do not use for a voltage input other than that specified. Make sure that the voltage input (115V or 230V) is correctly set.

10 Care and maintenance

The term "Maintenance" means any task performed to ensure that a medical device is in good working order and ready for use as intended. Maintenance encompasses different areas, such as everyday care and cleaning, inspection checks, repair tasks and refurbishment.



NOTE:

Have your vehicle checked once a year by an authorised Invacare® dealer in order to maintain it's driving safety and roadworthiness.

10.1 Cleaning the mobility device

When cleaning the mobility device, pay attention to the following points:

- Only use a damp cloth and gentle detergent.
- Do not use any abrasive or scouring agents.
- Do not subject the electronic components to any direct contact with water.
- Do not use any high-pressure cleaning devices.

Disinfection

Spray or wipe disinfection using a tested and recognised product is permitted. A list of the current permitted disinfectants is available from the Robert Koch Institute at http://www.rki.de.

10.2 Inspection checks

The following table lists inspection checks that should be performed by the user and their intervals. If the mobility device fails to pass one of the inspection checks, please refer to the chapter indicated or contact your authorised Invacare® dealer. A more comprehensive list of inspection checks and instructions for maintenance work can be found in the Service Manual for this device, which can be obtained from Invacare®. That Manual, however, is intended to be used by trained and authorised service technicians, and describes tasks which are not intended to be performed by the user.

Maintenance Jobs	When Delivered	Weekly	Monthly
Seat and backrest padding:			
Check for perfect condition.			✓
Tyres:			
Have tyres checked for specified air pressure (2,5 bar).	√	\checkmark	
Front wheels			
Front wheels must spin smoothly.		\checkmark	
If wheels wobble or do not spin easily, adjust steering pivot pin or front wheel bearing.			✓
Rear wheels:			
Test wheel for firm seat on the axle drive shaft.			\checkmark
Rear wheels must spin without wobbling			√
Electronics / Electrical System:			
Check all plug connections for condition and firm connection.			\checkmark

Have batteries been fully charged before the daily operate.	ation? Befo	Before every trip	
Are all holders, screws firmly fixed, tight and safe?		\checkmark	
Are all electric bulbs of the lighting system (if applicable) in working order?		Before each trip	
Cleaning:			
Clean all parts carefully.	Whe	n necessary	

11 Repair instructions

The following are instructions on repairs that can be performed by the user. For the specifications of spare parts please see "Technical specifications" on page 80, or consult the Service Manual, available from Invacare® (in this connection please see the addresses and phone numbers in section "How can you get in touch with Invacare®?" on page 3). In case you require assistance, please contact your Invacare® Dealer.

11.1 Repairing a flat tyre



WARNING: Danger of damage or injury if the vehicle is accidentally set into motion during repairs!

- Switch the power off (ON/OFF Button)!
- Engage the motors!
- Secure the vehicle against rolling away by placing wedges under the wheels!

11.1.1 Removing/fitting tyres (4-wheel version and rear wheels of 3-wheel version)



Required items:

- 17 mm spanner
- Rubber mallet
- Screwdriver
- 17 mm safety nut

Removing the tyres

- Jack the vehicle up and place a block of wood underneath it to prop it up.
- · Remove the plastic cap.
- Remove the wheel safety nut (1) using the 17 mm spanner.
- Remove the wheel by giving it a few light taps on the back with the rubber mallet to release it from the axle.


Fitting the tyres

- Fit the tyres following the sequence described above in reverse order.
- Use a new wheel safety nut.



Problems removing the wheel?

You may need to use a special tool. Please contact your authorised Invacare® dealer for assistance.

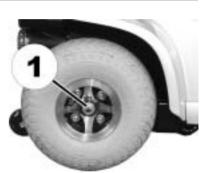

For a description of the procedure for repairing a tyre, see section "**Mending punctures**" on page **73**.

11.1.2 Removing/fitting tyres (castor wheel of 3-wheel version)



WARNING: Lifting the vehicle by the plastic cladding may cause damage to the cladding.

• Only lift the vehicle by the chassis.




Required items:

- 12 mm spanner
- 13 mm spanner
- 13 mm safety nut

Removing the tyres

- Lift the vehicle at the front of the chassis and tip it backwards with the anti-tipper wheels until it is lying on the backrest.
- Remove the wheel safety nut (1) using the 13 mm spanner.
 Counterhold the bolt (2) using the 12 mm spanner.
- Remove the bolt.
- Remove the wheel together with the spacers and the washer.

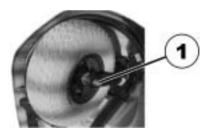


Fitting the tyres

 Fit the tyres following the sequence described above in reverse order.

Make sure that the spacers and the washer are positioned correctly.

• Use a new wheel safety nut.

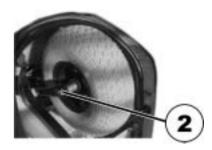


For a description of the procedure for repairing a tyre, see section "**Mending punctures**" on page **73**.

11.1.3 Mending punctures



Requirements:

- Open spanner, 17 mm.
- Rubber hammer
- Raise the vehicle and place wooden blocks underneath to support it.
- Remove the wheel locknut (1) with a 17 mm open-ended spanner.
- Remove the wheel by tapping it gently with the rubber hammer on the rear face to carefully loosen it from the axle.

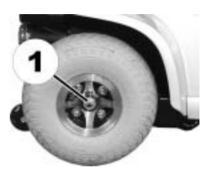



Problems when removing wheel?

It may be necessary to use a special tool Please ask your Invacare® dealer to help you.

11.1.3.1 Repairing punctured tyres



Requirements

- inner tube repair set **or** a new inner tube
- talcum powder
- · open-ended spanner, 13 mm
- jaw spanner 13 mm
- Remove valve cap.
- De-inflate the tyre by pressing in the centre valve pin.
- Loosen the 4 nuts (1) with the socket spanner and remove them. Counter the bolts from the other side with the jaw spanner to prevent turning.
- Remove both wheel rim halves out of the tyre and remove the inner tube.
- Repair the inner tube and refit in the wheel, or replace it with a new inner tube.





Did the old inner tube get wet during the repair?

If you repaired the old inner tube and reused it, and it became wet during repair, it is much easier to refit it into the wheel if you powder it lightly with talcum powder.

- Refit the wheel rim parts from outside into the tyre.
- Pump up the tyre lightly.
- Reinsert the nuts and bolts which hold the wheel rim together and tighten fully.
- Make sure that the tyre is properly located on the wheel rim.
- Inflate the tyre up to the recommended tyre pressure (2.8 bar).
- Check to make sure that the tyre is still located properly on the wheel rim.
- Screw the valve cap back on.
- Reassemble the wheel.

12 Dismantling the Scooter for transport

Please proceed as follows to dismantle the Scooter for transport:

- Remove the seat
- Take out batteries
- Remove the drive unit

Please proceed in reverse order to reassemble the Scooter.

12.1 Removing the seat

The lever for releasing the seat so that the seat can be rotated and removed is located on the left under the seat (1).

- Press the lever forwards to disengage the seat.
- Turn the seat to the side.
- To remove the seat, hold it firmly by the backrest and the front edge and pull it upwards.



12.2 Removing the battery and drive unit


Once you have removed the seat, remove the battery and motor compartment cover panel.
 The cover panel is secured using a velcro band, so you only need to pull it upwards to remove it.

12.2.1 Removing the batteries


Please see chapter "Removing and fitting batteries" on page 60 for information on removing the batteries.

12.2.2 Removing the drive unit

 Separate the main wiring harness connecting plug (1).

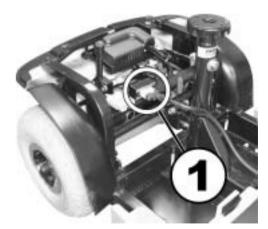

- Pull the disengaging lever (A) to the rear. The drive unit lock is now released.
- Lift up the chassis with the seating tube. The drive unit can now be released from the chassis.

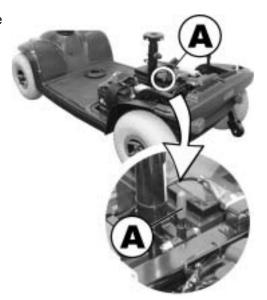

13 Disposal

- The equipment wrapping is potentially recyclable.
- The metal parts are used for scrap metal recycling.
- The plastic parts are used for plastic recycling.
- Electric components and printed circuit boards are disposed of as electronic scrap.
- Exhausted or damaged batteries can be returned to your medical equipment supplier or Invacare®.
- Disposal must be carried out in accordance with the respective national legal provisions.
- Ask your city or district council for details of the local waste management companies.

14 Technical specifications

The technical information provided hereafter applies to a standard configuration or represents maximum achievable values. These can change if accessories are added. The precise changes to these values are detailed in the sections for the respective accessories.

Permissible operating and storage conditions	
Temperature range for operation according to ISO 7176-9:	• -25° +50 °C
Temperature range for storage according to ISO 7176-9:	• -40° +65 °C

Electrical system	
Motors	• 1 x 240 W
Batteries	• 2 x 12 V/36 Ah (C20) leakproof/AGM
	• 2 x 12 V/40 Ah (C20) leakproof/AGM
Main fuse	• 70 A

Charging device	
Output current	• 5 A ±
Output voltage	24 V nominal (12 cells)
Input voltage	• 200 – 250 V nominal
Operating temperature (surroundings)	• -25° +50 °C
Storage temperature	• -40° +65 °C

Tyres	
Tyre type	• 3.00 - 4 pneumatic
	The recommended maximum tyre pressure in bar or kpa is marked on the side
	wall of the tyre or the rim. If more than one value is listed, the lower one in the
	corresponding units applies.
	(Tolerance = -0.3 bar,1 bar = 100 kpa)

Tyres	
Tyre type	10" puncture-proof

Driving characteristics	
Speed (dependent on country - please ask your	• 6 km/h
dealer which speed is available in your country.)	• 8 km/h
Min. braking distance	• 1000 mm (6 km/h)
	• 1500 mm (8 km/h)
Max. safe slope *****	• 10° (17,5 %)
Max. climbable obstacle height	• 60 mm
Turning diameter	2620 mm (4-wheel version)
	2320 mm (3-wheel version)
Turning width	• 1520 mm
Drive range in accordance with ISO 7176-4:2008 ***	• 38 km

Dimensions	
Overall length	• 1220 mm
Drive unit width	• 590 mm
Overall width (armrest adjustment range)	• 580 730 mm
Total height	• 990 mm
Seat width	• 470 mm
Seat depth	• 410 mm
Backrest height ****	• 475 mm
Backrest angle	• +9,5°
Armrest height	• 200 mm

Weight	
Kerb weight	• 83.5 kg

Component weights	
Chassis	• 3-wheel: ca. 40.5 kg
	• 4-wheel: ca. 46 kg
Seat unit	• ca. 14 kg
Batteries	ca. 12 kg per battery

Loading	
Max. load	• 136 kg

Axle loads	
Max. front axle load	• 85 kg
Max. rear axle load	• 160 kg

^{***} Note: The drive range of an electric wheelchair is strongly influenced by external factors, such as the charging state of the batteries, surrounding temperature, local topography, road surface characteristics, tyre pressure, weight of driver, drive style and use of batteries for lighting, servos etc.

The specified values are theoretical maximum achievable values measured according to ISO 7176-4:2008.

****** Static stability according to ISO 7176-1 = 9° (15,8 %)
Dynamic stability according to ISO 7176-2 = 6° (10,5 %)

^{****} Measured without seat cushion

15 Inspections Performed

It is confirmed by stamp and signature that all jobs listed in the inspection schedule of the Service and Repair Instructions have been properly performed. The list of the inspection jobs to be performed can be found in the Service Manual which is available through Invacare®.

Delivery Inspection	1 st Annual Inspection
	·
Stamp of authorised Dealer / Date / Signature	Stamp of authorised Dealer / Date / Signature
2 nd Annual Inspection	3 rd Annual Inspection
Stamp of authorised Dealer / Date / Signature	Stamp of authorised Dealer / Date / Signature
4 th Annual Inspection	5 th Annual Inspection
Stamp of authorised Dealer / Date / Signature	Stamp of authorised Dealer / Date / Signature

Hersteller / Manufacturer / Fabricante / Proizvođač:

CHIEN TI ENTERPRISE CO. LTD.

No. 13, Lane 227, Fu Ying Road Hsin Chuang, Taipei, Taiwan **R.O.C**.

Inverkehrbringer / European Representative / Representante europeo / Predstavništvo u Europi:

EMERGO EUROPE

Molenstraat 15 2513 BH, The Hague **The Netherlands**

Distribution / Distribución / Distribucija:

France, Italia, España, Portugal

Invacare® Poirier SAS

Route de St Roch F-37230 Fondettes

France

Tel: (33) (0) 247 - 62 64 66, Fax: (33) (0) 247 - 42 12 24

United Kingdom, Ireland

Invacare® Ltd

Pencoed Technology Park Pencoed Bridgend CF35 5AQ

United Kingdom

Tel: (44) (0) 1656 - 77 62 22, Fax: (44) (0) 1656 - 77 62 20

Sweden, Danmark, Nederland Invacare REA

Växjövägen 303 S-343 71 DIÖ

Sweden

Tel: (46) (0) 476 - 535 00, Fax: (46) (0) 476 - 535 99

Deutschland, Eastern Europe Invacare Deutschland GmbH

Kleiststraße 49 32457 Porta Westfalica

Deutschland

Tel: (49) (0) 5731 - 754 0, Fax: (49) (0) 5731 - 754 52 191

