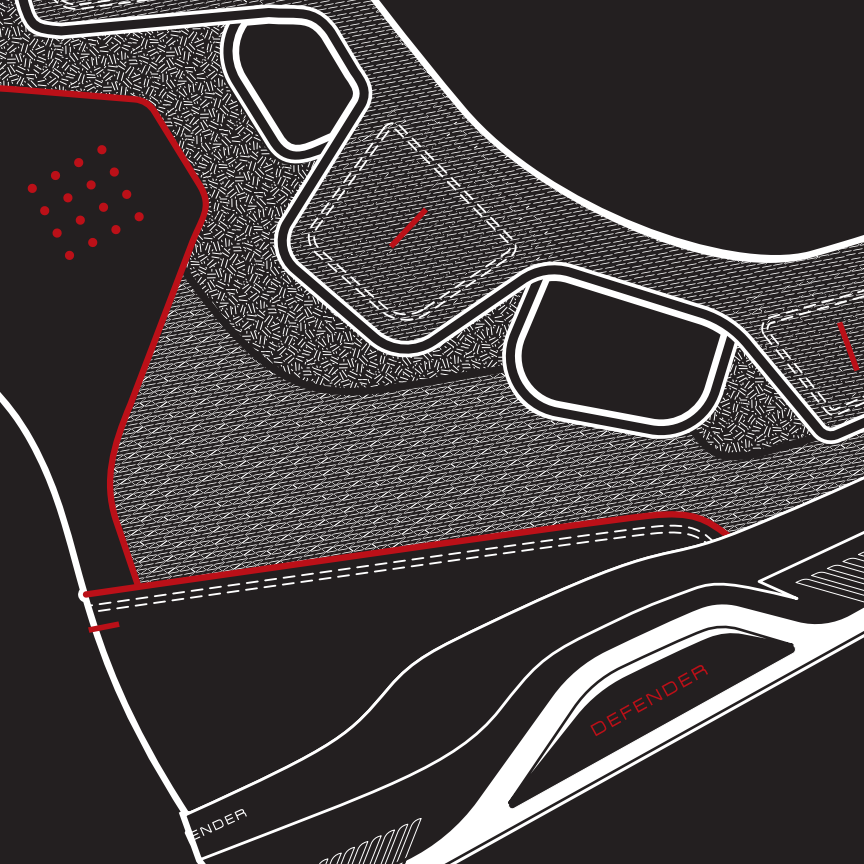
DEFENDER

Foot Defender Manual

ENDER

DEFENDER

Contents

- 02 What Is In The Box
- 03 Your Foot Defender
- 04 Putting Your Foot Defender On
- 10 Removing Your Foot Defender
- 11 Cleaning
- 11 Duration Of Use
- 12 Things To Remember
- 13 Customer Service

What Is In The Box

EN

The Foot Defender is a foam lined brace with pneumatic fit adjustment, a rigid outer sole, lower leg bracing system, and high performance shock absorbing insole. It is used to relieve and protect the injured foot and ankle. The Foot Defender reduces pressure and friction on the bottom of the foot to provide a healing environment for Diabetic Foot Ulcers. The Foot Defender works in a similar way to a cast with the benefit of being able to be removed for bandage changes or cleaning of the foot and ankle.

The footbed height of the Foot Defender is slightly higher than a normal pair of running shoes. We suggest that you wear running shoes or something similar on the other foot. Try to keep your feet as evenly balanced as you can.

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EN

The Foot Defender was designed to be easy to put on and take off, fit as many foot and ankle shapes as possible, and work with a minimum of fuss. It is of the utmost importance to wear your boot properly so that it functions optimally.

Follow these steps to use your Foot Defender:

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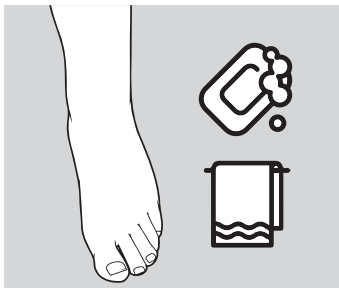
EN

Step 1: Make sure the leg and foot are clean and dry.

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EN


Step 2: Put on a sock. Always wear your Foot Defender with a sock and/or compression bandage as directed by your healthcare professional.

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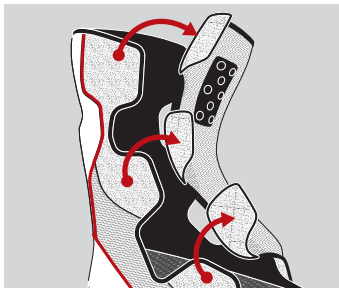EN

Step 3: Unfasten the three Velcro tabs of the Anterior Spat from one side of the Foot Defender.

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EN


Step 4: Place foot inside the Foot Defender. Make sure the heel is firmly on the footbed and all the way back against the vertical portion inside of the Foot Defender.

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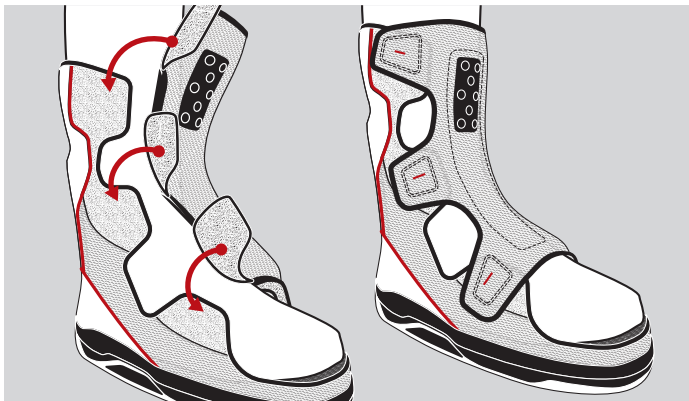EN


Step 5: Reposition the Anterior Spat on Foot Defender. Ensure all Velcro tabs are aligned and firmly fastened between corresponding wings of the Anterior Spat and the Foot Defender.

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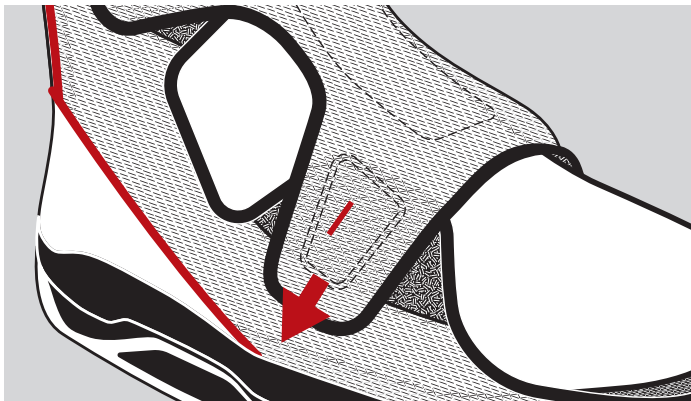EN

Step 6: Pull each Velcro tab of the anterior spat until the Foot Defender fits snugly. Ensure that the Velcro tabs are pressed down and engaged securely. The foot should not be able to move much.

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EN

Step 7: Locate the pneumatic pump on the side of the Foot Defender. Repeatedly press the pump until the pneumatic air bladder fills the gaps between the leg and the vertical shaft of the Foot Defender completing a secure and snug fit.

CAUTION: If you have reduced sensation be sure to have someone inspect the fit to reduce the risk of inappropriate fit leading to areas rubbing or high pressure.

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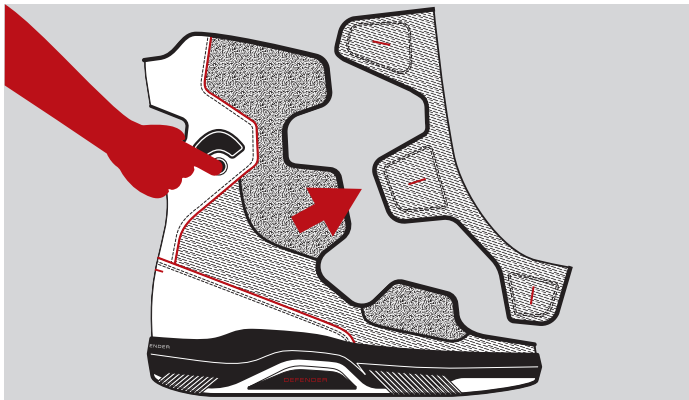
Removing Your Foot Defender

EN

Locate and press the release valve next to the pneumatic pump on the Foot Defender to loosen. Unfasten all Velcro tabs and remove the Anterior Spat.

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EN

If the Foot Defender gets soiled or dirty it can be wiped down with a disposable baby wipe or disinfecting wipe. Let air dry before using again.

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EN

Your doctor will decide how long you need to wear the Foot Defender. Please follow the doctor's instructions even if you feel improvement and would like to stop wearing it sooner. Your doctor will be checking your progress and will decide what is in your best long-term interest.

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EN

You should inspect your skin under the brace daily to check for any breakdown. If you notice areas of breakdown, localized redness, or have any issues please call your doctor. Always use the Foot Defender in accordance with your doctor's instructions and guidance. Your health care team may have given you this device as part of your care. If so, please use it as directed.

If this information was not given to you as part of your care, please check with your doctor. This is not medical advice. This is not to be used for diagnosis or treatment of any medical condition. Because each person's health needs are different, you should talk with your doctor or others on your health care team when using this information. **If you have an emergency, please call 911.**

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EN

Customer Service:

Always discuss any potential issues with your healthcare professional first.

Contact Defender operations with questions or concerns.
www.defenderoperations.com/customerservice

Copyright © 2021 Defender Operations. All rights reserved.

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PEEL

FOOT DEFENDER

Pneumatic Lower Extremity
Ambulatory Protective Device
XS SKU: 865634000495
L4361

PEEL

FOOT DEFENDER

Pneumatic Lower Extremity
Ambulatory Protective Device
S SKU: 860001567601
L4361

PEEL

FOOT DEFENDER

Pneumatic Lower Extremity
Ambulatory Protective Device
M SKU: 00860001567618
L4361

PEEL

FOOT DEFENDER

Pneumatic Lower Extremity
Ambulatory Protective Device
L SKU: 860001567625
L4361

PEEL

FOOT DEFENDER

Pneumatic Lower Extremity
Ambulatory Protective Device
XL SKU: 860001567632
L4361