



FOOT DEFENDER STRAP EXTENSION (2 pieces) INSTRUCTIONS

We have designed the Foot Defender to have a wide opening and a roomy fit but sometimes we still need more space. These strap extenders help expand the size of the calf area of the boot. These extenders come in one size, with two pieces in the kit.

Simply attach them to the boot at the top strap location. Then apply the front closure onto the strap extension and put the boot on as normal.

Sold as a pair.

Dimensions:
4" (10cm) x 1 3/4" (4.5cm)

Instructions:

1. Remove the Strap Extenders out of the kit and separate the 2 extender straps.
2. Remove the front closure from your Foot Defender. (See fig. 1)
3. Find the loop side of the Extender and attach the Extender to the top of the Foot Defender boot with about a 50% overlap. (see fig. 2)



Figure 2

4. Put your foot in the Foot Defender and apply the Foot Defender as normal.

Do not over tighten the top straps with your new extenders. You should be able to get 2 fingers spacing at your shin. (see fig 3)

Any question – please call Technical Service at 305-204-7203 x3



Figure 1



Figure 3