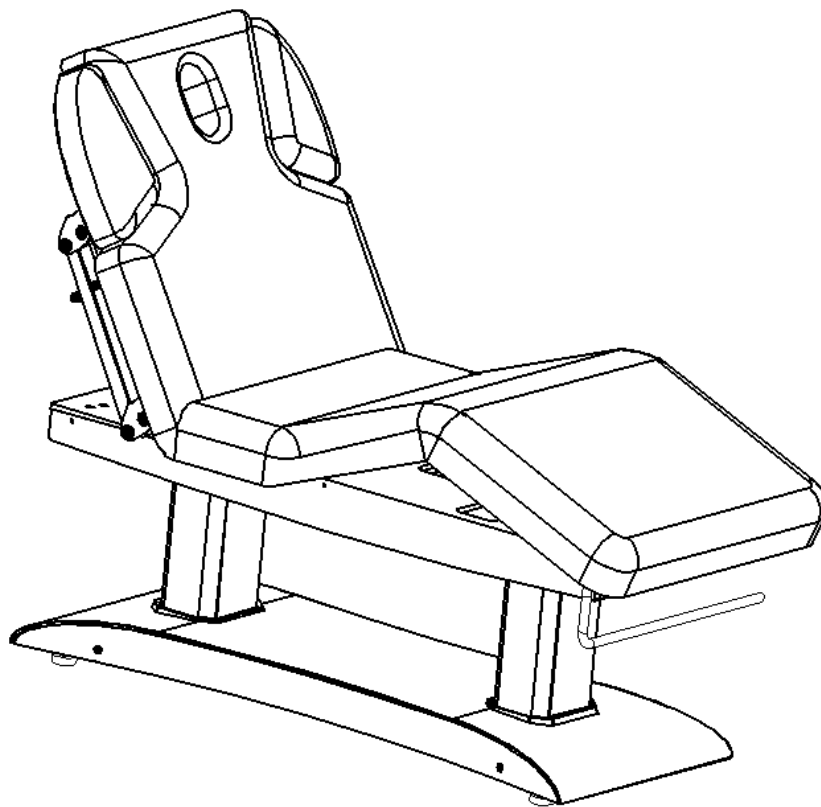
CHARA WARM

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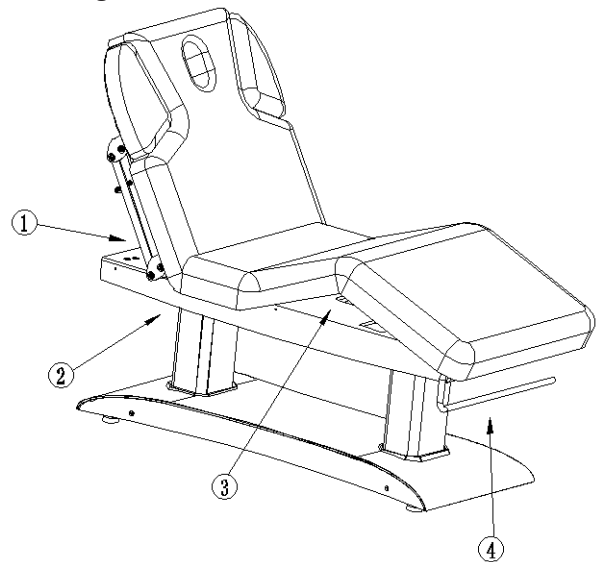
- The maintenance should not be dealt with without official training.

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Größe: 2000 x 760 x 660 mm

Mindestens: 660 mm

Maximal: 840 mm

Nettogewicht: 103 kg

Bruttogewicht: 128 kg

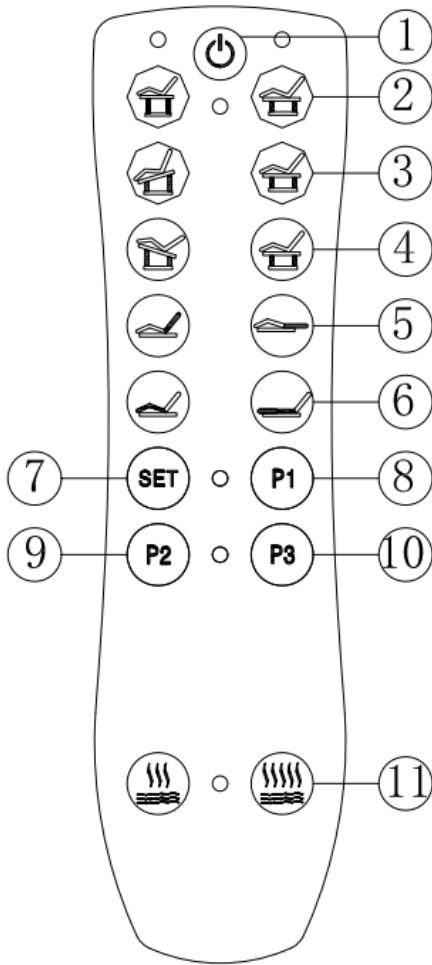
Kapazität: 200 kg

Eingangsspannung: AC100~130V/220~240V,
50/60Hz

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Leistung: 160 W

• REGLER

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① SW1 button: turn on/off hand control, press button, turn on hand control, press again button, turn off hand control, bed back to original position, press again button, bed stop recover.

② SW2 press button, bed rise up, press button, bed descend.

③ SW3 press , to adjust the bed tilt forward,

press , to adjust the bed flat.

④ SW4 press , to adjust the bed tilt backward;

press , to adjust the bed flat.

⑤ SW5 press , to adjust backrest up

press , to adjust backrest down.

⑥ SW6 press , Increased footrest angle,

press , reduced footrest angle.

⑦ SW7 press , setting Memory function.

⑧-⑩ Auto mode button, the bed will automatically adjust to the pre-set height and angle when any key is pressed.

Remark: Automatic mode setting to adjust the status of the lift, tilt, seat length, backrest and footrest sections, long press until the blue light next to it is on to activate the memory cartridge function, press set memory state. are the same method as .

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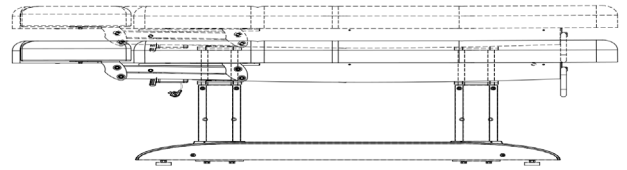
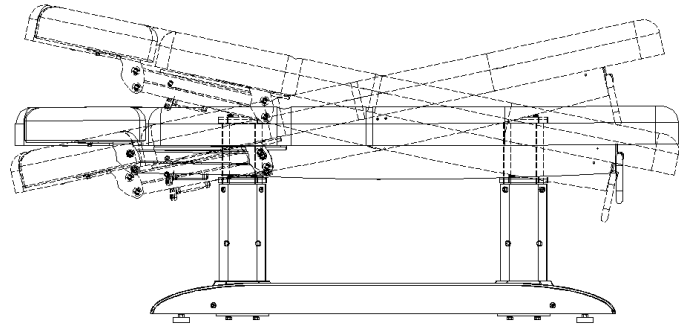
backward;press  , to adjust

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
press  button, backrest incline angle rise up,press

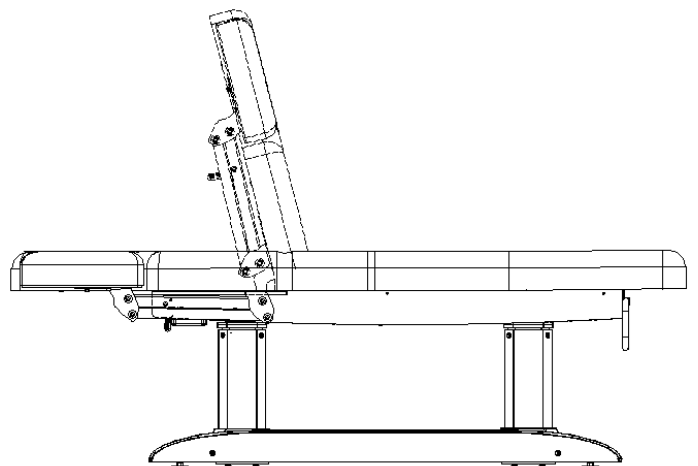button, reduce backrest incline angle, as pic shows :

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
press  button, footrest
incline angle rise up, press



 button, reduce
footrest incline angle.

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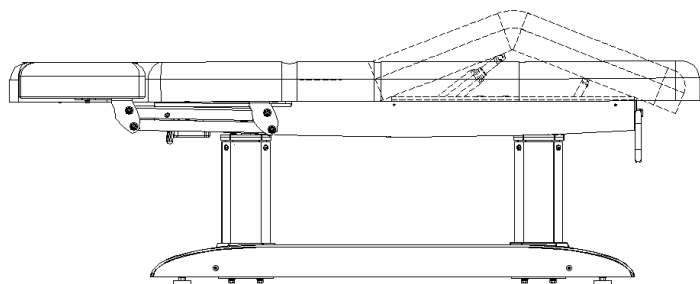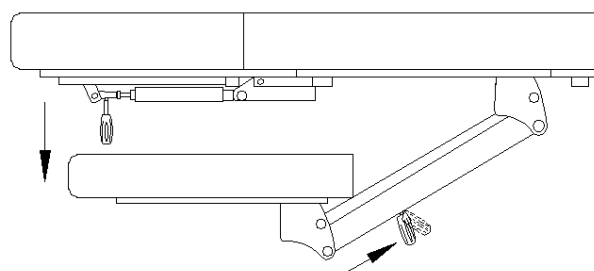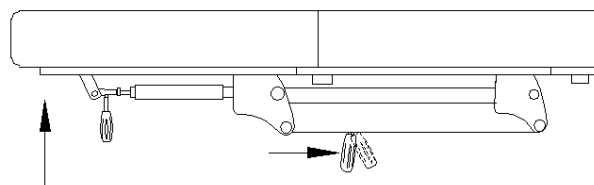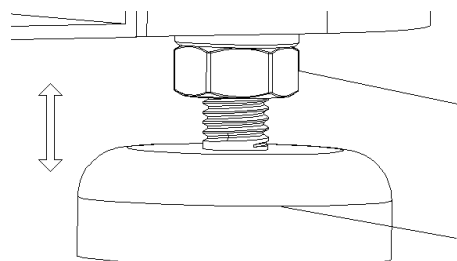
Phenomenon	Diagnosis	Operation of elimination
Press the pedal but without any responses.	1.Controller power switch start or not.	1.Try to re-start the controller power switch.
	2.To check if the pedal has reached to maximum.	2.Try other buttons workable or not.
	3. To check if the power plug is connected.	3.Disconnect the power plug and re-plug it again.
	4.To check if each connecting plug of the transformer is connected.	4.Re-plug and connect it again.
Letting out noises and shaking while going up and down	Screw loosen	Tighten all the screws

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