

Drugs of Abuse Tests

Cross-Reaction Reference Guide



COMPOUND	RESULTS
Acetyhydrocodone	Positive for Opiates (MOP,OPI) and Oxycodone (OXY)
Acetaminophen (4-Acetamidophenol)	Non-reactive
Acetophenetidin	Non-reactive
Acetylcodeine	Positive for Opiates (MOP,OPI)
Acetylsalicylic Acid	Non-reactive
Aciclovir Tablets	Non-reactive
Acid Reducer (Ranitidine Hydrochloride Tablets)	Non-reactive
Acyclovir	Non-reactive
Advil	Non-reactive
Afrin	Non-reactive
Airshield	Positive for EDDP, Methamphetamine (MET/mAMP) and Tricyclic (TCA)
Aleve	Non-reactive
Alprazolam	Positive for Benzodiazepines (BZO)
Aminophylline Tablets	Positive for Benzodiazepines (BZO)
Aminopyrine	Non-reactive
Amiodarone Hydrochloride Tablets	Non-reactive
Amitriptyline Hydrochloride Tablets	Positive for Tricyclic (TCA)
Amlodipine Mesylate Tablets	Non-reactive
Amoxicillin	Non-reactive
Amphetamine	Positive for Amphetamine (AMP), Ecstasy (MDMA) and Methamphetamine (MET/mAMP)
Ampicillin	Non-reactive
Ampicillin Capsules	Non-reactive
Ampicillin Sodium Injection	Non-reactive
Apomorphine	Non-reactive
Aripiprazole	Non-reactive
Aspartame	Non-reactive
Aspirin	Non-reactive
Atomoxetine	Positive for Benzodiazepines (BZO)

COMPOUND	RESULTS
Atorvastatin Calcium Tablets	Non-reactive
Atropine	Non-reactive
Atropine Sulfate	Non-reactive
Barbital	Positive for Barbiturates (BAR)
Benadryl (Diphenhydramine Hydrochloride)	Non-reactive
Benzilic Acid	Non-reactive
Benzodiazepine	Positive for Benzodiazepines (BZO)
Benzoic Acid	Non-reactive
Benzoylcegonine	Positive for Cocaine (COC)
Bilirubin	Non-reactive
Buprenorphine	Non-reactive
Bupropion	Non-reactive
Captopril Tablets	Non-reactive
Carbamazepine Tablets	Non-reactive
Cefaclor Capsules	Non-reactive
Cefradine Capsules	Non-reactive
Cephalexin	Non-reactive
Cephradine	Non-reactive
Chloral Hydrate	Non-reactive
Chloramphenicol	Non-reactive
Chlorothiazide	Non-reactive
Chlorpromazine	Non-reactive
Chlorpromazine Hydrochloride Tablets	Positive for Tricyclic (TCA)
Chlorpromazine Tablets	Positive for Tricyclic (TCA)
Chloroquine	Non-reactive
Cholesterol	Non-reactive
Chondroitin	Non-reactive
Cimetidine	Non-reactive
Ciprofloxacin Hydrochloride Tablets	Non-reactive
Citalopram	Positive for EDDP

COMPOUND	RESULTS
Clarithromycin Capsules	Non-reactive
Claritin	Non-reactive
Clomipramine Hydrochloride Tablets	Positive for Tricyclic (TCA)
Clonazepam	Positive for Benzodiazepines (BZO)
Clonidine	Non-reactive
Clopidogrel Hydrogen Sulphate Tablets	Non-reactive
Clozapine Tablets	Non-reactive
Cocaine Hydrochloride	Positive for Cocaine (COC)
Codeine	Positive for Opiates (MOP,OPI)
Codeine Phosphate	Positive for Benzodiazepines (BZO), Opiates (MOP,OPI) and Oxycodone (OXY)
Conjugated Estrogens Tablets	Non-reactive
Cortisone	Non-reactive
Creatinine	Non-reactive
D,L-Octopamine	Non-reactive
D,L-Propranolol	Non-reactive
D,L-Tryptophan	Positive for Amphetamine (AMP) and Barbiturates (BAR)
D,L-Tyrosine	Non-reactive
DayQuil	Positive for Benzodiazepines (BZO)
Delsym	Non-reactive
Deoxycorticosterone	Non-reactive
Depakote	Non-reactive
Dextromethorphan	Non-reactive
Dextromethorphan Hydrobromide Soft Capsules	Non-reactive
Dextropropoxyphene Napsylate	Positive for Propoxyphene (PPX)
Diazepam Injection	Positive for Benzodiazepines (BZO)
Diclofenac Sodium Enteric-coated Sustained Release Capsules	Non-reactive
Diclofenac	Non-reactive
Diflunisal	Non-reactive
Digoxigenin	Non-reactive

COMPOUND	RESULTS
Digoxin	Non-reactive
Diphenhydramine	Non-reactive
Diphenoxylate Hydrochloride	Non-reactive
Dirithromycin Enteric-coated Capsules	Non-reactive
D-Norpropoxyphene	Non-reactive
Domperidone Tablets	Non-reactive
Dopamine Hydrochloride	Positive for Amphetamine (AMP)
Doxepin Hydrochloride Tablets	Positive for Tricyclic (TCA)
Doxylamine	Positive for Tramadol (TRA)
D-Pseudoephedrine	Non-reactive
Duloxetine	Non-reactive
Ecgonine Methyl Ester	Non-reactive
Ecstasy Hydrochloride	Positive for Ecstasy (MDMA) and Methamphetamine (MET/mAMP)
Effexor	Non-reactive
Enalapril Maleate Tablets	Non-reactive
Ephedrine Hydrochloride	Positive for Methamphetamine (MET/mAMP)
Epinephrine Hydrochloride	Non-reactive
Erythromycin	Positive for Benzodiazepines (BZO)
Esomeprazole Magnesium Enteric-coated Tablets	Non-reactive
Estroven	Non-reactive
Extenze	Positive for Oxycodone (OXY)
Fenofibrate	Non-reactive
Fenoprofen	Non-reactive
Fentanyl Citrate	Positive for Ecstasy (MDMA)
Flunitrazepam	Positive for Benzodiazepines (BZO)
Fluoxetine Hydrochloride	Positive for Benzodiazepines (BZO)
Fluvoxamine	Non-reactive
Furosemide	Non-reactive
Furosemide Tablets	Positive for Benzodiazepines (BZO)
Gabapentin	Non-reactive

COMPOUND	RESULTS
Gentisic Acid	Non-reactive
Glibenclamide Tablets	Non-reactive
Gliclazide Tablets	Non-reactive
Glipizide Controlled-release Tablets	Non-reactive
Glucose	Non-reactive
Haloperidol Tablets	Non-reactive
Heartburn Relief	Non-reactive
Hemoglobin	Non-reactive
Hydralazine	Positive for Benzodiazepines (BZO)
Hydrochlorothiazide	Non-reactive
Hydrocortisone	Non-reactive
Hydrocortisone Butyrate, Hydrocortisone Injection	Non-reactive
I Caps	Non-reactive
Isosorbide Dinitrate Tablets	Non-reactive
Isoxsuprine	Non-reactive
Ketoconazole Tablets	Non-reactive
Ketoprofen	Non-reactive
Labetalol	Non-reactive
Lamotrigine	Positive for Phencyclidine (PCP)
Lansoprazole Tablets	Positive for Benzodiazepines (BZO)
Levofloxacin Hydrochloride Tablets	Non-reactive
Levonorgestrel Tablets	Non-reactive
Levothyroxine Sodium	Non-reactive
Lidocaine Hydrochloride Injection	Non-reactive
Lisinopril Tablets	Non-reactive
Lithium Carbonate Tablets	Non-reactive
Liverite	Positive for Cannabinoids (THC)
Lofexidine Hydrochloride	Positive for Oxycodone (OXY)
Loperamide	Non-reactive
Loperamide Hydrochloride Capsules	Non-reactive

COMPOUND	RESULTS
Loratadine Tablets	Non-reactive
Lorazepam	Positive for Benzodiazepines (BZO)
L-y-Ephedrine	Positive for Methamphetamine (MET/mAMP)
Magnesium	Non-reactive
Mega-T Plus (Phytosterols, Vitamin B6)	Non-reactive
Meperidine	Non-reactive
Meprobamate	Non-reactive
Methadone	Positive for EDDP and Methadone (MTD)
Methamphetamine	Positive for Ecstasy (MDMA), Methamphetamine (MET/mAMP) and Tricyclic (TCA)
Methamphetamine Hydrochloride	Positive for Ecstasy (MDMA), Methamphetamine (MET/mAMP) and Tricyclic (TCA)
Methoxyphenamine	Positive for Amphetamine (AMP) and Barbiturates (BAR)
Metoprolol Tartrate Tablets	Non-reactive
Midazolam	Positive for Benzodiazepines (BZO)
Mifepristone Tablets	Non-reactive
Mirtazapine Tablets	Non-reactive
Montelukast Sodium Tablets	Non-reactive
Morphine	Positive for Opiates (MOP,OPI)
Mosapride Citrate Dispersible Tablets	Non-reactive
Mucinex	Non-reactive
Nalidixic Acid	Non-reactive
Naloxone	Non-reactive
Naloxone Hydrochloride	Positive for Oxycodone (OXY)
Naltrexone	Non-reactive
Naltrexone Hydrochloride	Positive for EDDP, Oxycodone (OXY) and Tramadol (TRA)
Naphthalene Sulfonic Propoxyphene	Positive for Propoxyphene (PPX)
Naproxen	Non-reactive
Niacinamide	Non-reactive
Nifedipine	Non-reactive

COMPOUND	RESULTS
Nifedipine Tablets	Non-reactive
Nikethamide Injection	Non-reactive
Nimetazepam	Positive for Benzodiazepines (BZO)
Nimodipine Tablets	Non-reactive
Nitrazepam	Positive for Benzodiazepines (BZO)
Nitroglycerin Tablets	Non-reactive
Nordiazepam	Positive for Benzodiazepines (BZO)
Norethindrone	Non-reactive
Noscapine	Non-reactive
O(3)-Monoacetylmorphine Sulfamate	Positive for Opiates (MOP,OPI)
O-Hydroxyhippuric Acid	Non-reactive
Olanzapine Tablets	Non-reactive
Omeprazole	Non-reactive
Omeprazole Enteric-coated Tablets	Non-reactive
Oxalic Acid	Non-reactive
Oxazepam	Positive for Benzodiazepines (BZO)
Oxolinic Acid	Non-reactive
Oxycodone Hydrochloride, Acetaminophen	Positive for Benzodiazepines (BZO), Opiates (MOP,OPI), Oxycodone (OXY) and Tramadol (TRA)
Oxymetazoline	Non-reactive
Paliperidone	Non-reactive
Pantoprazole	Positive for Cannabinoids (THC)
Papaverine	Non-reactive
Paracetamol, Pseudoephedrine Hydrochloride, Dextromethorphan Hydrobromide and Chlorpheniramine Maleate Tablets	Positive for Benzodiazepines (BZO), Methamphetamine (MET/mAMP) and Phencyclidine (PCP)
Paracetamol Tablets	Non-reactive
Paroxetine	Non-reactive
Penfluridol Tablets	Non-reactive
Penicillin V Potassium	Non-reactive
Penicillin-G	Non-reactive

COMPOUND	RESULTS
Perphenazine	Positive for Tricyclic (TCA)
Perphenazine Tablets	Positive for Tricyclic (TCA)
Pethidine Hydrochloride	Non-reactive
Phenelzine	Non-reactive
Phenobarbital	Positive for Barbiturates (BAR)
Phenol Asami-sensitized	Non-reactive
Phenoxyethyl Penicillin Potassium Tablets	Non-reactive
Phentolamine	Positive for EDDP and Tramadol (TRA)
Phenylephrine	Positive for Methamphetamine (MET/mAMP)
Phenytoin Sodium Tablets	Positive for Barbiturates (BAR)
Pholcodine	Positive for Opiates (MOP,OPI)
Pioglitazone Hydrochloride Dispersible Tablets	Non-reactive
Piracetam Tablets	Non-reactive
Pravastatin Sodium	Non-reactive
Prednisone	Non-reactive
Prednisone Acetate Tablets	Non-reactive
Procaine	Positive for Cocaine (COC)
Procaine Hydrochloride Injection	Positive for Cocaine (COC) and Methamphetamine (MET/mAMP)
Propranolol Hydrochloride	Non-reactive
Propylthiouracil Tablets	Non-reactive
Pseudoephedrine Hydrochloride	Positive for Methamphetamine (MET/mAMP)
Quetiapine Fumarate	Positive for Tricyclic (TCA)
Quinine	Non-reactive
Ranitidine	Non-reactive
Rifampicin Capsules	Non-reactive
Risperidone Tablets	Non-reactive
Salicylic Acid	Non-reactive
Secobarbital	Positive for Barbiturates (BAR) and Methamphetamine (MET/mAMP)
Serotonin	Non-reactive
Sertraline Hydrochloride Tablets	Non-reactive

COMPOUND	RESULTS
Sildenafil Citrate Tablets	Non-reactive
Simvastatin Tablets	Non-reactive
Sinus & Allergy	Positive for Benzodiazepines (BZO) and Methamphetamine (MET/mAMP)
Sodium Valproate Tablets	Non-reactive
Spirolactone Tablets	Non-reactive
Sulfamethazine	Non-reactive
Sulindac	Non-reactive
Temazepam	Positive for Benzodiazepines (BZO)
Tetracycline	Non-reactive
Tetracycline Tablets	Non-reactive
Tetrahydrocortisone 3 -acetate	Non-reactive
Tetrahydrocortisone 3-(BD-*glucuronide)	Non-reactive
Tetrahydrozoline	Non-reactive
Triazolam	Positive for Benzodiazepines (BZO)
Thebaine	Positive for Opiates (MOP,OPI)
Thiamine	Non-reactive
Thioridazine	Non-reactive
Topiramate	Non-reactive
Tramadol Hydrochloride	Positive for Tramadol (TRA)
Trazodone Hydrochloride Tablets	Non-reactive
Triamterene	Non-reactive
Trifluoperazine	Non-reactive
Trimethoprim	Non-reactive
Tyramine	Positive for Amphetamine (AMP) and Barbiturates (BAR)
Urine Acid	Non-reactive
Valproate	Non-reactive
Venlafaxine Hydrochloride	Positive for Phencyclidine (PCP)
Verapamil	Non-reactive
Vitamin B1	Non-reactive
Vitamin B12, Proprietary Blend	Non-reactive

COMPOUND	RESULTS
Vitamin B2	Non-reactive
Vitamin C Tablets	Non-reactive
Zencore Plus 2	Non-reactive
Zoloft	Non-reactive
Zomepirac	Non-reactive
B-Estradiol	Non-reactive
3-Hydroxytyramine	Non-reactive

Version 04/10/2015