

# KN90 Face Mask

## User Manual

Model Number: FM1-4plus

Easy@Home® KN90 Face Mask protects from dust and other large particles, and anti-inhalation allergens such as pollen.

### Filtration efficiency

BFE(Bacterial filtration efficiency) >98%

PFE(Nonoil particle filtration efficiency) >90%

### Product component

A mask body, strap, the nose clip.

### Instructions for use



**Step 1:** Remove the protective clip and stretch the mask, make sure the position of the metal nosepiece is on top.



**Step 2:** Cover the mask on the nose and mouth, and pull the straps around both ears.



**Step 3:** Fix metal nosepiece to mode the nose area, then check air leakage by covering the mouth and nose with both hands, and exhale. Readjust nosepiece if air leaks around the nose.

### Storage and usage recommendations

1. Wash your hands before wearing and avoid touching the inside of the mask when wearing.
2. If needed, the mask can be recycled and should be stored in a clean, open paper bag; If necessary, it can be rinsed with clean water and hung in a clean, dry and ventilated place;
3. The mask should be in sealed in individual package and avoid being touched.
4. If the mask is dirty, deformed, damaged, or has an odor, it should be replaced in time.
5. If the mask comes into contact with or is in close contact with the patient's blood, respiratory / nasal secretions or is contaminated by other bodily fluid, please immediately replace it.
6. This product is prohibited to be washed by hot water above 140°F (60°C) or disinfected by detergent, ultraviolet rays, microwave oven, steam, alcohol or disinfectant etc, as this can permanently damage the mask(s).

### Precautions

The following people should not wear masks:

- 1) People with heart or respiratory problems (such as asthma, emphysema), and please try not to be exposed to the hazy environment for a long time.
- 2) Pregnant women.
- 3) Dizziness, dyspnea and those with sensitive skin after wearing.
- 4) Children under 3 years old.

### Storage conditions

Store at room temperature, avoid moisture and water.

**Packing** 2 Masks.

**Production date / batch number** See packaging.

**Shelf life** 36 months

**Healthcare-Manager.com**

Questions or comments?

Please call toll-free:

1-855-822-6999 M-F 9 a.m.-5 p.m. CST

E-mail: [service@healthcare-manager.com](mailto:service@healthcare-manager.com)

Easy Healthcare Corporation

360 Shore Dr. Burr Ridge, IL USA 60527