

FEMALE URINATION DEVICE

Q1: How to use it?

First, turn the end of the urinal marked "front" toward the front of body, then hold the urinal as indicated on the diagram.

Then, lean slightly forward and start peeing (can extend with extension tube).

Tips: Before your first use outside, you should practice in the bathroom a few times.



Q2: What kind of people is this suitable for?

This peeing funnel is suitable for females of all ages. Especially for those who are in late-pregnancy, post-surgery, doing outdoor activities, or who suffer from leg pain that have difficulty sitting and squatting.

Q3: What situations is this female urination device used for?

It is helpful when you are doing outdoor activities such as going hiking, camping, boating, going on a road-trip, attending concerts, getting caught in traffic jams, using dirty toilets, etc.

Q4: What kind of material is this peeing cup made of?

It is made of high quality silicone that is medical grade. It's soft, comfortable, hygienic, and does not irritate the skin.

Q5: Is it safe for a little girl to use this product?

Yes, of course.

Q6: What is the size of it?

The approximate size for pee funnel: unfolded length 9.6-inch, folded length 5.5-inch. And the length of the extension tube is approximately 3.8-inch.



Q7: Is the drawstring bag waterproof?

Yes, but you should shake off as much water as possible before storing the funnel in the bag.

Q8: How should I clean and store it?

After each use, wash thoroughly with water or soapy water, and then clean with rubbing alcohol.

Q9: What does the package include?

- 1 x female urination device
- 1 x extension tube
- 1 x waterproof drawstring bag
- 1 x instructions