

Cleaning and Maintenance

The unit should be cleaned with a damp soft cloth. A non abrasive cleaning agent may be used for stubborn stains. **Make sure that no liquid enters the unit.**

The unit has been designed for use by people with special needs however this does NOT mean it is unbreakable. You must ensure the unit is not kicked or otherwise abused, or saturated in liquid.

Troubleshooting

Q. The unit doesn't work.

A1. Is it plugged in? If so,

A2. Is the unit paired to an IRiS Listener? If so,

A3. Try pairing the unit to another IRiS Listener. Does this work? If not, please contact us.

Contact Details

For USA please email office@experia-usa.com

For UK and rest of the world email support@experia-innovations.co.uk

Telephone: (800) 882-4045

Please note: email is our support team's preferred communication and this takes precedence over telephone calls.

QUESTIONS & IDEAS?

If you have an idea on how to improve this product or want to see your product idea realized, please email: ideas@experia-usa.com.

All ideas are presented to our OT, technical and finance department and if all procedures are passed we will contact you to discuss your idea further.

We can't promise to develop every product but if it's a great idea, we'll try!

Manufactured proudly in the UK by:

Experia

Acorn Phase 3, High Street, Grimethorpe
Barnsley, South Yorkshire, S72 7BD United Kingdom

© Copyright 2010-2013 Experia

All rights reserved.

Experia is a registered trademark.



experia **USA**
multisensory for everyone



IRiS Balance Beam



WARNING: CHOKING HAZARD

Small Parts: Not suitable for children under 3 years.



3+ years

Congratulations on the purchase of your IRiS Balance Beam!

The IRiS Balance Beam has been designed specifically for people with special needs and should, if used in accordance with these guidelines, offer years of switching learning and fun!

The IRiS Balance Beam offers opportunities to develop numerous skills including color recognition, cause and effect, eye-hand coordination, and gross/fine motor skills and many more.

The IRiS Balance Beam is an IRiS Talker and can control any number of IRiS Listeners.

Please ensure you read and follow these instructions carefully before installing or operating the unit. Please keep this manual for future reference.

If upon receipt of this product, the user suspects that the product is defective, and/or requires service or repair due to malfunction or damage to the product, please contact the manufacturer immediately. Please be aware that improper installation and/or use or misuse of this product may result in injury, death, or adverse reactions to its user. Failure to comply with these precautions or with specific warnings elsewhere in this manual violates safety standards of design, manufacture, and intended uses of the product.

Features

- IRiS wirefree compatible
- Hi-grade, phthalate-free vinyl
- Durable & easy to use
- 1 Year manufacturers guarantee (Collect and Return)

Pack contents

- 1 x IRiS Balance Beam
- 4 x 'AA' alkaline batteries

Getting Started

1. Install the 'AA' batteries into the compartment on the bottom of the Balance Beam. Follow markings on the battery case.
2. Decide which IRiS Listeners you want to control and pair the unit to them – see 'Pairing to IRiS Listeners' section.
3. Now its ready to use, simply touch any panel on the unit. The IRiS Listener will change colors to match the color panel that was pressed.

Caution: Always know the limitations of the user, and employ proper spotting techniques and protective gear where necessary. Do not allow use of this product unsupervised

Pairing to IRiS Listeners

This procedure is split into 2 sections, one for those who do not have an IRiS Master Paired and for those who do.

Without a IRiS Master Paired: Every IRiS Listener has a button on it called a pairing button. To get the IRiS Balance Beam to work with a product, first find the pairing button on the product and press it. You then need to touch a color on the IRiS Balance Beam within 5 seconds. The IRiS Listener will change to the color you have pressed if correctly paired. The unit will now work with the IRiS Balance Beam.

With a IRiS Master Paired: To get the IRiS Balance Beam to work with a product, first decide which unit you want to control and find its pairing button on the IRiS Master Paired and press it (refer to your manual for the master paired). You then need to touch a color on the front of the IRiS Balance Beam within 5 seconds. The IRiS Listener will change to the color you have pressed if correctly paired. The unit will now work with the IRiS Balance Beam.

Installing Batteries: IMPORTANT: BATTERY INFORMATION

- Please retain this information for future reference.
- Batteries should be replaced by an adult.
- Turn the unit upside down to see the battery compartment.
- Apply pressure to the tab at either side.
- Remove the battery compartment drawer.
- Fit 4 x 1.5v AA batteries into the drawer following the marks thereon.
- Push the drawer back into the battery compartment.
- Check to ensure it is in fully and that both drawer tabs are flush with the battery compartment.

CAUTION:

- Always follow the instructions carefully. Use only batteries specified and be sure to insert them correctly by matching the + and – polarity markings.
- Do not mix old batteries and new batteries.
- Experia recommend the use of alkaline batteries.
- Remove exhausted or dead batteries from the product.
- Remove batteries if product is not to be in use for two weeks or longer.
- Do not short circuit the supply terminals.

DO NOT MIX ALKALINE BATTERIES WITH RECHARGEABLE BATTERIES