216 | Model D Loop | Gebrauchsanweisung

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Product name: Model D Loop

Product reference: 216 217

Categorization: Class I Medical Device accordingly Regulation (EU) 2017/745. Fulfills the requirements of the standard EN 14683:2019+AC:2019 for medical face masks as well as biological assessment for medical devices accordingly ISO 10993.

CHECK PRIOR TO USE: Inspect the mask, including any component parts, for holes, tears or damage prior to use. Should any damage be identified, DO NOT USE IT.

FITTING INSTRUCTIONS:

1. Before touching the mask, wash your hands with soap and water or clean your hands with alcohol-based hand disinfectant.
2. Hold the mask in front of your face: coloured side facing outwards and edge with nose wire at the top.
3. Attach ear loops behind your ears. Using both hands, bend the nose clip around the nose for a good fit on the face.
4. While holding the mask in place with one hand, pull the bottom of the mask under your chin, expanding the folds of the mask. The mask should now fully cover your nose, mouth and jaw.
5. When wearing glasses, place them over the mask.

IMPORTANT: Do not touch the contaminated outside of the mask during or after use. Pick up the elastic ear loops behind your ears to remove the mask. Immediately dispose of properly. Wash and disinfect your hands after use. If the mask becomes damp, replace immediately.

Storage: Store in a dry environment out of direct sunlight in original packaging, see also packaging.

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Hersteller: DACH Schutzbekleidung GmbH & Co. KG
Manufacturer: DACH Schutzbekleidung GmbH & Co. KG