# Thealoz® Duo UD

This product is a medical device. Please read the instructions before use.  
Keep this leaflet. You may need to read it again.

## INDICATIONS

THEALOZ DUO UD is indicated to protect, hydrate and lubricate the surface of your eyes, for treatment of moderate to severe dry eye syndrome. For ocular use only.

## SUITABLE FOR

- Adults,
- Children: Parents or carers should administer the product,
- Pregnant or breastfeeding women based on current knowledge,
- Wearers of any type of contact lenses.

## CONTRAINDICATIONS

Do not inject and do not swallow.  
Do not use THEALOZ DUO UD if you are allergic to any of its ingredients (see Device description).

## DEVICE DESCRIPTION

THEALOZ DUO UD is a solution containing trehalose (3%), sodium hyaluronate (0.15%), sodium chloride, trometamol, hydrochloric acid and water for injections. THEALOZ DUO UD is a sterile, phosphate-free, hypotonic and pH neutral solution.

THEALOZ DUO UD is gentle for eyes because it is preservative-free. Avoiding chemical preservatives is beneficial for the eye's surface long term and eliminates reactions amongst people who are sensitive to preservatives.

## BENEFIT, PERFORMANCE AND MECHANISM OF ACTION

THEALOZ DUO UD provides immediate comfort that lasts all day long, to relieve the symptoms of dry eyes. THEALOZ DUO UD is recommended if you have discomfort, stinging, foreign body sensations or irritation of your eye. These symptoms can be caused by factors such as wind, smoke, pollution, dust, sunny or cold conditions, dry heat, air conditioning, plane journeys, and also by long hours working in front of a computer screen.

THEALOZ DUO UD is compatible with the delicate surface of your eyes, and can also be used by people wearing any type of contact lenses.

The main ingredients of THEALOZ DUO UD are trehalose and sodium hyaluronate: this unique combination of trehalose and sodium hyaluronate in THEALOZ DUO provides long-lasting protection, hydration and lubrication of the eye surface.

Trehalose is a natural substance found in many plants and animals, which need to survive in extreme conditions of temperature and dryness. It provides protective and hydrating properties.

Sodium hyaluronate, naturally found in the human eye, holds water to hydrate and lubricate the surface of the eye. It keeps the solution on the surface of the eyes, giving long lasting relief and improving the rate of healing.

## GENERAL RECOMMENDATIONS FOR USE

Apply 1 drop to each eye, from 4 to 6 times daily.

There is no particular training required to use THEALOZ DUO UD.

Wash hands thoroughly before use.  
Open the foil sachet and take one single-use vial off the strip.  
Replace the rest of the strip back into the sachet.  
Do not use any tool to open the vial.

Turn it upside down and tap the vial until the solution comes down.

Open the vial by twisting the top tab off.  
To avoid contamination, do not touch the tip of the vial with anything.

Squeeze until you see the solution.  
Apply one drop in the pocket of the eyelid, while gently pulling the lower lid down and looking up. Close the eye gently after application.

Throw away the single use vial, even if there is some solution left, do not keep it.

## CAUTIONS FOR USE

Check the packaging is intact before first use: use only if the sachets and single-use vials are unopened.  
To avoid contamination, do not touch the tip of the vial with anything.

Do not reuse the vial, because after you open it the contents are not protected from microbial contamination. You must contact a healthcare professional if your symptoms get worse or do not improve.

## WARNINGS

Keep out of the sight and reach of children.  
After use, do not drive or use machines if your vision is blurred: wait for a few minutes until your vision clears.  
Wait at least 10 minutes between using two different eye products.  
Never dilute THEALOZ DUO UD or mix it with other solutions.

## SIDE EFFECTS

The following side effects have been reported: rare possibility of mild eye irritation and ocular redness. In such cases, contact lens wearers should remove their contact lenses.

When you use eye drops, you might experience disturbing symptoms such as burning sensation, stinging sensation, foreign body sensation in the eye and blurred vision for a short time.

If you note any side effects or unusual sensations after using this product, please contact a healthcare professional and report the information to the local distributor, to the manufacturer or to your local Health Authority (see contacts at the end of the instructions for use).

## STORAGE AND DISPOSAL CONDITIONS

Store between 8 °C and 30 °C.  
Do not expose your product to direct sunlight or humidity.

Throw away the single use vial immediately after use even if there is solution left in it.

Keep the unopened single-use vials in their sachet in order to protect them from light.

The unopened single-use vials can be used up to three months after first opening of the sachet.

Do not use after the expiry date which is marked on the single-use vial and on the outer box.

The expiry date refers to the last day of that month as long as the packaging is intact and it has been stored correctly.

Dispose of the product responsibly and in accordance with local guidance (the single use vial and the sachet should be disposed of as household waste, the outer carton and the leaflet are recyclable).

## Date of issue and version of the instruction for use:

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## Distributor:

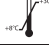
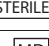
THEA Pharmaceuticals Ltd, MED IC5, Innovation Way,  
Keele University Science and Innovation Park, Keele,  
Newcastle Under Lyme, Staffordshire, ST5 5NT  
www.thea-pharmaceuticals.co.uk  
Tel.: 0345 521 1290

## Health authority:

Medicines and Healthcare products Regulatory Agency,  
10 South Colonnade, London, E14 4PU  
Tel.: 020 3080 6000

## MEANING OF THE PICTOGRAMS ON THE PACKAGING

	Manufacturer
	Batch number
	Date of manufacture
	Use-by date
	Temperature limits
	Sterilized using aseptic processing techniques
	Medical Device
	Consult instruction for use
	Single use
	Do not use if package is damaged
	Caution

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Théa Pharma GmbH, Schillerstraße 3, 10625 Berlin