

ouson⁺

CARE™



HOME ESSENTIAL



OUSON+ CARE

1 **Arm-Type Electronic
Blood Pressure Monitor**

Page 01 - 06

2 **Infrared Thermometer**

Page 07 - 14

3 **Fingertip Pulse Oximeter**

Page 15 - 18

4 **Disinfectant Spray**

Page 19 - 22

5 **Hand Sanitizer**

Page 23 - 26

6 **Botanica Lavandula**

- Anti-Mosquito
- Aromatherapy

Page 27 - 32

**HOME
ESSENTIAL**

OUSON+TM

CARE

“ On a visit to our local pharmacy, we realized that healthcare equipment came at a very high price. This sparked the idea of bringing affordable healthcare to the masses, and so we set out to achieve our mission to provide Malaysians access to affordable healthcare – anytime, anywhere; because we truly wanted to help everyone live healthier lives and attain a higher quality of life.

Founders of Ouson+ Care ”

Established in 2015, Ouson+ Care is a proud Malaysian brand distributor of affordable health care equipment suitable for both home and medical use. As a believer in the preventability of lifestyle diseases, we're committed to helping everyone lead healthier lives – starting in their own homes, which is why it is our mission to continuously deliver high-quality and reliable products for the management of health at home.

From blood pressure monitors, thermometers, disinfectant sprays, and more – our products are made with you in mind. Not only are Ouson+ Care products GDPMD (Good Distribution Practice for Medical Device) and KKM (Kementerian Kesihatan Malaysia) certified, we also adhere to a strict code of conduct to ensure the quality, safety, and performance of our products, especially our medical devices.



WINNER OF THE SUSTAINABLE BRAND AWARDS (SBA) 2021

SME 100 AWARDS 2021 RECIPIENT (MALAYSIA)





BSX556 / BSX523 ARM-TYPE ELECTRONIC BLOOD PRESSURE MONITOR



BSX556

MAIN UNIT

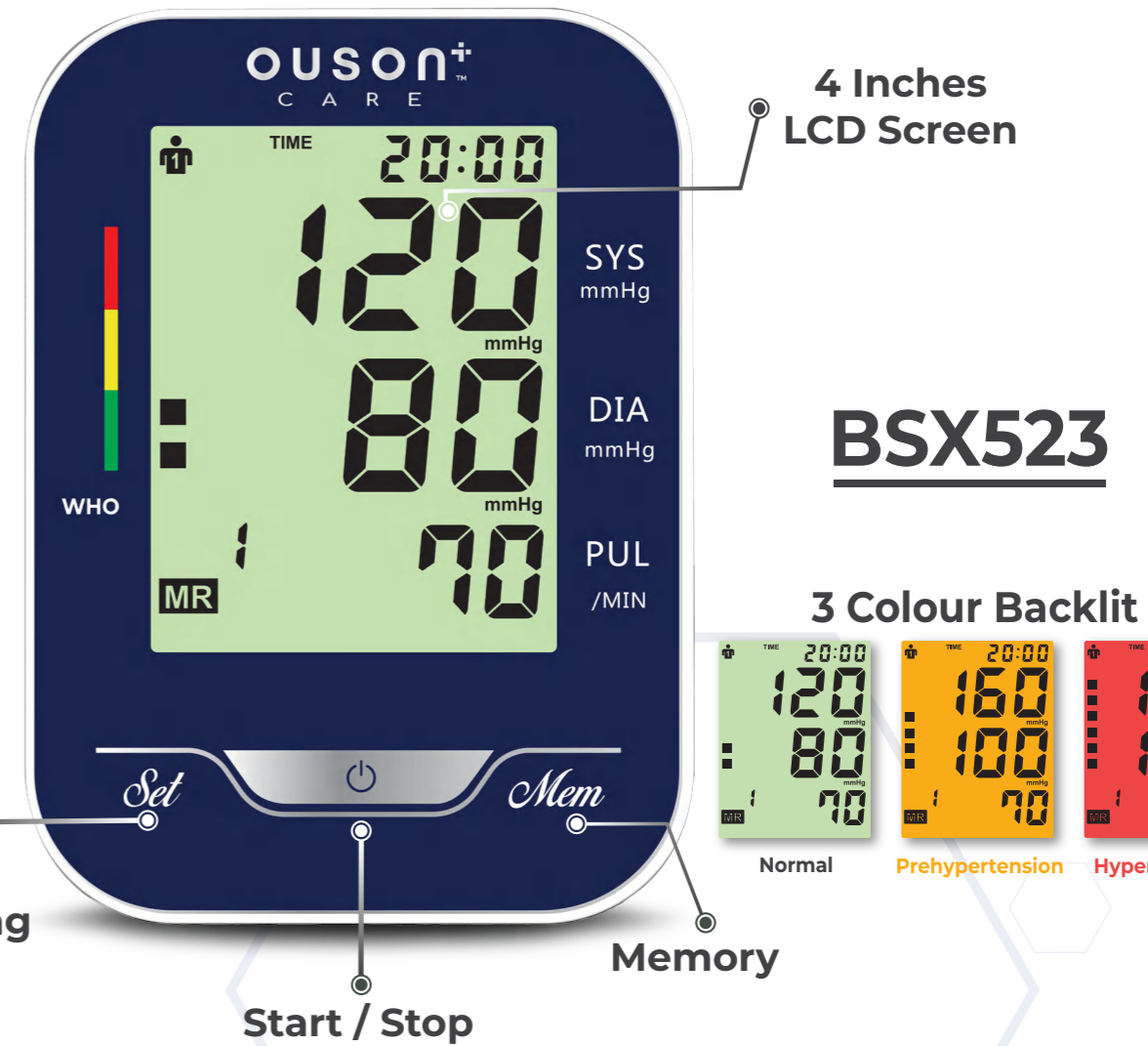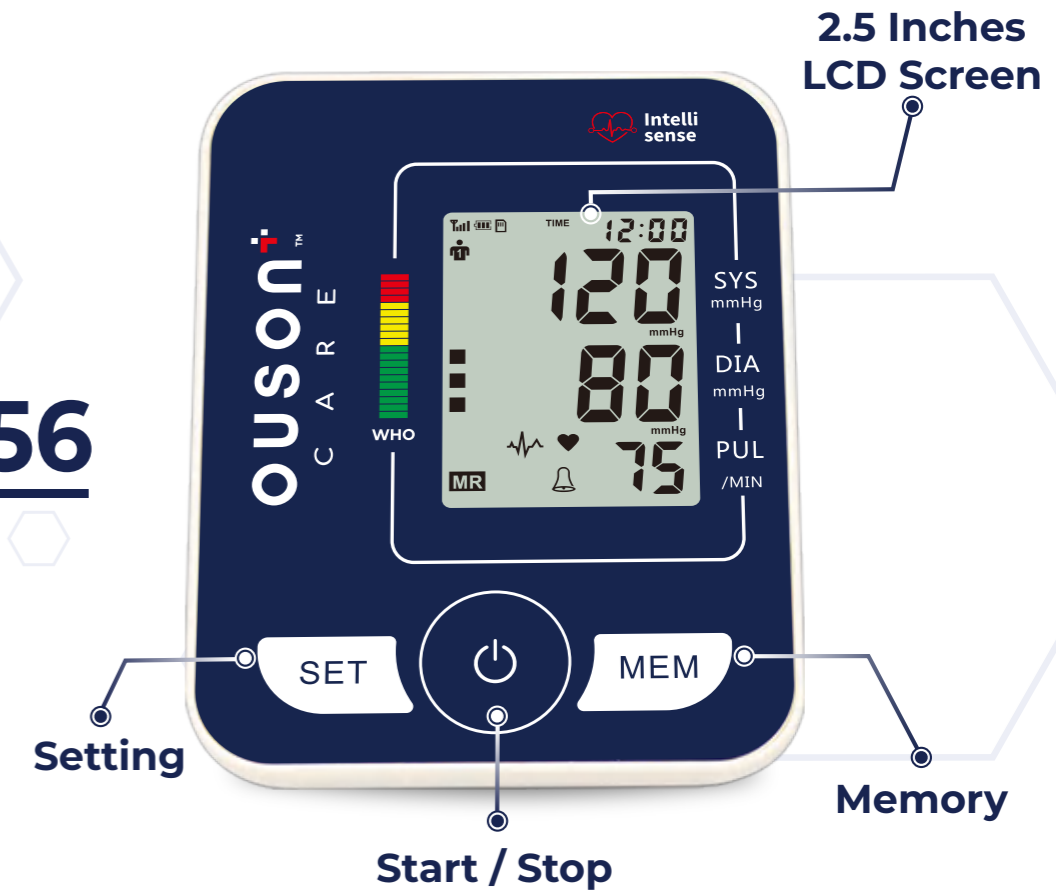

01

02

BLOOD PRESSURE UPPER ARM CUFF

INCLUDED

ADD-ON



Standard Size
22-32 CM

XS Size
17-22 CM
(Model No. : OC-XS-1722)

L Size
22-48 CM
(Model No. : OC-L-2248)

XL Size
22-52 CM
(Model No. : OC-XL-2252)

INSTRUCTIONS



Mirco USB Port

To supply power, you may use batteries or the Micro USB Port. To use the port, insert a 5V 1A AC Micro USB adapter into the port to the right of the monitor.



Battery Installation

To use batteries, turn the device so that the monitor is facing away from you. Press and slide in the direction indicated to remove the battery cover. Insert 4 AA batteries and slide the battery cover back into its original position.



Arm Cuff Hole

Insert the cuff air tube plug into the jack to the left of the monitor. Make sure the plug is fully inserted.

BSX556

BSX523



SPECIFICATIONS

	BSX556	BSX523
Cuff Circumference	22cm~32cm (±5mm)	
Memory	2*99 groups memory (Double users)	
Measurement Range	Blood pressure 0-290mmHg Pulse 40-180 beats/min	
Accuracy	Blood pressure: ±3mmHg Pulse: ±5% of the reading	
Power Source	Internal-DC 6V(4*AA) External-DC 6V 0.6A	
Operating Conditions	Temperature: 5°C to 40°C Humidity: 15% to 85%RH Pressure Altitude: 80Kpa~106Kpa	
Measurement Mode	Oscillographic measurement method	
LCD Screen Size	2.5 inches	4 inches
Automatic Shutdown	30s	60s
3 Colour Indicator	No	Yes

HOW TO USE

Two User Mode



Accurate Reading



Large LCD Screen



MDA Approved
Reg. No.: GB3302221-71460



WHO Standard



WHO

MAIN FEATURES

05



1
Wrap the cuff on the upper arm. Make sure the cuff lies around 2-3cm above the elbow.

You should be able to fit two fingers between your arm and the cuff.



2
Once you are ready, keep calm and turn on the device.



3
The cuff will expand and take several readings. Select your User ID (P1 or P2) to save the results.



BSX556



BSX523

06

INFRARED THERMOMETER



FRONT



BACK



SPECIFICATIONS



BSX976

Power Supply	DV 1.5V x 2(2 AAA 7#)	
Memory	20 sets of Memory	
Working Environment	(15.0~40.0)°C RH: (20~85)%RH	
Storage Environmnet	(-20.0~55.0)°C RH: <93%	
Display Screen	1.5 inch LCD Display	
Resolution	0.1°C (0.1°F)	
Measurement Range	<i>Body Temperature</i>	: 32.0~43°C
	<i>Object Temperature</i>	: 5.0~95.0°C
Accuracy	<i>Body Temperature</i>	: +0.3°c/+0.6°f
	<i>Object Temperature</i>	: +1.0°c

MAIN FEATURES

Quick Contactless Measurement



20 sets of Memory Storage



Forehead Temperature



Object Temperature



Automatic Shutdown




MDA Approved
Reg. No.




TEMPERATURE INDICATION



32°C - 37.5°C

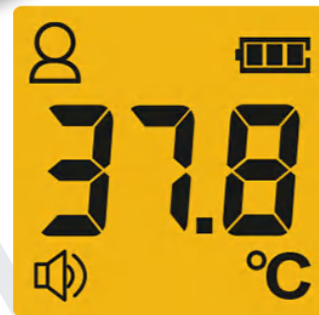

37.6°C - 38°C

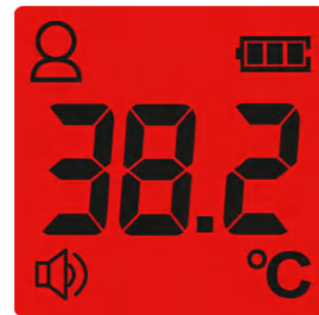

38.1°C - 42.9 °C


IMPORTANT SAFETY INSTRUCTIONS

Never use the thermometer for purposes other than those it has been intended for. Please follow the general safety precautions when using on children.


Never immerse the infrared Thermometer into water or other liquids (Not waterproof).

Keep the instrument away from direct exposure to sunlight and keep it in a dust-free and dry area at the temperature between (15~40) C.

MODE AND REFERENCES

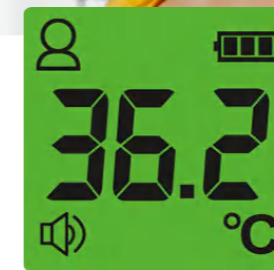

Celsius (°C)

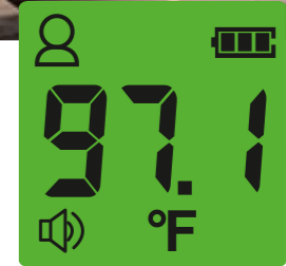


Fahrenheit (°F)



FINGERTIP PULSE OXIMETER



PRODUCT DESCRIPTION

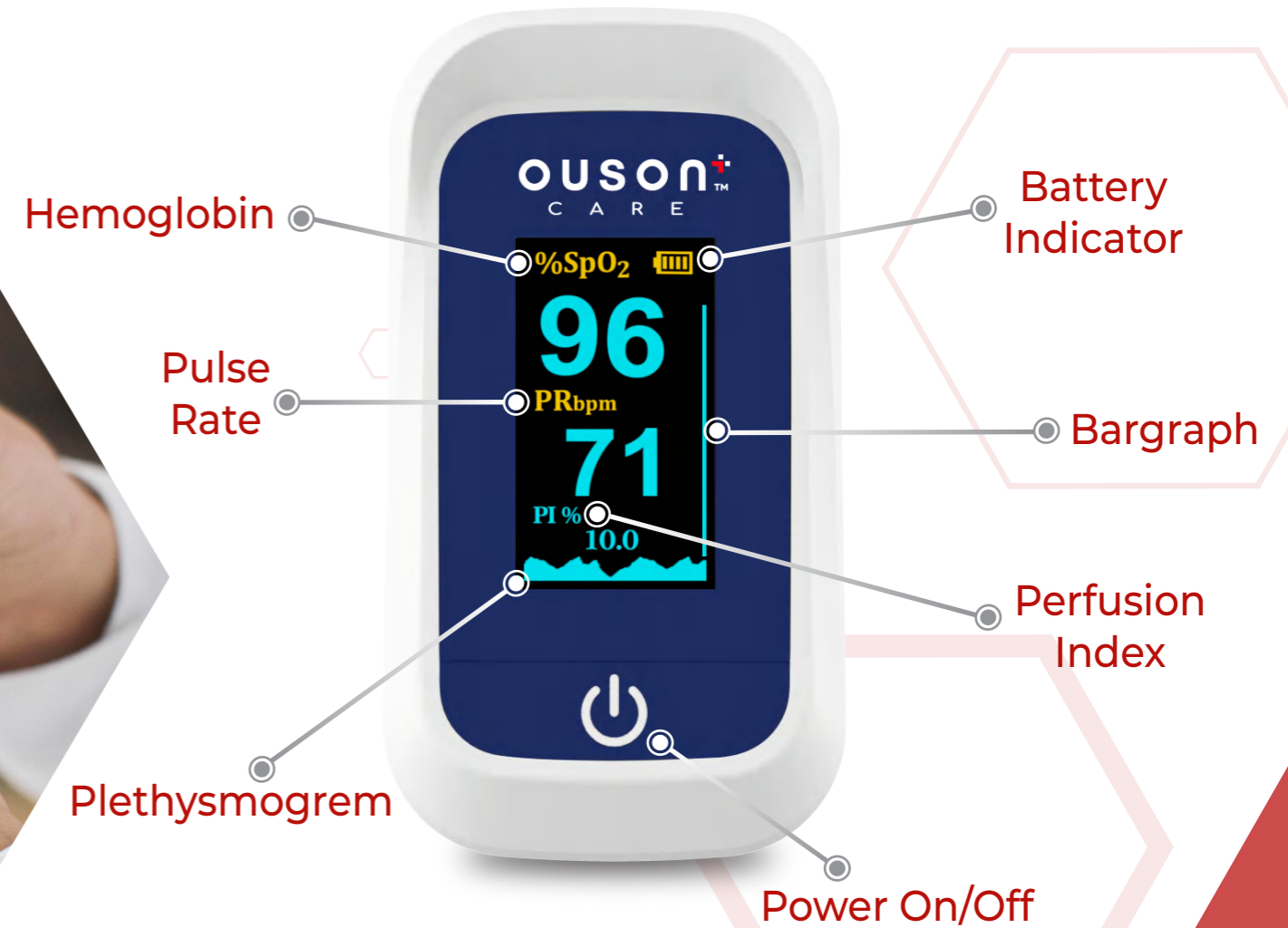

ONE TOUCH ROTATE SCREEN



360° View



270° View



90° View



180° View




SPECIFICATIONS



BSX231

Accuracy	SpO ₂ : 70~100%, ±2%; Unspecified(≤70%) Pulse Rate: +3 bpm
Working Environment	10~40°C, 70~106Kpa, 30-75%RH
Storage/Transport environment	-20~55°C, 70~106Kpa, ≤93%RH
Measurement Range	spO ₂ : 35~100%, Pulse Rate: 30~250bpm
Power supply	2x AAA Batteries

MAIN FUNCTION

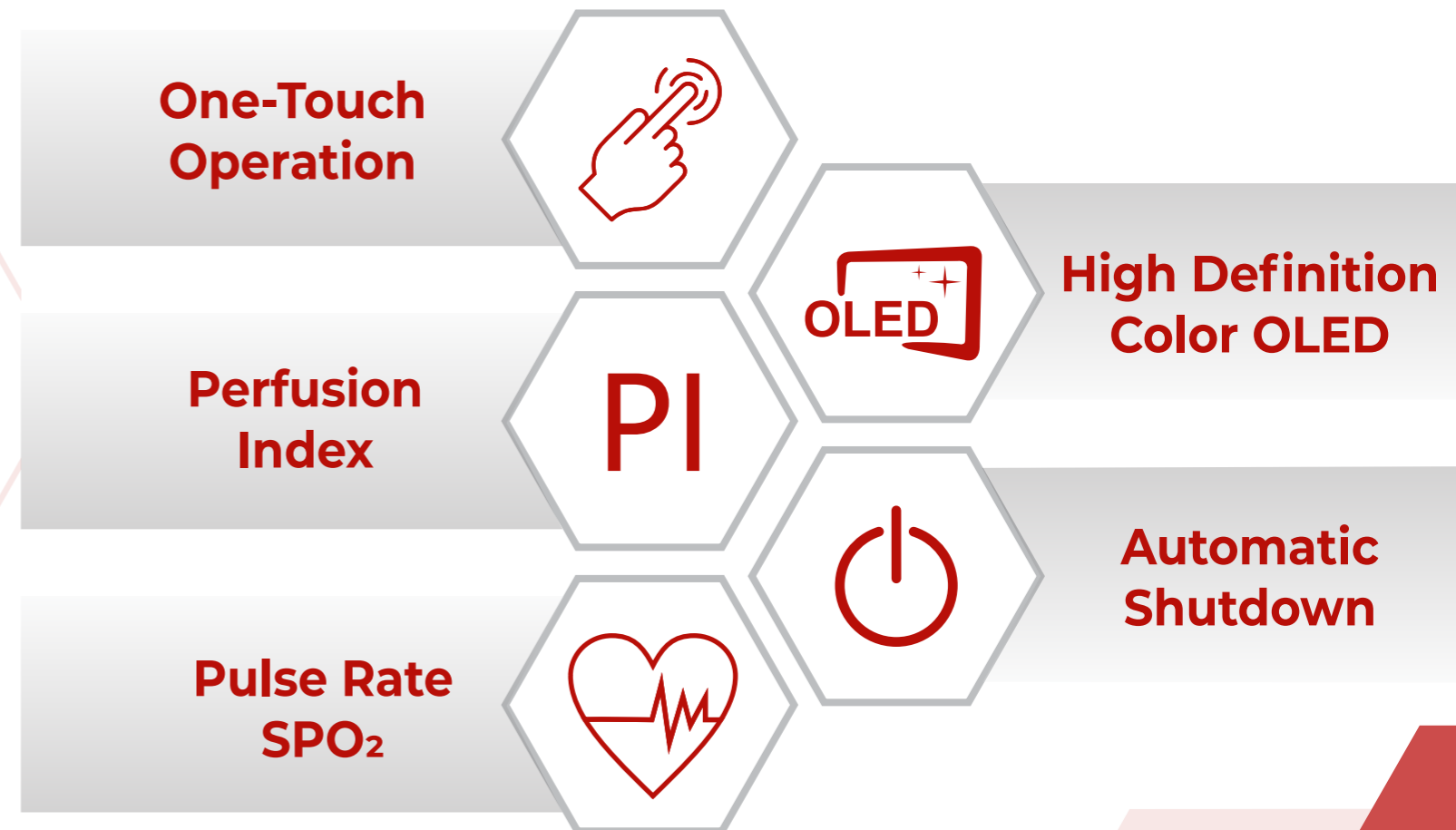


HOW TO USE



DISINFECTANT SPRAY




MAIN FUNCTION

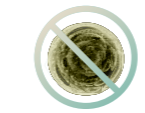

Eliminate odors



Protect all types of surfaces

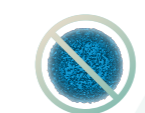

Kills 99.9% of germs



Prevents mold and mildew



New Formula
now with Fragrance

Flora Scent



New Formula
now with Essential Oil

Eucalyptus
+
Peppermint

RECOMMENDED USE FOR

Mattress



Toys



Key



Clothes



Toilet



Door



Sofa



Phone



Shoe



DETERMINATION OF BACTERICIDAL ACTIVITY



A. Test Organism: Staphylococcus aureus ATCC 6538

Sample	Contact time (minutes)	Bacterial Concentration (cfu/ml)	Log Reduction
Inoculum Control	-	2.3×10^7	-
Ouson Disinfectant Spray	1	<140	>5.21

B. Test Organism: Pseudomonas aeruginosa ATCC 15442

Sample	Contact time (minutes)	Bacterial Concentration (cfu/ml)	Log Reduction
Inoculum Control	-	2.0×10^7	-
Ouson Disinfectant Spray	1	<140	>5.15

HAND SANITIZER

MAIN FUNCTION



Anti-Bacteria 99.9%

Liquid Based

No Rinse

Non Sticky

Safe for Children

Approved by KKM

Notification No. : NOT210705656K


ACTIVE INGREDIENTS

Chlorhexidine Digluconate

Benzalkonium Chloride

INGREDIENTS:

Water, Alcohol, Fragrance, Glycerin, Polysorbate-20, PEG-7 Glyceryl Cocoate, Aloe Barbadensis Leaf Juice, Chlorhexidine Digluconate, Benzalkonium Chloride.

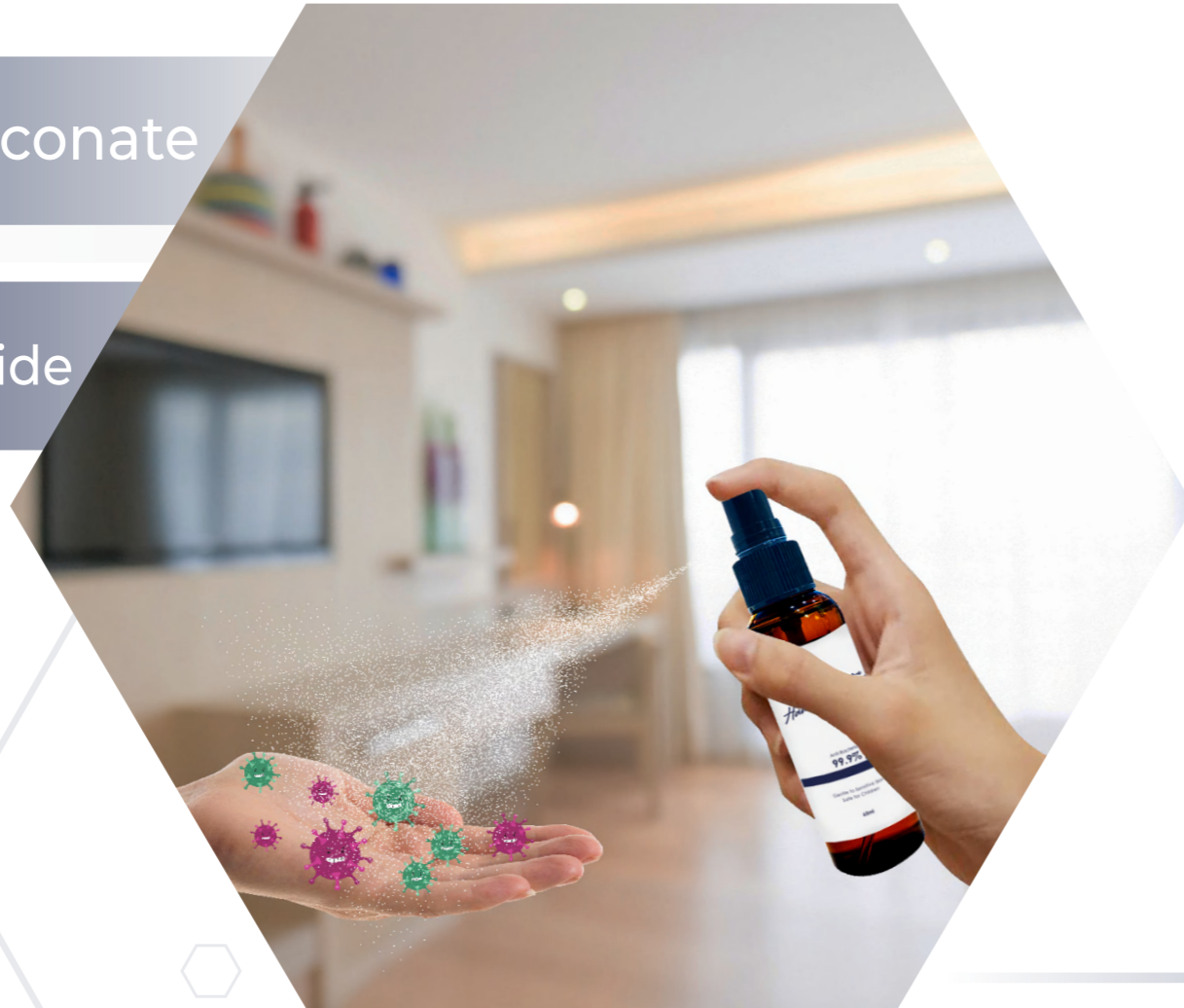

DETERMINATION OF BACTERICIDAL ACTIVITY



A. Test Organism: Staphylococcus aureus ATCC 6538

Sample	Contact time (minutes)	Bacterial Concentration (cfu/ml)	Log Reduction	% Kill Rate
Inoculum Control	-	2.1×10^7	-	-
Hand Sanitizer Spray (S1598)	1	<140	>5.17	99.999%

B. Test Organism: Pseudomonas aeruginosa ATCC 15442

Sample	Contact time (minutes)	Bacterial Concentration (cfu/ml)	Log Reduction	% Kill Rate
Inoculum Control	-	1.7×10^7	-	-
Hand Sanitizer Spray (S1598)	1	<140	>5.07	99.999%

WARNING



Avoid contact with eyes



Do not Swallow



For external use only



Stop using if irritation occurs

BOTANICA LAVANDULA

ANTI-MOSQUITO / AROMATHERAPY



Anti-Mosquito

WITH BOTANICA LAVANDULA



Anti-Stress

WITH BOTANICA LAVANDULA

INGREDIENTS



Bright and slightly citrusy, lemongrass oil leaves you feeling refreshed and restored while helping to relieve fatigue.

Cymbopogon Schoenanthus
(Lemongrass) Oil



Aromatic, earthy and invigorating, eucalyptus oil helps clear the mind to help relieve unwanted stress and tension.

Eucalyptus Globulus
(Eucalyptus) Oil



Classic, lightly floral and subtly sweet, lavender oil promotes a restful state of mind to help you experience calmness and feel completely relaxed.

Lavandula Angustifolia
(Lavender) Oil



**Plant Based
Essential Oil**



Non-Greasy



Deet Free



Alcohol Free



SGS Tested



**No Added
Paraben**

MAIN FEATURES

BOTANICA LAVANDULA



Anti-Mosquito

WITH BOTANICA LAVANDULA

Don't let mosquitoes and other pest-like bugs ruin your day. Protect your loved ones from unsightly bites and mosquito-borne diseases with Ouson+ Care's Anti Mosquito Spray made with essential oils proven to clinically repel mosquitoes and other insects.

Anti-Stress

WITH BOTANICA LAVANDULA

Life's too short to be stressed out. Experience a sense of calm and say goodbye to unwanted stress and overwhelming emotions with Ouson+ Care's Aromatherapy Spray made from pure essential oils.



SUITABLE FOR USE

Sport



ANTI-MOSQUITO

Home



Camping



Travel

Bed Sheet



AROMATHERAPY

Pillow



Curtain



Diffuser

HOW TO USE

ANTI-MOSQUITO



Spray 1-2 times on the disired area

if sprayed on the human skin, rub it repeatedly & evenly



AROMATHERAPY WITH AROMA DIFFUSER



Remove the bottle cap



Spray 1-2 times per 100ml water



Put in your favourite place to create a relaxing environment

OUSON⁺_{TM}

CARE

ReanneQ Sdn Bhd (1351295-w)

 C-G-2 & C-2-2, Boulevard Business Park, Pusat Komersial
Jalan Kuching, 51200 Kuala Lumpur.

 sales@ouson.care

 +603 6242 8944



www.ouson.care