

THE SUPPORT RAIL IS AN OPTIONAL ACCESSORY UTILIZED FOR ADDITIONAL ASSISTANCE TRANSITIONING IN AND OUT OF BED.

## STEP 1 (SUPPORT RAIL)


With the help of an able-bodied assistant, place the carton on the floor in its desired location. Remove all tape and binding straps, being careful not to puncture the box with any sharp objects.

## STEP 2

Open the box and remove the plastic bag. Locate and set aside any hardware.


## STEP 3

Use the remote control to raise the head of the adjustable base all the way. The Dawn House Bed has six threaded holes on each side of the base. Remove the 12x screws located in these holes and set aside to be used for installation.

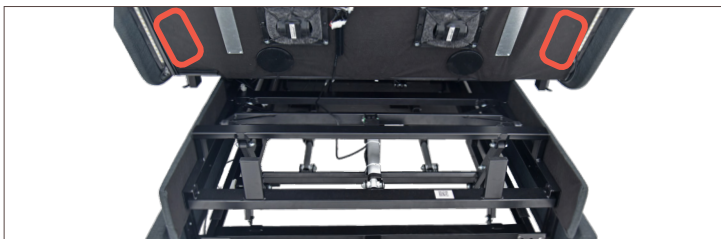

## STEP 4

While an able-bodied assistant holds the support rail in place, attach the support rail to the bottom of the deck with (6) hex head screws. **Be careful not to pinch the light strip.**

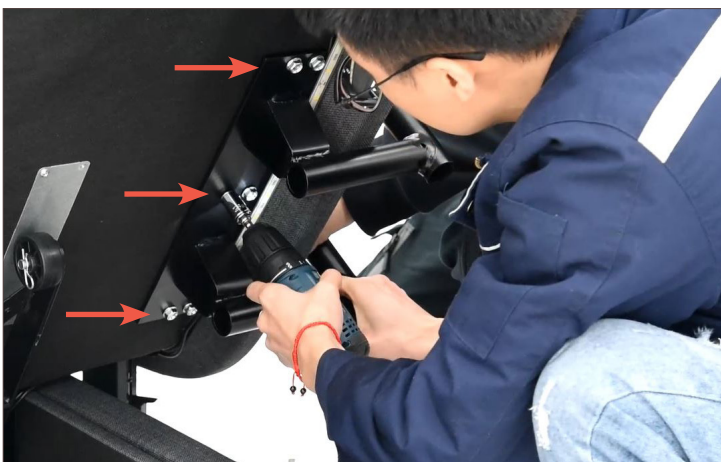



**ENTRAPMENT HAZARD:** Only remove the support rail cover to wash. Replace immediately before using. Using the bed without the support rail cover may increase the risk of serious injury.

## STEP 5 (BOLSTER)

If only installing one support rail, be sure to install the bolster on the opposite side to keep the mattress in its place and prevent entrapment hazards.



To install the bolster, use an able-bodied assistant to hold it in place, and attach the bolster to the bottom of the deck with (2) hex head screws. **Be careful not to pinch the light strip.**

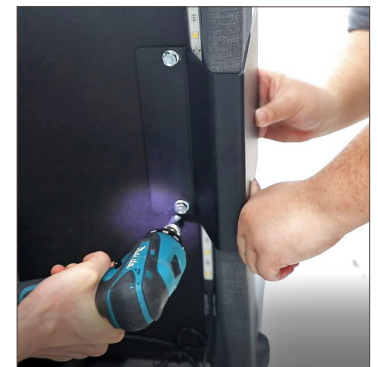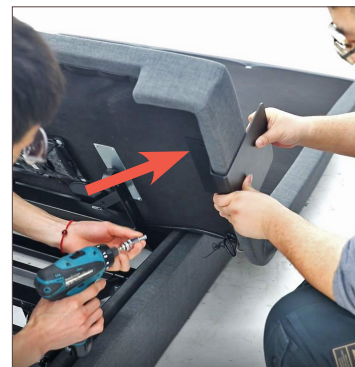

SETUP IS NOW COMPLETE.

