


Treatment guide for
BONE FRACTURES
 with the Handy Pulse Laser

info@pulselaserrelief.com
 1300 886 929



Session Procedure

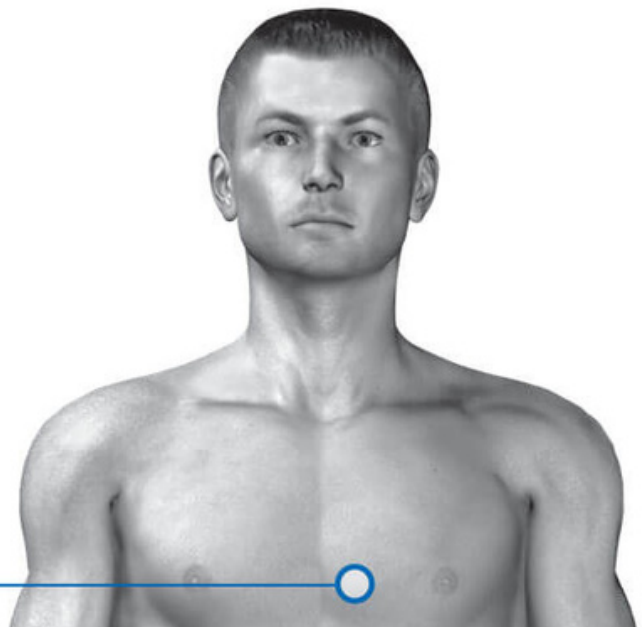

No.	Exposure Area	Frequency	Exposure Time
1	The 4th intercostal space at the left sternum edge	5 Hz	2 min
2	Transcutaneous Laser Blood Irradiation - TLBI (see Part 2 - coming soon) in projection zones of major vessels closest to the fracture location (not shown in the figure)	50 Hz	5 min per side
3	Fracture zone via the "window" in the cast* (not shown in the figure)	Variable Mode 1000Hz to 2500Hz	5 min

Session frequency.....1 session per day
 Course.....10-15 sessions

*** In case of rib fractures in the heart projection zone, laser therapy is conducted at 5 Hz!**