


## Treatment guide for **RADIOCARPAL JOINT** with the Handy Pulse Laser

info@pulselaserrelief.com 1300 886 929

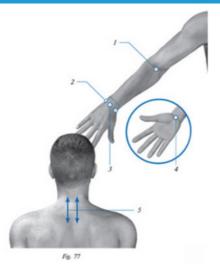

## **Session Procedure**

No.	Exposure Area	Frequency	Exposure Time
1	Cubital Fossa	50Hz	5 min
2	Side surfaces of the joint	VARIABLE	2 min per each area
3	Rear side of the joint	VARIABLE	2 min per each area
4	Palm side of the joint	VARIABLE	2 min per each area
5	Along the neck and chest section of the spine (C5–Th1), paravertebral, scanning	1000Hz	2 min on each side

Session Frequency: 1 session per day Course: 10-15 sessions

Repeated courses should be based on indications in 3-4 weeks. Up to 3 laser therapy courses should be conducted during the chronic process.