

Treatment guide for **TRAUMATIC AND POST-SURGICAL WOUNDS**

with the Handy Pulse Laser

info@pulselaserrelief.com

1300 886 929



Session Procedure:

No.	Exposure Area	Frequency	Exposure Time
1	<u>TLBI</u> (see page : https://www.pulselaserrelief.com.au/TLBI/) in projection zones of major vessels closest to the affected area	50 Hz	2 min per side
2	Along the wound or post-surgical suture capturing 2–3 cm of surrounding tissues to the sides, scanning at 1 cm per second at distance (at 0.5–1 cm over the body surface)	Variable Mode 1000Hz to 2500Hz	2 min each at 10 cm ² of area

Conventional dressings should be used after each session.

Session frequency.....1 session per day

Course.....7–10 sessions

Repeated courses in 3–4 weeks(with incomplete convalescence of the wound)

Weakened patients with intoxication or immune deficit should have the GSBS or URP. Please email us if you need more details on this.

*Treatment should start from healthy skin areas surrounding the wound gradually approaching the center of the wound. In presence of the dressing, contact (with the dressing) scanning methodology is used.