
STYLSMILE™

Invented With Love

Instructions / Mode d'emploi
Instructies / Anleitung
Instruksjoner / Anvisningar / Istruzioni
Instrucciones / Leiðbeiningar

Accelerator tray - 15 minutes, every day for 6 days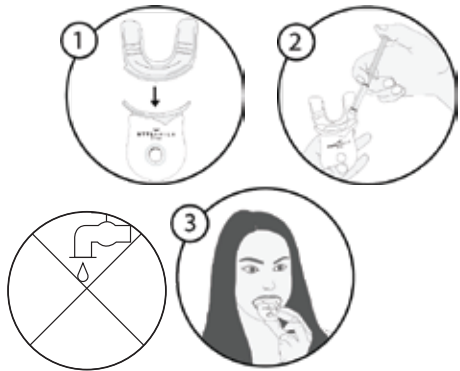

LED TRAY 6 v.

For more information and refill packs visit
www.stylsmile.com

@stylsmile_uk

teamtom@stylideas.com

+44 (0)20 3225 5559

www.STYLIDEAS.com

OBELIS S.A., Bd. Général Wahis 53,
B-1030 Brussels, Belgium.

STYLIDEAS Ltd, Amshold House, IG10 2RW, UK.

EN FR NL DE

NO SE IT IS

TB03B

TB03B_06_CN

(EN)

Accelerator tray - 15 minutes, every day for 6 days

Before first use remove the plastic battery tab from the battery compartment of the LED Accelerator, and rinse the mouth tray in warm water.

1. Connect the LED Accelerator to the mouth tray by gently easing it on and ensuring the silicone lip is over the plastic area on all sides.
2. Apply 1/4 Gel from syringe to the upper and 1/4 to the lower inside of the mouth tray (for ease it is suggested you hold the tray vertically).
3. Place mouth tray in mouth, ensuring each tooth is coated in gel. Wipe away any excess gel touching the gums or lips. Turn on the LED light and leave for 15 minutes. After 15 minutes turn off the LED light and remove the tray from mouth.

Repeat everyday for 6 days and wash the mouth guard clean after each use. Do not swallow gel. Rinse mouth after use. Keep LED Accelerator away from water. Wipe clean with dry cloth. Do not clean in dishwasher.

(EN) **ACCELERATOR TRAY SAFETY PRECAUTIONS:** THIS IS NOT A TOY. DO NOT SHINE IN EYES. THIS IS NOT INTENDED FOR USE BY PERSONS (INCLUDING CHILDREN) WITH REDUCED PHYSICAL, SENSORY OR MENTAL CAPABILITIES, OR LACK OF EXPERIENCE AND KNOWLEDGE, UNLESS THEY HAVE BEEN GIVEN SUPERVISION OR INSTRUCTION CONCERNING USE OF THE PRODUCTS BY A PERSON RESPONSIBLE FOR THEIR SAFETY. CHILDREN SHALL NOT PLAY WITH THE APPLIANCE. CLEANING AND USER MAINTENANCE SHALL NOT BE MADE BY CHILDREN WITHOUT SUPERVISION. PLEASE STORE WITH CARE AND OUT OF REACH OF CHILDREN. DO NOT TAMPER WITH ANY INTERNAL COMPONENTS. THERE ARE NO USER SERVICEABLE PARTS INSIDE THE UNIT. DO NOT ATTEMPT TO REPAIR THE PRODUCT YOURSELF. DO NOT PLACE ANY PART OF THE PRODUCT NEAR HEAT SOURCES SUCH AS RADIATORS, COOKERS, ETC. DO NOT USE THIS PRODUCT FOR ANYTHING OTHER THAN ITS INTENDED USE. DO NOT OPEN THE PRODUCT OTHER THAN TO REPLACE THE ACCELERATOR BATTERIES. IF THE APPLIANCE IS DAMAGED IN ANY WAY, STOP USING IT. IF THE APPLIANCE IS DAMAGED, CONTACT TEAMTOM@STYLIDEAS.COM. IF YOU HAVE A PACEMAKER OR OTHER IMPLANTED DEVICE, CONTACT YOUR PHYSICIAN OR THE DEVICE MANUFACTURER OF THE IMPLANTED DEVICE BEFORE YOU USE THE **STYLSMILE** ACCELERATOR. THE **STYLSMILE** ACCELERATOR HAS ONLY BEEN DESIGNED TO WHITEN TEETH. DO NOT USE FOR ANY OTHER PURPOSE. STOP USING THE APPLIANCE AND CONTACT YOUR DOCTOR IF YOU EXPERIENCE ANY DISCOMFORT OR PAIN. IF YOU HAVE MEDICAL CONCERNS, CONSULT YOUR DOCTOR BEFORE YOU USE THE **STYLSMILE** ACCELERATOR.

GEL WARNINGS: DO NOT SWALLOW GEL. AVOID CONTACT WITH THE EYES. NOT SUITABLE FOR CHILDREN UNDER THE AGE OF 7 YEARS. WHITENS NATURAL TEETH ONLY. DISCONTINUE TREATMENT IF YOU EXPERIENCE PAIN OR EXCESSIVE SENSITIVITY. THIS IS NOT A TOY. DO NOT USE THIS PRODUCT IF YOU ARE ALLERGIC TO ANY OF THE INGREDIENTS. STORE IN COOL DRY PLACE BETWEEN 5°C AND 30°C AWAY FROM HEAT AND DIRECT SUNLIGHT.

GEL INGREDIENTS: GLYCERIN, PEG-8, AQUA (WATER), PHTHALIMIDO-PEROXY-CAPROIC ACID, SODIUM POLYACRYLOYLDIMETHYL TAURATE, MENTHA ARVENSIS LEAF OIL, ALOE BARBADENSIS LEAF JUICE, LECITHIN, PVM/MA COPOLYMER, POLYSORBATE 20, MALTODEXTRIN, SODIUM SACCHARIN, MICA, SODIUM HYDROXIDE, CI 77891.

INSTRUCTIONS FOR DISPOSAL: DISPOSING OF THIS PRODUCT CORRECTLY WILL HELP SAVE VALUABLE RESOURCES AND PREVENT ANY POTENTIAL NEGATIVE EFFECTS ON HUMAN HEALTH AND THE ENVIRONMENT WHICH MIGHT OTHERWISE ARISE FROM INAPPROPRIATE WASTE HANDLING. PLEASE CONTACT YOUR LOCAL AUTHORITY FOR FURTHER DETAILS OF YOUR NEAREST DESIGNATED COLLECTION POINT. PENALTIES MAY BE APPLICABLE FOR INCORRECT DISPOSAL OF THIS WASTE, IN ACCORDANCE WITH YOUR NATIONAL LEGISLATION. USED ELECTRICAL AND ELECTRONIC EQUIPMENT (WEEE) SHOULD NOT BE MIXED WITH GENERAL HOUSEHOLD WASTE. FOR PROPER TREATMENT, RECOVERY AND RECYCLING, PLEASE TAKE THIS PRODUCT TO DESIGNATED COLLECTION POINT. PLEASE RECYCLE USED ACCELERATOR TRAY BATTERIES AND USED ACCELERATOR TRAY AT YOUR NEAREST RECYCLING CENTRE. **INSTRUCTIONS FOR DISPOSAL (GEL):** CORRECT DISPOSAL OF THIS PRODUCT. RECYCLE PACKAGING ACCORDING TO NATIONAL LEGISLATION.

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INGRÉDIENTS GEL: GLYCÉRIN, PEG-8, AQUA (WATER), PHTHALIMIDO-PEROXY-CAPROIC ACID, SODIUM POLYACRYLOYLDIMETHYL TAURATE, MENTHA ARVENSIS LEAF OIL, ALOE BARBADENSIS LEAF JUICE, LECITHIN, PVM/MA COPOLYMER, POLYSORBATE 20, MALTODEXTRIN, SODIUM SACCHARIN, MICA, SODIUM HYDROXIDE, CI 77891.

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GELINGREDIENSER: GLYCERIN, PEG-8, AQUA (WATER), PHTHALIMIDO-PEROXY-CAPROIC ACID, SODIUM POLYACRYLOYLDIMETHYL TAURATE, MENTHA ARVENSIS LEAF OIL, ALOE BARBADENSIS LEAF JUICE, LECITHIN, PVM/MA COPOLYMER, POLYSORBATE 20, MALTODEXTRIN, SODIUM SACCHARIN, MICA, SODIUM HYDROXIDE, CI 77891.

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GELINGREDIENTER: GLYCERIN, PEG-8, AQUA (WATER), PHTHALIMIDO-PEROXY-CAPROIC ACID, SODIUM POLYACRYLOYLDIMETHYL TAURATE, MENTHA ARVENSIS LEAF OIL, ALOE BARBADENSIS LEAF JUICE, LECITHIN, PVM/MA COPOLYMER, POLYSORBATE 20, MALTODEXTRIN, SODIUM SACCHARIN, MICA, SODIUM HYDROXIDE, CI 77891.

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