Thank you for purchasing Restform® Leg Pillow, the pillow that reduces pressure in your upper thighs and knees.

It offers excellent support for your hips, legs and knees. The thick end of the pillow reduces pressure in the knees and prevents the bones rubbing against each other.

Restform® Leg Pillow gives you freedom of movement and adapts to your posture.

The extra-soft viscoelastic foam keeps its shape and breathes, so it stays cool.

The Restform® Leg Pillow adapts to the contour of your legs, knees and thighs. It relieves back pain. The Restform® Leg Pillow helps restore the spine's natural alignment. It relieves back, hip and knee pain associated with poor posture while sleeping.

IMPORTANT:

- Because this product is made of new materials, it may have that "new" smell. The smell can be reduced by taking Restform® Leg Pillow out of the box to air it. This process may take a few days.

We recommend washing the pillow cover to eliminate the "new" smell. Follow the washing instructions on the label.

- This device may be used by children aged eight and over, by people with reduced physical, sensory or mental abilities, and by people lacking in experience and knowledge if they are appropriately supervised or given suitable instructions on the safe use of the device and understand the dangers its use entails.

- Consult a health professional in the event of increased discomfort or pain after use.

- Prevent the pillow from making contact with sharp objects, as they could tear the fabric.

In the event of a major incident with the product, please immediately inform INDUSTEX (see the box and instruction manual for contact details) and the competent health authority in the country where it was purchased.

ERGONOMIC DESIGN

Its curved shape provides for better pelvic posture while you sleep. It has ventilation holes to keep it cool all night long.

USE:

Place Restform® Leg Pillow between your legs. Make sure the thinnest part makes contact with your knees. Once Restform® Leg Pillow is in position, it should adapt naturally to the contour of your thighs without causing discomfort. (See Figure 1).

Pillow Cover Cleaning and Care:

The soft plush pillow cover is easily removed for washing. Open the zip on the pillow cover, remove the filling and wash the pillow cover in the washing machine on a delicate cycle with cold water. Allow it to air dry. Do not iron.

COMPOSITION: PILLOW: 100% polyurethane

COMPOSITION: PILLOW COVER: 60% polyester and 40% lycell.

Quality Warranty:

This product is covered by a warranty against manufacturing defects subject to the time periods stipulated by the legislation in force in each country. This warranty does not cover damages resulting from inadequate use, negligent commercial use, abnormal wear and tear, accidents or improper handling.

AUSTRALIA & NEW ZEALAND ONLY

This product has the benefit of certain Consumer Guarantees. These are prescribed by the Australian Consumer Law & the New Zealand Consumer Guarantees Act 1993 both of which provide protection for consumers. There is no express warranty for this product in Australia or New Zealand. The above paragraph refers to other countries.

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Restform® Leg Pillow is a registered EU/CTM trade mark. International patents pending. All imitations will be prosecuted.