

STAND UP SEAT

ELEVATING SEAT



USER MANUAL

Model: SX-AF81160S

▲ READ THIS MANUAL BEFORE USING THE PRODUCT AND KEEP IT FOR FUTURE REFERENCE.

Introduction

Congratulations! You have just purchased Stand Up Seat, a product designed with quality, safety and functionality in mind.

Please read this manual carefully before using this product, especially the safety chapter, so that you can use it correctly and enjoy your new Stand Up Seat to the fullest and safely.

Keep this manual for future reference when necessary.

Index

Introduction	2
Safety instructions	3
Attention!	4
Before using your elevator seat for the first time	4
Assembly of the seat	5
Steps for seat assembly	5
How to recline your seat, fold down the footrest and elevate the seat	6
Remote control	7
Operating modes	8
Remote control buttons	9
Technical specifications	10

Safety instructions



We recommend that you read all instructions carefully before you start using your new Stand Up Seat

- This product has an automatic massage program of 15 minutes duration.

This time can be reduced or extended, from 5 to 60 minutes. IN ANY CASE, OPERATION SHOULD NOT EXCEED 60 MINUTES.

- Check the compatibility of the mains and machine voltages before starting to use it. The seat operates at a voltage of 12 V, which is completely safe for human use.
- Unplug the product when not in use.
- Children should always use the seat under adult supervision.
- Stop using the seat immediately in case of discomfort.
- Do not stand on the seat.
- Do not clean with water or chemicals. When cleaning, do not forget to unplug it first.
- If the use of this elevator seat causes discomfort, it should be discontinued immediately without attempting to manipulate the mechanism.

Attention!



- This thermotherapy elevator seat generates heat when in use; it should be used with the utmost caution.
- Do not stand on the seat or use it as a ladder.
- Only one person can sit in the seat at a time.
- Do not use this seat if you notice that the cables, screws or anchors are in poor condition.
- Regularly check the condition of cables, screws and anchors to ensure the optimal functioning of your seat.
- Do not use the seat if any part is missing or broken.
- This seat should only be handled or repaired by an authorized technician.
- The box with the transformer and remote controls for your seat is located on the underside of the chair, between the seat's frame.

Before using your elevator seat for the first time



1. Clean the removable parts thoroughly with a damp cloth. Never use abrasive products to clean the seat.
2. Place the seat on a stable, dry surface away from heat sources.
3. Make sure that the main supply has sufficient power for the voltage required by the seat.

Assembly of the seat

- To facilitate handling Stand Up Seat comes in different parts that are easily assembled.
- Unpack all parts. Place them on the floor in an unobstructed location, preferably in the area where you want to place it permanently.
- The assembly is very simple. We recommend that it is done by two people.



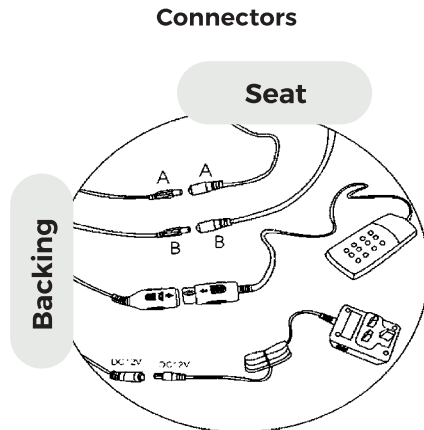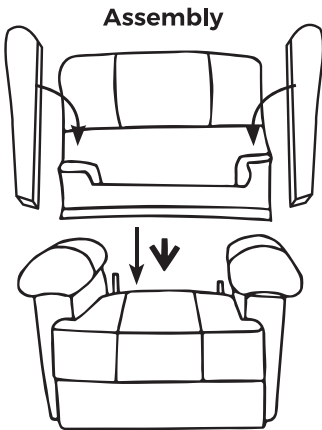

Steps for seat assembly



1. Carefully slide the backrest piece of the seat until it fits snugly into the base.
2. To mount the earmuffs on the seat, place them on the side rails and slide them down.
3. Connect the wires as shown in the figure (connectors). Do not force any connection as each plug only fits its mate (the one of the same color).



You can now start using your Stand Up Seat without any problems!

How to recline your seat, fold down the footrest and elevate the seat

- Both the backrest recline, footrest folding and elevation functions are automatic.
- You will only need to press the buttons on the controller that control these functions (Figure A).
- Using the button  you can recline the backrest at the same time as the footrest is lowered. As it is automatic, you only have to release the button when you reach the desired position.
- To return to the initial position, press the button . Holding the button down will start to raise the seat until the person occupying the seat is almost standing.
- Starting from the initial position, if you want to use the elevating function, press the button  until you reach the desired elevating position.
- To return to the initial position, press the button .

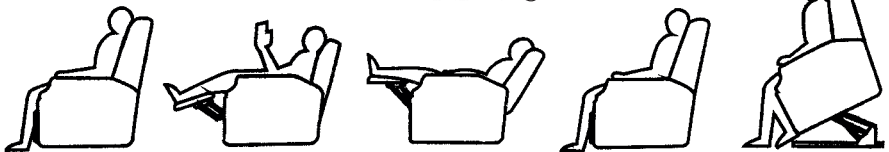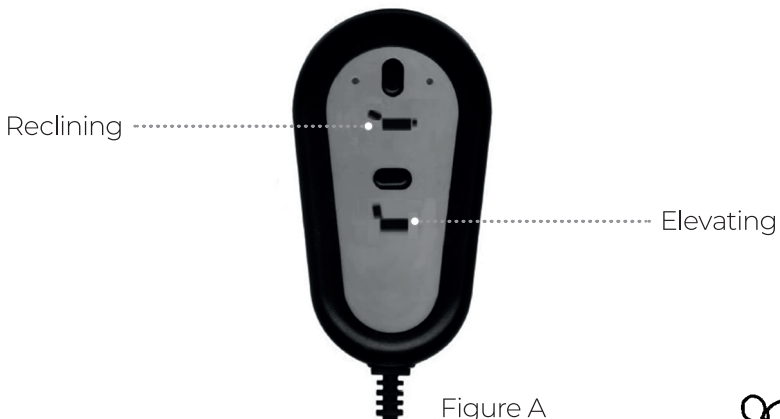

Remote control

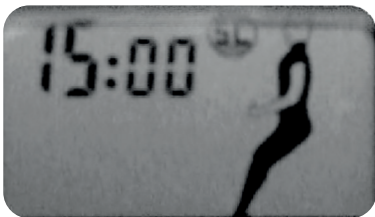
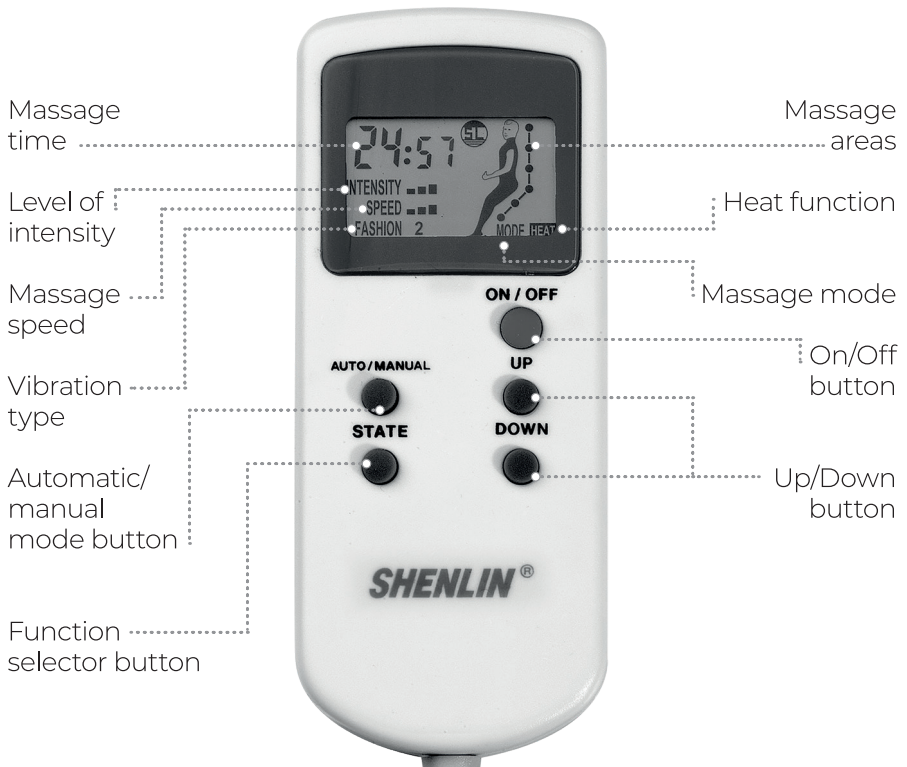


Figure B



Figure C

Operating modes

Automatic mode:

- The remote control display will show Figure B as soon as it is plugged in, indicating that the system is programmed for an automatic massage mode lasting 15 minutes, unless otherwise instructed by the user.
- Use the **Up/Down** buttons to set the time as desired (from 5 to 60 minutes, adjustable in 5-minute intervals).
- Pressing the **On/Off** button will start the chair to operate in the programmed mode and start the time countdown.

Manual mode:

- Press the **Auto/Manual** button to enter the menu to switch from automatic mode to manual mode, where you can "design" your own massage.
- Vibration intensity, massage speed, type of vibration, massage mode (piecemeal, full or spot massage in one area) and the "heat" option are selectable for your comfort and relaxation (Figure C).

The selection of functions is simple:

- Once the manual mode is selected, press the **State** button to switch from one function to another. The function that is selected will be the one that flashes on the display.
- When no function flashes, the massage time is selected.
- **Up/Down** are the buttons with which, once a function has been selected, you can control the different values of that function.

Remote control buttons

1. Press the **Up/Down** button to adjust the massage intensity levels (vibration intensity), once the intensity function is selected, by pressing **State** until Intensity flashes on the display.
2. Press the **Up/Down** button to adjust the massage speed levels, once the speed function is selected, by pressing **State** until Speed flashes on the display.
3. Press the **Up/Down** button to select from the different vibration types, the one you like the most, once the Vibration Type function is selected, press **State** until Fashion flashes on the display.
4. Press the **Up/Down** button after pressing **State** to select the desired massage mode when Mode flashes on the display. The different massage modes are reflected in the body outline drawn on the display. As you can see, you can select either a full body massage or zone/area massage, which will light up on the display, indicating the precise location where you will receive the massage.
5. Press the **State** button to select Heat or Heat function and then the **Up/Down** button to set it. This mode is not affected by functions or features you have preset for other functions.
When the heat function is on, lines (like flames) will appear above the word Heat on the right side of the screen.
6. Press **Up/Down** to adjust the massage time when pressing **State** without selecting another function (no function flashes on the display). Remember that the massage time is selectable in 5-minute intervals, with a minimum of 5 minutes and a maximum of 60 minutes.

DO NOT FORGET TO UNPLUG THE MASSAGE SEAT WHEN NOT IN USE.

Technical specifications

Model	SX-AF81160S
Armchair Dimensions	91 cm (width) x 104 cm (height) x 94,5 cm (depth)
Seat Dimensions	51 x 50 cm
Backrest Dimensions	Without earmuffs: 56x65 cm. With earmuffs: 70x65 cm
Fully reclined length	162,5 cm
Fully elevated height	138 cm
Remote controls	1. Massage and Thermotherapy; 2. Lift function, reclining and folding of backrest and footrests
Weight	49 kg.
Material	Cow leather (parts in contact with skin)
Maximum user weight	136 kg.
Voltage (adapter)	Input: 230V-50Hz Output: 12V/800mA 10W
Motors	5 for massages + 1 elevator/ recliner
Heat function	Available

STANDUP
SEAT

STAND UP SEAT

Made in China
©2022. Stand Up Seat. All Rights Reserved.

