eZwell

Remedy Massage System

Vibration Massage System with Shiatsu and Heat therapy

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- Improve your blood circulation
- Reduce pain and muscle soreness
- Decrease stress and promote relaxation

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To prevent potential injury or damage to the product, please read this entire manual before operating your device. Keep this manual for future reference.

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Congratulations on your purchase of the eZwell Remedy Massage System!

The eZwell Remedy Massage System provides an easy and effective way to stimulate your blood circulation, reduce pain and muscle soreness, decrease stress and improve your well-being.

This compact and easy-to-use system provides the benefits of vibrational therapy combined with shiatsu massage and heat therapy, right in the comfort of your own home.

Choose one of the 5 presets created by our experts to give your full body the massage it has well earned, or select different settings to isolate your neck, back, lumbar or leg areas for a focused massage with your preferred intensity. Plus, the portable design means you can take it with you wherever you go.

With a convenient carry bag included, you can easily transport your Remedy Massage System from home to the office or on the go. Whether you want to relax at home while reading a book, or you want to relieve tension in your muscles after a long and stressful day at work, the eZwell Remedy Massage System will be your best friend.


Our engineers at eZwell are dedicated to design the best products, and craft them from high quality, durable materials to ensure maximum comfort.

A full-service massage for your body has never been this easy!

Let's get comfy,

The eZwell team

- 1 Shiatsu Massage Pillow
- 2 Switch
- 3 Adapter Connector
- 4 Din Connector
- 5 Adapter Connector
- 6 Remote Control
- 7 Massage Mat
- 8 Socket Adapter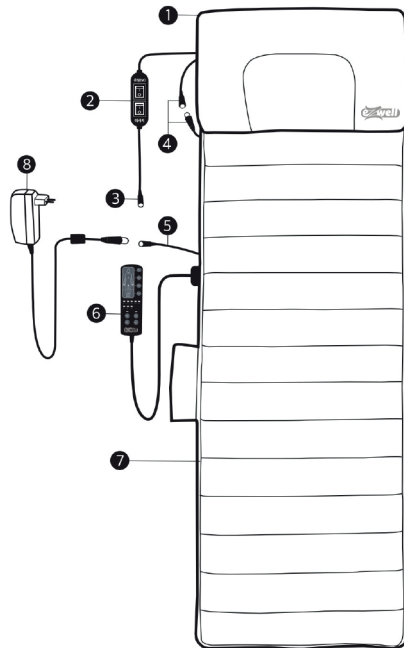


About the Remedy Massage System

The eZwell Remedy Massage System contains two highly effective massage methods.

1. Massage mat with vibrational massage and heat therapy. The mat features strategically placed massage motors throughout the surface to specifically target every part of the body. It is possible to select a full body massage, or isolate certain muscle groups. There is a choice of 5 massage programs and 3 intensity levels.

2. Massage pillow with shiatsu massage and infrared heat. The 6 rolling massage heads provide instant relieve in your head and neck.

The Massage Pillow and massage mat can be used in combination with each other, but it is also possible to use them separately. The Massage Pillow can also be used for specific muscle groups, such as the back, legs and feet.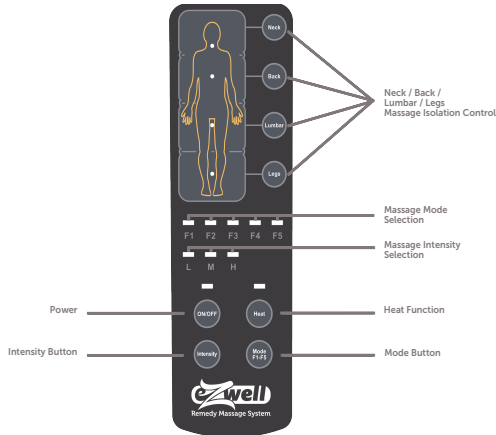


Three Intensity Levels:

1. **L** for low intensity massages
2. **M** for medium intensity massages
3. **H** for high intensity massage

Five Massage Modes

1. **Pulsate**
2. **Tapping**
3. **Rolling**
4. **Kneading**
5. **Intense**

(After several minutes, the program will switch to Auto mode, and change automatically through the various massage modes while in use)

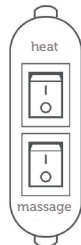How to use

1. Unpack the eZwell Remedy Massage System and all of its components.
2. Place the mat on a flat, stable surface such as the floor, a bed, sofa, or on a chair.
3. Plug the power adapter into the wall outlet and connect with the adapter connector of the Massage Pillow or the Massage Mat.
4. Turn on the device by pressing the ON/OFF button on the remote control.
5. Adjust the intensity, mode and optional heating on the remote control.
6. Lie down on the Remedy Massage System and position yourself comfortably. You can place a towel or a thin blanket between your body and the system to reduce the intensity of the vibration if necessary.
7. Relax and allow the vibrations to massage your body and neck. You can also choose to focus on specific areas such as your legs, lumbar, back and neck.
8. When you want to use the Massage Pillow or Massage Mat independently from each other, unzip the pillow from the mat, unplug the DIN Connector and connect the socket adapter with the device you like to use.
9. To turn off the device, press the ON/OFF button on the remote control.
10. When finished using the system, unplug the power cord and store the unit in a safe and dry place.

Safety tips

Caution: Please read all safety tips for your safety prior to use.

Keep this, and any other electronic products out of reach of children. Basic safety precautions should always be enforced.

Unplug device when not in use.

Be cautious not to overload your home's outlet.

Keep used and unused cords tidy and secure to prevent damage.

Do not use pins or other sharp objects on the Massage System.

Never use the Remedy Massage System if it is folded up or crumpled.

Do not stand on the unit.

Keep electronic devices and outlets away from water to prevent shock. Do not use when wet or moist. Do not use the Remedy Massager in damp rooms (e.g. when bathing or showering).

Do not use the Massage System as a substitute for medical treatment.

Consult your doctor before use if you are pregnant, have a pacemaker, have undergone recent surgery, suffer from blood clots or have any other medical condition. If you experience any pain or discomfort during the massage, stop immediately and consult your doctor.

Extensive use may lead to excessive heating and may shorten the lifespan of the unit. After continuously operating for 30 minutes, user should stop it for at least 30 minutes before using it again.

If the massager temperature is too high for your personal preference, discontinue use and allow the system to cool

Cleaning & Storing

CLEANING YOUR UNIT

- To clean the outer shell of your massager, use a soft, slightly damp sponge.

- Do not allow water or any liquid to come in contact with the unit.

- DO NOT use any chemicals or harsh abrasives such as cleaners and brushes to clean.

- NEVER immerse your Remedy Massage System or any other electronic device in water.

- Never clean when the unit is turned on.

- Make sure the massager is turned off and unplugged from the adapter before cleaning.

STORING YOUR UNIT

- Unplug the cord from the power source.

- Loosely fold the massage mat.

- Place the massager, along with the cord, in its carry bag, or in a safe, dry and cool place.

- AVOID contact with sharp edges or pointed objects.

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