EN

INSTALIFE® INSTRUCTION MANUAL

Product Description

1. Pressure pad
2. Main adjustable strap
3. Adjustable side straps
4. Ring

Indicated to relieve lower back pain associated with:

- a. chronic low back pain
 - b. sciatica
 - c. piriformis syndrome
- For use left or right calf
- One size fits all
 - Adjustable
 - Unisex

Warnings:

- Do not wear Instalife® on both legs at the same time.
- Do not use Instalife® without consulting a doctor first.
- Some people may be allergic to neoprene. If you observe skin irritation, stop using the product and consult your doctor.
- If, after using the product, your legs feel sore or the back pain continues or increases, stop using Instalife® and consult your doctor.

In the event of a major incident with the product, please immediately inform INDUSTEX (see the box and instruction manual for contact details) and the competent health authority in the country where it was purchased.

STEP 1: Determine on which leg to wear Instalife®. If the back pain is on the right side, wear Instalife® on the right leg. If the pain is on the left side, wear it on the left leg. If the pain is in the middle, try to determine if it is more towards the right or the left and wear it on that leg. If you feel the pain is exactly in the middle of your back, wear Instalife® on the right leg first and, if the pain is not relieved, wear it on the left leg.

STEP 2: Open the product by pulling apart the main adjustable strap (2) and the adjustable side straps (3).

STEP 3: Wrap Instalife® around the leg just below the knee and make sure that the pressure pad (1) is in the centre of the calf, as shown in Figure A.

STEP 4: Then firmly attach Instalife® by inserting the main adjustable strap (2) through the ring (4). Pull and fasten it using the adjustable fastener. Ensure that it is wrapped tightly and that the pressure pad is right in the middle of the calf.

NB: Instalife® should be wrapped firmly but should not be uncomfortably tight. If it is uncomfortable, loosen it a bit while keeping it tight.

Instalife® should not be worn for more than 2 hours the first time it is used. If you feel comfortable wearing it, try wearing it for a longer period of time in the next session.

Material composition: 33% neoprene, 39% polyester, 18% nylon, 8% polypropylene, 1.5% EVA, 0.5% latex.

Care Instructions: Hand wash. Do not tumble dry. Do not iron.

Quality Warranty

This product is covered by a warranty against manufacturing defects subject to the time periods stipulated by the legislation in force in each country.

This warranty does not cover damages resulting from inadequate use, negligent commercial use, abnormal wear and tear, accidents or improper handling.

Australia & New Zealand only

This product has the benefit of certain Consumer Guarantees. These are prescribed by the Australian Consumer Law & the New Zealand Consumer Guarantees Act 1993 both of which provide protection for consumers. There is no express warranty for this product in Australia or New Zealand. The above paragraph refers to other countries.

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