

ATTO ACCESSORIES

SEATING & SUPPORT



ATTO /ATTO
SPORT Armrests



Seat Cushion



Seat Cushion
Leather



Luxury Swivel,
Seat Leather



Seat & Cushion
Support Belt



Luxury Orthopedic
Backrest

STORAGE



Essentials
Pouch



Front & Rear
Basket



Carryall
& Cushion



Carryall
& Cushion, Leather



Pet Carrier &
Cushion, Black



Flight Kit



Oxygen Bottle
Holder & Case

TECHNICAL



Cane / Crutches
Holders



Mobile Phone
Holder



Rear Light Lifting
Handle



ATTO Left Hand
Throttle Kit



Standard Flight
Battery



Long Distance Battery
Not Flight Approved



Sun & Rain
Foldable Canopy



Rear View Mirror



Cable Kit


SCAN TO LEARN
HOW TO COMBINE
OUR ACCESSORIES

CARRYALL & CUSHION LEATHER

DON'T THINK TWICE,
JUST BRING IT ALONG!




How to Use CARRYALL & CUSHION LEATHER

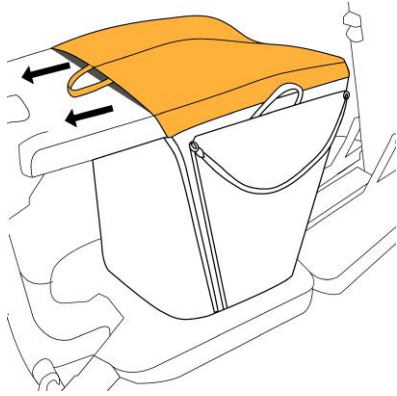

01

Slip the cushion over the seat (use handles to pull if needed)

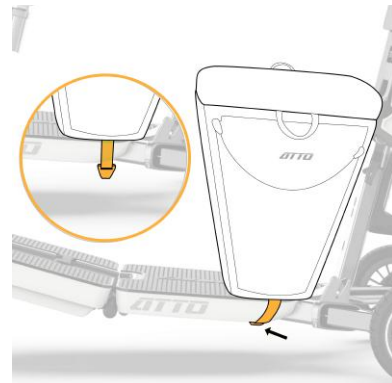


02

Simply pull hook on elastic strap to fold under and connect to ATTO chassis

NOTE!

Take off before folding ATTO

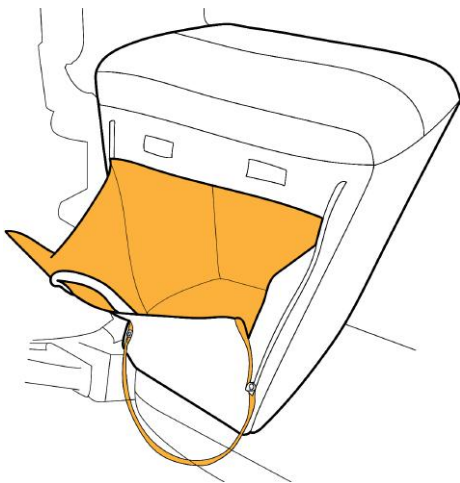

03

One-hand "pull to open" secure funnel access, so nothing falls out.
Quick "no zip" flap, to ensure everything stays inside when closing.



04

Attach to ATTO Seat Cushion
dedicated upper zip