Reflexx S.p.A., Via Passeri 2, 46019 Viadana (MN) | ITALY

DISPOSABLE EXAMINATION NITRILE GLOVE

Reflexx N71

USER INFORMATION SHEET

rev. UIS 00 nov. 2019

CE 0465

PPE cat. III Reg. EU 2016/425 EN ISO 374-1:2016 TYPE B, EN ISO 374-5:2016 VIRUS (ISO 16604:2004)


Chemical compound :	EN ISO 374-1:2016 TYPE B		EN 374-4:2013
	Performance level	Breakthrough time	% degradation
Sodium Hydroxide 40%	K 3	>60'	-3,5%
Hydrogen Peroxide 30%	P 3	>60'	22,4%
Formaldehyde 37%	T 3	>60'	27,1%

EU type examination by 0465 CIMAC Via Aguzzafame 60/b, IT-27029 Vigevano (PV)

OTHER COMPLIANCE: - Medical Device CLASS I | Reg. EU 2017/745 - EN 455 1,2,3 & 4

Reg. CE 1935/2004 - Reg. UE 10/2011 - DM 21/03/1973
FOOD SAFE certified by: Chelab S.r.l., a Merieux NutriScience company Via Fratta, 25, 31023 RESANA (TV) Italy

EN 374-4: 2013 Degradation results indicate the change in puncture resistance of the gloves after exposure to the challenge chemical.

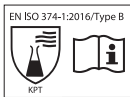EN

Declaration of Conformity on www.reflexx.com

■ Disposable examination nitrile glove ■ Powder free, non-sterile ■ Shape: ambidextrous, five fingers ■ Sizes: 7-10 ■ Please read this USER INFORMATION sheet carefully before using the glove.

RECOMMENDATIONS OF USE

■ Multi-purpose glove to protect the hand and/or the handled product, can also be used for general uses ■ This information does not reflect the actual duration of protection in the workplace and the differentiation between mixtures and pure chemicals ■ The chemical resistance has been assessed under laboratory conditions from samples taken from the palm only (except in cases where the glove is equal to or over 400 mm – where the cuff is tested also) and relates only to the chemical tested. It can be different if the chemical is used in a mixture. ■ It is recommended to check that the gloves are suitable for the intended use because the conditions at the workplace may differ from the type test depending on temperature, abrasion and degradation. ■ When used, protective gloves may provide less resistance to the dangerous chemical due to changes in physical properties. Movements, snagging, rubbing, degradation caused by the chemical contact etc. may reduce the actual use time significantly. For corrosive chemicals, degradation can be the most important factor to consider in selection of chemical resistant gloves. ■ Before usage, inspect the gloves for any

defect or imperfections. ■ The penetration resistance has been assessed under laboratory conditions and relates only to the tested specimen. ■ These gloves are designed to protect the hand surface from chemical splashes. Therefore, the length of the gloves is below EN requirements of total minimum glove length, and deems as 'Fit for Special Purpose' (EN 420:2003, Clause 5.1.3).

WARNINGS

■ Before using the product, try it in the real condition of use ■ The glove is a device subject to wear, so frequent replacement is recommended and in all cases where imperfections are observed ■ Wear gloves with dry, clean hands ■ In rare cases, transient skin reactions and hypersensitivity reactions may occur ■ In case of reaction, discontinue use and consult a doctor.

STORAGE AND DISPOSAL

■ Store the gloves in their original packaging in a dry place. ■ Do not expose to direct sunlight and heat sources. ■ The expiry date is 3 years from the date of manufacture indicated on the sales package, and is valid for the product properly stored ■ Gloves must be disposed of according to local regulations.

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