# Magicgel

## Migraine Ice Cap (Magic Gel)

REF migrainegelpack

Document code: TF-IFU-02 / Revision: 01

/ Issue Date: 24.12.2019

EN

## Instruction for Use Ice pack

IT

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#### 1 Introduction

##### 1.1 Intended use

A cold ice pack to be placed against the body to promote pain relief.

##### 1.2 Identification of the target population

The ice pack is intended for use by adults (all aged 18 years and older) and children under parental/adult supervision.

##### 1.3 Warnings

- Never place pack directly onto the skin as extreme temperatures can cause burns.
- Always wrap pack in a cloth or protective sleeve before use.
- Always use under adult supervision.

- Keep out of the reach of children.
- If you have diabetes, or you are affected by any circulatory issues seek medical advice before use.
- If any redness or irritation persists, consult a physician.
- Cease use if pack causes any discomfort.

- If the pack is punctured or opened and the solution is accidentally swallowed, drink plenty of water and consult a physician immediately.
- If solution contacts eyes, wash out thoroughly with water.

- Consult your doctor if you have any concerns about your health.

- Check pack for leakages before use.
- Clean pack before use.
- Always read the instructions before use.
- If you experience any pain or discomfort STOP and contact a medical professional immediately.

- Cold client. If it is important that cold clients have an ice treatment, warm them first by using such applications as hot footbaths, hot packs, blankets, hot water bottles, and/or warmed linens.

- Elders, that is, clients who are over 60 years old, may not be able to keep warm during cold treatments; be careful not to chill them.
- Children, especially preschool-age children, have difficulty describing their bodily sensations and may not be able to tell you if cold is causing pain, so use local cold with caution.
- Use caution when applying cold over superficial nerves.

Do not exceed recommended times, and do not tie cold applications over superficial nerves.

- Clients may have decreased muscle strength and slowed reflexes for 30 minutes after ice treatment and may be prone to injury if they exercise right away

#### 1.4 Limitations

- The patient should use Product family Hot and Cold Gel Packs in presence of a Healthcare Professional (HCP) for the first time.

- Product family Hot and Cold Gel Packs is not intended for use when users can't follow instructions, due to excess movement, inability to read or hear instructions.

- Intense use of the device may result in chafing or irritation of the skin at the points of contact.

- Intensive use of the device could lead to muscle fatigue or even tendinitis and/or muscle contractures

- Product family Hot and Cold Gel Packs is not intended for use when users have exposed to a cold environment before the measurement or has very cold hands.

#### 2 General safety Instructions

This Instruction for Use (IFU) deals with the specifications and way of use of an ice pack. Stretton Online / Gelpacks.com is by no means responsible for any malfunction or accident arising from a user ignoring the instructions described in this IFU.

This IFU contains description of all the functions available, so read this IFU carefully and familiarize yourself thoroughly with its contents before operating the unit.

#### 3 Product Placement

##### 3.1 For Cold Use

- Store in fridge or freezer for 3 hours or longer
- Place the gel pack in the strapping provided, or a light item of clothing, avoiding direct contact with the skin

- Apply to the body for 15-20 minutes.

- Repeat as many times as necessary ensuring at least 2 hours between applications.

For regular use please contact a medical professional to make sure that cold treatment is suitable for your particular injury. This pack is not suitable for warm use. Do not expose to high temperatures, hot water, oven, microwave or other heating methods.

##### 3.2 For Warm Use

Do not expose to high temperatures, hot water, oven, microwave or other heating methods.

**Title/Meaning:** Manufacturer  
**Function/description:** To identify the manufacturer of a product. This symbol shall be used filled in all applications to differentiate it from ISO 7000-2497

**Title/Meaning:** Date of manufacture  
**Function/description:** To indicate the date on which a product was manufactured.

**Title/Meaning:** Catalogue number  
**Function/description:** To identify the manufacturer's catalogue number, for example on a medical device or the corresponding packaging. The catalogue number shall be placed adjacent to the symbol.

**Title/Meaning:** Batch code  
**Function/description:** To identify the manufacturer's batch or lot code, for example on a medical device or the corresponding packaging. The code shall be placed adjacent to the symbol.

Mark of conformity according to the European Medical Device Directive 93/42/EEC

**Title/Meaning:** Operator's manual; operating instructions  
**Function/description:** To identify the location where the operator's manual is stored or to identify information that relates to the operating instructions. To indicate that the operating instructions should be considered when operating the device or control close to where the symbol is placed.

**Title/Meaning:** Do not use if package is damaged  
**Function/description:** To indicate that the device must not be used if the package holding the device is damaged, for example on packaging of medical devices.

**Title/Meaning:** Keep away from sunlight  
**Function/description:** To indicate that transport package shall not be exposed to sunlight.

**Title/Meaning:** Keep away from rain  
**Function/description:** To indicate that the transport package shall be kept away from rain and in dry conditions.

**Title/Meaning:** Temperature limit  
**Function/description:** To indicate the maximum and minimum temperature limits at which the item shall be stored, transported or used.

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