Before You Start

- Unpack the contents.
 Before you begin, read the INNOVO® Garment Exchange Policy.
- Charge the battery.
 - Watch the instructional video on myinnovo.com.
 - It is recommended to wear thong underwear or no underwear with INNOVO®.
 - Prior to treatment, always use the restroom and empty your bladder.
- Turn the shorts inside out so the electrodes are visible.

Preparing INNOVO®

- Hang shorts up prior to using the spray.
- Remove the cap from the spray bottle. Hold the bottle 6-8 inches from the shorts. Thoroughly coat each panel with 4-6 sprays to ensure even coverage.
- Ensure to spray all 8 electrode panels.
The Shorts must be sprayed before every use

Put On The Shorts

- Reverse the shorts so the electrode panels are on the inside.
- Pull on the shorts until the waistband is sitting comfortably above your hips. The red and blue connectors should be to the front, with the wash label to the rear. **Do not pull on the connectors.**
- Ensure the whole surface of each electrode is in full contact with your skin.
 Double check the positioning of the dotted lines printed on the rear of the shorts.
Dots must be on the crease between your thigh and your bottom

Before Using INNOVO®

- Connect the red cables together.
 Connect the blue cables together.
Important: You will hear the cables securely clicking together.
- Connect the cable to the controller. You may need to slide the doors open.
- Choose whether you will carry out your session in the standing, reclining or lying position. **For best results, we recommend you use INNOVO® in the standing position.**
Both knees must be bent

Using The Controller

- When in position, turn on the controller by pressing the power button.
- Press the + button to increase the intensity.

INNOVO® Size	2	3	4	5	6	7	8	9	10
Minimum Target Intensity	38	42	46	50	54	58	62	66	70

As you increase the intensity on the controller you will start to feel your pelvic floor contracting. With an effective pelvic floor contraction you will feel a sensation of "squeezing", "tightening" and or "lifting up" between your legs.

The sensation changes as you increase the intensity:
 1. Low intensity - gentle tingling;
 2. Moderate intensity - contraction of the buttocks;
 3. Therapeutic intensity - contraction of the pelvic floor.

It is important to experience a pelvic floor contraction to benefit from the therapy, and you should aim to use INNOVO® at the strongest intensity you can comfortably tolerate.

During And After Your Session

- Attach neck strap if needed for hands free option during your 30 minute treatment.
- If you need to move or change position during your treatment, remember to pause the treatment by pressing the on-off button. After your session, press the on-off button for 2 seconds to switch the controller off.
- Remove the shorts cable from their socket.
- Remove the shorts by pulling the waistband down. The shorts will be inside out when removed.
- Fold the shorts in half so that the electrodes remain flat.
 Store the product for your next treatment session.
 For cleaning, reference the washing instructions found in the INNOVO® manual.

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