

USER GUIDE: Malem Wireless (MO12)




WEAR OWN UNDERWEAR

EXTRA RECEIVERS

VOLUME CONTROL

Set-Up

- Clip the Easy-Clip sensor (attached to transmitter) to the area of fabric likely to get wet first
- The transmitter has a magnetic strip that attaches to the waistband on a child's own underwear
- Turn the receiver on and place on a bedside table
- Additional receivers can be placed in other rooms

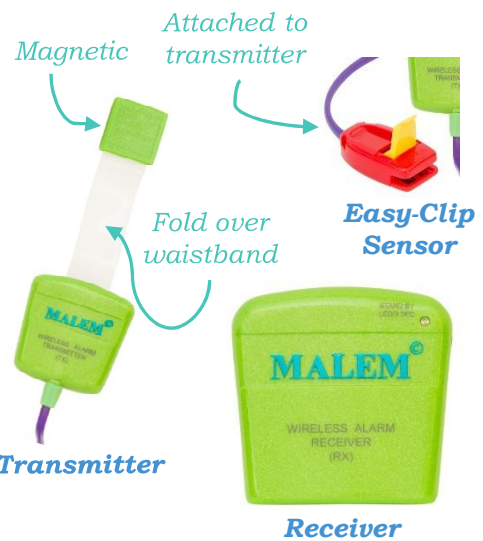


Flashing light every 3 secs when paired

Additional receivers can be linked to the same transmitter so that the alarm sounds in multiple rooms, e.g. parents' bedroom



Pairing The Transmitter & Receiver


- To re-tune the transmitter & receiver, use a paperclip to press through the hole on the back of the receiver
- The green LED will glow. Whilst it is glowing, activate the transmitter by lowering the Easy-Clip sensor lever.
- The green LED will flash then stop - the pairing is complete.

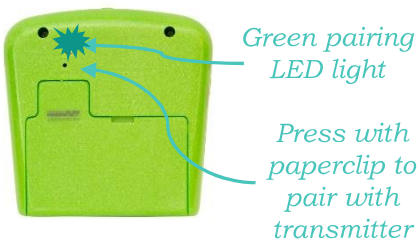

Changing The Sound Settings

- Can choose:
 - 8-tone (a different sound plays each time the alarm is triggered)
 - Single-tone (always the same tone)

8-tone settings:

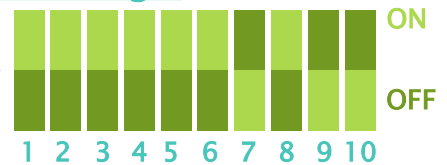

Single-tone settings:

