

USER GUIDE: Malem Audio (MO3)



SMALL

EASY TO USE

CHEAPEST MALEM ALARM

LIGHT

Set-Up

- Connect the sensor to the base of the alarm
- Clip the sensor onto your child's underwear, in the place most likely to receive the first drops of urine
- Attach the alarm to your child's pyjama top, near the collar bone where they will hear it sound



The MO3 alarm is nice and light, so your child should feel comfortable with it attached

Resetting

- When the alarm sounds, lift the clip on the sensor
- Press the reset button on the side of the alarm
- The alarm should stop sounding



⊘ *Don't stop the alarm by detaching the sensor from the alarm!*

Single tone alarm: Always plays the same alarm sound

8 tone alarm: Cycles through 8 different alarm sounds (helps to prevent auditory accommodation)

Batteries

- The alarm uses alkaline cell batteries and should last for a continuous hour of the alarm sounding
- To change the batteries, remove the cover on the back of the alarm with a screwdriver
- Replace all 3 batteries, ensuring they are the right way round



the **bedwetting** doctor

www.thebedwettingdoctor.com