

# USER GUIDE: Malem Ultimate (MO4)



**POPULAR**

**SOUND + VIBRATE + LIGHT**

**WAKES EVEN DEEP SLEEPERS**


## Set-Up

- Connect the sensor to the base of the alarm
- Clip the sensor onto your child's underwear, in the place most likely to receive the first drops of urine
- Attach the alarm to your child's pyjama top, near the collar bone where they will hear it sound & feel it vibrate



## Changing The Settings

- The alarm can be set to alert by:
  - SOUND & LIGHT (1)
  - SOUND, VIBRATE & LIGHT (2)
  - VIBRATION & LIGHT (3)
- The tamper proof setting switch is located in the battery compartment
- Remove the battery cover with a screwdriver and adjust the switch to the preferred setting

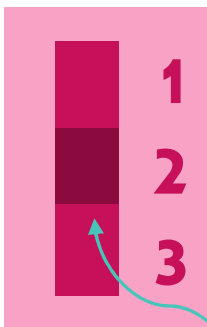

*The MO4 alarm is designed to wake up even the deepest sleepers!*

**Single tone alarm:** Always plays the same alarm sound

**8 tone alarm:** Cycles through 8 different alarm sounds (helps to prevent auditory accommodation)

## Resetting

- When the alarm sounds, lift the clip on the sensor
- Press the reset button on the side of the alarm
- The alarm should stop sounding



*Don't stop the alarm by detaching the sensor from the alarm!*

the **bedwetting** doctor

www.thebedwettingdoctor.com