

# USER GUIDE: Malem Ultimate Selectable (MO4S)



VERSATILE

SINGLE TONE & 8-TONE

SOUND + VIBRATE + LIGHT



*The MO4S alarm is equivalent to 10 alarms in 1! Change the settings to suit your child's needs...*


## Set-Up

- Connect the sensor to the base of the alarm
- Clip the sensor onto your child's underwear, in the place most likely to receive the first drops of urine
- Attach the alarm to your child's pyjama top, near the collar bone where they will hear it sound & feel it vibrate


## Changing The Alert Settings

- The setting switch for sound & vibration options is located on the **side** of the alarm
- The alarm can be set to alert by:
  - SOUND & LIGHT (S)
  - SOUND, VIBRATE & LIGHT (S&V)
  - VIBRATION & LIGHT (V)

*The combination of sound & vibration together will help to wake even deep sleepers!*

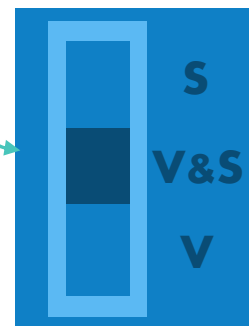

*The vibration only setting can be used in a situation where you don't wish to disturb others (e.g. shared room)*

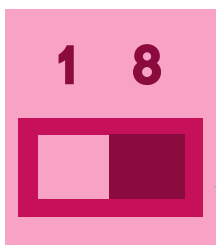

*The 8-Tone setting plays a different sound each time the alarm is activated so that your child doesn't become too used to the same sound and stop responding*

## Changing The Sound Settings

- The setting switch for different sound options is located on the **top** of the alarm
- The alarm can be set to be:
  - (1) **Single tone:** Always plays the same alarm sound
  - (8) **8-tone:** Cycles through 8 different alarm sounds (helps to prevent auditory accommodation)

the **bedwetting**doctor

[www.thebedwettingdoctor.com](http://www.thebedwettingdoctor.com)