

USER GUIDE: Malem Ultimate 1+ Record (MO5)



PERSONALISED RECORDING

REASSURING

SOUND + VIBRATE + LIGHT

Set-Up

- Connect the sensor to the base of the alarm
- Clip the sensor onto your child's underwear, in the place most likely to receive the first drops of urine
- Attach the alarm to your child's pyjama top, near the collar bone where they will hear it sound & feel it vibrate



A 10 second message can be recorded




Recording A Message

- Hold down the record button, which is located inside the battery compartment
- Speak into the microphone or record a favourite piece of music

Changing The Alert Settings

- The tamper-proof setting switches for are located inside the battery compartment
- The alarm can be set to alert by:
 - MESSAGE, LIGHT (& VIBRATE*) (M)
 - SOUND, LIGHT (& VIBRATE*) (S)
 - LIGHT (& VIBRATE*) (V)

** If vibrate setting is also turned ON (see below)*
- The vibration setting can be switched:
 - ON
 - OFF

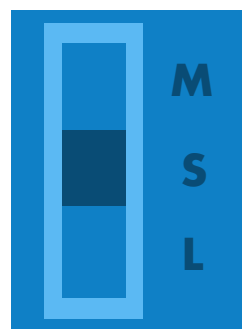

A familiar voice or favourite music sound is reassuring for children to wake up to