

USER GUIDE: Malem Bed-Side (MO6)



NOTHING ATTACHED


RECORDABLE

LOUDEST MALEM ALARM

Set-Up

- Place the bed-mat on the mattress and cover with a cotton sheet and/or bath towel
- Connect the wire from the bed-mat to the alarm, which is placed beside the bed

Make sure that the wire is tucked in to prevent tripping over it!

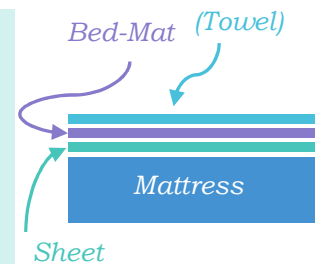

Volume Control

Adjusting The Volume

- The volume control is located on the side of the alarm unit

Changing The Settings

- Change the sound setting (A):
 - 8 tones
 - Single sound
 - Recorded message
- Choose the single sound that's played by turning on the selected sound (B)
- Record a personalised message with (C)
- Adjust the sound and light settings (D)
 - Light + Sound
 - Light only
 - No light or sound

