## EN Product description

The YOU-M4 respiratory protection mask without valve complies with the requirements of the European standard EN 149:2001+A1:2009 (Respiratory protective devices). It is used to protect the wearer from solid and non-vaporisable liquid particles and also gives effective respiratory protection against harmful particles, pathogens and aerosols. The mask is made of high quality non-woven fabric and performs at least 94% filtration. The nose clip and elastic earbands ensure an optimal fit.

## EN Instructions for fitting and removing the FFP2 mask:

Before fitting the YOU-M4, wash and/or disinfect your hands thoroughly and check the mask for any damage.

- (1) Unfold the mask, pre-shape the nose clip by slightly bending it and place the mask at the bottom of the chin.
- (2) Now pull the elastic earbands over the ears. Make sure that they are not twisted.
- (3) Using both hands, adjust both the mask and the nose clip to the shape of your face or nose so that it fits comfortably against your face and seals.
- (4) Before usage, e.g. at the workplace, a leakage check should be done. A tight fit must be ensured. **If necessary, the earbands can be shortened by knotting it.** Facial or head hair in the area of the sealing edge can prevent the mask from fitting correctly and therefore preventing its protective function.
- (5) To remove the mask, grab the elastic earbands, take them off and properly dispose them according to local regulations.

### Leakage check:

Place both hands carefully over the mask. Breathe out strongly. If there is any air leakage in the nose area, adjust the nose clip. If there are leaks at the edge of the mask, correct the fit of the mask and the elastic earbands. The respiratory protection mask is only effective if it is properly fitted, adjusted and worn for the entire time the wearer requires protection.

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EN  
Notes / Warnings:

**Do not use the respiratory protection mask, if:**

- metal in the environment may cause a hazard, because the nose clip contains metal
- the oxygen concentration is less than 19.5 %
- you smell or taste a contaminant or hazardous substance
- it is a fire and explosion hazard area

**Warning:** Do not use the particle filtering half mask more than one shift (NR = non reusable).

Leave the workplace or contaminated area immediately if dizziness, irritation or other symptoms occurs or if breathing becomes difficult.

Remove and replace the respiratory mask if it has been damaged or contaminated with blood or other infectious materials. Wearing the mask for too long may cause irritation and/or pressure marks on the ears.

Do not make any modifications or repairs to the mask. The YOU-M4 does not protect the user from gases or vapours, does not provide oxygen and cannot eliminate the risk of infection or contagion.

The product is only intended to be used for a maximum of one shift. Ignoring these instructions may affect the efficiency of the YOU-M4.

Storage instructions are printed on the packaging unit.

The mask is not intended for any other use than described and should not be used in such cases.

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EN

**Notes on certification & declaration of conformity**

The YOU-M4 respiratory protection mask complies with the requirements of European standard EN 149:2001+A1:2009 (respiratory protective devices) and Regulation (EU) 2016/425 (personal protective equipment) and is therefore CE marked. The applicable provisions can be viewed in the certificate as well as the declaration of conformity at: [www.yourmask-berlin.de](http://www.yourmask-berlin.de)

The certification issued is subject to the procedure of module C2 of Regulation (EU) 2016/425 supervised by DEKRA Testing & Certification GmbH, Handwerkstrasse 15, 70565 Stuttgart, Germany, NB 0158.