
stylies®

Comfort & Care

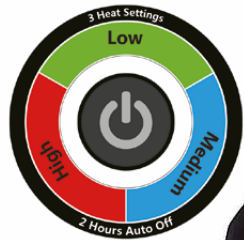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Congratulations!

You have chosen well by purchasing this appliance. It will provide you with many years service if you take proper care of it. However, before using appliance for the first time, we request that you read these instructions thoroughly, and most of all to note the safety instructions below. Any persons who are unfamiliar with these instructions must not use the appliance. Keep the packaging for later use. However, destroy all plastic bags, as these can become dangerous toys for children.

Sicherheitshinweise.....	4–11
Consignes de sécurité	
Avvertenze di sicurezza	
Safety Instructions	
Beschreibung.....	12
Description	
Descrizione	
Description	
Reinigung.....	13
Nettoyage	
Pulizia	
Cleaning	
Sortimentsüberblick.....	22–23
Vue d'ensemble de l'assortiment	
Panoramica dell'assortimento	
Overview whole assortment	

Produktübersicht.....	14
Anwendung.....	15
Présentation de l'appareil.....	16
Utilisation.....	17
Panoramica del prodotto.....	18
Uso.....	19
Product presentation.....	20
Use.....	21
Garantie.....	24
Garantie	
Garanzia	
Warranty	

-  Sicherheitshinweise Bandage: **Wichtig!** Zuvor beachten!
Consignes de sécurité bandage: **Important!** A observer préalablement!
Avvertenze di sicurezza bendaggio: **Importante!** Osservare anticipatamente!
Safety instructions bandage: **Important!** To observe first!
-  Sicherheitshinweise Bandage: **Wichtig!** Sicherheitshinweise für die Anwendung
Consignes de sécurité bandage: **Important!** Instruction de sécurité pour l'utilisation
Avvertenze di sicurezza bendaggio: **Importante!** Istruzioni di sicurezza per l'uso
Safety instructions bandage: **Important!** Safety instruction for the use
-  Entsorgung
Élimination
Smaltimento
Disposal
-  Der Grüne Punkt – europaweites Kennzeichnungssystem für Verpackungsrecycling
Le point vert – label marquant l'adhésion à un système de contribution au recyclage des déchets
Punto verde – logo utilizzato per individuare un particolare sistema per lo smaltimento degli imballaggi
The green dot – label marking the adhesion to a system contribution to the recycling of waste

-  Sicherheitshinweise Powerbank: **Wichtig!** Zuvor beachten
Consignes de sécurité Powerbank: **Important!** A observer préalablement
Avvertenze di sicurezza Powerbank: **Importante!** Osservare anticipatamente
Safety instructions Powerbank: **Important!** To observe first
-  Sicherheitshinweise Powerbank: **Warnung!** Wichtig!
Consignes de sécurité Powerbank: **Avvertimento!** Important!
Avvertenze di sicurezza Powerbank: **Avvertimento!** Important!
Safety instructions Powerbank: **Warning!** Important!
-  In Übereinstimmung mit den Europäischen Richtlinien für Sicherheit und EMV
Conforme aux Directives européennes en matière de sécurité et de CEM
In conformità alle Direttive Europee per la Sicurezza e EMC
Complies with European Directives on safety and EMC

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If you suffer from a medical problem (e.g. varicose veins, thrombosis, etc.), consult a doctor prior to using the product.

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Never apply to helpless, infants or heat-sensitive individuals.

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Inspect that all components of the product are in a faultless condition prior to each start-up of the product. The individual components of this product may not be disassembled, repaired or modified.

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Do not insert any needles or pointy objects into the product.

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The product is not intended for professional, medical or hospital use.

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Persons with reduced physical, sensory or mental capabilities, lack of experience or knowledge may only use the appliance if they are supervised by a person who is responsible for their safety or if they have been shown how to use the appliance safely and they have understood the risks inherent in its use.

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Children should not play with the product. Keep packaging (e.g. plastic bags) away from children.

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Always switch off the appliance: whenever it is left unattended, before cleaning, when it fails to work properly, and after use.

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For safety reasons, do not use accessories that are not recommended or sold by the manufacturer.

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If the product is used for purposes other than those specified by the manufacturer, operated incorrectly, or not repaired professionally, we will not accept any liability for any resulting damage. In this case, the warranty claim will be void.

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Never use the product with open wounds, burns, hematoma, swellings, or similar conditions.

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Stop immediately, if you notice any unpleasant sensations, redness of the skin, rashes or inflammation. Heatsensitive persons in particular should use the product with care and interrupt the treatment in a timely manner.

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For sensitive skin, it is strongly recommended not to use directly on the skin. Constantly check the condition of the skin to avoid any redness or burns.

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Use the product only in dry conditions and with a dry body – never in the bathtub, shower, above filled sinks, in pools, saunas, or during rain.

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Do not fall asleep during use. Don't use overnight.

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Don't use the product while folded or wrinkled. Avoid over-folding the bandage. Do not put heavy objects on the bandage.

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The product should not be used in the following cases:

- Health or disability issues
- Insensitivity to heat
- Injury, sunburn, scarring, inflamed areas or other damage due to disease
- Phlebitis, arthritis or other inflammatory processes
- Under the effects of alcohol, sleeping pills, sedatives or drugs
- Pacemaker

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Use only a 5V power adapter for charging, otherwise the device may get damaged.

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Children should not play with the appliance. Keep the appliance and power cable away from children under 8 years. Keep packaging (e.g. plastic bags) away from children.

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If the appliance is used for purposes other than those specified by the manufacturer, operated incorrectly, or not repaired professionally, we will not accept any liability for any resulting damage. In this case, the warranty claim will be void.

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Don't directly connect any contacts as this can lead to short circuits.

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Never use close to water. Do not expose to rain or humidity. Never immerse the appliance into water or other liquids.

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Check the appliance and power cable for damage. Never switch on a damaged appliance (including power cable) – have it repaired or replaced by the manufacturer, the manufacturer's service centre or by a qualified service technician. Never open the appliance yourself – risk of injury!

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Never place the appliance or the power cable on a hot surface (hob). Keep the appliance away from open flames. Do not expose the appliance to intense heat (heat sources, radiators, sunshine).

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Load the appliance only at a socket with correct voltage. Never use extension leads.

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For the first time use or when the Powerbank has been out of use for a long time, about 5–6 full charges and discharges are needed to achieve full performance.

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






For simple cleaning of the bandage, use a soft brush. The heating bandage can be washed in the washing machine. Use the fine washing program of your machine, which washes at a maximum of 30 °C. The washing machine should only be lightly loaded, as the washing program requires a lot of water for cleaning. Please observe the washing instructions below.

Warning: please remove the Powerbank before washing! Before washing, slide the USB cable into the pocket for the power bank.

Info: the cable and the USB connection will not be damaged. The controllers are sealed so that no water gets inside when washing.

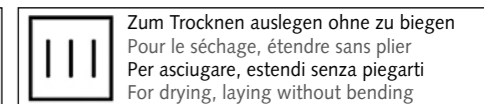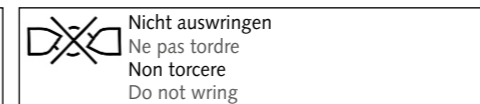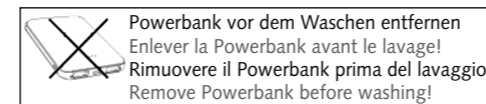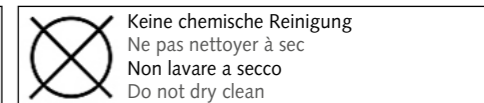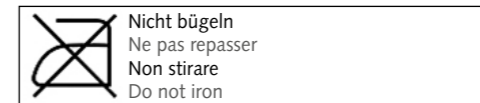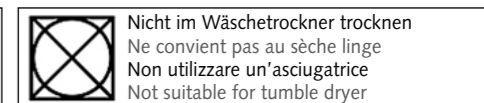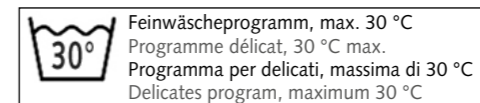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Heating bandage

A Bandage with heating wire

B Power switch and temperature adjustment on 3 levels:
 Green LED indicator: approx. 35°C
 Blue LED indicator: approx. 40°C
 Red LED indicator: approx. 60°C
 LED indicator without light: OFF

C Rechargeable battery (Powerbank)

D Pocket for the Powerbank

E USB A power cord integrated in the pocket

F Velcro straps to close and secure

Powerbank

1 Power switch


2 Battery level indicator (LED)

3 Micro USB charging port (input)

4 Output 1: USB A

5 Output 2: USB A

6 USB cable

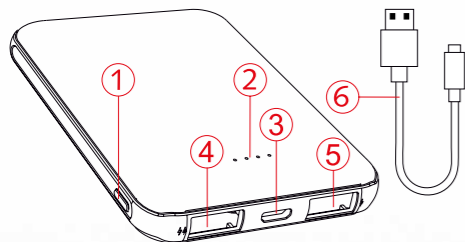Before use

1. Fully charge the Powerbank before using it for the first time:

- Connect the micro USB port of the Powerbank (3). Use the included USB cable (6).
- An appropriate power source is the USB port on your computer or an optional AC adapter with USB output (5V).
Tip: Often the original charger of your mobile device, such as the power supply of the iPhone, is also suitable.
- The active charging function is indicated by the flashing of the LED (2).
- **Info:** Charging time of the Powerbank: 3 hours. Use time of the Powerbank: up to 6 hours.

Use of the heating bandage

2. Connect the USB cable (6) which is in the bandage pocket (D) to the Powerbank (4 or 5).

3. Put on the bandage and secure it with the velcro strips (F).

Please ensure that the heating bandage sits loosely, i.e. some air circulates between the heating bandage and your skin/clothes during use. Accumulation of heat must be prevented.

4. Switch on by pressing the power switch (B). The device starts at the lowest temperature (green light / 35°C).

To change the temperature, press the power switch (B) again as follows:

- 1 x for medium temperature (blue light / 48°C)
- 2 x for high temperature (red light / 60°C)

5. To switch off, press the power switch (B) till the light turns off.

Info:

We recommend using the bandage for 30 to 45 minutes on the highest heat or 2 hours on the lowest heat and this no more than 2–3 times per day. Due to the automatic shutdown, the heating bandage turns off automatically after 2 hours.

Tip:

On the inside of the bandage is a net pocket for the use of a cold pack (not included). Follow the original instructions of the cool pack manufacturer for use.

Important:

Avoid over-folding the bandage. Do not put heavy objects on the bandage.

Wärmebandage Nacken
 Bandage chauffant pour la nuque
Bendaggio riscaldante per il collo
 Heating neck bandage

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The product guarantee corresponds with the respective local law and regulations and applies for at least 2 years from date of purchase. It covers faults in construction, workmanship and material. Excluded from this guarantee are all wear parts as well as inappropriate use or maintenance as well as modifications or repairs done by unauthorised third parties.

Please keep your proof of purchase to verify any claims under guaranty.

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TAVORA Brands AG