
# stylies®

*Comfort & Care*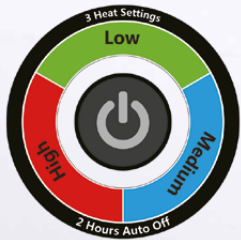

Wärmegürtel Menstruationsbeschwerden | Ceinture chauffante pour douleurs menstruelles  
Cintura riscaldante per dolori mestruali | Heating belt for menstrual pain

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### Congratulations!

You have chosen well by purchasing this appliance. It will provide you with many years service if you take proper care of it. However, before using appliance for the first time, we request that you read these instructions thoroughly, and most of all to note the safety instructions below. Any persons who are unfamiliar with these instructions must not use the appliance. Keep the packaging for later use. However, destroy all plastic bags, as these can become dangerous toys for children.

Sicherheitshinweise.....	4–11
Consignes de sécurité	
Avvertenze di sicurezza	
Safety Instructions	
<b>Beschreibung.....</b>	<b>12</b>
Description	
Descrizione	
Description	
<b>Reinigung.....</b>	<b>13</b>
Nettoyage	
Pulizia	
Cleaning	
<b>Sortimentsüberblick.....</b>	<b>22–23</b>
Vue d'ensemble de l'assortiment	
Panoramica dell'assortimento	
Overview whole assortment	

-  **Sicherheitshinweise Gürtel: Wichtig! Zuvor beachten!**  
Consignes de sécurité ceinture: **Important!** A observer préalablement!  
Avvertenze di sicurezza cintura: **Importante!** Osservare anticipatamente!  
Safety instructions belt: **Important!** To observe first!
-  **Sicherheitshinweise Gürtel: Wichtig! Sicherheitshinweise für die Anwendung**  
Consignes de sécurité ceinture: **Important!** Instruction de sécurité pour l'utilisation  
Avvertenze di sicurezza cintura: **Importante!** Istruzioni di sicurezza per l'uso  
Safety instructions belt: **Important!** Safety instruction for the use
-  **Entsorgung**  
Élimination  
Smaltimento  
Disposal
-  **Der Grüne Punkt – europaweites Kennzeichnungssystem für Verpackungsrecycling**  
Le point vert – label marquant l'adhésion à un système de contribution au recyclage des déchets  
Punto verde – logo utilizzato per individuare un particolare sistema per lo smaltimento degli imballaggi  
The green dot – label marking the adhesion to a system contribution to the recycling of waste

Produktübersicht.....	14
Anwendung.....	15
Présentation de l'appareil.....	16
Utilisation.....	17
Panoramica del prodotto.....	18
Uso.....	19
Product presentation.....	20
Use.....	21
<b>Garantie.....</b>	<b>24</b>
Garantie	
Garanzia	
Warranty	

-  **Sicherheitshinweise Powerbank: Wichtig! Zuvor beachten**  
Consignes de sécurité Powerbank: **Important!** A observer préalablement  
Avvertenze di sicurezza Powerbank: **Importante!** Osservare anticipatamente  
Safety instructions Powerbank: **Important!** To observe first
-  **Sicherheitshinweise Powerbank: Warnung! Wichtig!**  
Consignes de sécurité Powerbank: **Avvertimento!** Important!  
Avvertenze di sicurezza Powerbank: **Avvertimento!** Important!  
Safety instructions Powerbank: **Warning!** Important!
-  **In Übereinstimmung mit den Europäischen Richtlinien für Sicherheit und EMV**  
Conforme aux Directives européennes en matière de sécurité et de CEM  
In conformità alle Direttive Europee per la Sicurezza e EMC  
Complies with European Directives on safety and EMC

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If you suffer from a medical problem (e.g. varicose veins, thrombosis, etc.), consult a doctor prior to using the product.

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Never apply to helpless, infants or heat-sensitive individuals.

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Inspect that all components of the product are in a faultless condition prior to each start-up of the product. The individual components of this product may not be disassembled, repaired or modified.

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Do not insert any needles or pointy objects into the product.

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The product is not intended for professional, medical or hospital use.

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Persons with reduced physical, sensory or mental capabilities, lack of experience or knowledge may only use the appliance if they are supervised by a person who is responsible for their safety or if they have been shown how to use the appliance safely and they have understood the risks inherent in its use.

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Children should not play with the product. Keep packaging (e.g. plastic bags) away from children.

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Always switch off the appliance: whenever it is left unattended, before cleaning, when it fails to work properly, and after use.

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For safety reasons, do not use accessories that are not recommended or sold by the manufacturer.

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If the product is used for purposes other than those specified by the manufacturer, operated incorrectly, or not repaired professionally, we will not accept any liability for any resulting damage. In this case, the warranty claim will be void.

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Never use the product with open wounds, burns, hematoma, swellings, or similar conditions.

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Stop immediately, if you notice any unpleasant sensations, redness of the skin, rashes or inflammation. Heatsensitive persons in particular should use the product with care and interrupt the treatment in a timely manner.

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For sensitive skin, it is strongly recommended not to use directly on the skin. Constantly check the condition of the skin to avoid any redness or burns.

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Use the product only in dry conditions and with a dry body – never in the bathtub, shower, above filled sinks, in pools, saunas, or during rain.

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Do not fall asleep during use. Don't use overnight.

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Don't use the product while folded or wrinkled. Avoid over-folding the belt. Do not put heavy objects on the belt.

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The product should not be used in the following cases:

- Health or disability issues
- Insensitivity to heat
- Injury, sunburn, scarring, inflamed areas or other damage due to disease
- Phlebitis, arthritis or other inflammatory processes
- Under the effects of alcohol, sleeping pills, sedatives or drugs
- Pacemaker

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Use only a 5V power adapter for charging, otherwise the device may get damaged.

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Children should not play with the appliance. Keep the appliance and power cable away from children under 8 years. Keep packaging (e.g. plastic bags) away from children.

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If the appliance is used for purposes other than those specified by the manufacturer, operated incorrectly, or not repaired professionally, we will not accept any liability for any resulting damage. In this case, the warranty claim will be void.

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Don't directly connect any contacts as this can lead to short circuits.

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Never use close to water. Do not expose to rain or humidity. Never immerse the appliance into water or other liquids.

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Check the appliance and power cable for damage. Never switch on a damaged appliance (including power cable) – have it repaired or replaced by the manufacturer, the manufacturer's service centre or by a qualified service technician. Never open the appliance yourself – risk of injury!

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Never place the appliance or the power cable on a hot surface (hob). Keep the appliance away from open flames. Do not expose the appliance to intense heat (heat sources, radiators, sunshine).

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Load the appliance only at a socket with correct voltage. Never use extension leads.

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For the first time use or when the Powerbank has been out of use for a long time, about 5–6 full charges and discharges are needed to achieve full performance.

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






For simple cleaning of the belt, use a soft brush. The heating belt can be washed in the washing machine. Use the fine washing program of your machine, which washes at a maximum of 30 °C. The washing machine should only be lightly loaded, as the washing program requires a lot of water for cleaning. Please observe the washing instructions below.

**Warning:** please remove the Powerbank before washing! Before washing, slide the USB cable into the pocket for the Powerbank.

**Info:** the cable and the USB connection will not be damaged. The controllers are sealed so that no water gets inside when washing.

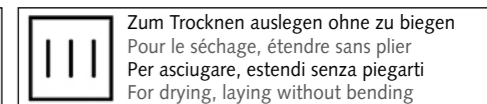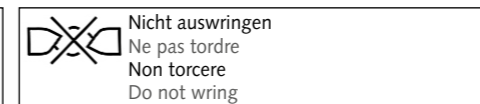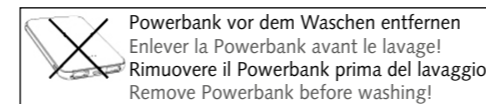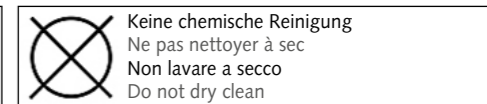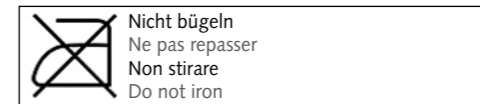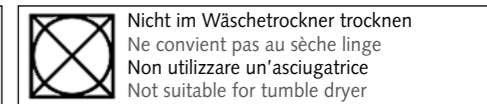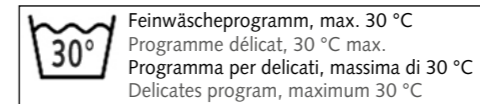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
**Heating belt**

- A** Belt with heating wire
- B** Power switch and temperature adjustment on 3 levels:  
Green LED indicator: approx. 35°C  
Blue LED indicator: approx. 40°C  
Red LED indicator: approx. 60°C  
LED indicator without light: OFF
- C** Rechargeable battery (Powerbank)
- D** Pocket for the Powerbank
- E** USB A power cord integrated in the pocket
- F** Velcro straps to close and secure

**Powerbank**

- 1** Power switch
- 2** Battery level indicator (LED)
- 3** Micro USB charging port (input)
- 4** Output 1: USB A
- 5** Output 2: USB A
- 6** USB cable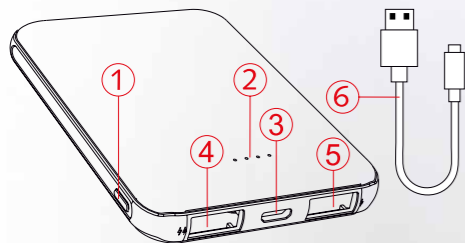

**Before use**

1. Fully charge the Powerbank before using it for the first time:
    - Connect the micro USB port of the Powerbank (3). Use the included USB cable (6).
    - An appropriate power source is the USB port on your computer or an optional AC adapter with USB output (5V).
- Tip:**  
Often the original charger of your mobile device, such as the power supply of the iPhone, is also suitable.
- The active charging function is indicated by the flashing of the LED (2).
  - **Info:**  
Charging time of the Powerbank: 3 hours. Use time of the Powerbank: up to 6 hours.

**Use of the heating belt**

2. Connect the USB cable (6) which is in the belt pocket (D) to the Powerbank (4 or 5).
3. Put on the belt and secure it with the velcro strips (F).  
Please ensure that the heating belt sits loosely, i.e. some air circulates between the heating belt and your skin/clothes during use. Accumulation of heat must be prevented.
4. Switch on by pressing the power switch (B). The device starts at the lowest temperature (green light / 35°C).  
To change the temperature, press the power switch (B) again as follows:
  - 1 x for medium temperature (blue light / 48°C)
  - 2 x for high temperature (red light / 60°C)
5. To switch off, press the power switch (B) till the light turns off.

**Info:**

We recommend using the belt for 30 to 45 minutes on the highest heat or 2 hours on the lowest heat and this no more than 2-3 times per day. Due to the automatic shutdown, the heating belt turns off automatically after 2 hours.

**Important:**

Avoid over-folding the belt. Do not put heavy objects on the belt.

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Bandage chauffant pour la nuque  
 Bendaggio riscaldante per il collo  
 Heating neck bandage

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The product guarantee corresponds with the respective local law and regulations and applies for at least 2 years from date of purchase. It covers faults in construction, workmanship and material. Excluded from this guarantee are all wear parts as well as inappropriate use or maintenance as well as modifications or repairs done by unauthorised third parties.

Please keep your proof of purchase to verify any claims under guaranty.

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**TAVORA Brands AG**