

HIFU LUXURY BEAUTY MACHINE

USER GUIDE

camz



ANTI-AGEING

REJUVENATE SKIN

FACE LIFTING | SKIN TIGHTENING | WRINKLE REMOVER

Instrument Guide



USER GUIDE

Precaution

Women in childbirth are not recommended to use the instrument

Avoid the intra-orbit region, Adam's apple and wounded parts when using the probe

It is not recommended to operate instruments for patients with malignant tumors, hemophilia, or severe bleeding

Patients with major diseases, skin diseases and infectious diseases should cautiously use the instrument

Patients with serious diseases such as heart disease and mental disease should cautiously use the instrument

Wounds, acnes or pustule should be avoided when use the instrument

The skin which has injected hyaluronic acid, Shuiguang needle, botulinum toxin etc should not use the machine unless a month have passed

Women's physiological period is not recommended to use the instrument

Avoid the parts with prosthesis (metal prosthesis) when using the instrument

If the skin is at an allergic period, and it is recommended to wait for a week after the end



SPECIFICATION:


TECHNOLOGY: HIFU
VOLTAGE: AC110V-240V
POWER: 20W
MACHINE SIZE: 190*46*80MM

Level adjustment button

Start button

Color screen display

Adjustment button

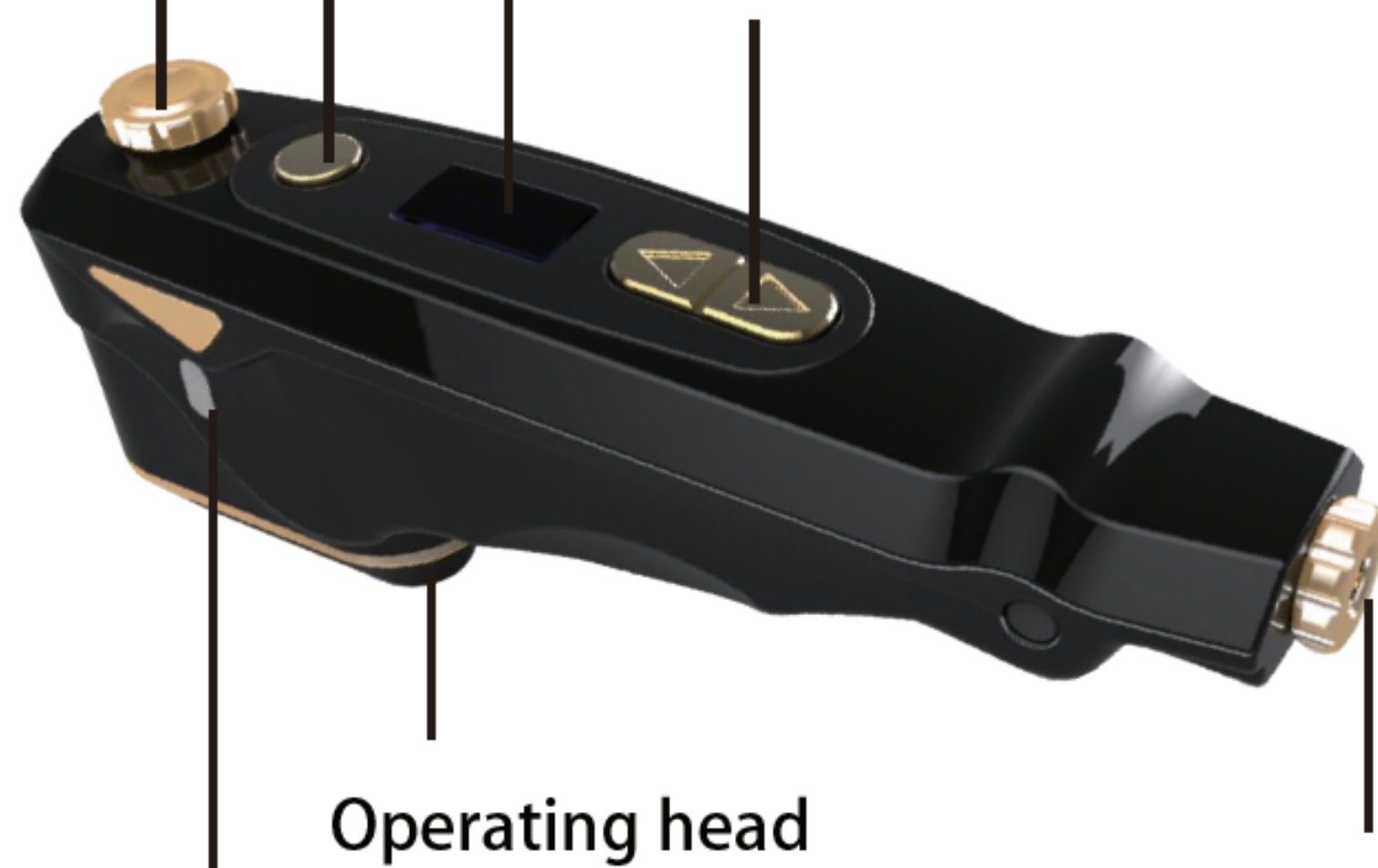


Operating head

Scale display

power outlet

Operation procedure

1. Before the operation, clean the face first, (if you have makeup, please remove the makeup with cleansing oil and wash the face with cleanser again)
2. After the power supply is connected, the instrument automatically enters the standby state
3. Level adjustment: After the power is connected, the circular knob on the top of the handle is used to adjust the level. The counterclockwise adjustment is 3 or 4.5 grades, and the clockwise adjustment is 3 or 1.5 grades.
4. Energy regulation: It is suggested that the first operation start from the lowest energy (the lowest energy of the instrument is 1J, and the highest is 9J)

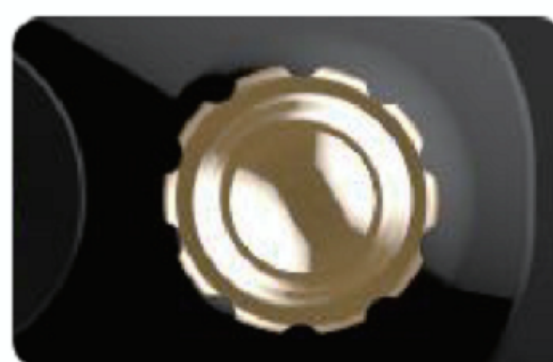

Step 1: Test whether the energy is suitable Operate the instrument at the lower part of the right corner of the mouth first. If there is no sensation, please adjust the energy to one or two level until you can feel a slight hotness or small needling in the operated part. A minority group has soreness feelings near the periodontal.

Step 2: Test until you find the suitable energy intensity and start the operation with the intensity. Lower it when you move to the nose wing part because the part where many bones locates is more sensitive, the energy intensity to bear is relatively lower. (Different parts may bear different intensity, adjust the energy according to self-conditions)

After the level and energy are adjusted, apply the medium to the operated parts and operating probe, and then operate the instrument.

During the operation, keep the operated parts with medium. If the medium is volatile, do not operate until the medium is replenished. Please be noted that the probe must firmly adhere to the skin. There should be no space between the probe and the skin.

The whole face operation take about 30-45 minutes. The specific time depends on the facial situation of each person

Area Techniques




Neck

Use 4.5/3.0 level and start the operation from the lowest energy. Move the probe sideward close to the top of the clavicle along the neck to the jaw with fixed-point operation (operate 3-4 times sideward); (Remember that the position of Adam's apple is not recommended to use the instrument)



Eyes

Use 1.5 level and start from the lowest energy. Move the probe from the lower part of the eyes sideward to the temple, and from the paropia lifting to the above of the eyebrow tail with fixed-point operation. Pass the temple and avoid the intra-orbital, eyelids and eyebrows during the operation; (you may operate 3-5 times sideward)

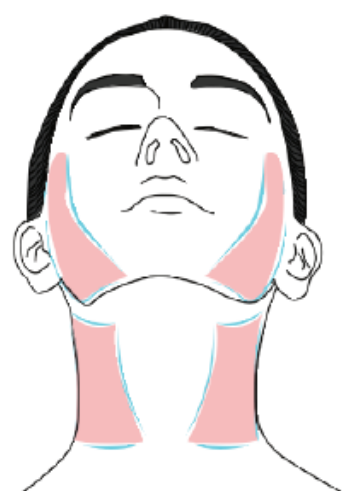

Face

Use 4.5 level and start from the lowest energy. Move the probe along the contours and operate in the shape of "#". Use 3.0 level to operate again after 4.5 level is used. If the nasolabial folds and wrinkles at the corner of the mouth are deep, use 4.5 level to operate 3-5times (do not operate in a fixed point).



Forehead

Use 1.5 level and start the operation from the lowest energy. Move the probe sideward from the position between the eyebrows (fixed point operation) to the hair line. Operate from the bottom up from the position at the eyebrow to hair line and from the position above the eyebrow to hair line. Do not touch the eyebrow. (You may operate 3-4 times sideward)

Notice: During the operation, the probe can't be kept in one position. It is not recommended to use the highest energy at the very beginning but start with the lowest energy and adjust the energy intensity according to the feelings of the operated part.

After Using

After the operation, wipe the operation head with wet towel. Then clean the face with clean water or warm water. Immediately apply a moisturizing mask and remove it in ten minutes. If the moisture on the mask paper is dried halfway, it needs timely replenishment ; Notice: If you need to operate on other parts, consult first before the operation.

Liquid filling of the probe

The liquid in the probe is distilled water / mineral water. When the liquid in the erected probe occupies only half of the space, it should be filled in the probe.



Replacement of the probe film

If there is scratch, wrinkle or breakage on the probe film, then replace the film on the operation head. Tear the old film first, then paste a new one when replacing the film.

Instrument maintenance

1. Each time the instrument is used, clean the instrument and keep it properly.
2. Make sure that the voltage of the instrument is suitable. If the local power supply voltage is unstable, we suggest that the user finds the stable supply with the matching power.
3. In order to ensure the operating effect and the normal service life of the instrument, please use the fittings by the original manufacturer.
4. Do not put the instrument in the wet or water source, as well as directly in the sun.
5. Do not place the instrument near the heat source, which may affect the life and normal use of the instrument.
6. Water, oil and other media must not penetrate into the instrument. Do not strike, knock or heavily fall the instrument to avoid the damage.
7. After use, please disconnect the power and clean the probe.
8. It is forbidden to use product containing alcohol to wipe or clean the instrument.

LAMZ

USER GUIDE

Precaution

Women in childbirth are not recommended to use the instrument

Avoid the intra-orbit region, Adam's apple and wounded parts when using the probe

It is not recommended to operate instruments for patients with malignant , tumors hemophilia, or severe bleeding

Patients with major diseases, skin diseases and infectious diseases should cautiously use the instrument

Patients with serious diseases such as heart disease and mental disease should cautiously use the instrument

Wounds, acnes or pustule should be avoided when use the instrument

The skin which has injected hyaluronic acid, Shuiguang needle, botulinum toxin etc should not use the machine unless a month have passed

Women's physiological period is not recommended to use the instrument

Avoid the parts with prosthesis (metal prosthesis) when using the instrument

If the skin is at an allergic period, and it is recommended to wait for a week after the end of the allergy



SPECIFICATION:

TECHNOLOGY: HIFU

VOLTAGE: AC110V-240V

POWER: 20W

MACHINE SIZE: 190*46*80MM

famz

USER GUIDE