## Instruction Manual / Manuel d'instructions / Manual de instrucciones

**EN** Thank you for purchasing the Sleep Noodle UP. This is the first step in enjoying a good night's sleep, for both you and your sleep partner. Follow the directions below to ensure you receive the most benefits from this innovative device.

You have already chosen your size by following the measuring instructions listed on our website before you purchased it. Confirm that you have chosen the correct size by measuring the distance around your chest, just below the underarm. Compare your measurement to the sizing chart on our packaging.

1. Remove the Sleep Noodle UP pouch and belt from the plastic bag. Unzip the pouch, and pull out the hook and loop straps inside, laying them on either side of the pouch. Carefully insert the black foam into the Sleep Noodle UP pouch starting at the end with the zipper pull tab, ensuring that the flat side of the foam faces toward the bottom of the pouch. Secure the noodle in place by wrapping the inner hook and loop straps around the foam. Once secured, gently zip up the Sleep Noodle UP pouch.

### 2. Wear the belt one of two ways!

**Horizontal:** The Sleep Noodle UP comes packaged to be worn in the horizontal position. Wrap the belt around your body at chest level, just below the underarm. If correctly positioned, the Sleep Noodle UP pouch should be centered horizontally on your back with the logo right side up, and the belt attached at the front. Do not tighten the Sleep Noodle UP belt to the point that you are uncomfortable. It should be loose enough to place two fingers between your chest and belt, while not allowing the belt to rotate around your chest. The inside of the belt is made of a non-slip material to help avoid this rotation.

**Vertical:** Unfasten Sleep Noodle UP pouch and strap from each other. Slide the strap out of the horizontal position of the 2-way belt, turn the strap 90 degrees, and re-insert it into the vertical position opening of the belt until you're able to reattach the Sleep Noodle UP pouch to its hook and loop strap. The vertical position allows the belt to be worn across your chest, similar to the horizontal position, or down lower, just above your waist. Simply adjust the height of the Sleep Noodle UP pouch by moving it higher or lower in the sleeve of the 2-way belt. Remember to adjust the position of the belt so that the Sleep Noodle UP pouch is centered vertically on your back, along your spine.

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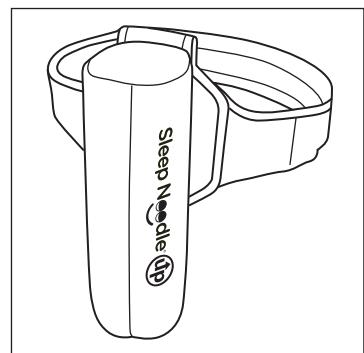
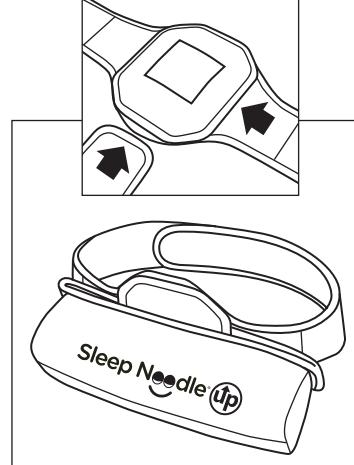
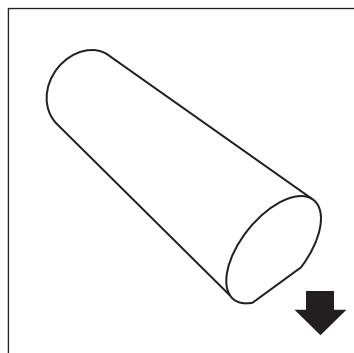
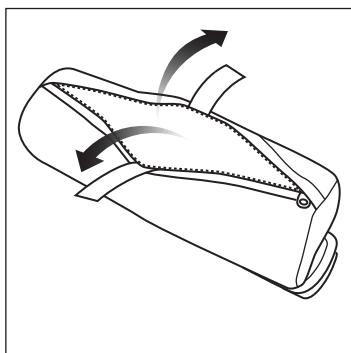
- When not in use, it is a good practice to attach the two sides of the belt together; this will reduce the amount of external debris such as hair, lint, and dirt from attaching to the hook & loop fastener.

#### Cleaning:

- Always remove the internal foam noodle before washing the Sleep Noodle UP. This is done by unzipping the pouch, and pulling the noodle outwards, past the zipper. The Sleep Noodle UP includes a self-healing zipper. If the zipper separates, simply zip and re-zip to repair. You may be required to bend the foam slightly to pass the zippered area. Never force the zipper open or closed. Always store foam noodle in a clean dry area while cleaning belt.
- Do not place in washing machine. Hand washing will extend the life of the Sleep Noodle UP and keep hook & loop fastener from picking up material from adjacent clothes. Prepare a cold bath in a sink or container, with a mild detergent. Do not use any type of bleach to clean. Immerse the Sleep Noodle UP belt into the bath for a short period of time. Scrub areas with a soft brush in areas requiring additional attention. Once clean, remove Sleep Noodle UP from bath, and allow to drain excess water. Replace the soapy bath water with clean fresh water. Immerse the Sleep Noodle UP once again for a short period of time, allowing any soap residue to be removed. Remove from water, allowing time to air dry before reinserting the foam noodle and closing the pouch with the zipper. Do not iron or place in dryer.

#### Warranty:

The Sleep Noodle UP includes a 1 year limited manufacturer warranty from time of purchase. In the case of a warranty issue, return the entire Sleep Noodle UP to the location of purchase with the original receipt. At the manufacturer's option, the belt will be replaced or repaired.

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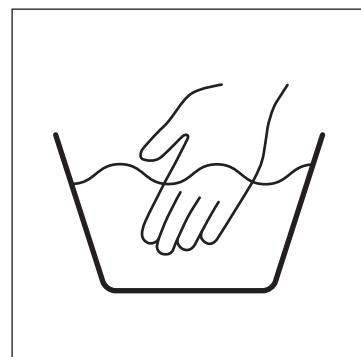
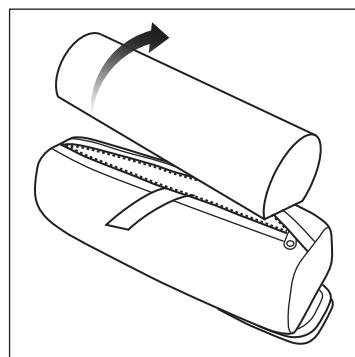
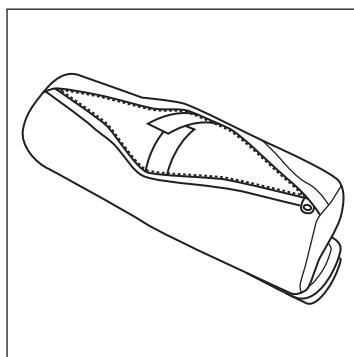
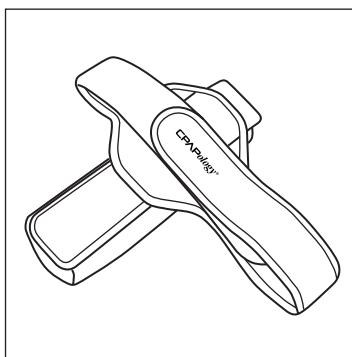
NOTE: Sleep Noodle helps you to develop the habit of sleeping on your side. We recommend that you give the Sleep Noodle a try for a minimum of 2 weeks.

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**Sleep Noodle®**

[www.sleepnoodle.com](http://www.sleepnoodle.com)

Sleep Noodle® is a brand of KEGO Corporation.  
Tel: 833.817.7027