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User Manual

MANO XL

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Declaration of conformity

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Contents

I. General information	27
Notes on the electromagnetic compatibility (EMC).....	27
Storage of the user manual.....	28
Product description	28
Medical device.....	28
Purpose	28
Evolution and lifetime of products	28
Caution when using around medical devices	29
Precautions by alteration of the performance of the device	29
Reinstatement, installation and maintenance.....	29
Cleaning of the system	29
Special advice for transport and storage.....	30
Disposal of the unit according to EU directive 2002/96/EEC	30
Power Supply	30
Safety instructions	31
II Instructions on the adequate use of the electronic magnifier MANO XL	32
Scope of delivery	32
Commissioning	32
Compositions.....	32
Battery Charging.....	34
Power LED	34
Stand.....	35
Operation.....	35
Power Button	35
Beep Type	36
Beep Control.....	36

Magnification	36
Color Mode	36
Brightness	38
Lighting	38
Focus Control	38
Image Freeze	38
Image Storage	39
Image Playback	39
Line & Mask	39
Normal, no line & mask	40
Find	40
Live Panning	41
Memory	41
Change frequency (60Hz to 50Hz)	41
HDMI Output	42
Auto Power Saving	42
Restore	42
Hardware	42
Troubleshooting	43
The unit does not turn on	43
Black screen or White screen	43
The image does not change	43
Smudges or blemishes on the screen	43
Specifications	44
Warranty Information	45

I. General information

Important notes on the operation of the reading magnifier MANO XL. Please read them carefully prior to the operation of the reading magnifier!

Notes on the electromagnetic compatibility (EMC)

This reading system is suited for use in all set-ups including the domicile that are connected directly to a public power supply for buildings with residential purposes.

This reading system is designed for the use in the indicated electromagnetic environment. The user should ensure the use in the indicated environment.

This reading system operates on HF-Energy exclusively for its own function. Therefore the emitted interference of high frequency disturbances is very low and the interference with other appliances operated close-by is improbable. When operating the reading system simultaneously with other systems, however, the possibility of disturbances of the system or other systems should not be neglected.

For this reason it should be ensured that the system is not placed directly beside or on top of another system. If the set-up of this system in the proximity of medical, measuring devices cannot be avoided, the operator of these devices has to be informed that it is necessary to observe the performance of the system to control the conventional use of the system in the chosen set-up.

Electronic devices are sensitive towards electric discharge and towards high frequency electromagnetic fields.

Attention:

In order to avoid high frequency disturbances the operator should respect a minimum distance of one meter to portable and mobile HF-telecommunication devices.

The device may not be used in the patients environment.

Storage of the user manual

The user manual is an essential part of MANO XL. Please keep the booklet in a safe place for future reference.

Product description

MANO XL is an opto-electronic magnifier for private appliance, which magnifies text and pictures.

- 7" LCD-Display (approx. 17,8 cm).
- Continuous magnification ranges from approx. 2,4x – 28x.
- ergonomic and compact design with fold-out stand for easy use.
- light weight (420g) for comfortable use at home, or on the go.

Medical device

MANO XL is classified as a medical device, class I, according to 93/42/EEC, Annexe IX, standard 12. For the distribution to countries outside the EU the respective country-specific guidelines have to be regarded.

Purpose

This electronic magnifier is a medical device to assist with the magnification of printed text and pictures. Visually impaired people can compensate for their loss of visual acuity by using this device. For private appliance only.

Evolution and lifetime of products

The Medical Device Directive regulates by law that manufacturers of medical devices shall determine a period of safety operation of a product. We fulfill hereby legal requirements and determine - for formal reasons - a period of three years after date of first purchase.

Note:

The mentioned life span does not reflect the complete life span of the product. It is rather an indication on when the product might be obsolete and substitution might be recommended due to new product developments.

Caution when using around medical devices

In case the end user intends to use this device in combination with another medical devices, it is recommended to get in contact with Reinecker Vision GmbH or with a licensed distributor.

Precautions by alteration of the performance of the device

Should any of the technical features or performances change, the device must be turned OFF and the battery must be taken out. **Please do not switch on the device again.** Refer directly to your authorized dealer or to Reinecker Vision GmbH Service Center.

Reinstatement, installation and maintenance

This device may be technically maintained and reconditioned by Reinecker Vision GmbH only or by an authorized technician. Reinstatement may only take place after the system has been cleaned, disinfected and safety checked.

Cleaning of the system

- MANO XL has to be disconnected from the power supply before cleaning.
- For cleaning please use the enclosed screen cleaning cloth or any other lint-free, lightly moistened cloth.
- Interior of device must NOT come in contact with liquids.

Special advice for transport and storage

For safety reasons, the unit must only be transported and stored in the original packing. The unit must be stored in dry and closed rooms in the original packing.

The following conditions for transport and storage have to be respected:

- Air pressure: 700 to 1000 hPa
- Temperature range: +5°C to +40°C
- Relative air humidity: 30% to 80%

Disposal of the unit according to EU directive 2002/96/EEC

MANO XL contains of electronic components. To avoid damage to the environment, the country-specific rules have to be respected.

Power Supply

Only use the original power supply and cables which are included in the delivery by Reinecker Vision GmbH.

Caution:

The use of any other equipment might lead to an increased electromagnetic emission and to a product damage.

Safety instructions

Please respect the following safety instructions:

1. Do not plug the AC adapter into a faulty electrical outlet. Doing so may cause electrical shock or fire.
2. Do not touch the AC adapter with wet hands.
3. Only use the original power supply which is included in the delivery by Reinecker Vision GmbH. The use of any other equipment might lead to a product damage.
4. Only use the original battery which is included in the delivery by Reinecker Vision GmbH. The use of any other equipment might lead to a product damage.
5. Do not expose MANO XL to direct sunlight or to very hot or cold environment, as this would harm the picture quality.
6. Do not use this device below +10°C or above +35°C (50° or above 95° F).
7. No liquids must reach the interior of the system. Only use the device in a dry environment.
8. The device may not be used next to medical devices (patients environment).
9. Do not press down on the screen or lay heavy objects on the device. This may cause damage to the device, particularly, the LCD.
10. Do not try to repair or to dismantle the device by your own. This would automatically void the warranty. If you need any technical assistance please get in contact with the local distributor or with Reinecker Vision GmbH.
11. Always switch OFF the MANO XL and disconnect it from the power supply before cleaning it. For cleaning please use the screen cleaning cloth or any other soft, lint-free, lightly moistened cloth. Do not use alcohol containing or strong detergents.

II Instructions on the adequate use of the electronic magnifier MANO XL

Scope of delivery

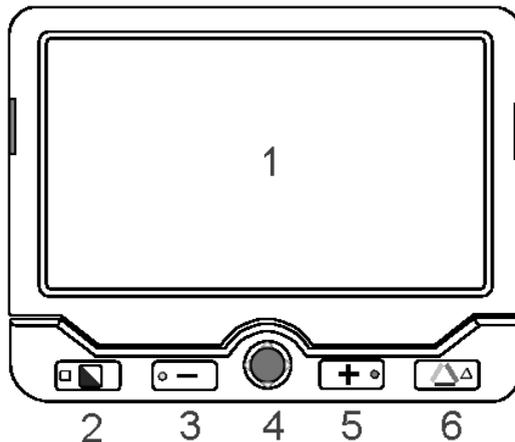
The following components are integrated in the delivery:

- HD-magnifier MANO XL
- Power supply incl. connection cable
- HDMI-cable (1,5 m)
- Protection bag
- Micro fibre cloth
- User manual

Commissioning


Before using MANO XL for the first time, you have to charge the battery. Further information can be found in the chapter “charging the battery”.

Compositions


1. LCD Screen
2. Enhanced Color
3. Zoom Out

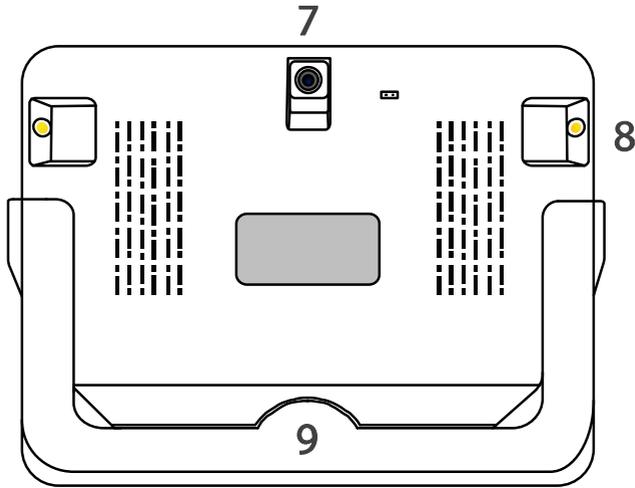
4. Joystick
5. Zoom In
6. True Color

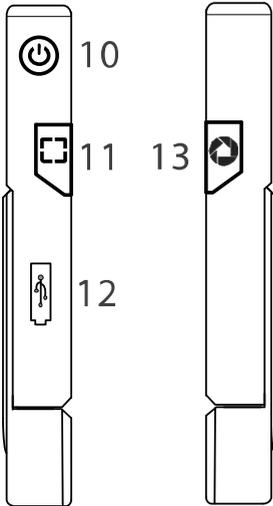7. Rear Camera 8. LED Lights 9. Reading-/Writing-Stand

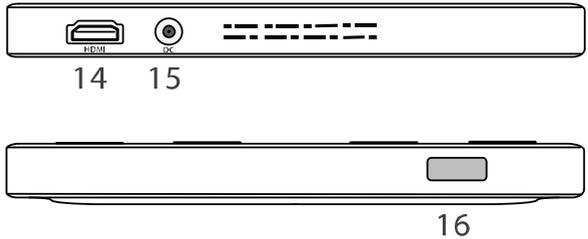
10. Power Button 13. Freeze Button
 11. Line marker and shadowing
 +search function 14. HDMI Port
 12. USB Port 15. Power Port
 16. Focus Control

Battery Charging

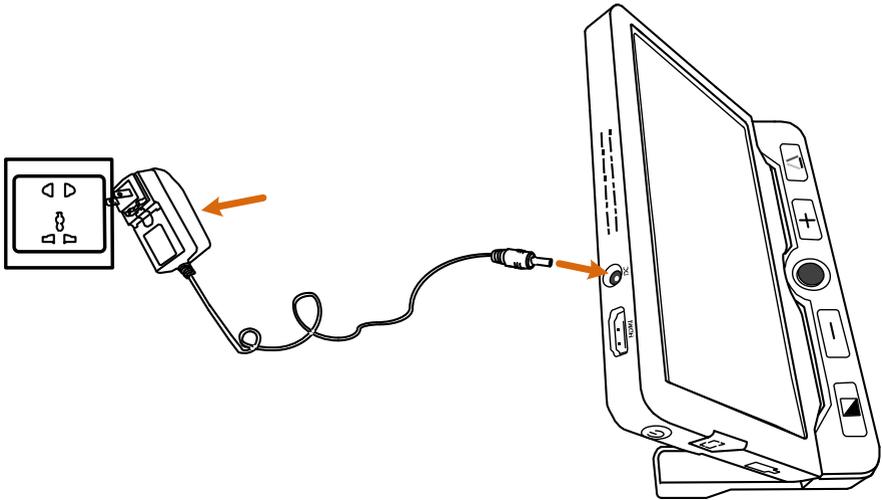- Connect the supplied power cable to the power adapter.
- Connect the supplied power cable to the unit's power port.
- Plug the power adapter into an AC outlet and start charging.


Power LED

- Charging when the unit is off, the power LED will be green, and then auto turn off when fully charged. A full Charge will take about 4 hours and will support the unit working for about 3 hours (depending on operation mode).
- Charging when the unit is on, the power LED will be green.
- When the unit is on without charging, the power LED will be green when the battery level is high and will be red when the battery is dying. It is a signal to charge the battery as long as the red light starts to flash.
- You can also check the battery status by pressing the power button. Refer to the following instructions for more details. Further information can be found in chapter "operation".

Stand

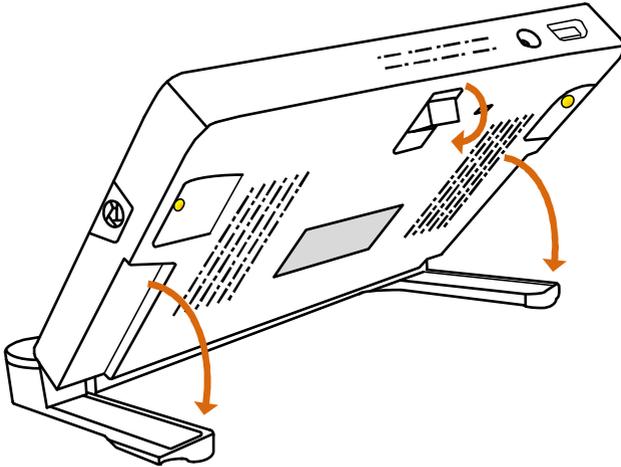- With integrated fold-away writing stand at an ergonomic viewing angle, MANO XL can satisfy users' desire to read more and for a long time in a comfortable position. Also, there's enough space under the camera to work the pen. Unfold the stand as the following figure shows.
- The rear camera is designed to auto rotate with the stand, so that both the Near View and Far View are supported on this unit. Please fold the stand when view in a distance.

Operation

Power Button

- Press and hold the Power Button  for 3 seconds to turn on/off the unit.
- When the unit is on, press the Power Button  to check the current status of battery capacity, freeze mode, focus status, and beep.
- Press other buttons when holding the power button will make combo functions. Refer to the following instructions for more details.

Beep Type

- Unit on/off beep
- Button beep
- Threshold beep
- Low battery beep

Beep Control

- Press and hold the Power Button  and press the True Color Button  to turn on/off the beep.

Magnification

- Press the Zoom In Button  to increase the magnification step by step or hold the Zoom In Button  for continuous magnification.
- Press the Zoom Out Button  to decrease magnification and hold the Zoom Out Button  for continuous decrease.

Magnification range when standing on table: 2.4x – 28x

Color Mode

A color mode is a predefined color combination. Full-color mode is suitable for viewing pictures or maps.

Enhanced Color mode is combined with different foreground and background colors which can help to read more clearly.

- Press the Enhanced Color Button  to go through the Enhanced-color combination list.
- Press the True Color Button  to return to the Full-color mode.

- While in the Enhanced Color Mode, long press the Color Mode Button  for 3 seconds, a Red Heart Icon  appears with two beeps which indicates the current color mode is added to your Favorite Enhanced Color Combination List.
- Short press the color mode button  to switch among “White on Black”, “Black on White”, “Favorite front color on Favorite back color”, “Favorite back color on Favorite front color”.
- Long press the Color Mode Button  3 for 3 seconds, the Full View Icon  appears with one beep which indicates that you have changed to the Integrated Enhanced Color Combination List.

The factory default Favorite Enhanced Color Combination List (4 modes)

 White on Black

 Black on White

 Yellow on Black

 Black on Yellow

The Integrated Enhanced Color Combination List (18 modes)

 White on Black

 Black on White

 Yellow on Black

 Black on Yellow

 White on Blue

 Blue on White

 Yellow on Blue

 Blue on Yellow

 Green on Black

 Black on Green

 Violet on Black

 Black on Violet

 Blue on Black

 Black on Blue

 White on Green

 Green on White

 White on Red

 Red on Black

Brightness

Brightness of the screen can be adjusted in both Full-Color Mode and Enhanced-Contrast Mode.

- Press and hold the Power Button  and press the Zoom In Button  to increase the brightness. Factory default value will be highlighted green.
- Press and hold the Power Button  and press the Zoom Out Button  to decrease the brightness.

Lighting

- Press and hold the Power Button  and press the Freeze Button  to turn on/off the LED Lighting manually.

Focus Control

- Click the Focus Control Button  to lock the focus, a red frame appears around the screen, which is especially useful when hand-writing.
- Click the Focus Control Button  again to unlock the focus, the red frame disappears.

Image Freeze

- Freezing an image on the LCD screen can be useful for further reading. At this status, magnification as well as color change is still allowed for a better experience.
- Press Freeze Button  to lock the the active image.
- Press Freeze Button  again to unlock.
- For the locked image with increased magnification, you can use the Joystick  to navigate around freely.

Image Storage

- Long press the Freeze Button  for 3 seconds to store the current image. You can save up to 18 images.

Note: When the saved images reach to the maximum value (18 images, Memory Full ICON appears), you will need to delete some pictures first in Playback mode before the next successful saving.

Image Playback

- To enter the Playback Mode, long press the True Color Button  for 3 seconds.
- Slide the Joystick  to select the target image from the Nine Grids Thumbnail Image Gallery. Press the True Color Button  to view the target image. You can view and navigate the image with the same operations described in the Image Freeze section. Press the Freeze Button  to return to the Nine Grids Thumbnail.
- When in Nine Grids Thumbnail, press the Locate Button  to select the target Image which needs to be deleted. Press the Locate Button  again to confirm, or any other buttons to cancel.
- When in Nine Grids Thumbnail, hold the Locate Button  for 2 seconds to delete **all** the images. Press the Locate Button  again to confirm, or any other buttons to cancel.
- Long press the True Color Button  for 3 seconds to exit the Playback mode.

Line & Mask

Press the Locate Button  to switch among different Reading Line & Mask modes.

• One horizontal line

Slide the Joystick  up/down to change the position of horizontal reading line. Slide the Joystick  left/right to decrease/increase the width of the line.

- **One vertical line**

Slide the Joystick  left/right to change the position of vertical reading line. Slide the Joystick  up/down to increase/decrease the width of the line.

- **Horizontal mask**

Slide the Joystick  up/down to change the position of the masks.

Slide the Joystick  left/right to narrow/enlarge the distance between the masks.

- **Vertical mask**

Slide the Joystick  left/right to change the position of the masks.

Slide the Joystick  up/down to enlarge/narrow the distance between the masks.

Note: there is no live movement via joystick available in this mode.

Normal, no line & mask

- Press the Power Button  will quickly exit the Line & Mask modes.

Find

- Long press the Locate Button  for 1 second and hold not to release it, a location box would appear.
- Move and change the content in the location box, aiming it at the part you want to view, and then release the Locate Button , the image will return back to the previous magnification with the re-freshed content.

For example, you are viewing one part of a map with a certain magnification. Then you want to see another part without changing your favorable magnification. Then the function would be convenient and useful for you.

Live Panning

- In the state of amplification, users can slide the Joystick  when in the real-time video mode to navigate the image for seeing more without moving the unit.
- Press the Zoom in  and Zoom Out  Buttons at the same time to help you re-center the image (Red Dot appears).
- Two ways to quickly jump to the start/end point:
 - 1) press and hold the Power Button , and slide the Joystick 
 - 2) quickly push the Joystick  in the same direction twice.

For example, if you have finished reading a horizontal line and want to read from the beginning of the next line, please hold the Power Button  and slide the Joystick  left or just quickly push the Joystick left  twice. The start point or end point will be auto recorded and refreshed according to the maximum navigation range when reading.

- The navigation speed can be adjustable in three levels: slow, medium (the factory default) and fast. Push the Joystick  to one side and press the Zoom in/out to increase/decrease the reading speed.

Memory

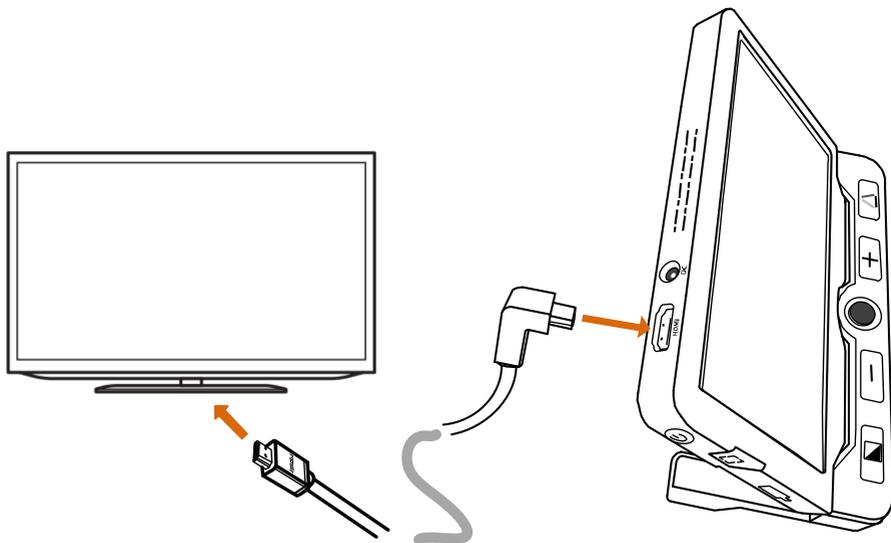
The unit will memorize automatically the most recent settings such as color mode, magnification, beep when power is off normally and apply them when the unit is on next time.

Change frequency (60Hz to 50Hz)

Press and hold the Power Button  and the Color Button  for about 3 seconds, to change frequency from 60Hz to 50Hz or the other way around. Change when necessary, only if there is a unpleasant flicker on the screen.

HDMI Output

Connect the unit to your external display device with HDMI cable for a better and larger viewing as following pictures show

Auto Power Saving

If no operation and no move for over 3 minutes, the unit will turn off automatically.

- Press and hold the Power Button  and click the Enhanced Color Button  to turn on/off the Auto Power Saving function manually (factory default is on).

(Pay attention to the Moon-symbol.)

Restore

Long press the Freeze Button  and the Locate Button  at the same time for 5 seconds to reset to the default factory settings. Auto power off when reset is done.

Hardware

When the unit is hung or can't be turned off normally, press the Power Button  + True Color Button  + Zoom In Button  at the same time to trigger the hardware reset.

Troubleshooting

The unit does not turn on

- Connect to the power adapter
- Charge the battery
- Hardware Reset

Black screen or White screen

- If the unit is sitting on a flat pure surface, the screen may be black or white. Lift the unit off to ensure change happens.
- Adjust the magnification
- Adjust the Color Mode
- Restart the unit
- Hardware Reset

The image does not change

- If the unit enter the freeze mode or playback mode, the image will not change.
- Press power button to check the current status
- Restart the unit
- Hardware Reset

Smudges or blemishes on the screen

- Adjust brightness
- Clean the LCD screen and camera

Please contact your local distributor if problem is not solved.

Specifications

- **Display:** 7-inch IPS LCD screen / resolution: 1280* 720
- **Camera:** 13M Pixels / video resolution: 1920* 1080
- **Size:** < = 188x138x21 mm
- **Weight:** < = 420 grams
- **Color Mode:** Nature color, favorite user defined mode; 18 Integrated enhanced color modes
- **Magnification:** 2.4x – 28x
- **Battery:** Integrated rechargeable Li battery; 3-hour continuous use after 4-hour charge
- **Operating time:** up to approx. 4 hours
- **Charging time:** approx. 3 hours (depending on the set mode)
- **Power Adapter:** Input: 110-240V; Output: 5V/3A

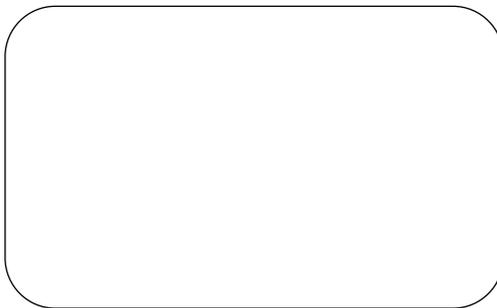
Warranty Information

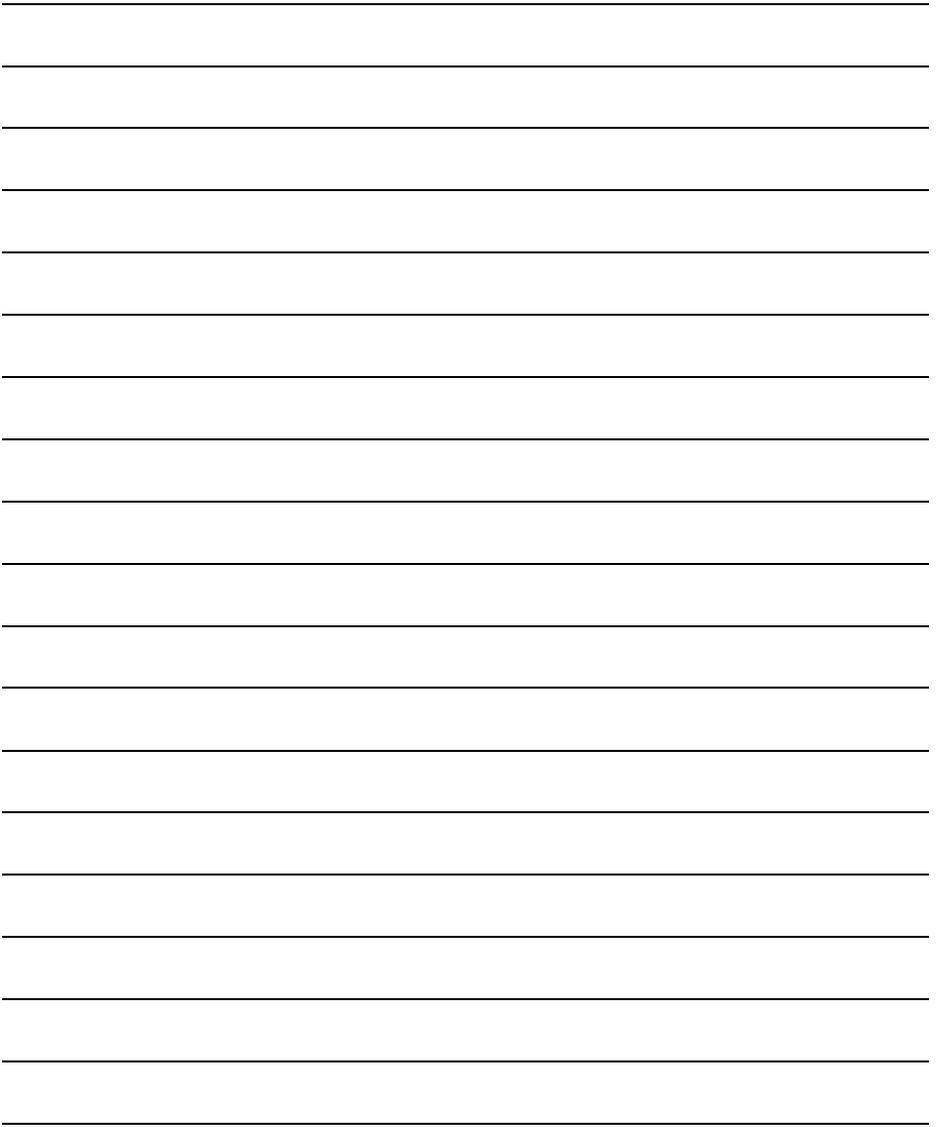
1. Reinecker Vision GmbH grants a warranty of 24 months (warranty period only valid within the European Union) against defects in manufacture counted from date of invoice ex Reinecker Vision GmbH.
2. All parts that have been replaced when repairing the unit will automatically become property of Reinecker Vision GmbH.
3. Damage caused by improper use or by non-compliance with the operating instructions will result in the invalidation of the warranty.
4. Interventions from persons other than Reinecker Vision GmbH authorised specialists may lead to invalidity of warranty.
5. Partial warranty is valid for light sources serving to illuminate the reading object, batteries and other parts subject to wear and tear.
6. If a warranty claim is necessary, the system has to be returned to the manufacturer in Alsbach-Hähnlein or to your authorised dealer in the original packaging including all components. The manufacturer does not carry the cost for transport, packaging and insurance.

Phone number: +49 6257 9311-0

View our General Terms and Conditions.

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E-Mail: info@reineckervision.de
www.reineckervision.de

Reinecker Vision GmbH International Contact

sales department, customer relations:

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In case service is necessary, please contact your authorized distributor.

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Reinecker Vision GmbH Zentrale