

GoodSleep^{co}



Sleep Image Workflow



Getting started

→ Setting up the test through ResDent




Use ResDent to set up a test for a new or existing patient

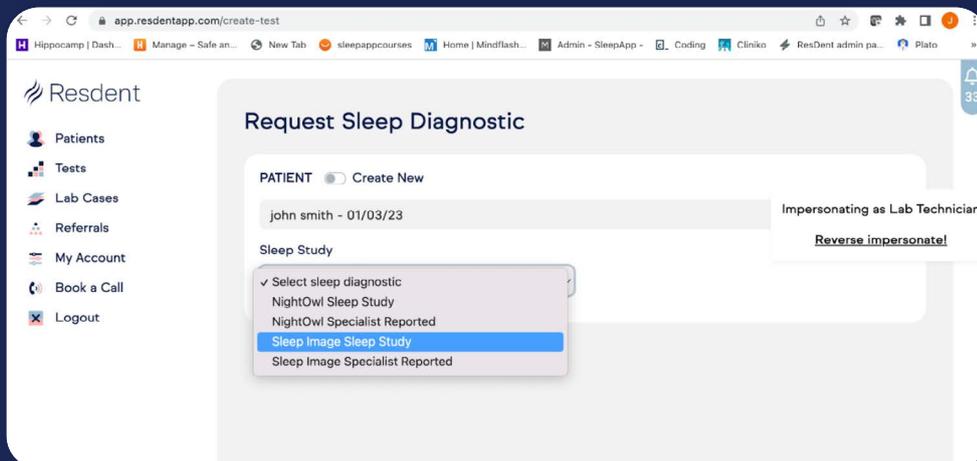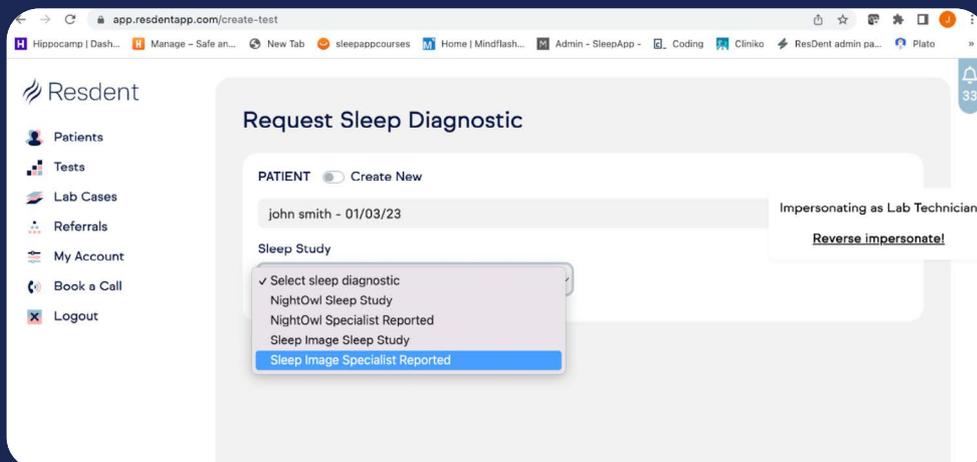

See over

GoodSleepco™

Sleep Image Workflow

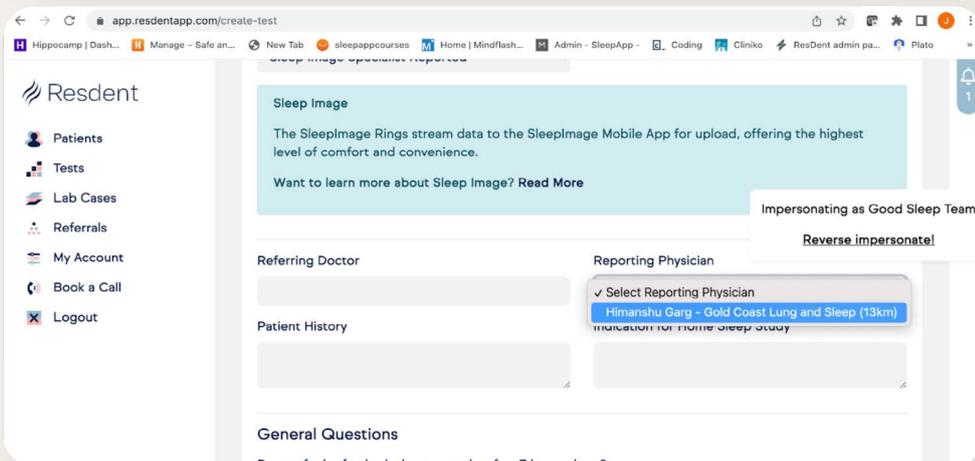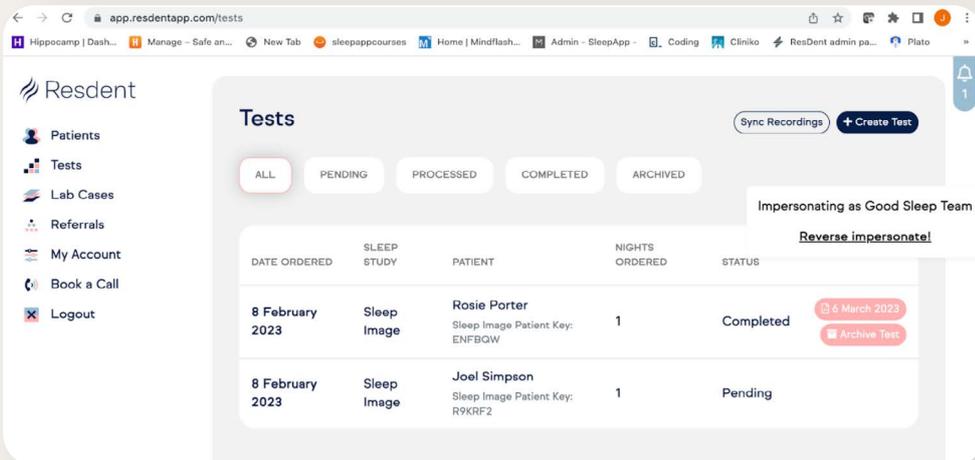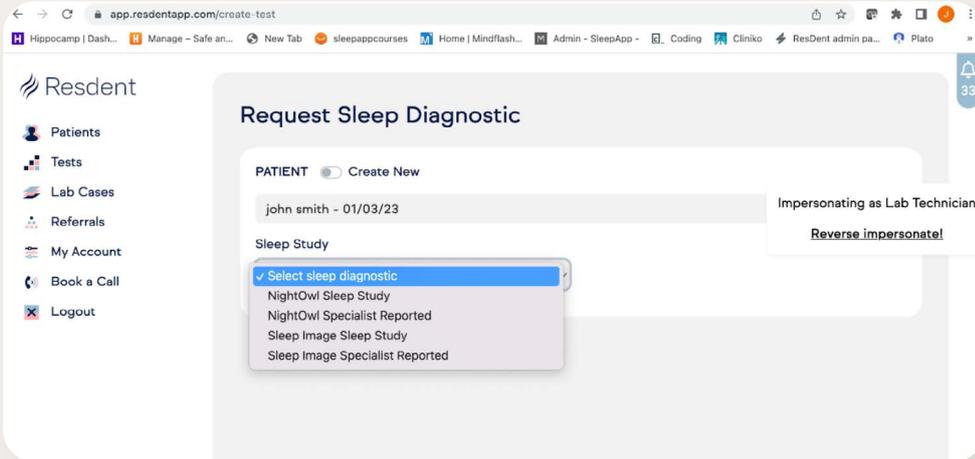


GoodSleepco™

Sleep Image Workflow

Set your patient up

→ Initiate a test

You are to initiate a test for the patient through ResDent so that the patient an email with details on downloading the Sleep Image app

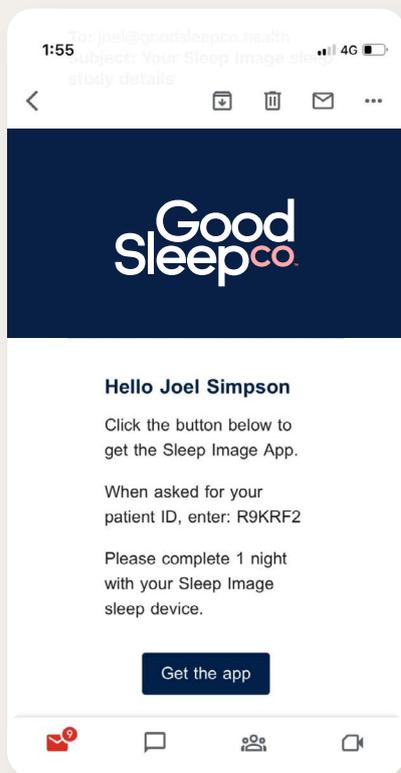


Ring handover & App

→ Sleep Image ring given to patient


You give the patient a Sleep Image ring before they leave

→ Patient to download Sleep Image app

Patients will download the Sleep Image app

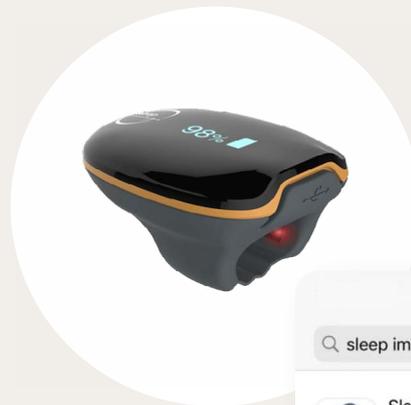


sleepimage

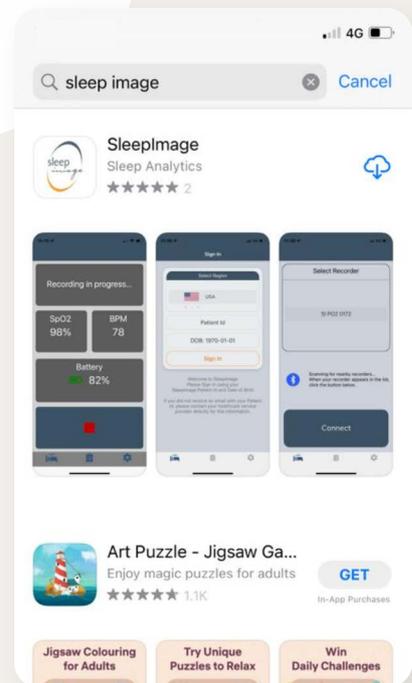

Patient - First night monitoring

→ Patient starts test

Patient puts the ring on, and starts the test before bed

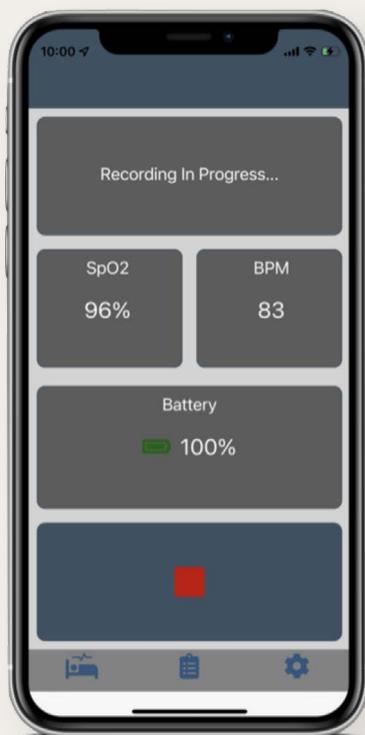

→ Patient ends test in morning

When patient wakes up they take the ring off and end the test

→ Email notification - Results ready

When the results are ready you will be notified by email

→ Show in ResDent

Autoscore plus specialist report options

→ Access the report in ResDent

If you are waiting on a specialist report this may take a few more days but don't worry, you'll get these back soon enough