

Cleaning


Option 1. Clean ContactEZ *Dental Strips* in an Automatic Washer.

Option 2. Clean ContactEZ *Dental Strips* by hand.

Step 1. Holding the ContactEZ *Dental Strips* under a faucet, run regular tap water over each strip, and scrub the stainless steel surface using a brush.



Step 2. Clean the ContactEZ *Dental Strips* using General Cleaning Solution in an Ultrasonic Tank for 10 - 20 minutes.

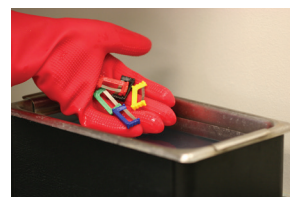

Step 3. Rinse the ContactEZ *Dental Strips* under a faucet with regular running tap water.



Sterilization

Step 1. Sterilize the ContactEZ *Dental Strips* in a standard steam heat autoclave (not dry heat) at 30 lbs of pressure for 15 minutes, with a maximum temperature of 286°F/141°C. ContactEZ recommends using distilled water in the autoclave.



Step 2. Allow the ContactEZ *Dental Strips* to cool down for a period of 5 minutes once the sterilization process is complete.



Step 3. Remove the ContactEZ *Dental Strips* from the autoclave by gloved hand. The strips are now ready to use.



Disposal

When the diamond powder has worn off of the stainless steel strip, ContactEZ *Dental Strips* may be disposed of with normal waste products.



Patented • 100% Guaranteed
contacez.com • 360.694.1000



EC

REP

EMERGO EUROPE
Molenstraat 15
2513 BH, The Hague
The Netherlands



ContactEZ, LLC
217 SE 136th Ave, Suite 105
Vancouver, WA 98684
United States