## **Height Adjustable Strolley Trolley**

**VG798**

## **Fixing and Maintenance Instructions**

## INTRODUCTION

Thank you for deciding to purchase the Strolley Trolley from Aidapt. The Strolley Trolley is manufactured from the finest materials and components available, when used correctly they are designed to give many years of reliable trouble-free service.

## IMPORTANT!

**N.B. This equipment must be installed by a competent person.**

**Please Note: Always consult with the prescriber/Physiotherapist if you are in any doubt that this product is suitable for your use.**

Maximum Weight on Tray: 15 kg

**Do not exceed the weight limit stated – doing so could put the user at risk.**

## BEFORE USE

Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the item.

Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

## ASSEMBLY

Your trolley is provided to you packed in cube form. Please examine all components for any damage before use. If you see any damage or suspect a fault, please contact your supplier immediately.

In this case, please do not use your product as it may compromise the safety of the user.

The package contains the following components:

- Main trolley body
- 4 x height adjustable legs with pin clips
- 2 x detachable arms
- 2 x clip on moulded table tops

**Please read and follow the instructions carefully:**

1. Remove all the contents from the box.
2. Ensure that you protect the paintwork of the trolley during assembly by placing the trolley on carpet or a large sheet of cardboard.
3. Take the trolley frame and place it upside down and carefully insert the tapered section of the brown painted leg assembly into the body of the trolley frame.
4. Insert the leg retention clip. Ensure that the retention clip passes all the way through.
5. Ensure that all four extension legs are adjusted to the same height and then turn the frame upright.
6. Test the trolley for stability thoroughly and visually check that all four leg retention clips are in place.

**NB. Do not use the leg retention clips to adjust the trolley height.**

7. Place the two detachable arms into the locating holes with the higher part of the arms towards the front of the trolley.

**N.B. Ensure both arms are fully located in place and the four brass buttons pass completely through the frame; they should protrude approximately 5 mm out from the main frame.**

8. Clip on both trolley trays with the flared edge towards the front of the trolley.

Your completed trolley should now look like the image to the right.


## HEIGHT ADJUSTMENT

Your trolley can be adjusted in height if required.

- To adjust the height remove the lower 'E' clips from the extension legs and pull each leg to the desired height. (See fig. 1.)
- Ensure that all four extension legs are evenly adjusted.
- Re-insert all 'E' clips facing the line of travel also ensuring that the 'E' clips have passed through the product frame and extension leg and are secure. (See fig 2.)
- Never adjust the legs unevenly; this is dangerous and can seriously compromise the safety of the user.

Check that the trolley is sitting evenly on the floor and there is no excessive rocking or the trolley is tilting to one side. If this is the case, please do not use the trolley, but repeat the above steps to correct the problem or contact your supplier for service support.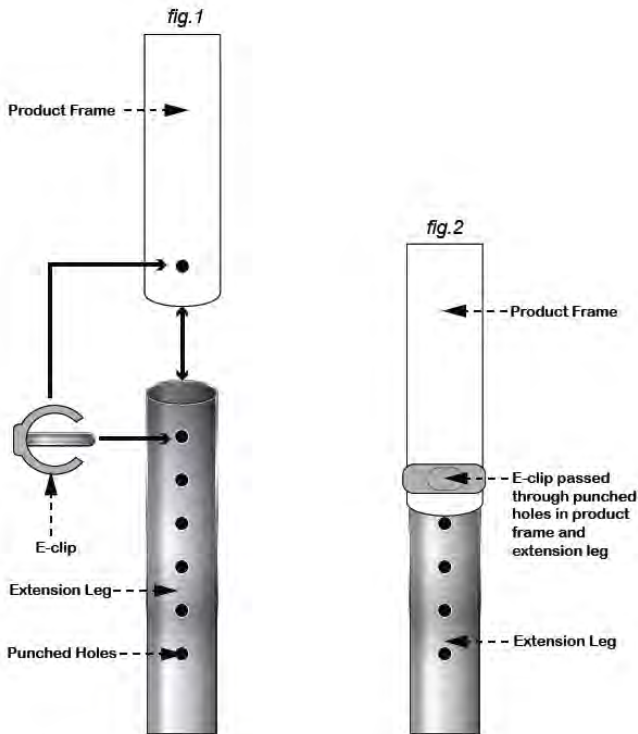

## **CLEANING**

The polypropylene clip on trays can be removed for cleaning. To remove, place your finger and thumb on the underside of the U clip over inverted U section, push upwards and remove.

The Aidapt tray can be cleaned with either a mild detergent or disinfectant, or alternatively pressure washed or autoclaved at a temperature of up to 100oC.

To refit the tray, simply push down on the inverted U sections ensuring that the tray is in line with the trolley frame. Check that all the engaging edges are sitting neatly, evenly and securely to the frame.

## **CLEANING THE CASTORS**

Your trolley is supplied with high quality metal castors with an integral wheel guard.

The castors can be removed from the frame by removing the upper 'E' clip retaining the extension leg. Once done, wash the castor in a bucket of mild detergent or disinfectant.

After cleaning ensure that you thoroughly dry the castor and extension leg prior to reassembly and finish by applying a water-dispersing agent such as WD40 over the castor wheel and body prior to reassembly.

## **CARE & MAINTENANCE**

Please make a safety check of the product at regular intervals or if you have any concerns.

## **REISSUE**

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

This includes:

- Tightness of all nuts/bolts/castors
- Other screw in/bolt in/push in components.

**If in any doubt, please do not issue or use, but immediately contact your supplier for service support.**

## **IMPORTANT INFORMATION**

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

**Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX**  
**Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: [www.aidapt.co.uk](http://www.aidapt.co.uk)**  
**Email: [accounts@aidapt.co.uk](mailto:accounts@aidapt.co.uk) • [adaptations@aidapt.co.uk](mailto:adaptations@aidapt.co.uk) • [sales@aidapt.co.uk](mailto:sales@aidapt.co.uk)**

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