# TESMED Trio 6.5

Mit TENS, EMS & Massage

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## TESMED Trio 6.5

### Operation Instruction Manual

Be sure to read this instruction manual before operation and keep properly!

Tesmed copyright - it is absolutely forbidden the partial or total reproduction of any present content

**This manual is valid for the Trio 6.5 stimulator**

This user manual is published by Tesmed, in full compliance with the original version of the manufacturer, Tesmed does not guarantee its content and reserves the right to improve it and modify it at any time without prior notice, subject to authorization by the manufacturer. The changes will however be published in a new edition of this user manual.

All rights reserved.  
Trio 6.5 Rev. V1.0 © 2018-Tesmed

# Table of Contents

<b>1. FOREWORD</b> .....	<b>104</b>
1.1 Introduction .....	104
1.2 Medical background.....	104
<b>2. SAFETY INFORMATION</b> .....	<b>105</b>
2.1 Intended use .....	105
2.2 Contraindications.....	105
2.3 Warning .....	106
2.4 Precautions .....	107
2.5 Adverse reactions.....	108
<b>3. GETTING TO KNOW YOUR DEVICE</b> .....	<b>108</b>
3.1 Front and Rear Panel.....	108
3.2 LCD Display .....	109
3.3 Accessories.....	109
<b>4. SPECIFICATION</b> .....	<b>109</b>
4.1 Technical information .....	109
4.2 Technical specification for TENS mode .....	110
4.3 Technical specification for EMS mode .....	111
4.3 Technical specification for MASSAGE mode.....	111
<b>5. OPERATION INSTRUCTION</b> .....	<b>112</b>
5.1 Low battery and charging .....	112
5.2 Connect electrodes to electrode cables .....	112
5.3 Connect electrode cables to device .....	112
5.4 Electrode.....	112
5.5 Electrode placement .....	113
<b>6. INSTRUCTIONS FOR USE</b> .....	<b>114</b>
6.1 Turn on.....	114
6.2 Select the treatment mode.....	114
6.3 Select the application program.....	114
6.4 Starting the treatment .....	115
6.5 Adjust channel intensity .....	115
6.6 Turn the device off .....	116
6.7 Load detect function .....	116
6.8 Low battery indication .....	116

6.9 Charging .....	117
6.10 Power saving function .....	117
6.11 Use of electrode .....	117
6.12 Where do I attach adhesive electrode? .....	118
<b>7. PROGRAM .....</b>	<b>119</b>
7.1 Application program .....	119
<b>8. CLEANING AND MAINTENANCE.....</b>	<b>122</b>
8.1 Cleaning and caring for the device.....	122
8.2 Cleaning and caring for the electrode and electrode cables .....	122
8.3 Maintenance .....	122
<b>9. TROUBLESHOOTING .....</b>	<b>123</b>
<b>10. STORAGE .....</b>	<b>124</b>
<b>11. DISPOSAL.....</b>	<b>124</b>
<b>12. EMC DECLARATION .....</b>	<b>124</b>
<b>13. Glossary of Symbols .....</b>	<b>124</b>
<b>14. SOFTWARE VERSION .....</b>	<b>125</b>
<b>15. GUARANTEE.....</b>	<b>125</b>
Treatment boards, program and positioning of electrodes .....	128

## 1. FOREWORD

### 1.1 Introduction

The device FDES116 is a dual channels portable electrical therapeutic instrument. Before using, please read all the instructions in this user manual carefully and keep it safe for future.

Portable electrical therapeutic instrument belongs to the group of electrical stimulation systems. It has three basic function – TENS (Transcutaneous Electrical Nerve Stimulation), EMS (Electrical Muscle Stimulation) and MASSAGE.

Function of the portable electrical therapeutic instrument has 36 application programs(12 TENS programs, 12 EMS programs and 12 massage programs). The respective application program controls the generated electric impulses, their intensity, frequency and frequency range. The values for the respective program are listed on page 19.

The operation principle of electrical stimulation equipment is based on simulating the body's own pulses which are transmitted transcutaneous to nerve or muscle fibers by means of electrodes. The electrodes can thereby be attached to many parts of the body, whereby the electrical stimuli are safe and practically painless. You merely feel a gentle pricking or vibrating in some applications. The electrical pulses transmitted to the tissue affect the transmission of stimulation in nerve conductions as well as Neuron and muscle tissue in the field of application. Electrical stimulation does not replace regular exercising of the muscle, but is able to reasonably supplement the effect thereof

### 1.2 Medical background

#### **1.2.1 EXPLANATION OF PAIN**

Pain is a warning system and the body's method of reminding us that something is wrong. Pain is important signal; without it abnormal conditions may go undetected, causing damage or injury to vital parts of our bodies. Even though pain is a necessary warning signal of trauma or malfunction in the body, nature may have gone too far in its design.

Aside from its value in diagnosis, long-lasting persistent pain serves useless purpose. Pain does not begin until coded message travels to the brain where it is decoded, analyzed, and then reacted to. The pain message travels from the injured area along the small nerves leading to the spinal cord. Here the message is switched to different nerves that travel up the spinal cord to the brain. The pain message is then interpreted, referred back and the pain is felt.

#### **1.2.2 EXPLANATION OF TENS**

TENS (Transcutaneous Electrical Nerve Stimulation) gives good results in relief of pain. It is clinically proven and used daily by physiotherapists, other caregivers and top athletes around the world. High-frequency TENS activates the pain-inhibiting mechanisms of the nervous system. Electrical impulses from electrodes, placed on the skin over or near the painful area, stimulate

the nerves to block the pain signals to the brain, and the pain is not perceived. Low-frequency TENS stimulates the release of endorphins, the body's natural painkillers.

### 1.2.2 EXPLANATION OF EMS

Electrical Muscle Stimulation is an internationally accepted and proven way of treating muscular injuries. It works by sending electronic pulses to the muscle needing treatment; that causes the muscle to exercise passively. It is a product derived from the square waveform, originally invented by John Faraday in 1831. Through the square wave pattern it is able to work directly on muscle motor neurons. The Ultra EMS System has low frequency and this in conjunction with the square wave pattern allows direct work on muscle groupings.

### 1.2.3 EXPLANATION OF MASSAGE

Massage, to stimulate the muscle motor nerves of healthy persons to improve their muscular performance, to achieve a relaxing effect or to improve circulation exchanges.

## 2. SAFETY INFORMATION

### 2.1 Intended use

---

#### For TENS mode:

- To be used for temporary relief of pain associated with sore and aching muscles due to strain from exercise or normal household work activities.

#### For EMS mode:

- To be used to stimulate healthy muscles in order to improve and facilitate muscle performance.

#### For MASSAGE mode:

- To be used for recovery of fatigue, improves blood circulation neuralgia.

### 2.2 Contraindications

---

- Do not use this device if you have a cardiac pacemaker, implanted defibrillator, or other implanted metallic or electronic device. Such use could cause electric shock, burns, electrical interference, or death.
- The device should not be used when cancerous lesions or other lesions are present in the treatment area.
- Stimulation should not be applied over swollen, infected, inflamed areas or skin eruptions (e.g. phlebitis, thrombophlebitis, varicose veins, etc.).
- Electrode placements must be avoided that apply current to the carotid sinus region (anterior neck) or transcranially (through the head).
- This device should not be used over poorly innervated areas.
- Inguinal hernia.

- Do not use on scarred areas following surgery for at least 10 months after the operation.
- Do not use with serious arterial circulatory problems in the lower limbs.

### 2.3 Warning

---

- If you have had medical or physical treatment for your pain, consult with your physician before using this device.
- If your pain does not improve, becomes more than mild, or continues for more than five days, stop using the device and consult with your physician.
- Do not apply stimulation over your neck because this could cause severe muscle spasms resulting in closure of your airway, difficulty in breathing, or adverse effects on heart rhythm or blood pressure.
- Do not apply stimulation across your chest because the introduction of electrical current into the chest may cause rhythm disturbances to your heart, which could be lethal.
- Do not apply stimulation over, or in proximity to, cancerous lesions.
- Do not apply stimulation in the presence of electronic monitoring equipment (e.g., cardiac monitors, ECG alarms), which may not operate properly when electrical stimulation device is in use.
- Do not apply stimulation when in the bath or shower.
- Do not apply stimulation while sleeping.
- Do not apply stimulation while driving, operating machinery, or during any activity in which electrical stimulation can put you at risk of injury; and
- Apply stimulation only to normal, intact, clean, healthy skin.
- The long-term effects electrical stimulation are unknown. Electrical stimulation device do not have any curative value.
- Stimulation should not be applied transthoracically in that the introduction of electrical current into the heart may cause cardiac arrhythmias.
- Stimulation should not take place while the user is connected to high-frequency surgical equipment, which may cause burn injuries on the skin under the electrodes, as well as problems with the stimulator.
- Do not use the stimulator in the vicinity of shortwave or microwave therapy equipment, since this may affect the output power of the stimulator.
- Never use near the heart. Stimulation electrodes should never be placed anywhere on the front of the thorax (marked by ribs and breastbone), but above all not on the two large pectoral muscles. Here it can increase the risk of ventricular fibrillation and lead to cardiac arrest.
- Never use on the eye area.
- Never use near the genitals.
- Never use on the areas of the skin which lack normal sensation
- Keep electrodes separate during treatment, electrode in contact with other could

result in improper stimulation or skin burns.

- Keep the stimulator out of reach of children.
- Consult your doctor if you are in any doubt whatsoever.
- Discontinue and do not increase the intensity level if you feel discomfort during use.

## 2.4 Precautions

---

- TENS is not effective for pain of central origin including headache.
- TENS is not a substitute for pain medications and other pain management therapies.
- TENS devices have no curative value.
- TENS is a symptomatic treatment and, as such, suppresses the sensation of pain that would otherwise serve as a protective mechanism.
- Effectiveness is highly dependent upon patient selection by a practitioner qualified in the management of pain patients.
- The long-term effects of electrical stimulation are unknown.
- Since the effects of stimulation of the brain are unknown, stimulation should not be applied across your head, and electrodes should not be placed on opposite sides of your head.
- The safety of electrical stimulation during pregnancy has not been established.
- You may experience skin irritation or hypersensitivity due to the electrical stimulation or electrical conductive medium (silica gel).
- If you have suspected or diagnosed heart disease, you should follow precautions recommended by your physician.
- If you have suspected or diagnosed epilepsy, you should follow precautions recommended by your physician.
- Use caution if you have a tendency to bleed internally, such as following an injury of fracture.
- Consult with your physician prior to using the device after a recent surgical procedure, because stimulation may disrupt the healing process.
- Use caution if stimulation is applied over the menstruation or pregnant uterus; and
- Use caution if stimulation is applied over areas of skin that lack normal sensation.
- For single patient use only.
- Keep yourself informed of the contraindications.
- This stimulator is never use by patients who have noncompliant, emotionally disturbed, dementia, or low IQ.
- The instruction of use was listed; any improper use may be dangerous.
- Caution should be used for patients with suspected or diagnosed heart problems.
- Isolated cases of skin irritation may occur at the site of the electrode placement following long-term application.
- Do not use this device at the same time as other equipment which sends electrical pulses to your body.
- Do not use sharp objects such as pencil point or ballpoint pen to operate the buttons

on the control panel.

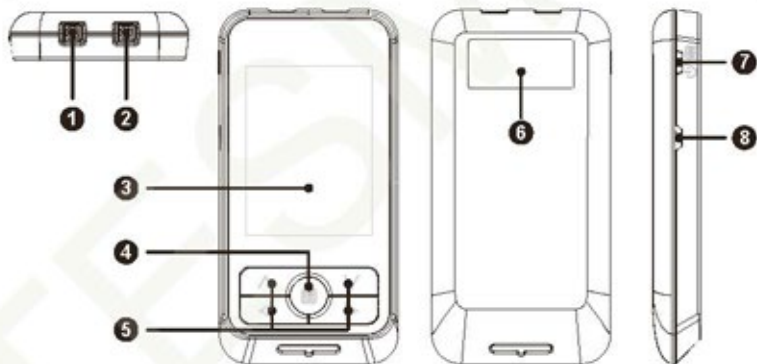
- To check the electrode connections before each use.
- Electrical stimulators should be used only with the electrodes recommended for use by the manufacturer.

## 2.5 Adverse Reactions

- Possible skin irritation or electrode burn under the electrodes may occur.
- On very rare occasions, first-time users of EMS have reported feeling light-headed or faint. We recommend that you use the product while seated until you become accustomed to the sensation.
- If the stimulation levels are uncomfortable or become uncomfortable, reduce the stimulation intensity to a comfortable level and contact your physician if problems persist.

## 3. GETTING TO KNOW YOUR DEVICE

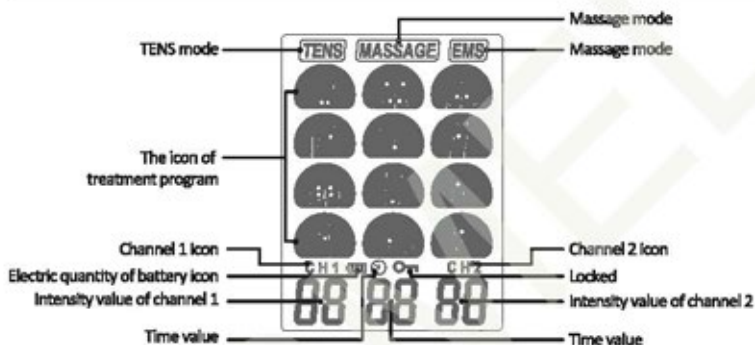
### 3.1 Front and Rear Panel

1. Channel 1 output socket: electric signal output of channel 1 after connection of the cable with adhesive electrodes.
2. Channel 2 output socket: electric signal output of channel 2 after connection of the cable with adhesive electrodes.
3. LCD display: Display the user parameters and information.
4. Press the [ ] button to select the treatment program.
5. Press the [ ] button to increase output intensity of channel 1 or channel 2, press the [ ]

- ✓ ] to button decrease output intensity of channel 1 or channel 2.
- Label
  - Press the [ ] button of turn on or turn off the unit, and in the standby mode press can be select the treatment mode.
  - USB charging socket

### 3.2 LCD DISPLAY

### 3.3 Accessories

No.	DESCRIPTION		Part No.	QTY	Remark
1	Stimulator unit	FDES116	116-01	1	Type BF
2	Electrode cables	L=1.2M, black	116-02-12B	2	
3	USB cable	L=0.2M, black, Micro 5 Pin	116-03-M02B	1	
4	Electrode pad	50*50mm, Adhesive Electrodes	116-04-5050U	4	
5	User manual	FDES116 V1.0	116-05	1	

## 4. SPECIFICATION

### 4.1 Technical information

Device name	New stimu
Model	FDES116

Power supply	DC3.7V/400mAh Polymer lithium battery (Built-in)
Output channel	Dual, isolated between output channels
Wave sharp	Bi-phase square-wave pulse
Output voltage	Max. 40Vpp (at 500ohm load)
Output current	Max. 80mA (at 500ohm load)
DC capacity of output	0
Treatment time	10-30 minutes
Output intensity	0 to 40 levels, adjustable
Treatment mode	TENS, EMS and Massage
Number of treatment program	TENS: 12 programs; EMS: 12 programs; Massage: 12 programs
Operating conditions	5°C to 40°C with a relative humidity of 15%-93%, atmospheric pressure from 700 hPa to 1060 hPa
Transportation and storage environment:	-25-55°C (-13-131°F); ≤95% RH; 700hPa – 1060hPa
Dimensions	110*60*16.2mm (L*W*H)
Weight	95g
Automatic shutoff	3min.
Electrode detection function	The amplitude level will be reset to 0 V, when the amplitude level is 1 or greater and an open circuit at either channel is detected.
Output precision	All the output parameters allow ±20% error for the specification.
Grade of waterproof	IP22
Electric shock	Internally powered ME equipment
Applied part	Type BF applied part, including the whole unit
Mode of operation	Continuous operation
Device service life	2 years (Normally)
<b>Note: Not intended to be sterilized.</b>	
<b>Not for use in an OXYGEN RICH ENVIRONMENT</b>	

#### 4.2 Technical specification for TENS mode

P.W. (pulse width)	80-300us
P.R. (Frequency)	1-150Hz (Hz=vibration per second)
Output characteristics	Constant Voltage (CV)

Program output type	Continues, Hans, Modulation, FM, IM and PM
Treatment time	30 minutes

#### 4.3 Technical specification for EMS mode

P.W. (pulse width)	180,200,250 and 300us
P.R. (Frequency)	2, 50, 65 and 75Hz (Hz=vibration per second)
Output characteristics	Constant Voltage (CV)
Program output type	Continues and IM
Treatment time	30 minutes
Contraction (on) time	1-10sec.
Relaxation (off) time	4-6sec.
Ramp up and ramp down time	2-5sec.

#### 4.4 Technical specification for Massage mode

P.W. (pulse width)	50-300us
P.R. (Frequency)	2-120Hz (Hz=vibration per second)
Output characteristics	Constant Voltage (CV)
Program output type	IM
Treatment time	10-30 minutes
Contraction (on) time	0, 7sec. or keep output
Relaxation (off) time	0, 1 and 3sec.
Ramp up time	0, 4-6sec.
Ramp down time	0-6sec.

## 5. OPERATION INSTRUCTION

### 5.1 Low battery and charging

---

#### 5.1.1 Check/Replace the battery

Press the [⏻] button to turn on the device at power off mode. If the battery is low battery (LCD showing  icon), connect the Micro USB cable to charging. In any mode, connect the Micro USB cable should be into charging mode and exit the current mode. When charging, the battery icon will be showing the LCD and flicker.

### 5.2 Connect electrodes to electrode cables

---

Insert the electrode cables into electrode pad connector (ad flow picture). Make sure they are properly connected to ensure the good performance.

### 5.3 Connect electrode cables to device

---

1. Before proceeding to this step, be sure the device is completely switch OFF;
2. Holding the insulated portion of the electrode cable connector, insert the plug into the receptacle on the top of the main device;
3. The device has two output channels at the top of the unit controlled by Channel 1 and Channel 2;
4. You may choose to use one channel with one pair of electrode cable or both channels with two pairs of electrode cables. Using both channels gives the user the advantage of stimulating two different areas at the same time.

**Caution:**

**Do not insert the plug of electrode cables into any AC power supply socket.**

### 5.4 Electrode

---

#### 5.4.1 Electrode options

The electrodes (electrode pad) are disposable and should be routinely replaced when they start to lose their adhesive nature. If you are unsure of your electrode adhesive properties, please order new replacement electrodes. Replacement electrodes should be re-ordered under the advice of your physician or the device manufacturer to ensure proper quality. Following

application procedures outlined on electrode packing when using the new replacement electrodes to maintain optimal stimulation and to prevent skin irritation.

**Caution:**

Always use the electrodes which comply with the requirements of the IEC/EN60601-1, ISO10993-1/-5/-10 and IEC/ EN60601-1-2, such as the electrodes with CE mark, or which are legally marketed in the US under 510(K) procedure.

#### 5.4.2 Place electrodes on skin

Place the electrode on the part you need treatment, and base on the instructions of this user manual. Before use, please make the skin that need treatment and the silica gel of electrode clean and humid, to prevent the discomfort that cause by dry skin. Please ensure the skin and electrode connection well.

**Caution:**

1. Please turn off to prevent electric shock when pre-moving the treatment position of the electrode.
2. Before applying the electrode, it is recommended to wash and degrease the skin, and then dry it.
3. Never remove the electrode from the skin while the stimulator is still treating.
4. Only use the electrodes that provided by the manufacturer, use other accessories could result in injury to the patient.
5. It is recommended that, at minimum 16mm<sup>2</sup> self-adhering electrodes are used.

### 5.5 Electrode placement

FDES116 is a kind of OTC stimulator, suitable for home use, you only need to use according to the user manual, place the electrode on the position where you feel pain and need to exercise, treating and adjusting base on your own feeling.

The applicable treatment part of each program was shown on LCD. You can follow up this indication for treatment.

Please refer to the electrode treatment, program and placement sheets from page 128 to page 160.

## 6. INSTRUCTIONS FOR USE

### 6.1 Turn on

When use for the first time, press the [  ] button to turn on the device. LCD will be full-display, into standby mode.

### 6.2 Select the treatment mode

You can press the [  ] button to switch select the treatment mode, at the standby mode. Each press can be switched the one treatment mode between the TENS, EMS and MASSAGE. The selected treatment mode icon will flash.

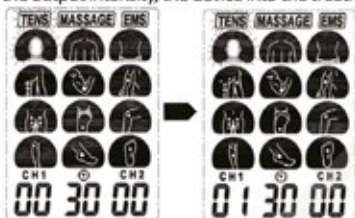
### 6.3 Select the application program

When you selected the treatment mode, press the [  ] button to switch select the application program. Each press can be switched the one application program between the 12 programs. The selected program icon will flash.

## 6.4 Starting the treatment

According the sub-clause 5.2 to 5.4 to connected the electrode and electrode cables. Press the [^] button to increase the output intensity, the device into the treatment mode.

The left [^] button to control the Channel 1 intensity, the right [^] button to control the Channel 2 intensity.

## 6.5 Adjust channel intensity

Place the electrodes stick the application of body parts, press the [^] button to increase output intensity. The output intensity is increasing a level in each press the [^] key, and the LCD will be displayed the output intensity. It has 40 levels output intensity. Please adjust the intensity in your feel comfortable condition.

If you fell the output intensity too strong, you can press [v] button to decrease the output intensity. The output intensity is decrease a level in each press the [v] button. When the output intensity decreased to zero, it will return the wait mode.

In treatment, if no further operation after 15 seconds, the operation system will be locked automatically, in order to prevent unintended wrong operation, and LCD will show the lock icon. You can press V button to unlock operation.

#### Caution:

If the stimulation levels are uncomfortable or become uncomfortable, reduce the stimulation intensity to a comfortable level and contact your medical practitioner if problems persist.

## 6.6 Turn the device off

At the wait mode, press the [  ] button for 2 sec. to turn off the device, and the LCD will be blank.

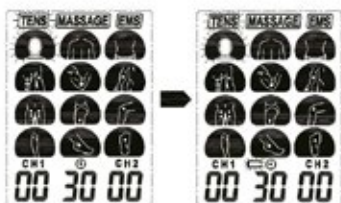
## 6.7 Load detect function

When the device haven't detect the load or the electrode with skin connect not good, it will automatically detect the load if the intensity is added to 1 level, if it is judged to no-load, the intensity will automatically return to zero and the device will return to wait mode.

## 6.8 Low battery indication

When the battery voltage less than d.c. 3.3V  $\pm$ 0.2V the low battery icon will be blink to showing the LCD display. In the status, you should be connect the micro USB cables to charging.

## 6.9 Charging

Any mode, if you connect the micro USB cables to charging, the LCD will be showing the charging process icon as follows:

### Caution:

1. In first use, please run out of battery power, then charging 12 hours.
2. If not use the product for a long time, sometimes can't switch on the device is normal, because the battery has run out. If you connected the charger, the interface will show the icon as picture of 6.6, that means the device is good, just don't have power.
3. If not use the product more than 6 months, please charge and discharge the device once a month at least to maintain the battery life.
4. Do not use the device during charging.
5. In order to ensure the product safety, please choose the charger approved by IEC60601-1 and IEC60601-1-2 or have UL MARK or GS certification.

## 6.10 Power saving function

If no button press within 3min. at wait mode, the device will be auto. into power saving mode.

## 6.11 Use of electrode

1. The electrode may only be connection with the New Stimu device. Make sure that the device is turned off when attaching or removing the adhesive electrode.
2. If you want to reposition the electrode during the application, turn the device off first.
3. The use of electrode may lead to skin irritations. If you experience such skin irritations, e.g. redness, blistering or itching, discontinue using the adhesive electrode. Do not use the New Stimu device permanently on the same body part, as this may also lead to skin irritations.
4. Electrode is personal belongings, just for single person use. Please note that prevent cross

use.

5. The electrode must connect entirely to the skin surface to prevent high local currents, which may lead to skin burns.
6. Do not use the adhesive electrodes for more than approx. 20 times, as contact between the electrodes and the skin deteriorates over time.
7. The adhesive force of the electrodes depends on the skin properties, storage, and the number of applications. If your adhesive electrodes no longer fully stick to the skin's surface, replace them with new electrodes. The adhesive electrodes must adhere entirely to the skin surface to prevent high local currents, which may lead to skin burns. Stick the adhesive electrodes back onto the protective foil after use and store them in the storage bag to prevent them from drying out. This retains the adhesive force for a longer period.

**Caution:**

1. Before applying the electrode, it is recommended to wash and degrease the skin, and then dry it.
2. Never remove the electrode from the skin while the device is still switched on.
3. Only use the electrode that provided by the manufacturer. Use other products could result in injury to the user.

## 6.12 Where do I attach adhesive electrode?

---

1. Each person reacts differently to electric nerve stimulation. Therefore, the placement of the electrodes may deviate from the standard. If application is not successful, contact your physician to find out which placement techniques are best for you.
2. Do not use any adhesive electrodes with an electrode size smaller than the original attachment electrode issue by manufacturer, otherwise the current density may be too high and cause injuries.
3. The size of the adhesive pads may not be changed, e.g. by dipping off parts of them.
4. Make sure that the region radiating the pain is enclosed by the electrodes. In case of painful muscle groups, attach the electrodes in such a way that the affected muscles are also enclosed by the electrodes.

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## **8. CLEANING AND MAINTENANCE**

Fully comply with the following daily maintenance requirements are necessary, to make sure your product operate intact and guarantee the product Long-term performance and safety.

### **8.1 Cleaning and caring for the device**

---

- 8.1.1 Clean the device with a soft, slightly damp cloth. In case of heavier dirt build-up, you may also add a mild detergent.
- 8.1.2 Do not subject the New Stimu to moisture or dampness. Do not hold the New Stimu under running water, do not submerge it in water or other liquids.
- 8.1.3 The New Stimu is sensitive to heat and may not be exposed to direct sunlight. Do not place the New Stimu on hot surfaces.
- 8.1.4 Carefully clean the surface of the adhesive electrode with a damp cloth. Make sure the device is turn off!
- 8.1.5 For reasons of hygiene, each user should use his/ her own set of electrode.
- 8.1.6 Do not use any chemical cleaners or abrasive agents for cleaning.
- 8.1.7 Ensure that no water penetrates into the machine. If this should ever happen, only use the device again once it is completely dry.

### **8.2 Cleaning and caring for the electrodes and electrode cables**

---

- 8.2.1 Clean the electrode cords by wiping them with damp cloth. Coating them lightly with talcum powder will reduce tangles and prolong the life.
- 8.2.2 After used the electrode pads, please keep them in the sealed bag properly. Improperly storage will lead to the gel moisture loss and viscosity reduced, breeding ground for bacteria.

### **8.3 Maintenance**

---

- 8.3.1 The manufacturer didn't authorize to any maintenance agencies abroad. If your device has any problem, please contact the distributor. The manufacturer will not be held responsible for the results of maintenance or repairs by unauthorized persons.
- 8.3.2 The user must not attempt any repairs to the device or any of its accessories. Please contact the retailer for repair.
- 8.3.3 Opening of the equipment by unauthorized agencies is not allowed and will terminate any claim to warranty.

Each product in manufacturing has been through the systematic validation. The performance is stable and did not need to undertake the performance parameters of calibration and validation. If your product can't reach the expected performance and the basic function have change in normal use, please contact the retailer.

## 9. TROUBLESHOOTING

If your device does not seem to be operating correctly, refer to the chart below to determine what may be wrong. If none of these measures can solve the problem, the device should be sent to service.

Problem	Possible Cause	Solution
Displays fail to light up	Battery electrical quantity is too low, can not turn on.	Connect the micro USB cables to charging.
Weak stimulation	Electrodes 1. Dried out or contaminated. 2. Placement electrode wires. 3. Old/worn/damaged	Replace and re-connect
Stimulation is uncomfortable	1. Intensity is too high. 2. Electrodes are too close Together. 3. Electrodes or lead wires are damaged or worn 4. Electrode active area size is too small.	1. Decrease intensity. 2. Reposition the electrodes. 3. Replace. 4. Replace electrodes with the one that have an active area no less than 25cm <sup>2</sup>
Intermittent output	Electrode wires  Program option in use	1. Verify connection is secure. Insure the connection is firm. 2. Turn down the intensity. Rotate electrode wires in socket 900. If still intermittent, replace electrode wires. 3. If still intermittent after replacing electrode wires, a component may have failed. Call the repair department. Some program is intended to stimulate in intermittent way. Refer to the Program Option Controls in the operation section to get the description of the program option.
Invalid stimulation	Electrodes and the application position is abnormal Unknown	Change electrodes and the application position. Contact your doctor

## 10. STORAGE

1. Store the device in a dry room and protect it against heat, sunshine and moisture.
2. Store the device in a cool, well-ventilated place.
3. Detach the connecting cable from the electrodes.
4. Never place any heavy objects on the device.

## 11. DISPOSAL

Spent batteries do not belong in the household waste. Dispose of the battery according to the current regulations. As a consumer, you are under obligation to dispose of batteries correctly. Consult your municipal authority or your dealer for information about disposal.

## 12. EMC DECLARATION

1. The New Stimu needs special precautions regarding EMC and needs to be installed and put into service according to the EMC information provided in the ACCOMPANYING DOCUMENTS.
2. Wireless communications equipment such as wireless home network devices, mobile phones, cordless telephones and their base stations, walkie-talkies can affect this equipment and should be kept at least a distance  $d = 3,3$  m away from the equipment.

**(Note: As indicated in Table 6 of IEC 60601-1-2:2007 for ME EQUIPMENT, a typical cell phone with a maximum output power of 2 W yields  $d = 3.3$  m at an IMMUNITY LEVEL of 3 V/m)**

## 13. Glossary of Symbols

	Serial number.....0000001
	Attention: Read the operating instruction for use!

	<p>Meaning of the symbols on the product, the packaging or in the operating instructions: electric devices are recyclable material and should not be disposed of with household waste after their useful life! Help us to protect the environment and save resources and take this device to the appropriate collection points. Please contact the organization which is responsible for waste disposal in your area if you have any questions.</p>
	<p>Type BF applied part</p>
	<p>Manufacturer information:</p>
<p><b>IP22</b></p>	<p>IP code of the device: this device's grade of against Ingress of solid foreign objects – <math>\geq 12.5</math>mm diameter (and the against access to hazardous parts with finger); the grade of waterproof is dripping (15° tilted)</p>
	<p>Date of manufacture</p>

## 14. SOFTWARE VERSION

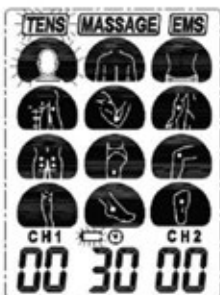
The device's software version is Ver. 1.0.

## 15. GUARANTEE

We grant you a lifetime guarantee after the date of purchase. Any damage caused by improper handling shall not be covered by the guarantee. Battery and packaging are also excluded from the guarantee. All other damage claims excluded. A guarantee claim must be submitted with the purchase receipt. Please pack your defective instrument well and send with sufficient postage to the distributor.

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**GB MAINTENANCE:** there are no maintenance instructions for certain treatments. Left up to the discretion of the Client.

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**GB BEAUTY, SPORT, FITNESS TREATMENTS** (Key: pag.63)

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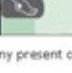
Trismed copyright - it is absolutely forbidden the partial or total reproduction of any present content

I GB D F E P	TRATTAMENTO MASSAGE TREATMENT MASSAGE BEHANDLUNG MASSAGE TRAITEMENT MASSAGE TRATAMIENTO MASSAGE TRATAMENTO MASSAGE	FIG.	TREATMENT	
			Prog.	Days
I GB D F E P	MASSAGGIO ADDOME MESSAGE: ABDOMEN MESSAGE BAUCH MESSAGE ABDOMEN MASAJE ABDOMEN MASSAGEM ABDOMEN	31	3 	30
I GB D F E P	MASSAGGIO ADDOME MESSAGE: ABDOMEN MESSAGE BAUCH MESSAGE ABDOMEN MASAJE ABDOMEN MASSAGEM ABDOMEN	32	3 	30
I GB D F E P	MASSAGGIO FIANCHI MESSAGE: SIDES MESSAGE HÜFTEN MESSAGE HANCHES MASAJE CADERAS MASSAGEM CINTURA	33	3 	30
I GB D F E P	MASSAGGIO FIANCHI MESSAGE: SIDES MESSAGE HÜFTEN MESSAGE HANCHES MASAJE CADERAS MASSAGEM CINTURA	34	8 	30
I GB D F E P	MASSAGGIO GLUTEI MESSAGE: GLUTEI MESSAGE GESÄSS MESSAGE MUSCLES FESSIERS MASAJE GLUTEOS MASSAGEM GLÚTEOS	35	7 	30
I GB D F E P	MASSAGGIO SPALLE E COLLO MESSAGE SHOULDERS AND NECK SCHULTER- UND HALSMASSAGE MESSAGE DE L'ÉPAULE ET DU COL MASAJE DE HOMBRO Y CUELLO MASSAGEM DE HOMBROS E DE PESCOÇO	61	1 	10
I GB D F E P	MASSAGGIO MUSCOLI DEL PIEDE MESSAGE MUSCLES OF THE FOOT MESSAGE DERMAL, RICE, EDEL R. JOES MESSAGE DES MUSCLES DUPIED MASAJE DE LOS MUSCULOS DEL PIE MASSAGEM DOS MUSCULOS DO PE	27	12 	10
I GB D F E P	MASSAGGIO BRACCIA MESSAGE ARMS ARMS MASSAGE MESSAGE DES BRAS MASAJE DE LOS BRAZOS MASSAGEM DOS BRACOS	2	5 	30
I GB D F E P	MASSAGGIO DIFERENZIALE ZONA CAVITA DIFFERENTIAL MASSAGE ANKLE ZONE DIFFERENTIAL MASSAGE ANKLE-ZONE MESSAGE DE LA ZONE DE LA CHEVILLE MASAJE DIFERENCIAL AREA DEL TODILLO MASSAGEM DIFERENCIAL ZONA DE TORNOZUELO	26	11 	5

Termed copyright - it is absolutely forbidden the partial or total reproduction of any present content

I GB D F E P	TRATTAMENTO MASSAGE TREATMENT MASSAGE BEHANDLUNG MASSAGE TRAITEMENT MASSAGE TRATAMIENTO MASSAGE TRATAMENTO MASSAGE	FIG.	TREATMENT	
			Prog.	Days
I MASSAGGIO GLUTEI GB MASSAGE: GLUTEI D MASSAGE GESÄSS F MASSAGE MUSCLES FESSIERS E MASAJE GLÚTEOS P MASSAGEM GLÚTEOS		36	7 	30
I MASSAGGIO GLUTEI GB MASSAGE: GLUTEI D MASSAGE GESÄSS F MASSAGE MUSCLES FESSIERS E MASAJE GLÚTEOS P MASSAGEM GLÚTEOS		37	7 	30
I MASSAGGIO GLUTEI E COSCIA GB MASSAGE: GLUTEI & THIGHS D MASSAGE GESÄSS - SCHENKEL F MASSAGE MUSCLES FESSIERS-CUISSSE E MASAJE GLÚTEOS - MUSLO P MASSAGEM GLÚTEOS E COXA		38	7 	30
I MASSAGGIO INTERNO COSCIA GB MASSAGE: INSIDE THIGHS D MASSAGE SCHENKEL INNENSEITEN F MASSAGE CUISSSES INTERNES E MASAJE INTERIORES MUSLOS P MASSAGEM INTERNO COXA		39	9 	30
I MASSAGGIO INTERNO COSCIA GB MASSAGE: INSIDE THIGHS D MASSAGE SCHENKEL INNENSEITEN F MASSAGE CUISSSES INTERNES E MASAJE INTERIORES MUSLOS P MASSAGEM INTERNO COXA		40	9 	30
I MASSAGGIO ESTERNO COSCIA GB MASSAGE: OUTSIDE THIGHS D MASSAGE SCHENKEL AUSSEN F MASSAGE CUISSSES EXTERNE E MASAJE EXTERIOR MUSLOS P MASSAGEM ESTERNO COXA		41	7 	30
I MASSAGGIO ESTERNO COSCIA GB MASSAGE: OUTSIDE THIGHS D MASSAGE SCHENKEL AUSSEN F MASSAGE CUISSSES EXTERNE E MASAJE EXTERIOR MUSLOS P MASSAGEM ESTERNO COXA		34	8 	30

Termed copyright - it is absolutely forbidden the partial or total reproduction of any present content

I GB D F E P	TRATTAMENTO MASSAGE TREATMENT MASSAGE BEHANDLUNG MASSAGE TRAITEMENT MASSAGE TRATAMIENTO MASSAGE TRATAMENTO MASSAGE	FIG.	TREATMENT	
			Prog.	Days
I GB D F E P	MASSAGGIO ESTERNO COSCIA MESSAGE: OUTSIDE THIGHS MESSAGE SCHENKEL AUSSEN MESSAGE CUISSES EXTERNE MASAJE EXTERIOR MUSLOS MASSAGEM ESTERNO COXA	42	9 	30
I GB D F E P	MASSAGGIO PETTORALI UOMO MESSAGE: PECTORALS MAN MESSAGE BRUSTMUSKELN MANN MESSAGE PECTORAUX HOMME MASAJE PECTORAL HOMBRE MASSAGEM PEITORAIS MASCULINO	43	4 	30
I GB D F E P	MASSAGGIO BRACCIO UOMO MESSAGE: ARM MAN MESSAGE ARME MANN MESSAGE BRAS HOMME MASAJE BRAZO HOMBRE MASSAGEM BRAÇO MASCULINO	44	2 	30
I GB D F E P	MASSAGGIO AVAMBRACCIO UOMO MESSAGE: FOREARM MAN MESSAGE UNTERARM MANN MESSAGE AVANT-BRAS HOMME MASAJE ANTEBRAZO HOMBRE MASSAGEM ANTEBRAÇO MASCULINO	45	5 	30
I GB D F E P	MASSAGGIO DENTATI MESSAGE: SERRATUS MESSAGE DENTAMUSKELN MESSAGE MUSCLES DENTELES MASAJE SERRATOSO MASSAGEM DENTEADOS	46	4 	30
I GB D F E P	MASSAGGIO FIANCHI UOMO MESSAGE: SIDES MAN MESSAGE HÜFTEN MANN MESSAGE HANCHES HOMME MASAJE CADERAS HOMBRE MASSAGEM CINTURA MASCULINO	47	4 	30
I GB D F E P	MASSAGGIO ADDUTTORI UOMO MESSAGE: ADDUCTORS MAN MESSAGE ADUKTOREN MANN MESSAGE MUSCLES ADDUCTEURS HOMME MASAJE ABDUCTORES HOMBRE MASSAGEM ADUTORES MASCULINO	48	9 	30
I GB D F E P	MASSAGGIO QUADRICIPITI UOMO MESSAGE: QUADRICEPS MAN MESSAGE QUADRIZEPS MANN MESSAGE QUADRICEPS HOMME MASAJE CUADRICEPS HOMBRE MASSAGEM QUADRICEPS MASCULINO	49	9 	30
I GB D F E P	MASSAGGIO GAMBE CICLISMO UOMO DONNA MESSAGE: CYCLING LEGS MAN WOMAN MESSAGE BEINE RADSPORT MANN FRAU MESSAGE JAMBES CYCLISME HOMME FEMME MASAJE PIERNAS CICLISMO HOMBRE MUJER MASSAGEM PERNAS CICLISMO MASCULINO FEMININO	50	10 	30

I GB D F E P	TRATTAMENTO MASSAGE TREATMENT MASSAGE BEHANDLUNG MASSAGE TRAITEMENT MASSAGE TRATAMIENTO MASSAGE TRATAMENTO MASSAGE	FIG.	TREATMENT	
			Prog.	Days
I GB D F E P	I MASSAGGIO GAMBE CICLISMO UOMO DONNA MESSAGE: CYCLING LEGS MAN WOMAN D MESSAGE BEINE RADSPORT MANN FRAU F MESSAGE JAMBES CYCLISME HOMME FEMME E MASAJE PIERNAS CICLISMO HOMBRE MUJER P MASSAGEM PERNAS CICLISMO MASCULINO FEMININO	51	9 	30
I GB D F E P	I MASSAGGIO GAMBE CICLISMO UOMO DONNA MESSAGE: CYCLING LEGS MAN WOMAN D MESSAGE BEINE RADSPORT MANN FRAU F MESSAGE JAMBES CYCLISME HOMME FEMME E MASAJE PIERNAS CICLISMO HOMBRE MUJER P MASSAGEM PERNAS CICLISMO MASCULINO FEMININO	52	9 	30
I GB D F E P	I MASSAGGIO GAMBE PALLAVOLO UOMO DONNA MESSAGE: VOLLEYBALL LEGS MAN WOMAN D MESSAGE BEINE HANDBALL MANN FRAU F MESSAGE JAMBES VOLLEY-BALL HOMME FEMME E MASAJE PIERNAS VOLEIBOL HOMBRE MUJER P MASSAGEM PERNAS VOLEI MASCULINO FEMININO	53	10 	30
I GB D F E P	I TONIFICAZIONE GAMBE SCI UOMO DONNA TONE-UP: SKIING LEGS MAN WOMAN D STÄRKUNG BEINE SKISPORT MANN FRAU F TONIFICATION JAMBES SKI HOMME FEMME E TONIFICACION PIERNAS ESQUI HOMBRE MUJER P TONIFICACAO PERNAS SKI MASCULINO FEMININO	54	10 	30
I GB D F E P	I MASSAGGIO GAMBE SCI UOMO DONNA MESSAGE: SKIING LEGS MAN WOMAN D MESSAGE BEINE SKISPORT MANN FRAU F MESSAGE JAMBES SKI HOMME FEMME E MASAJE PIERNAS ESQUI HOMBRE MUJER P MASSAGEM PERNAS SKI MASCULINO FEMININO	55	9 	30
I GB D F E P	I MASSAGGIO GAMBE SCI UOMO DONNA MESSAGE: SKIING LEGS MAN WOMAN D MESSAGE BEINE SKISPORT MANN FRAU F MESSAGE JAMBES SKI HOMME FEMME E MASAJE PIERNAS ESQUI HOMBRE MUJER P MASSAGEM PERNAS SKI MASCULINO FEMININO	56	8 	30
I GB D F E P	I MASSAGGIO GAMBE TENNIS UOMO DONNA MESSAGE: TENNIS LEGS MAN WOMAN D MESSAGE BEINE TENNISSPORT MANN FRAU F MESSAGE JAMBES TENNIS HOMME FEMME E MASAJE PIERNAS TENIS HOMBRE MUJER P MASSAGEM PERNAS TENIS MASCULINO FEMININO	57	9 	30
I GB D F E P	I MASSAGGIO MUSCOLI FLESSORI UOMO MESSAGE: FLEXORS MAN D MESSAGE BEUGEMUSKELN MANN F MESSAGE MUSCLES FLÉCHISSEURS HOMME E MASAJE MÚSCULOS FLEXORES HOMBRE P MASSAGEM MÚSCULOS FLEXORES MASCULINO	58	9 	30
I GB D F E P	I MASSAGGIO GINOCCHIO MESSAGE: KNEE D MESSAGE KNIE F MESSAGE GENOU E MASAJE RODILLA P MASSAGEM JOELHO	59	8 	30

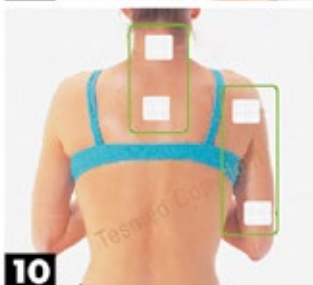
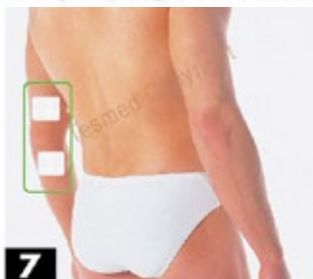
<b>I TRATTAMENTO</b> MASSAGE <b>GB TREATMENT</b> MASSAGE <b>D BEHANDLUNG</b> MASSAGE <b>F TRAITEMENT</b> MASSAGE <b>E TRATAMIENTO</b> MASSAGE <b>P TRATAMENTO</b> MASSAGE	<b>FIG.</b>	<b>TREATMENT</b> <b>Prog. Days</b>	
<b>I MASSAGGIO DEFATICANTE</b> <b>GB RELAXING MASSAGE</b> <b>D ERHOLUNGSMASSAGE</b> <b>F MASSAGE DEFATIGANT</b> <b>E MASAJE RELAJANTE</b> <b>P MASSAGEM RELAXANTE</b>	<b>60</b>	<b>10</b> 	<b>20</b>
<b>I MASSAGGIO DEFATICANTE SPALLE</b> <b>GB RELAXING MASSAGE: SHOULDERS</b> <b>D ERHOLUNGSMASSAGE SCHULTERN</b> <b>F MASSAGE DEFATIGANT EPAULES</b> <b>E MASAJE RELAJANTE CERVICAL Y HOMBROS</b> <b>P MASSAGEM RELAXANTE COSTAS</b>	<b>61</b>	<b>2</b> 	<b>30</b>
<b>I MASSAGGIO SPORT SCI UOMO DONNA</b> <b>GB MESSAGE SPORT: SKIING MAN WOMAN</b> <b>D MESSAGE SKISPORT MANN FRAU</b> <b>F MESSAGE SPORT SKI HOMME FEMME</b> <b>E MASAJE DEPORTE ESQUI HOMBRE MUJER</b> <b>P MASSAGEM ESPORTE SKI MASCULINO-FEMININO</b>	<b>62</b>	<b>2</b> 	<b>30</b>
<b>I MASSAGGIO SPORT PALLAVOLO PALLACANESTRO UOMO DONNA</b> <b>GB MESSAGE SPORT: VOLLEYBALL / BASKETBALL MAN WOMAN</b> <b>D MESSAGE HANDBALL BASKETBALL SPORT MANN FRAU</b> <b>F MESSAGE VOLLEY-BALL BASKET HOMME FEMME</b> <b>E MASAJE VOLEIBOL BALONCESTO HOMBRE MUJER</b> <b>P MASSAGEM ESPORTE VOLEI BASQUETE MASCULINO - FEMININO</b>	<b>63</b>	<b>10</b> 	<b>30</b>
<b>I MASSAGGIO SPORT PALLACANESTRO DONNA</b> <b>GB MESSAGE SPORT: BASKETBALL WOMAN</b> <b>D MESSAGE BASKETBALL SPORT FRAU</b> <b>F MESSAGE SPORT BASKET FEMME</b> <b>E MASAJE DEPORTE BALONCESTO MUJER</b> <b>P MASSAGEM ESPORTE BASQUETE FEMININO</b>	<b>64</b>	<b>2</b> 	<b>30</b>
<b>I MASSAGGIO SPORT PALLAVOLO DONNA</b> <b>GB MESSAGE SPORT: VOLLEYBALL WOMAN</b> <b>D MESSAGE SPORT HANDBALL FRAU</b> <b>F MESSAGE SPORT VOLLEY-BALL FEMME</b> <b>E MASAJE DEPORTE VOLEIBOL MUJER</b> <b>P MASSAGEM ESPORTE VOLEI FEMININO</b>	<b>65</b>	<b>2</b> 	<b>30</b>
<b>I MASSAGGIO SPORT PALLACANESTRO UOMO</b> <b>GB MESSAGE SPORT: BASKETBALL MAN</b> <b>D MESSAGE BASKETBALL SPORT MANN</b> <b>F MESSAGE SPORT BASKET HOMME</b> <b>E MASAJE DEPORTE BALONCESTO HOMBRE</b> <b>P MASSAGEM ESPORTE BASQUETE MASCULINO</b>	<b>66</b>	<b>2</b> 	<b>30</b>
<b>I MASSAGGIO SPORT PALLAVOLO UOMO</b> <b>GB MESSAGE SPORT: VOLLEYBALL MAN</b> <b>D MESSAGE SPORT HANDBALL MANN</b> <b>F MESSAGE SPORT VOLLEY-BALL HOMME</b> <b>E MASAJE DEPORTE VOLEIBOL HOMBRE</b> <b>P MASSAGEM ESPORTE VOLEI MASCULINO</b>	<b>67</b>	<b>2</b> 	<b>30</b>

- I** Posizionamento elettrodi
- GB** Positioning of electrodes
- D** Positionierung der Elektroden
- F** Positionnement électrodes
- E** Colocación de los electrodos
- P** Figura posição dos eletrodos

Termed copyright - it is absolutely forbidden the partial or total reproduction of any present content

- I** Posizionamento elettrodi  
**GB** Positioning of electrodes  
**D** Positionierung der Elektroden  
**F** Positionnement électrodes  
**E** Colocación de los electrodos  
**P** Figura posição dos eletrodos

Tesmed copyright - It is absolutely forbidden the partial or total reproduction of any present content

- I** Posizionamento elettrodi
- GB** Positioning of electrodes
- D** Positionierung der Elektroden
- F** Positionnement électrodes
- E** Colocación de los electrodos
- P** Figura posição dos eletrodos

- I** Posizionamento elettrodi  
**GB** Positioning of electrodes  
**D** Positionierung der Elektroden  
**F** Positionnement électrodes  
**E** Colocación de los electrodos  
**P** Figura posição dos eletrodos

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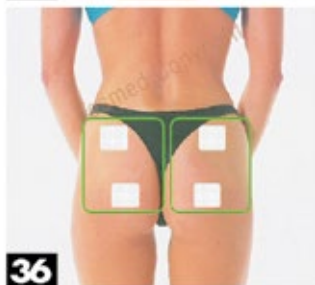
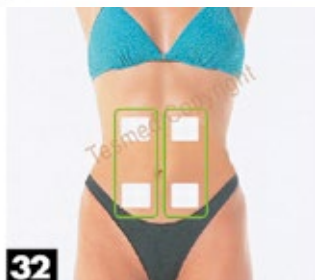
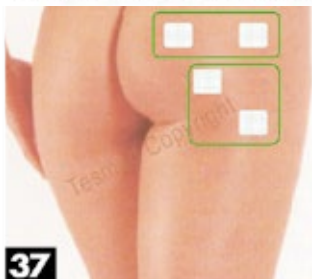
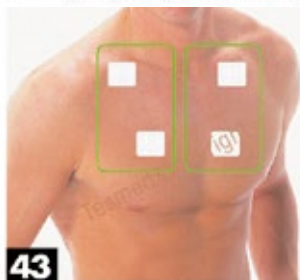
Termed copyright - it is absolutely forbidden the partial or total reproduction of any present content

- I** Posizionamento elettrodi
- GB** Positioning of electrodes
- D** Positionierung der Elektroden
- F** Positionnement électrodes
- E** Colocación de los electrodos
- P** Figura posição dos eletrodos

Termed copyright - it is absolutely forbidden the partial or total reproduction of any present content

- I** Posizionamento elettrodi  
**GB** Positioning of electrodes  
**D** Positionierung der Elektroden  
**F** Positionnement électrodes  
**E** Colocación de los electrodos  
**P** Figura posição dos eletrodos

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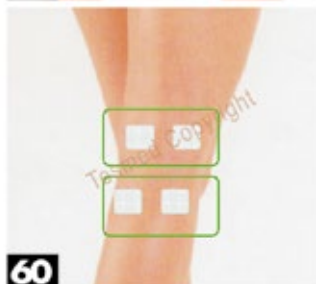
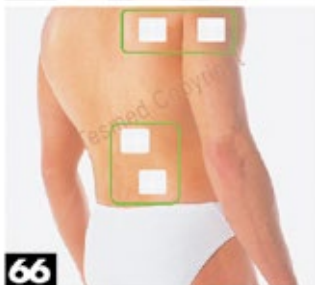
Teamed copyright - it is absolutely forbidden the partial or total reproduction of any present content

- I** Posizionamento elettrodi  
**GB** Positioning of electrodes  
**D** Positionierung der Elektroden  
**F** Positionnement électrodes  
**E** Colocación de los electrodos  
**P** Figura posição dos eletrodos

Teomed copyright - it is absolutely forbidden the partial or total reproduction of any present content

- I** Posizionamento elettrodi
- GB** Positioning of electrodes
- D** Positionierung der Elektroden
- F** Positionnement électrodes
- E** Colocación de los electrodos
- P** Figura posição dos eletrodos

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Famidoc Technology Co., Ltd.

Add: No. 212 Yilong Road, Hexi Industrial Zone, Jinxia, Changan Town,  
Dongguan, Guangdong Province, China

EC REP

Shanghai International Holding Corp. GmbH (Europe)

Add: Eiffestrasse 80, 20537 Hamburg Germany

[www.tesmed.com](http://www.tesmed.com)  
[info@tesmed.com](mailto:info@tesmed.com)