



The Turtle Measuring Tape

Perfect fits help fight and correct Flat Head Syndrome!

The Turtle Air and Turtle Lucky Elephant are FDA-cleared medical devices.

What does that mean?

Only one baby can wear the beanie, so if it's returned, it cannot be resold. Make sure you get the right size for your baby by using this handy head measuring tape!

HEAD'S UP!

Make sure you print your Turtle Tape at "Actual Size" and be sure to check the accuracy of your measuring tape with a ruler as certain printer settings may cause the tape to print at sizes other than intended.

What You'll Need:



Printer



Scissors



Tape

How to Use:

- 01 — **Print** your Turtle Measuring Tape.
- 02 — **Cut** along the dotted lines.
- 03 — **Align** the two strips side by side and attach them using clear tape.
- 04 — For an accurate measurement of your baby's head, place the Turtle Tape across your baby's forehead. Then, measure around the full circumference of your baby's head.

Sizing Chart

Measure at an angle with the ends of the tape meeting at the base of your baby's head, right above the neck.

Small: 12"-14"

Medium: 14"-16"

Large: 16"-18"

