

How to Attach Nasal Cannula



1. Remove one pair of velcro cannula keys from package.
2. Use the loop side of velcro and wrap around the cannula, threading the tail end through the opening.
3. Tighten cannula keys around the tubing and using the tension on both ends of the cannula key, attach to the Midliner Max at the lower edge of the strap and in front of the ear.

Velcro keys can be used to secure other devices, such as feeding tubes, to the Midliner Max.

How to Reposition with nCPAP or Cannula in Place



Side Lying:

- Respiratory devices should be attached while in midline. Then, log roll infant gently on the bed surface while simultaneously tucking the lower roll under the beanie. (Both rolls will then be positioned posteriorly.)
- Secure rolls together with velcro back straps included in package.
- This helps counterbalance the weight of the nCPAP.



Prone:

- Begin with baby in side lying.
- Turn baby's body gently into prone, keeping the head turned on the side and both rolls remaining posterior.
- A side positioning body roll is helpful to allow baby to wrap around as shown.



Tortle Products LLC

5460 S. Quebec Street, Suite 150, Greenwood Village, CO 80111 | tortle.com

For questions or more information, call 1-720-428-2147 or email customerservice@tortle.com.



SMALL

Head
Circumference
34cm - 42cm



Midliner Max

FP

Infant Head Positioner

Supports early infant development and cooling therapy.
Excellent head support for the infant with neonatal abstinence syndrome.
Helps stabilize patient during procedures such as retinal exams and line placement.
Helps cranial and neck asymmetry.
Attaches to most respiratory devices.

About Your Midliner Max, FP

- Quantity: One
- Cotton Blend with Nylon/Lycra & Foam
- Single Patient, Multiple Use
- Non-Prescription
- Latex-Free
- Phthalates-Free
- Formaldehyde-Free
- X-Ray Friendly
- Hand Wash, Air Dry



Manufactured for:
Tortle Products LLC
5460 S. Quebec St., Ste 150
Greenwood Village, CO 80111
720-428-2147

CEPartner4U
Esdoornlaan 13
3951 DB Maarn
Netherlands



Reorder #
65931-FP

Made in China
LOT| 2022-3
Rev 06/2022

SKU# 65931-FP



(01)10819245011316

Patent # 8,096,304, Additional US and International patents pending

How to Correctly Place Turtle Midliner Max, FP on an Infant




1

Position baby in supine midline. Open front of the Midliner Max and slide it under baby's head as shown.



2a

Make sure the support rolls are almost touching baby's shoulders.

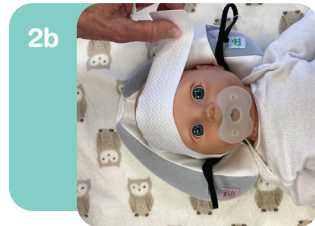

2b

To close Midliner Max straps, gently stretch from the base, and contour around the ears and up over the brows.

Testing Tightness & Slippage

Open the velcro front tab. If it recoils, close strap after tension is released. Pull the support rolls upward from the bottom to ensure a snug fit for baby.

Chin Strap Accessory



Using the central perforated opening, fit over the bony prominence of the mandible and then attach chin strap to the Midliner Max using the velcro attachments, bilaterally, as shown.

Designed to support baby's chin and close mouth, especially when receiving positive pressure from respiratory devices.

How to Attach Turtle Midliner Max, FP Using Toploader & Foam Block System



1

Place the black Toploader, velcro side down, on the front of Midliner Max. (Over baby's forehead area.)



2

Place interface with foam block centrally on Toploader strap. Secure in place by closing black strap over block and using velcro as shown.



3

Adjust interface so that nasal prongs, or mask, fit correctly with baby's nares.

Taking the plastic D-rings located on the black side straps, secure onto the blue slide attached to the interface. Fold the black strap onto itself, attaching the Midliner Max with the velcro tab while adjusting in place. Repeat on the other side.

BiliBans (Eye Shades) Accessory



- These soft eye shades can be used anytime to protect baby's eyes, even when using nCPAP, by wearing under interface.
- Place gently over baby's eyes and attach to the Midliner Max as shown using the small velcro tabs on either side.
- Shades can be reused multiple times on single patient.