


HIV 1 & 2

Rapid Blood Test

FEATURING

Touch-to-Test Technology

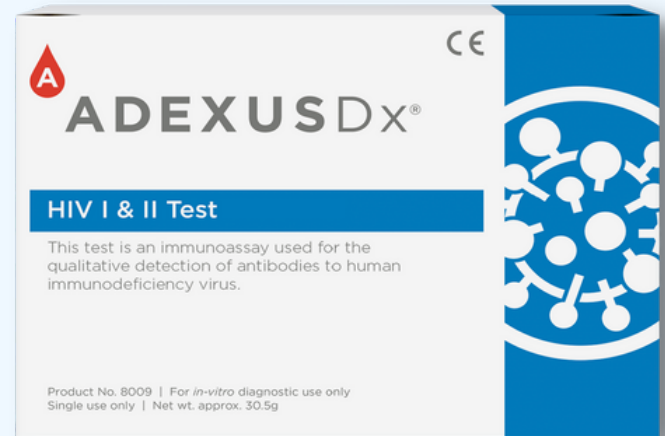

WHAT IS THE ADEXUSDx[®] HIV 1 & 2 Test?

The ADEXUSDx[®] HIV 1/2 Test is an immunoassay used for the qualitative detection of antibodies to human immunodeficiency virus in whole blood.

Human immunodeficiency virus is a slow replicating retrovirus that causes acquired immunodeficiency syndrome (AIDS). HIV-1 is more prevalent within North America and in most other regions worldwide. However, HIV2 has not been widely seen outside of Africa. There are a number of tests that are used to determine whether a person is infected with HIV. These include the HIV antibody test, P24 antigen test and PCR test for HIV RNA or DNA. There are other tests that are used to measure the severity or monitor therapy of HIV infection such as CD4 count and the viral load test. HIV antibody detection based rapid test is the most appropriate test for routine diagnosis of HIV infection among adults. Most people develop detectable HIV antibodies within 6 to 12 weeks of infection. The 'window period' is a term used to describe the period of time between HIV infection and the production of antibodies. During this time, an antibody test may give a 'false negative' result for HIV infection, which means the test will be negative, even though a person is infected with HIV.

Simple & Accurate

Features & Benefits:

- Used capillary or venous whole blood, plasma or serum
- Store at room temperature
- Self-contained

Not Needed:

- External reagents
- Lab equipment or readers
- In demand ancillary items
- Refrigeration

Learn more at nowdx.com
phone 479.966.4560 | website nowdx.com

Administering the ADEXUSDx[®] HIV 1 & 2 Test*

FINGER STICK METHOD

1. Perform finger stick.



2. Touch the drop of blood to the tip of the test.




3. Set timer for 15 minutes and read result.



TRANSFER DEVICE METHOD

1. Collect 40 µl of sample using a liquid transfer device.

2. Apply sample to the test.

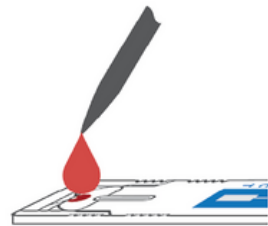

3. Set timer for 15 minutes and read result.






INTERPRETATION OF RESULTS




Negative Result



Positive Result

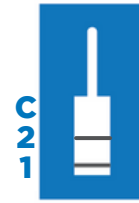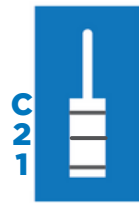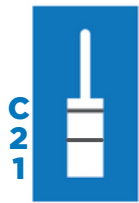

Invalid Result

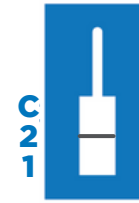

* For complete instructions, refer to the Product Insert before administering the test.

Visit nowdx.com for more information.

Description

ADEXUSDx[®] HIV 1 & 2 Test, Box

ADEXUSDx[®] HIV 1 & 2 Test, Box

Quantity

25

50

Catalog Number

8018-25

8018-50



1200 Stewart Place | Springdale AR 72764
phone 479.966.4560 | website nowdx.com

