


hCG


Rapid Blood Test & Control Set

FEATURING
Touch-to-Test
Technology

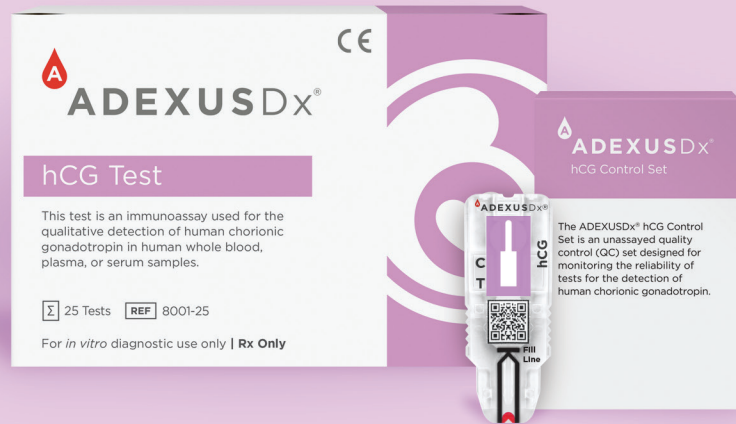

WHAT IS THE ADEXUSDx[®] hCG Test?

The ADEXUSDx[®] hCG Test is a rapid blood-based immunoassay that can detect pregnancy earlier than urine-based tests. It provides in-field, lab accurate results within 10 minutes, using as little as a drop of blood.

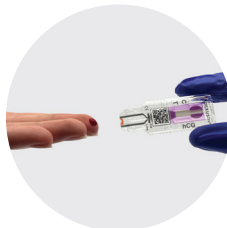

WHAT IS THE ADEXUSDx[®] hCG Control Set?

The ADEXUSDx[®] hCG Control Set is an unassayed quality control (QC) set designed for monitoring the reliability of tests for the detection of human chorionic gonadotropin.



Simple, Accurate & Low Cost

Features & Benefits:

- >99% sensitivity and specificity
- Fewer false negatives
- Uses capillary or venous whole blood, plasma or serum
- Store at room temperature
- Self-contained and portable
- 10mIU/mL detection


Not Needed:

- External reagents
- Lab equipment or readers
- Waiting on urine samples

Learn more at nowdx.com


phone 479.966.4530 | website nowdx.com

Administering the ADEXUSDx[®] hCG Test*

FDA Cleared | 



Transfer Device Method

1. Collect 40 µL of sample using a liquid transfer device.
2. Apply sample to the test.
3. Set timer for 10 minutes and read result.

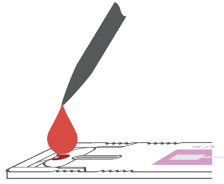

Finger Stick Method

1. Perform finger stick.
2. Touch the drop of blood to the tip of the test.
3. Set timer for 10 minutes and read result.

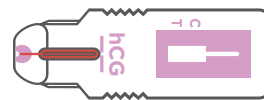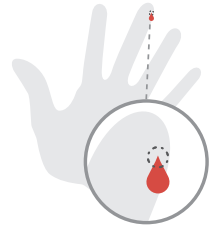

Interpretation of Results


A positive result has TWO LINES, one at the “C” (Control Line) and one at the “T” (Test Line). The intensity of the lines may vary. Look carefully as lines might be faint.



Positive Result



Negative Result

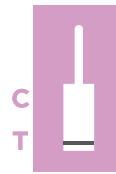

Invalid Result



Invalid Result

* For complete instructions, refer to the Product Insert before administering the test.

Visit nowdx.com for more information.

Description	Quantity	Catalog Number
ADEXUSDx [®] hCG Test, Box	25	8001-25
ADEXUSDx [®] hCG Test, Box	50	8001-50
ADEXUSDx [®] hCG Test, Boxes of 25, Case	1000	8001-1000-25
ADEXUSDx [®] hCG Test, Boxes of 50, Case	1000	8001-1000-50
ADEXUSDx [®] hCG Control Set, Box	1	401.009

 **NOWDiagnostics[®]**

1200 Stewart Place | Springdale AR 72764
phone 479.966.4530 | website nowdx.com

