

Syphilis Test & Control Set

FEATURING

Revolutionary Lateral
Flow Technology



WHAT IS THE ADEXUSDx® Syphilis Test?

The ADEXUSDx® Syphilis Test is a rapid immunochromatographic assay used for the qualitative detection of the presence or absence of antibodies to syphilis in human whole blood, plasma, or serum to aid in the diagnosis of *Treponema pallidum*.



WHAT IS THE ADEXUSDx® Syphilis Control Set?

The ADEXUSDx® Syphilis Control Set is an unassayed quality control (QC) set designed for monitoring the reliability of tests for the detection of antibodies to syphilis.



Simple, Accurate & Low Cost

Features & Benefits:

- High sensitivity and specificity
- Detected syphilis as early or earlier than 12 other tests in seroconversion panel

Not Needed:

- Buffers or diluents
- In demand ancillary items
- Refrigeration


[Learn more at nowdx.com](http://nowdx.com)

phone 479.966.4530 | [website nowdx.com](http://website.nowdx.com)

Administering the ADEXUSDx[®] Syphilis Test*


1

Collect 40 µL of sample using a transfer device.

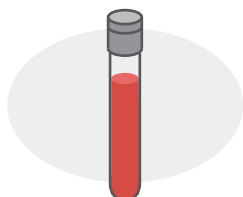

2

Apply sample to the test.

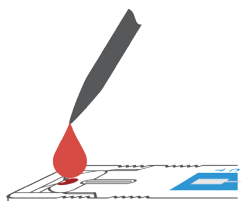


3

Set timer for 15 minutes and read result.



Interpretation of Results

A positive result has TWO LINES, one at the “C” (Control Line) and one at the “T” (Test Line). The intensity of the lines may vary. Look carefully as **lines might be faint.**

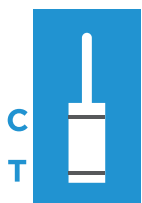

Positive Result



Negative Result



Invalid Result



Invalid Result

* For complete instructions, refer to the Package Insert before administering the test.

Visit nowdx.com for more information.

Description

ADEXUSDx[®] Syphilis Test, Box

Quantity

25

Catalog Number

8057-25

ADEXUSDx[®] Syphilis Test, Box

50

8057-50

ADEXUSDx[®] Syphilis Test, Boxes of 25, Case

1000

8057-1000-25

ADEXUSDx[®] Syphilis Test, Boxes of 50, Case

1000

8057-1000-50

ADEXUSDx[®] Syphilis Control Set, Box

1

457.006

 **NOWDiagnostics[®]**

1200 Stewart Place | Springdale AR 72764
phone 479.966.4530 | website nowdx.com



**MADE
IN THE
USA**