MENSTRUAL CUP GUIDE

Before you start

- Always wash your menstrual cup and hands before and after use with warm water and unscented soap and rinse thoroughly. Before the first use we recommend to sterilize the cup in boiling water for 3-5 minutes.
- Make sure you relax your muscles when you insert the menstrual cup.
- For easy insertion you can use a waterbased lubricant.

Folding and inserting

- Before inserting you have to fold the menstrual cup. There are many different ways to fold, most common is the C-shape.
- Insert the folded cup into the vagina. Inside the vagina, the menstrual cup unfolds automatically and rests against your vagina wall. You can push the menstrual cup further till it's completely inside, forming a seal to prevent leaking.
- When the menstrual cup is in place, turn gently the base of the cup. When the menstrual cup rotates properly, it is properly unfolded.

Good to know!

- There is no need to push the menstrual cup as far as a tampon.
- When the stem is too long, you can trim a small amount of stem by using scissors. Not while inserted!

Removing

- You can leave a menstrual cup on for up to 12 hours. This means that you only need to change the menstrual cup in the morning and evening. Except for heavy fluids. A menstrual cup can contain more liquid than 3 to 5 regular tampons.
- Always wash your hands before and after use with warm water and a mild soap and rinse thoroughly.
- Make sure you relax your muscles when you remove the menstrual cup.
- If you have not cut it off completely, you can use the stem to gently pull the menstrual cup slightly lower, so you can reach the base, but DO NOT use the stem to pull out your menstrual cup!
- Squeeze the bottom of the menstrual cup, to release the suction seal. When the suction seal is released you can gently pull the menstrual cup out.
- After removal, you empty the collected fluid into the toilet, or other suitable place. Rinse the menstrual cup with water and unscented soap before you re-insert the cup.

Store

At the end of your period, rinse the menstrual cup with warm water and unscented soap. Dry with a clean towel. Store your menstrual cup in the pouch, it needs some ventilation. If you want, you can sterilize the menstrual cup after each period by boiling it in water.

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