

CHAIR LIFT

INSTRUCTION MANUAL




-  Hangzhou Guoyuan Yanglao Technology Co., Ltd
-  Room 1603, Building 1, Xixitang Business Center, Cangqian Street, Yuhang District, Hangzhou, Zhejiang, China
-  amzmaidesite@163.com

Table of Contents

| | |
|---|---|
| I. Safety Instructions for Correct Use..... | 1 |
| II. Charger..... | 2 |
| III. Controller..... | 2 |
| IV. Chair Lifts..... | 3 |
| V. Product Structure..... | 4 |
| VI. Applicable Scene..... | 5 |
| VII. Technical Specifications..... | 5 |
| VIII. Maintenance & Repair..... | 6 |

Safety Instructions

I. Safety Instructions For Correct Use

1. Please charge the controller before use.
2. Do not stand on the lift chair.
3. Please sit on the lift chair steadily and use it only after ensuring the environment is safe.
4. Do not use it on surfaces that are excessively inclined (more than 5°) or have steps.
5. Do not use it if any components are missing.
6. Do not allow disabled individuals to use it alone.
7. Do not submerge the lift chair in water when not in use.
8. The maximum weight capacity of the lift chair is 300 pounds.
9. Do not place the controller in a overheated location as it contains a battery.

Before using the chair lifts, it is important to assess the user's suitability, taking into consideration factors such as weight, size, cognitive ability, and joint mobility, among others. It is crucial to prevent unsuitable individuals from using the chair. Disabled individuals should be accompanied by someone or be in close proximity to alarm equipment when using it.

• List

1.Charger



2.Hand Controller



3.Chair Lifts



Operating instructions

II. Charger

It is charged by connecting the charger and the hand controller.

| BearingWeight | Usage Count | Charge Time | Charging interval |
|---------------|-------------|-------------|-------------------|
| 300LBS | 10 | 2.5h | 15 Days |

Please assess the frequency of use based on the user's weight.

Ensure that the handheld controller is fully charged before use. Check the battery level by pressing the buttons on the handheld controller. (When the battery level is above 30%, the indicator light will be green. When the battery level is between 10% and 30%, the indicator light will be blue. When the battery level is below 10%, the indicator light will be red, accompanied by a beeping sound.)




During charging, only the indicator light on the charger will display red. The handheld controller does not have an indicator light.



Please ensure that the charger indicator light turns green to indicate that the handheld controller is fully charged.

III. Controller

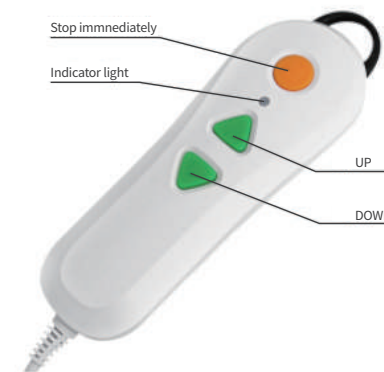

Friendly Reminder


We have summarized the common issues that occur during use. Please pay special attention to the following:

1. Please open the backrest of the lift chair and tilt it 45°. Then connect the hand controller and press the up button to lift the chair.

⚠️ If the lift chair cannot tilt backward, do not force it to start!

⚠️ Check if the bottom slot of the back push rod is correctly reset.

Refer to the diagram for more details.

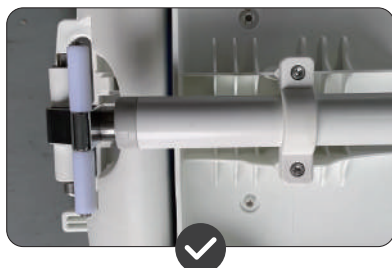

The long fixed rod should be in the upper horizontal position to allow proper tilting of the backrest.



If the short fixed rod is in the wrong position, the chair will not function properly. Please reset it.

2. The lift chair has a weight limit of 300 pounds. Do not exceed the weight limit, as it can be dangerous for you! If there are any functional failures or damages, please contact us for assistance.

3. The indicator light on the hand controller will not illuminate during charging. The charger will inform you about the charging progress, and the charger indicator light will turn from red to green when fully charged. You can also check the battery level by pressing the up and down buttons on the hand controller. (Battery level above 30% will display a green indicator light, 10-30% will display a blue indicator light, and below 10% will display a red indicator light and emit a beep sound.)

Please remember to charge the hand controller in a timely manner. We recommend charging it once a week for 2.5 hours each time.

We provide replacement parts for after-sales service, so please don't worry about any damage issues. We will help you solve any problems properly.

Thank you for choosing our product.

You can contact us anytime.

Email: amzmaidesite@163.com

Phone: 800-620-6618

Operating instructions

IV. Chair Lifts

Pre-check for using the lift chair:

1. Open the lift chair, recline the backrest to a 45° angle, and open the side panels.
2. Connect the handheld controller to the seat lift mechanism.
3. Press the "Down" button to ensure that it automatically tilts backward.
4. Press the "Up" button to ensure that it rises properly.

Before you sit on the lift chair, please operate the lift chair once using the handheld controller to familiarize yourself with its functioning.




Tilt back 45° at the lowest position.



Up/down through electric control.

V. Product Structure

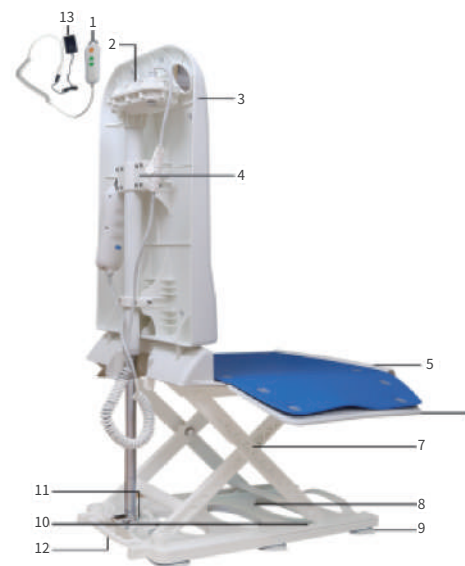

1. Hand Controller
2. Linear actuator
3. Back parts of chair
4. Safety pin
5. Seat parts of chair
6. Board parts
7. Support parts
8. Base parts
9. Sucker parts
10. Slip parts
11. Lock parts
12. Handle
13. Charger

Operating instructions

If there is damage, please check the position according to this page and contact us.

VI. Applicable Scene

Bathroom (Please measure the size of the bathtub to confirm that the bathtub can accommodate the chair lifts.)



Bedroom (Please confirm that the environment is safe before using.)



VII. Technical Specifications

| | |
|-----------------------------|--------------------|
| Controller Charging Voltage | 100-240VAC 50/60Hz |
| Input Voltage | 14.8VDC |
| Rated Load | 1400N |
| Load Speed | 7mm/s |
| Lowest Height | 86 |
| Highest Height | 487 |
| IP Code | IPX8 |

Maintenance & Repair

VIII. Maintenance & Repair

We provide a warranty of 12 months (from the date of shipment) If there is any problem, please contact us for repair, we will reply within 24 hours. Lifetime maintenance is assured. (Please keep the invoice properly.)

After 12 months, we will provide accessory for purchasing, please consult our staff for details.

✉ amzmaidesite@163.com

☎ 800-620-6618